

Instruction Manual

MTR 221 ACS - MTR 221 ACS DHHS

AUTOMATIC RADIOLOGICAL COLLIMATOR

Confidential Information



Ralco
X-RAY COLLIMATORS



SERIES R 221 ACS DHHS

DATE OF ISSUE:

25.05.2019

REVISION LEVEL:

H - 04.10.2021

ISSUED BY:



LEGAL WARNING






THE INFORMATION CONTAINED IN THIS MANUAL IS CONFIDENTIAL. THE CONTENTS OF THIS MANUAL, INCLUDING RALCO SRL SOFTWARE AND THE COLLIMATOR ITSELF CANNOT BE COPIED NOR REPRODUCED IN ANY WAY.

THE CONTENTS OF THIS MANUAL MUST ONLY BE USED FOR THE PROPER INTENDED USE, FUNCTION AND MAINTENANCE OF THE RALCO COLLIMATOR.


RALCO DOES NOT EXPRESSLY OR IMPLICITLY GRANT ANY PROPRIETARY RIGHTS OVER THE RALCO NAME NOR RIGHTS TO MARKET THE RALCO COLLIMATOR BY ANY OTHER NAME.

RALCO SRL WILL PURSUE ANY AND ALL LEGAL REMEDIES AVAILABLE UNDER THE LAW TO RECOVER DAMAGES INCURRED BASED UPON THE ABOVE VIOLATION(S).

Model		Certifications
R 221 ACS	STANDARD VERSION	
R 221 ACS DHHS	FDA CERTIFIED VERSION AVAILABLE ON REQUEST	 

Revision Level:

Rev. N	Date	N
A	29.10.2019	MRD/042/19
B	15.11.2019	MRD/044/19
C	17.04.2020	MRE/006/20
D	01.09.2020	MRM/004/20 MRE/002/20
E	08.04.2021	MRD/001/21
F	29.07.2021	MRM/055/21
G	13.09.2021	MRD/016/21
H	04.10.2021	ACD/012/21

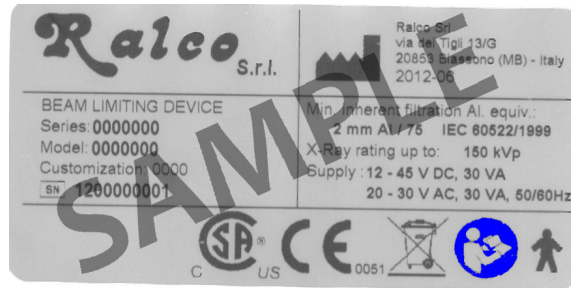

THIS INSTRUCTION MANUAL PROVIDES THE SPECIFICATIONS, DIMENSIONS, AND FUNCTIONS FOR A STANDARD COLLIMATOR. PERSONALIZATIONS ARE AVAILABLE UPON REQUEST. WITH PERSONALIZED COLLIMATORS, THE CUSTOMER MUST ENSURE THE FOLLOWING HAS BEEN PROVIDED:

- YOUR SPECIFIC CODE IS CLEARLY VISIBLE ON THE BACK LABEL AFTER "CUSTOMIZATION", ABOVE THE SERIAL NUMBER, TO ENSURE THE CORRECT PERSONALIZED COLLIMATOR HAS BEEN PROVIDED (SEE ABOVE).
- THE CHAPTER ENTITLED "PERSONALIZATIONS," WHICH INCLUDES RELEVANT INFORMATION REGARDING YOUR PERSONALIZATION.
- OTHER PROVIDED ATTACHMENTS WHICH REFER TO YOUR PERSONALIZATIONS.

CONTENTS

INTRODUCTION

DESCRIPTION

CHARACTERISTICS	8
-----------------------	---

SPECIFICATIONS

INSTALLATION

X-RAY TUBE COMPATIBILITY	16
MOUNTING THE FLANGE TO THE X-RAY TUBE	17
ALIGNMENT OF X-RAY TUBE FOCUS AND COLLIMATOR.....	19
Alignment Device	20
Mounting the Collimator to the Flange	20
VERIFICATION OF CORRECT INSTALLATION.....	23
ELECTRICAL CONNECTION.....	25
Power Supply Connection	26
Wiring Diagram	27

OPERATION INSTRUCTIONS

RADIOGRAPHY	32
Automatic Operation of Square Field (Long and Cross)	32
TOMOGRAPHY.....	33
Tomography Mode.....	33
Exposure Hold Mode	33
Inclinometer HOLD Mode	33
Light Field Operation	33

CALIBRATION

CENTERING X-RAY BEAM WITH COLLIMATOR LIGHT	35
LIGHT FIELD TO X-RAY FIELD ALIGNMENT	36
LIGHT FIELD ADJUSTMENT	37
Vertical Alignment.....	37
Longitudinal Alignment (Long)	37
Light Field Adjustment	38
Transversal Calibration (CROSS)*	38

ELECTRONIC SYSTEM

GC 007 BOARD.....	39
Characteristics	39
Characteristics	43
GC-LED-4A.....	47
GC 24 V Power Supply Converter	52

SYSTEM START-UP

DESCRIPTION OF CONFIGURATION CONTROLS.....	54
UPDATING OF FIRMWARE	55
Functional Overview	55
Bill of Material	55
Installation.....	56
Workflow	56
HW Setup	57
Copy	57
Install	58
Issues List.....	58
CONFIGURATION VIA SERIAL INTERFACE	59
POTENTIOMETER POSITIONING ALARMS	59

CAN BUS MESSAGES

MESSAGE LIST OVERVIEW	60
COLLIMATOR CONFIGURATION.....	61

QUICK START GUIDE

BOARD GC 007.....	93
Description.....	93

COMPLIANCE VERIFICATION

MINIMUM FILTRATION REQUIREMENT	95
Visual Determination of Half-Value Layer (HVL)	96
Quick-Check of Minimum Filtration Requirement at a Particular kVp.....	96
Standard Absorber Method	96
VISUAL DEFINITION OF X-RAY VERSUS LIGHT FIELD	97
FIELD SIZE INDICATION	97
CROSSHAIR ALIGNMENT.....	98
CHAPTER- ADJUSTMENTS, PARAGRAPH CROSSHAIR ALIGNMENT.....	98
LIGHT FIELD ILLUMINATION INTENSITY.....	98
EMC COMPLIANCE	98
Electromagnetic Emissions	99
Electromagnetic Immunity for All Equipment and Systems.....	100
Frequency Range and Level: RF wireless communication EQUIPMENT	100

COVER REMOVAL

ADJUSTMENTS

CROSSHAIR ADJUSTMENT	105
LASER ADJUSTMENT	105
MOTOR ADJUSTMENT.....	106

TROUBLESHOOTING

SUBSTITUTIONS

LED SUBSTITUTION	112
Replacing the Previous Version LED	113
Replacing the New Version LED	114
SUBSTITUTION OF THE ELECTRONIC BOARD FOR LED.....	115
SUBSTITUTION OF THE ELECTRONIC BOARD GC 007	116
SUBSTITUTION OF RETRACTABLE TAPE MEASURE	117
SUBSTITUTION OF THE LASER	118
SUBSTITUTION OF THE MIRROR	119
PHOTOCELL SUBSTITUTIONS	120

SPARE PARTS

LABELS	121
Parts Breakdown	125

OPTIONAL ITEMS

RO 002 Iron Mounting Flange Spacer	126
RO 054 Square Iron Mounting Flange Spacer	127

RO 063 Final Quality Test Report Documentation.....	127	RO 526 Semiautomatic mode (front panel with push buttons included).....	193
RO 074 External housing and guide rails in customized color.....	127	MANUAL CONFIGURATION.....	193
RO 082 Glass Mirror.....	128	Calibration Start-Up.....	193
RO 096 Wiring Customization.....	128	Configuration Procedure.....	194
RO 107 Knob Color Customization.....	128	RO 530 Resin Rotating Mounting Flange.....	203
RO 109 Front panel frame color customization.....	128	RO 533 Glass Mirror.....	203
RO 111 Front panel customization.....	128	RO 537 Addition of electronics and motorized shutter movement to render automatic collimator into manual collimator.....	203
RO 203 Self-centering top-cover bracket for Siemens X-Ray tube.....	128	RO 544 7" Touchscreen Display.....	203
RO 210 Aluminium Accessory Guides.....	130	RO 556 Measuring Tape Removal.....	212
RO 240 Focal-spot to skin spacer.....	130	RO 570 Shutter position verified by encoders.....	212
RO 242/1 Single laser line to align collimator and detector center: Class 2.....	131	RO 582 Metal Fixed Mounting Flange.....	213
RO 242/2 Two lasers (one mounted externally) forming a crosshair to center the patient to the collimator: Class 2.....	133	RO 587/1 Two lasers forming a single line at 1-meter SID: Class 1.....	214
RO 253 Accessory guide rail spacers in standard housing color.....	137	RO 587/2 Two lasers (one mounted externally) forming a crosshair to center the patient to the collimator: Class 1.....	216
RO 278 Fixed Near Focal Shutters.....	137	RO 597 Asymmetric longitudinal and transversal shutters.....	219
RO 305 Additional Variable Filtration - Automatic Selection.....	138	RO 609 Self-Centering Top-Cover Bracket with Zero Position Output Signal.....	219
RO 305/1 Additional Variable Filtration.....	139	RO 619 Resin Rotating Mounting Flange with Position Indicator.....	229
RO 308 External Board ASR003.....	139		
RO 329 Internal Proximity Sensor.....	166	MAINTENANCE	
RO 330 Connecting cable extension.....	168	CLEANING RECOMMENDATIONS.....	230
RO 332/A Resin Rotating Mounting Flange.....	169	DISINFECTION.....	230
RO 336 Aluminium accessory guide rails.....	169	RECOMMENDED MAINTENANCE PROGRAM.....	230
RO 344 Accessory guide rail spacers with reinforced support.....	170		
RO 356 Shutter position verified by potentiometer.....	170	GENERAL	
RO 405 Housing Customization.....	170	WARRANTY.....	232
RO 409 Resin Rotating Mounting Flange.....	171	REPAIRS.....	232
RO 441 Resin Rotating Mounting Flange.....	171	END OF LIFE DISPOSAL.....	233
RO 442 Resin Rotating Mounting Flange.....	172	DISASSEMBLY.....	233
RO 444 Can Open Protocol.....	172	TRANSPORT AND STORAGE.....	233
RO 445 Metal Rotating Mounting Flange.....	173	SAFETY/RESPONSIBILITY.....	234
RO 456 Metal Rotating Mounting Flange.....	174	RESIDUAL RISKS.....	235
RO 476 Asymmetric longitudinal and transversal shutters.....	174		
RO 484 Infrared Remote Control Box.....	176		
RO 489 Resin Rotating Mounting Flange.....	177		
RO 490 Metal Rotating Mounting Flange.....	178		
RO 492 Adjustable Top-Cover Bracket.....	178		
Alignment Device.....	182		
Mounting the Collimator to the Flange.....	183		
RO 493 Dual Energy Additional Variable Filtration.....	185		
RO 494 Customized Additional Variable Filtration.....	186		
RO 495 Camera assembled internally for patient monitoring.....	186		
RO 496 Software Customization.....	192		
RO 510/A Resin Rotating Mounting Flange.....	192		
RO 525 Customized Knob design.....	193		

INTRODUCTION

General Safety Information

Ralco products are designed and manufactured to meet the international safety standards for medical equipment. However, all medical electrical equipment requires proper installation, operation and servicing, particularly with regard to human safety. Read, note, and strictly observe all safety tags on the equipment. Strictly observe all safety directions, all warnings and all cautions that are mentioned in this chapter and throughout this service documentation. In order to protect the personal health of service personnel, operators and patients, ensure the safe servicing and operation of the system.

Intended Use of the Collimator

This collimator is designed to be used with a Radiological system producing ionizing radiation for medical or veterinary use. Whomever is authorized to operate or service the radiological equipment must be thoroughly familiar with the procedures related to radiation protection and equipment use and maintenance. Ralco is not responsible for any personal injuries or damage to property from misuse or unintended use of this collimator. This manual must be made available to the installer and operator.

Installer and Operator Responsibility

The installer and operator must verify that ALL safety standards are followed immediately after installation of the collimator and before any subsequent use. All procedures regarding the installation and safe use of this collimator must be strictly followed.

X-ray equipment manufacturer responsibility

If the X-ray equipment manufacturer does not use the READY signal, allowing X-ray exposures regardless of collimator state or status, all liability falls upon the X-ray equipment manufacturer. In the event of any emergency and the READY or ERROR signals are bypassed allowing X-ray exposures, all the liability falls upon the X-ray equipment manufacturer.

The collimator electronics supplies a signal defined as READY. The X-ray equipment manufacturer will manage this signal as the X-ray consent. In the event of a collimator failure, the liability for the correct bypass of this signal, in order to always ensure the X-ray consent, devolves upon the X-ray equipment manufacturer.

Reporting

Information regarding accidents that have occurred while using this collimator must be reported immediately to Ralco, srl.

Limitation of Liability

Ralco is not liable if the provided instructions are not complied with. Furthermore, Ralco is not liable if one or several of the following instances apply:
If the unit is specifically designed per client specifications and the certification was the duty of the client;

The collimator was modified in any way by the OEM or operator;
The collimator was installed without respecting the instructions, as provided in this manual;
The collimator was used in a way outside its intended use;

The collimator was not installed by competent personnel;
The collimator was not operated safely or in a way contrary to the instructions in this manual;
The collimator was not subject to routine inspection and maintenance by competent personnel;
The collimator was repaired with non-Ralco spare parts;

The Collimator was Used in a Way Not Reasonably Foreseen by Ralco

Ralco is not liable for any direct or indirect damage caused if the procedures in this manual are not followed.

The collimator complies with current standards for static load, a non-wearing class. This statement is based on studies and tests performed on the collimator performance and its weight. If the final system, through tilting movements or other dynamic movements, increases the risk factor other than those studied and tested, it is the responsibility of the customer to ensure that there are no dangerous situations.

The collimator described here, is used on Radiological systems and is classified as a type IIB according to Attachment IX. The collimator has been designed and manufactured in compliance to Attachment II of Legislative Decree, 2 February 1997, N. 46, Implementation of Directive 93/42/CEE of 14 June 1993 and successive modifications applying Directive 2007/47/CE.

List Applicable Standards and to which Ralco adheres to:

IEC 60601-1:2005 + Am1:2012 (ed. 3.1), North American market deflections (US+CA)
IEC 60601-1-2:2014 (ed 4)
IEC 60601-1-3:2008 + Am1:2013 (ed. 2.1)
IEC 60601-2-54:2009 + Am1:2015 (ed. 1.1)
IEC 60601-2-45:2011 + Am1:2015 (ed. 3.1) Applicable in Mammography instead of IEC 60601-2-54:2009
IEC 60601-1-6:2010 + Am1:2010 (ed. 3.1) (Usability)
IEC 62304:2006 + Am1:2015 (ed. 1.1) (Software)
IEC 60825-1:2014 (LASER - if applicable)
IEC 62471:2006 (LED - if applicable)
ISO 9001:2015
ISO 13485:2016

21 CFR subchapter J (FDA)
8750 01 & 8750 81 (CSA)

CE

*Ralco can provide documentation regarding its adherence to any of the above standards.

EN 60601-1 par. 5

Protection against electric hazards: "Class I" equipment.

Protection against direct and indirect contacts: Type B equipment with applied parts.

Protection against water see page: "Common equipment".

Safety of operation in the presence of inflammable anaesthetics with air or oxygen or nitrous oxide: Equipment not suited to application in the presence of inflammable anaesthetic mixtures containing air or oxygen or nitrous oxide.

- Operation conditions: Equipment for continuous operation at intermittent loads - See Chapter Operation Instructions.
- Should label data on the collimator not correspond to the specifications herein, inform Ralco of the non conformity.
- Verifications of the specifications are to be performed according to the indicated equipment standards.



Inherent Danger of X-Rays

The collimator has been constructed to current standards to meet the safety requisites of Directive 2007/47/CE and all other applicable standards. However, due to Radiology being an inherent dangerous activity which cannot be completely safe guarded against, it is crucial that all safety procedures are followed. The installer and operator must follow ALL established procedures (including those mentioned in this manual) to reduce the inherent danger of X-Rays.

The inherent risk of using collimators in Radiological systems is deemed reasonable as determined by applicable standards. The use of collimators for Radiological systems is strictly regulated. Ralco follows all applicable standards. It is up to the installer and operator to ensure that all possible steps are followed to ensure the health and safety of the patient and operator.

The contact with the moving parts can be considered impossible and may occur only during installation or maintenance (low hazard). The internal parts are protected by a mechanical safeguard in the form of a metal or plastic sheath which impedes access. Also motor units are not accessible as knobs are fixed using the Allen screws and can be removed only by using a tool, in accordance with paragraph 5.92.3 of EN 60601-1 and EN 60601-2-54 par. 201.9.2.2.5. In addition the collimator has been tested in accordance with EN 60601-1, paragraph 4.8.2



Read all information in this chapter carefully.

You are responsible to take safety precautions and to follow the safety instructions. Thus, you can prevent harm form yourself and other persons.

TECHNICALLY QUALIFIED PERSONNEL ONLY!

- Only technically qualified Field Service Engineers (FSEs) must do the service work. The FSEs must be appropriately and successfully trained and instructed.
- Perform all service tasks such as installation, maintenance, or repair in strict compliance with the provided safety instructions!
- Perform all action as for example unpacking, installation, setting to work, testing and replacement according to the sequence stated in this manual.
- Perform all service tasks in strict compliance with the local regulations concerning safety, health, accident prevention, and medical X-ray devices!
- Never change any system parameters or components that reduce the mechanical safety, electrical safety, radiation protection properties of the product!
- Do not modify this equipment without authorization of Ralco.
Any unauthorized modification can cause malfunction or deterioration of performance and quality and can therefore lead to personal injury, clinical misdiagnosis or clinical mistreatment.
- Never leave problems unsolved that may affect the safety of the product!
- Order spare parts via Ralco distribution channel.
- Always perform all relevant safety checks before handing the product over to the customer!
- Keep in mind that certain details or procedures change since you previously installed or serviced a similar product!

If you do not obey these instructions, there is a risk of death or serious injury.



A temperature of the collimator cover, under normal operating conditions and at the maximum ambient operating temperature (40°C), could exceed 41°C. According to the Table 24 of the General IEC 60601-1 standard, contact with the collimator cover must be less than 1 minute, both for the operator and for the patient. During the diagnostic test the operator must pay attention that the patient does not come into contact with the collimator surface for more than 1 minute.

If you do not obey these instructions, there is a risk of serious injury.

Precautions Against Mechanical Hazards

SAFE HANDLING OR ROTATING MACHINERY

- Never service rotating machinery, bands or chains when rotational movements are activated.
- Make sure that the rotational movements are switched off.
- Make sure that nobody can switch on the movements accidentally.
- Block the movement, if necessary.

Safe handling of tilting machinery

- Never service tilting machinery when tilting movements are activated or brakes are released.
- Make sure that the tilting movements are switched off.
- Make sure that nobody can switch on the movements accidentally.
- Block the movements, if necessary.

If you tilt the machinery during installation or during other service activities:

- Comply with the specific service instructions.
- Ask a second person to release the brake and to keep the tilting movement under control.
- Avoid uncontrolled tilting movement.
- Never release the brake longer than necessary.



Hazard of physical injury caused by heavy loads

- Wear appropriate protective clothing, such as safety boots and gloves.
- Take care that heavy loads are correctly lifted or carried to avoid any physical injury.
- Ask other persons for help to handle very heavy or awkwardly shaped loads.
- Use mechanical devices whenever it is possible.
- Stick to the installation instructions regarding:
 - suitable tools
 - lifting devices
 - lifting/support points
 - particular safety measures

If you do not obey these instructions, there is a risk of property damage.

Safe handling of light source

Light source and adjacent parts can be very hot. They can cause severe burns.

- Never touch the light source, light source socket, or light source bracket with your fingers immediately after use.
- Leave the light source and adjacent parts to cool down before handling them.

Precautions Against Electrical Hazards



Hazard of electrical shock during service tasks

During installation, maintenance, and other service tasks there is a risk of electrical hazards.

- Follow the general instructions provided in this chapter.

If you do not obey these instructions, there is a risk of property damage.

General precautions against electrical hazards

- Switch off the mains supply.
- Make sure that nobody can switch on the system accidentally before you start.
- Make sure that all UPS-powered components are powerless.
- Use a voltmeter to verify that all line disconnects are opened.
- Remove all jewellery, such as bracelets, or rings to prevent short circuits.

Precautions concerning installation and service

Electrical installations of medically used rooms must comply with the requirements of each country.

Follow the instructions provided on the installations sheets for the specific project.

- Always use a separate conductor for ground connection. The neutral conductor is not considered as ground connection.
- Do not use a three-to-two pins adapter to connect equipment provided with a power cord. Always connect the equipment to properly grounded, three-pin power outlets.
- Connect the system to the mains as late as possible during installation.
- Make sure that all protective earth conductors are installed and connected before first switching on the system.
- Always be sure that the power is switched off when installing any parts to the equipment.
- Make sure that the large internal capacitors are completely discharged. Discharging can take several minutes.
- Keep away water or other liquids from the inside of the equipment to avoid short-circuits or corrosion.
- After installation or service work on electrical components the following tests have to be performed to ensure the compliance with IEC 62353:
 - Touch voltage
 - Protective earth conductor resistance
 - Earth leakage measuring

Working when connected to voltages

- Switch off the main power supply when working at the equipment. This rule does not apply for certain measuring and adjustment procedures that are only possible when the product is switched on.
- Never measure or adjust equipment with power-on unless another person is present.
- Be very careful when working close to live contacts.
- Never perform any work on live parts (> 50 V).
- Prevent unwanted or sudden movements of the system because they are dangerous. If the procedure involves mechanical movements, take every precaution especially when working with moving and rotating parts. Use the service switch or the EMERGENCY STOP button before working within the dangerous movement area of a product.
- Check the ground connections before touching conductive system parts.
- Use tools and measuring instruments, which are suitable for the respective procedure only.
- Make sure that test and adjustment points are accessible without any risk of injury. If safe access is not possible, switch off the system.
- Turn off the power supply immediately after finishing the test or adjustment procedure.

Safe handling of high-voltage cables

- When the mains and auxiliary power supplies are disconnected: Wait at least two minutes before removing the flexible high-voltage cables from the X-ray tube housing or the generator.
- Discharge any residual charge before touching the contact pins (briefly connect them to ground).

Precaution Against Material Damage

ENVIRONMENTAL CONDITIONS FOR SAFE OPERATION

- The oxygen content of the ambient air during operation must be lower than 25%!
- The X-ray equipment must not be operated along with combustible anaesthesia substances!
- Ensure that the indoor temperature is within +10 - +40°C.
- Ensure that the indoor relative humidity is within 10 - 75%.
- Ensure that the air pressure is within 700 - 1060 hPa

Cleaning and disinfection

- Do not use aggressive cleaning agents to clean the product!
- When you use cleaning agents such as detergents and disinfectants, make sure that they do not contain explosive substances as they can create explosive gas mixtures!
- Obey the manufacturer's data of the cleaning agent and disinfectant!
- Obey the manufacturer's data of the cleaning clothes!



Malfunction through electromagnetic fields

The equipment fulfils all requirements concerning electromagnetic compatibility (EMC). nevertheless, powerful high frequency transmitters used near the electronic components can cause malfunction of electronic components under unusual circumstances. This malfunction causes unwanted risks for patients, operators, and service personnel.

- Switch off your mobile phone in designated areas.

If you do not obey these instructions, there is a risk of property damage.



Processor overheating

The processor can get damaged within a few seconds.

- Do not operate the processor without cooling.

If you do not obey these instructions, there is a risk of property damage.



Damage by electrostatic discharge (ESD)

ESD can destroy or partially damage electrostatic sensitive devices such as printed circuit boards (PCB). The damage can lead to instant or later failures of the device.

- Read all the following instructions concerning ESD protection.
- Take all necessary precautions to protect electrostatic sensitive devices.

If you do not obey these instructions, there is a risk of property damage.



The rules concerning ESD protection apply to new devices and old devices and old devices. Old devices are replaced and sent back to the service depot.




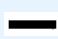

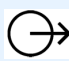





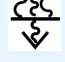

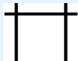

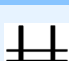



Additional symbols on the collimator (if applicable)



or **AC** Alternating Current No. 01-14 Ref. IEC 417-IEC 503



or **DC** Direct Current No. 01-18 Ref. IEC 417-IEC 5031

	or AC/DC Alternating and Direct Current No. 01-19 Ref. IEC 417-IEC 5033
	Protective Ground No. 01-20 Ref. IEC 417-IEC 5019
	Plus, Positive Polarity No. 01-27 Ref. IEC 417-IEC 5019
	Minus, Negative Polarity No. 01-28 Ref. IEC 417-IEC 5006
	Input No. 01-36 Ref. IEC 417-IEC 5006
	Output No. 01-37 Ref. IEC 417-IEC 5034
	Remote Control No. 01-38 Ref.
	Manual Control No. 01-45 ISO 7000-096
	Automatic Control (Closed Loop) No. 01-46 ISO 7000-0017
	Iris Diaphragm: Open No. 01-69 ISO 7000-0017
	Iris Diaphragm: Closed No. 01-70 Ref. 417-IEC 5324
	Radiation Filter or Filtration No. 04-51 Ref. 417-IEC 5381
	Light Indicator of Radiation Field No. 04-51 Ref. 417-IEC 5381
	Beam Limiting Device: Open No. 04-55 Ref. 417-IEC 5385
	Beam Limiting Device: Closed No. 04-56 Ref. 417-IEC 5386
	Beam Limiting Device: Closed with Separate Opening of the Shutters No. 04-57 Ref. 417-IEC 5387
	Beam Limiting Device with Separate Closing of the Shutters No. 04-58 Ref. 417-IEC 5388
	Cassette Size Sensing Device
	Device Requiring Proper Disposal, Attch. 4 Ref. 2002/95/CE



Type B Applied Part No. 02-02 Ref. 601-I-IEC (Not applicable for R 915, R 915 S and R 915 S DHHS)



Follow Instructions for Use Ref. ISO 7010-M002 (see the collimator label)



General Warning Sign No. 03-02 Ref. IEC 601-1



Caution: Laser Radiation Ref. 60825-1

This symbol identifies electrostatic sensitive devices.

- Always transport electrostatic sensitive devices (such as PCB) in their static shielding bags or boxes.
- Always use the ESD protection service kit when handling electrostatic sensitive devices.
 - Attach the wristband to your wrist (1) and connect the ground cable of the wristband to the conductive work surface (2).
 - Connect the second ground cable (3) to the conductive work surface (2).
 - Attach the crocodile clip to a reliable ground of the cabinet or console, where the PCB has to be installed.
- Place the PCB on the conductive work surface before installing them.
- Place the PCB on the conductive work surface for hardware settings or replacement of components.
- Never remove or disconnect the wristband before all electrostatic sensitive devices are installed.
- Never remove or disconnect the wristband before all removed electrostatic sensitive devices are safely packed in their static shielding bags or boxes.

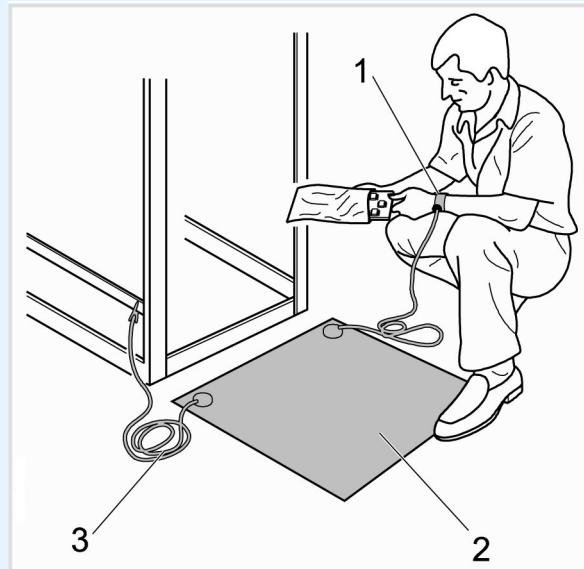


Fig. 1 ESD protection kit

Safety Signs on the Product

- Do not remove or change safety signs!
- Replace illegible safety signs by genuine spare parts!
- Clean soiled safety signs!

Refer to the labelling drawings of the components to see the positions of the labels.



Safety of Personal Protective Equipment

- The collimator does not contain polluting materials or products with the exception of the lead that composes the shutters and an inner lining of the collimator cover, if properly disposed of at the end of its life cycle.
- Avoid direct contact with lead especially for prolonged periods.
- Gloves must be worn when handling the disassembled collimator, especially the parts mentioned above.
- Do not use for any other work.
- Disposable gloves may also be worn and can be disposed with normal trash.



Disposal

- Dispose any material, batteries and X-ray equipment in accordance with the requirements of national legislation!
- Do not dispose X-ray equipment together with domestic waste!



- All Ralco's products conform to RoHS



- All Ralco's products conform to REACH

Safety Messages Reported Into The Manual



WARNING

This symbol combined with the signal word **WARNING** indicates a hazardous situation which, if not avoided, could result in death or serious injury.

If you do not obey these instructions, there is a risk of death or serious injury.



CAUTION

This symbol combined with the signal word **CAUTION** indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

If you do not obey these instructions, there is a risk of minor or moderate injury.

The shown symbols are sometimes replaced by other symbols which indicate the concrete risk:

Material Damage



NOTICE

This symbol combined with the signal word **NOTICE** indicates a hazardous situation which, if not avoided, could result in damages such as material damage. This damage is not related to personal injury.

If you do not obey these instructions, there is a risk of property damage.

Other Messages



Important

This symbol combined with the signal word **Important** indicates an important advice that points out that certain guidelines, parameters, conditions or restrictions must be observed.

For any inquires or notifications, please contact us at:

Ralco, srl
Via dei Tigli 13/G
20853 Biassono, (MB) Italia
Telephone: +39 039 2497925
Fax: +39 039 2497799
E-mail: ralco@ralco.it

DESCRIPTION

Multilayer, square field, automatic collimation system with a lightweight and compact design intended for installation on mobile and stationary X-ray equipment. This device has been designed and manufactured for skeletal investigations and ER applications.

The X-ray field is defined by six pairs of shutters, four of which are lead-lined. The six pairs of shutters move perpendicularly within the X-ray field. Two pairs of brass shutters are located near the focus, two are located near the entrance window and two are located near the exit window of the X-ray beam from the collimator. The latter shutters serve to accurately define the X-ray field edges.

Shutter movements are motorized, controlled by stepper motors. Shutters may be adjusted manually with the two knobs on the collimator front panel or remotely via CanBus controls.

CHARACTERISTICS

Mounting Plane at 80 mm (3.14") from the focus.

Continuous Film Coverage from Min: 00 x 00 cm to Max: 48 x 48 cm at 100 cm (40") SID.

Maximum Radiation Leakage: 150 kVp - 4 mA

High luminosity provided by a **White LED** simulating the X-ray field. The light field is controlled by an electronic timer.

Minimum Inherent Filtration 2 mm aluminium equivalent.

Internal Accessory Guides used for DAP chamber, if present.

Retractable Tape mounted on the radiological unit, measures the distance between the focus and the patient.

Resin Rotating Mounting Flange allows the collimator to rotate, 20mm thickness, 0° detent, 140mm diameter.

Single Laser for the alignment of the collimator to the receptor (Class 1).


GC 007 electronic board which controls the two stepper motors of square field and one motor for filter positioning.

GC 009 electronic board which controls the display, inclinometer and 2 encoders.

GC-LED-4A timer board for light source supply and operation.

Power Cable 10 m.

Fig. R 221 ACS - R 221 ACS DHHS Front Panel with Exterior Dimensions

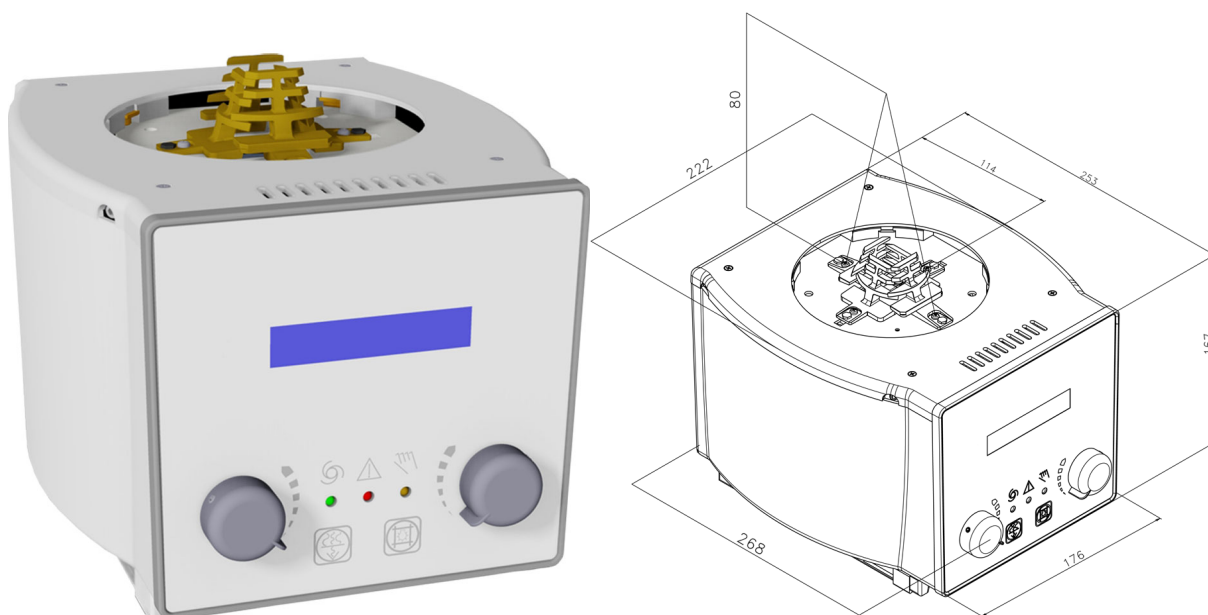

MTR 221 ACS - MTR 221 ACS DHHS - Description

This collimator may have the following optional items; a detailed description is provided in the chapter **OPTIONAL ITEMS**.

RO	DESCRIPTION
RO 002	Iron mounting flange spacer: 1.5mm thickness
RO 054	Square iron mounting flange spacer: 1.5mm thickness
RO 063	Final quality test report documentation (Light field, luminosity, light to X-ray field correspondence, light field border contrast ratio, x-ray leakage, control of general functions)
RO 074	External housing and guide rails in customized color
RO 082	Glass mirror with minimum internal inherent filtration: 1mm Al equivalent
RO 096	Wiring customization
RO 107	Knob color customization
RO 109	Front panel frame color customization
RO 111	Front panel customization
RO 203	Self-centering top-cover bracket for Siemens X-ray tube only (mounting flange provided by Siemens only)
RO 210	Aluminium accessory guide rails painted in standard housing color (unless RO 074 selected)
RO 240	Focal-spot to skin spacer
RO 242/1	Single laser line to align collimator and detector center: Class 2
RO 242/2	Two lasers (one mounted externally) forming a crosshair to center the patient to the detector: Class 2
RO 253	Accessory guide rail spacers painted in standard housing color (unless RO 074 selected)
RO 278	Fixed near focal shutters (single layer collimation) for use with a mounting plane of 80mm from the X-ray focus
RO 305	Additional variable filtration - automatic selection. 4 position rotating wheel with selectable filters (clockwise): (1) empty or (2) 0.1mm Cu+1mm Al or (3) 0.2mm Cu+1mm Al or (4) 2mm Al

MTR 221 ACS - MTR 221 ACS DHHS - Description

RO	DESCRIPTION
RO 305/1	Additional variable filtration - automatic selection. 4 position rotating wheel with selectable filters (clockwise): (1) empty or (2) 0.1mm Cu or (3) 0.2mm Cu or (4) 0.3mm Cu
RO 308	External analogic/digital interface in metal housing with 10 meter connecting cable
RO 329	Internal proximity sensor
RO 330	Connecting cable extension
RO 332/A	Resin rotating mounting flange: 15mm thickness, 0° detent/stop, 140mm diameter including fixing screws (not available with RO 492 or RO 498)
RO 336	Aluminium accessory guide rails (rail distance 150mm and 176mm)
RO 344	Accessory guide rail spacers with support painted in standard housing color (unless RO 074 selected)
RO 356	Shutter position verified by potentiometer: Output signal managed by customer
RO 405	Housing customization: Aluminum
RO 409	Resin rotating mounting flange milled to form a square: 20mm thickness, +/-0° detent, 140mm diameter + three iron mounting flange spacers milled to form a square: 1.5mm thickness (not available with RO 492 or RO 498)
RO 441	Resin rotating mounting flange: 20mm thickness, +/-50° detent, 140mm diameter (not available with RO 492 or RO 498)
RO 442	Resin rotating mounting flange: 20mm thickness, +/-90° detent, 140mm diameter (not available with RO 492 or RO 498)
RO 444	Collimator control protocol: CANopen (CiA 412-2)
RO 445	Metal rotating mounting flange, 20mm thickness, +/-90° mechanical stop, 140mm diameter (not available with RO 492 or RO 498)
RO 456	Metal rotating mounting flange: 20mm thickness, +/-90° mechanical stop, 136mm diameter (only available with RO 492)
RO 476	Asymmetric longitudinal and transversal shutters (increased housing dimensions: 244x282x216cm - R 225 ACS housing)
RO 484	Infrared control box to command collimator functions (only available with RO 476)

MTR 221 ACS - MTR 221 ACS DHHS - Description

RO	DESCRIPTION
RO 489	Resin rotating mounting flange: 20mm thickness, +/-45° detent, 140mm diameter (not available with RO 492 or RO 498)
RO 490	Metal rotating mounting flange, 18mm thickness, +/-90° mechanical stop, 140mm diameter with countersunk mounts (only available with RO 492)
RO 492	Adjustable top-cover bracket for use with metal mounting flange (mounting flange not included)
RO 493	Dual energy additional variable filtration: 200ms per 90° movement (filters not included- only available with RO 494)
RO 494	Customized additional variable filtration
RO 495	Camera assembled internally for patient monitoring: IP ethernet interface
RO 496	Software customization
RO 510/A	Resin rotating mounting flange: 18mm thickness, +/-90° detent, 140mm diameter (not available with RO 492 or RO 498)
RO 525	Knob customization
RO 526	Semiautomatic mode (front panel with push buttons included)
RO 530	Resin rotating mounting flange: 17mm thickness, +/-0° detent, 140mm diameter (not available with RO 492 or RO 498)
RO 533	Glass mirror + aluminum plate with minimum internal inherent filtration: 2.3mm Al equivalent
RO 537	Addition of electronics and motorized shutter movement to render automatic collimator into manual collimator
RO 544	7" touchscreen display
RO 556	Measuring tape removal
RO 570	Shutter position verified by encoders
RO 582	Metal fixed mounting flange: 16mm thickness, 136mm diameter (only available with RO 492)
RO 587/1	Two lasers forming a single line at 1-meter SID: Class 1

MTR 221 ACS - MTR 221 ACS DHHS - Description

RO	DESCRIPTION
RO 587/2	Two lasers (one mounted externally) forming a crosshair to center the patient to the detector: Class 1
RO 597	Asymmetric longitudinal and transversal shutters
RO 609	Self-centering top-cover bracket with zero position output signal using a microswitch (resin mounting flange not included)
RO 619	Resin rotating mounting flange: 15mm thickness, +/-90° detent, 140mm diameter, 0° position tolerance +/-3° with fixing screws (only available with RO 609)

SPECIFICATIONS

NOTE



SPECIFICATIONS OF THE BASIC VERSION OF THE COLLIMATOR ARE DESCRIBED HERE BELOW.

POWER SUPPLY

Power Supply	24V AC/DC, 50/60 Hz, 3.5A
Fuse for power supply protection collimator. Not supplied by Ralco.	T 4 A
Motor Supplied by Collimator Board	GC 007
Fuse for Power Supply Protection Motor. Not supplied by Ralco	T 4 A
Potentiometers/Encoders: Square Field Round Field	Optional item Optional item

ELECTRONIC SPECIFICATIONS

Software GC 007, GC 009, GC-LED-4A	The software version of the collimator boards is documented by the Final Control and Testing Form attached to the collimator.
Power Supply, External PCB	n.a.
Fuse for External Board	n.a.

RADIOLOGICAL SPECIFICATION

Inherent Filtration Al. Equivalent X-ray beam = 75 kV EN 60601-1-3 par. 7.3; 7.4	Min. Al Eq. 2 mm (1 mm on request)
----------------------------------------------------------------------------------------	------------------------------------

RADIOLOGICAL SPECIFICATION

Filtration, Additional
X-ray beam = 75 kV
EN 60601-1-3 par. 7.5

Optional item

Limitation of Extra Focal Radiation
Set Focus Distance, SID 100 cm (40")
EN 60601-2-54

< 150 mm

Square X-ray Field Selection
100 cm (40") SID - ($\pm 1\%$ SID)
EN 60601-2-54

Min: 00 x 00 cm
Max: 48 x 48 cm

Round X-ray Field Selection
1 m SID - ($\pm 1\%$ SID)
EN 60601-2-54

n.a.

Light Field Indicator
Luminosity at 1 m from the focus, field size set at 35x35 cm.
EN 60601-2-54

> 160 lx

Light Field Indicator - Contrast
Edge Contrast Setting 35x35 cm at 1 m SID
EN 60601-2-54

> 4:1

Light Field Indicator Precision
Light Field/X-ray Field Correspondence
EN 60601-2-54

< 1 % SID>

Accuracy of X-ray field as shown on the front display versus actual
X-ray field dimension.

< 2 mm per side>

X-ray Field Indication Precision
Settings on an Index Scale
EN 60601-2-54

< 2% SID>

SID: (optional)
Precision of Measurement with Retractable Tape
EN 60601-2-54

< 2% SID

Maximum Radiation Leakage
Measured at 100 cm with X-ray Beam = 150 kVp - 4 mA
EN 60601-1-3

< 40 mRh

GENERAL SPECIFICATIONS

Operation Environment
Ambient Temperature
Relative Humidity
Atmospheric Pressure

10° - 40°C
10% - 75%
700 - 1060 hPa

Storage
Temperature
Humidity
Atmospheric Pressure

-40° - +70°C
10% - 95%
500 - 1060 hPa

Weight*

* The weight may vary according to the optional items mounted to the collimator.

9.5 Kg

Dimensions

L: 268 mm
W: 222 mm
H: 167 mm

Maximum Load for Accessory Rails and Dimensions for Accessory

Static load: 70 N (about 7.1 Kg)
Dynamic load: 15 Nm (approx. 3.06 Kg)

Distance between Rail Slots

152 mm (+/- 0.5 mm)**

**



ATTENTION:

THE DISTANCE BETWEEN TWO RAILS VARIES AND DEPENDS ON THE COLLIMATOR MODEL.
PRIOR TO INSERTING ACCESSORIES IN THE RAILS MAKE SURE THAT THE DIMENSIONS OF THE
ACCESSORY MATCHES WITH THE DISTANCE BETWEEN TWO RAILS SLOTS (TOLERANCE MAX.
± 0,5 MM).

THE INCORRECT DIMENSION CAN LEAD TO DANGEROUS SITUATIONS AND/OR TO THE ACCESSORY
FALLING.

SEE THE CHAPTER **OPTIONAL ITEMS** IF THE RAILS DIFFER FROM THE STANDARD ONES.

INSTALLATION

WARNINGS



THE COLLIMATOR MUST BE INSTALLED TO THE X-RAY TUBE THROUGH A MOUNTING FLANGE. RALCO PROVIDES VARIOUS FLANGE OPTIONS WHICH MAY NOT BE INTERCHANGEABLE. ONLY FLANGES PROVIDED WITH THE COLLIMATOR MAY BE UTILIZED. THE END-USER MAY INSTALL THEIR OWN FLANGE, HOWEVER RALCO CANNOT GUARANTEE COMPATIBILITY. ANY PREEXISTING FLANGE ON THE END-USER SYSTEM MUST NOT BE USED.



WHEN A FLANGE IS PROVIDED WITH THE COLLIMATOR BEARING A MATCHING SERIAL NUMBER, ENSURE THEY ALWAYS REMAIN COUPLED. IT IS MANDATORY THEY REMAIN TOGETHER AND THE CORRECT FLANGE PART NUMBER IS USED.



THE CONTENTS OF THE INSTRUCTIONS BELOW SHOULD BE STRICTLY ADHERED TO. RALCO IS NOT LIABLE FOR ANY PROPERTY DAMAGE OR RESULTING HARM IF NON-RALCO COMPONENTS OR NON-COMPATIBLE RALCO COMPONENTS ARE USED DURING THE INSTALLATION PROCESS.

X-RAY TUBE COMPATIBILITY

- Using the dimensions in **Fig. Tube Compatibility** below, ensure the rear port shutters of the collimator are placed in the X-Ray tube port without interference.
 - Assembly values (X-Ray tube housing and collimator) must comply with a minimum value of inherent filtration and a maximum value of radiation leakage.
- The distance between the X-Ray tube focus and the flange mounting plane (collimator upper plate) must be: 80 mm (3.14"), tolerance ± 1 mm (0.04 ").

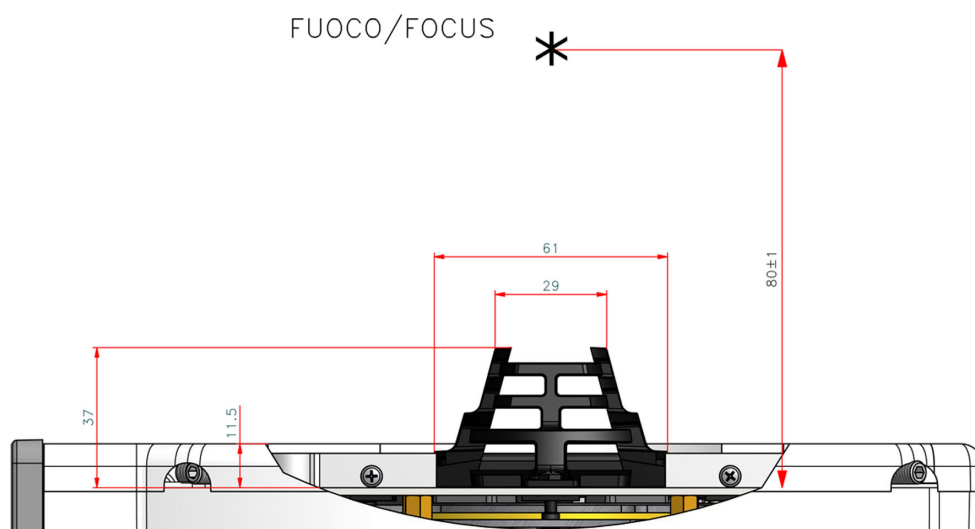


Fig. Tube Compatibility

NOTE



IF THE COLLIMATOR IS EQUIPPED WITH FOUR INDEPENDENT SHUTTERS, REMOVE THE LOCKING SCREWS BEFORE STARTING TO USE THE COLLIMATOR. FOLLOW THE PROCEDURE IN THE CHAPTER OPTIONAL ITEMS, SUBCHAPTER RO 476.

3. Carefully remove the collimator and the mounting flange from their packaging.
4. Use the X-Ray tube housing datasheet to determine the distance (**A**) from the focal spot to the X-Ray tube port, see **Fig. Collimator Installation**.
5. Subtract the resulting distance from the source flange distance (**B**) and determine the number of spacers (1.5 mm) which, combined with the thickness of the mounting flange, will make up the difference (**C**). Allowable tolerance is 1 mm. (0.04"), see **Fig. Collimator Installation**.
6. Once the mounting plane distance has been confirmed, continue with the mounting flange installation to the X-Ray tube.

The flange fixing screws and the spacers of the previous flange may be reused if the flange thickness is the same.

7.

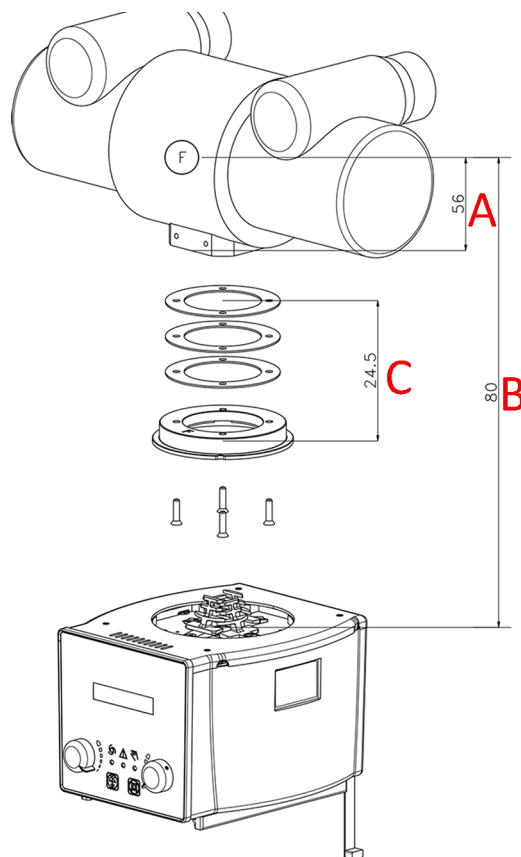


Fig. Collimator Installation

MOUNTING THE FLANGE TO THE X-RAY TUBE

NOTE



THE FOLLOWING MOUNTING INSTRUCTIONS ARE ONLY APPLICABLE FOR COMPATIBLE FLANGES. IF YOU ARE UNSURE THE MOUNTING FLANGE YOUR COLLIMATOR IS EQUIPPED WITH IS COMPATIBLE, PLEASE CONSULT YOUR PERSONALIZATION PAGE PROVIDED WITH THIS MANUAL TO LOCATE THE FLANGE PART NUMBER (RO REFERENCE). FOLLOW THE MOUNTING INSTRUCTIONS SET FORTH UNDER THE SPECIFIC RO REFERENCE IN THIS MANUAL.

WARNING



FLANGES MAY BE PROVIDED BY RALCO OR BY THE SYSTEM MANUFACTURER. FLANGES MAY NOT BE INTERCHANGEABLE. ONLY THE FLANGE PROVIDED WITH THE COLLIMATOR BEING INSTALLED WITH A SPECIFIC PART CODE MAY BE UTILIZED. ANY PRE-EXISTING FLANGES MAY NOT TO BE USED. IF THERE ARE ANY QUESTIONS REGARDING COMPATIBILITY, PLEASE CONTACT RALCO.

CAUTION



RALCO CANNOT GUARANTEE COMPLIANCE WITH RADIATION STANDARDS CONCERNING SAFETY IF THIS CONTROL HAS BEEN OMITTED.

1. Place the flange on the X-Ray tube port, see **Fig. Flange Installation**.
2. Mount the mounting flange and spacers (optional) to the X-Ray tube port using 4 screws.**

****Please ensure no conflicting information nor dangerous conditions exist due to adhering to these instructions or those provided by the X-Ray tube manufacturer. When in doubt please contact X-ray tube manufacturer and/or Ralco.**

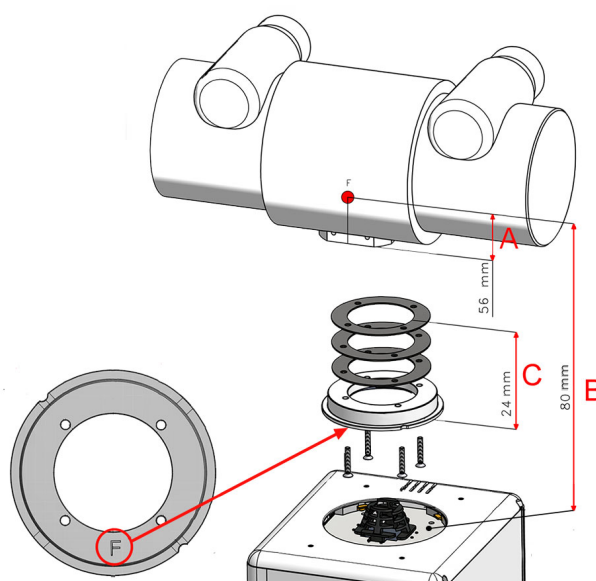


Fig. Flange Installation

*illustrative purpose only

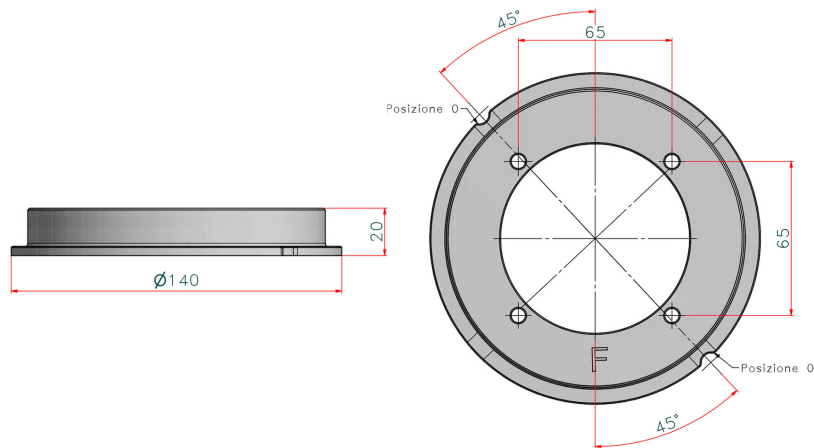


Fig. Mounting Flange

CAUTION



THE FLANGE MUST BE PLACED WITH THE LETTER "F" FACING TOWARD THE X-RAY TUBE PORT. INCORRECT POSITION MAY CAUSE THE COLLIMATOR AND FLANGE TO MALFUNCTION.



ENSURE THE SCREW HEAD IS INDEED CORRECT FOR THE FLANGE SELECTED. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE ALL SAFETY MEASURES ARE IMPLEMENTED TO ENSURE THE SCREWS ARE OPTIMALLY TIGHTENED INCLUDING THE USE OF APPROPRIATE LOCKTITE.



TIGHTEN THE 4 SCREWS TO THE X-RAY TUBE HEAD SECURELY, STRICTLY ACCORDING TO THE INSTRUCTIONS OF THE X-RAY TUBE MANUFACTURER. DO NOT EXCEED 0.45 NM OF FORCE.

ALIGNMENT OF X-RAY TUBE FOCUS AND COLLIMATOR

Ralco guarantees the correct collimator functionality, format compliance and light/X-Ray field alignment only if the mounting flange and the collimator have been installed exactly in the centre of the X-Ray beam.

All Ralco collimators are aligned on our test bench utilizing specific references/values for our X-Ray tube focus, detector and Source to Image Detector Distance (SID). The customer must know and verify all known variables which may influence the X-Ray tube focus and collimator alignment. These may include, the X-Ray tube focus position tolerance, distance from X-Ray tube focus to collimator mounting plane, or the SID.


Alignment Device

This device is used to ensure the collimator mounting flange is correctly aligned to the X-ray beam. The X-ray tube manufacturer provides a tolerance for the placement of X-ray tube focus.

Ralco recommends to use the Focal Alignment Device (jig) to ensure the correct flange alignment with the centre of the X-Ray beam, see **Fig. Focal Alignment Device**. By making an exposure, it is possible to verify the perpendicularity and concentricity using fixed references on the X-Ray image.

Once the mounting flange is aligned the collimator light/X-ray field should also be aligned (within specific tolerances).

Please consult the technical specifications of your X-Ray tube to find the maximum tolerance for the position of the focus. Should the use of an alignment device not be possible, Ralco collimators allow for the regulation of the light field.

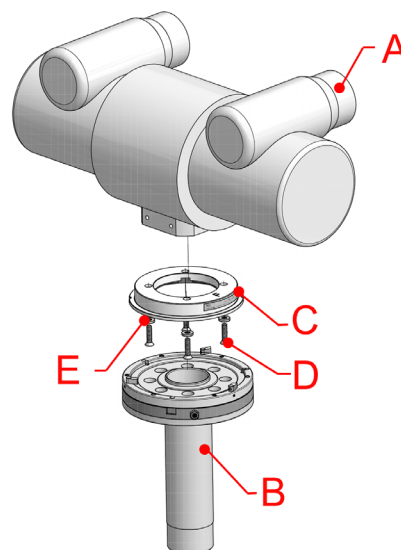

A - X-Ray Tube, B - Focal Adjustment Device
C - Mounting Flange, D - Screw, E - Washer

Fig. Focal Alignment Device

Mounting the Collimator to the Flange

NOTE



THE MOUNTING FLANGE PROVIDED (IF PURCHASED) WITH THE COLLIMATOR IS SUBJECTED TO TESTING PURSUANT TO ALL APPLICABLE STANDARDS.



MOUNTING BRACKET TABS CONFORM TO EN60601.

WARNINGS



PURSUANT TO APPLICABLE STANDARDS, RALCO HAS TESTED THE COLLIMATOR AND FLANGE APPLYING STATIC LOADS. RALCO IS NOT IN A POSITION TO KNOW THE DYNAMIC FORCES OF ALL END-USER SYSTEMS. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE DYNAMIC FORCES OF THE SYSTEM DO NOT CREATE A DANGEROUS CONDITION.

WARNINGS



IT IS THE RESPONSIBILITY OF THE SYSTEM MANUFACTURER TO ENSURE AND MITIGATE ANY DANGEROUS CONDITIONS WHICH MAY OCCUR DUE TO THE DYNAMIC FORCES CREATED BY THE SYSTEM. THE END-USER MUST PERFORM A SYSTEMATIC AND STRUCTURAL ANALYSIS DURING THE INSTALLATION AND USUAL MAINTENANCE.



SHOULD ANY DAMAGE TO THE COLLIMATOR OR FLANGE OCCUR A RISK ANALYSIS AND DAMAGE ASSESSMENT NEEDS TO BE CONDUCTED IMMEDIATELY. CONTACT RALCO IMMEDIATELY SHOULD THIS OCCUR. RALCO IS NOT LIABLE FOR RESULTING PROPERTY DAMAGE AND/OR HARM DUE TO AN UNREPORTED INCIDENT.



RALCO HAS DESIGNED AND TESTED THE COLLIMATOR FOR A LIFETIME OF 10 YEARS. AFTER THIS TIME PERIOD, IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE THE PROPER FUNCTIONING OF THE COLLIMATOR AND FLANGE. LIABILITY FOR ANY DANGEROUS CONDITIONS WHICH MAY BE PRESENT AFTER THE 10YEAR LIFETIME OF THE COLLIMATOR AND FLANGE RESTS WITH THE END-USER.



TO ENSURE THE SAFETY OF THE COLLIMATOR AND FLANGE AFTER 10 YEARS OF USE, RALCO HAS INSTITUTED A PROGRAM TO ASSESS THE SAFETY OF THE COLLIMATOR AND FLANGE. AFTER APPLYING A CHECK LIST OF QUALITY CONTROLS AND REFURBISHMENT ACTIVITIES (AT END-USER EXPENSE), RALCO MAY CERTIFY THE COLLIMATOR AND FLANGE FOR ADDITIONAL YEARS OF USE.

The mounting bracket has 2 tabs with springs in positions 2 and 4, while the 2 tabs in position 1 and 3 cover the surface of the outer ring of the flange, see **Fig. Mounting Bracket**.

1. Prepare the collimator to be installed by unscrewing the 4 hexagonal socket screws until the four tabs are completely withdrawn from the collimator top mounting plane, see **Fig. 4 Mounting Screws**.
2. If installing a manual collimator, adjust the collimator shutters to the fully open position using both knobs.

CAUTION



WHEN UNSCREWING THE HEXAGONAL SOCKET SCREW WHICH CONTROL THE TABS, DO NOT USE FORCE EXCEEDING 0,45 NM. UNSCREW WITH CARE SO AS NOT TO DAMAGE THE HEXAGONAL SCREW HEAD AND TABS.

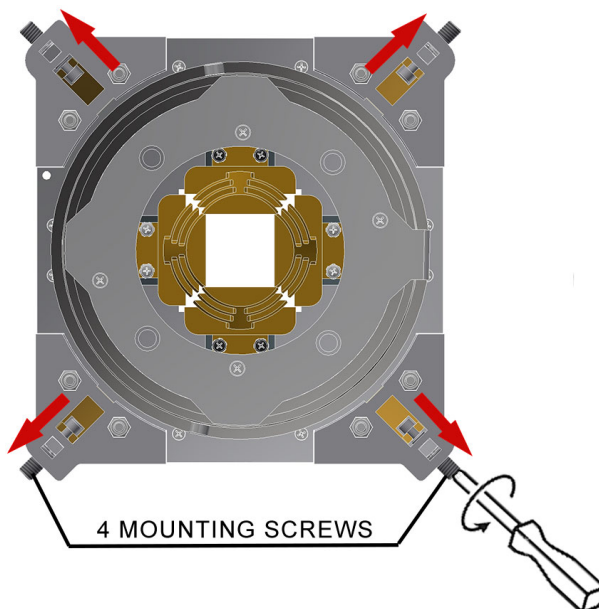


Fig. Collimator Mounting Bracket

3. With the tabs fully retracted, tabs in position 1 and 3 which overlap the outer ring must be tightening equally up to the end stroke with 0.45 Nm torque (at least 7 turns). See **Fig. Mounting Bracket**.
4. With the tabs fully retracted, the tabs in position 2 and 4 have an integrated adjustable spring and can be tightened in two different ways allowing the flange to be fixed or to rotate, see **Fig. Mounting Bracket**.

FIXED (NO ROTATION)

With the tabs fully retracted, tighten all tabs up to the end stroke with 0.45Nm torque at least 7 turns (if you are not able to respect the turns something is incorrect, repeat above instructions, if issues persist, please contact Ralco).

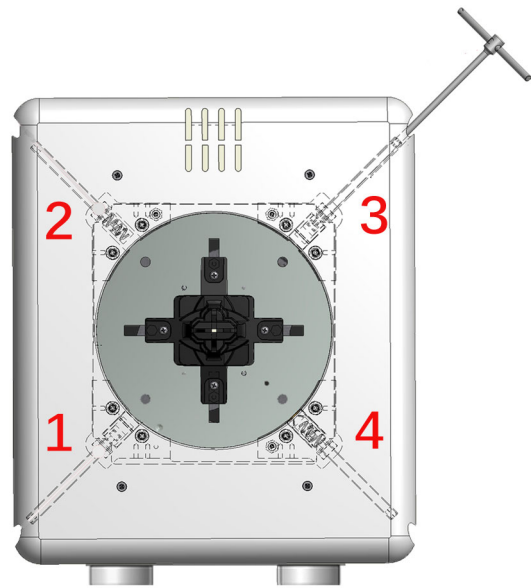


Fig. Mounting Bracket
*illustrative purpose only

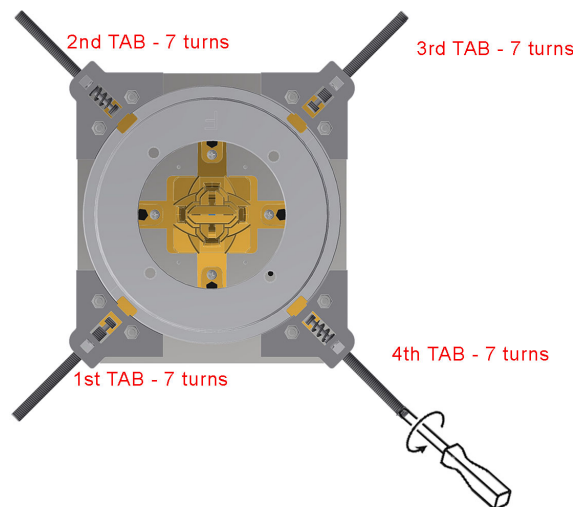


Fig. Mounting Bracket (Fixed)

WITH ROTATION

With the tabs fully retracted, tighten tabs in the following sequence (if you are not able to respect the turns something is incorrect, repeat above instructions, if issues persist, please contact Ralco):

- Tabs 1 and 3 - minimum of 7 turns
- Tabs 2 and 4 – exactly 5 ½ turns.

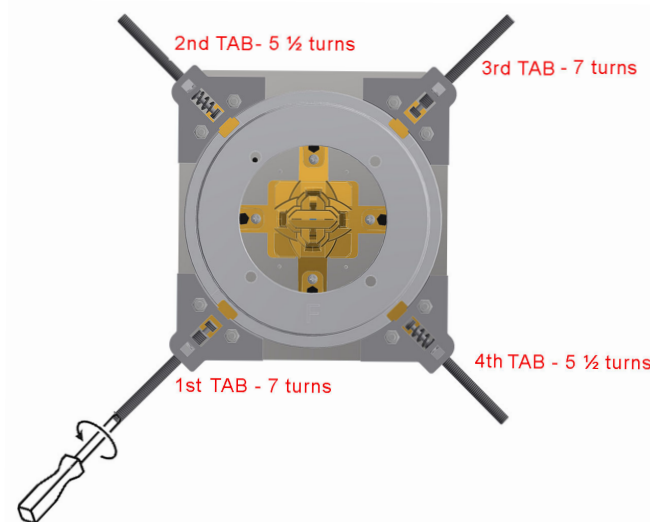


Fig. Mounting Bracket (Rotating)

The collimator tabs glide on the flange outer ring and the collimator rotates towards the X- Ray tube axis. If, however:

- The release force from the position 0° and the collimator rotation is too low, the two tabs of the tab 2 and 4 need to be tightened ½ turn.
- The release force from the position 0° and the collimator rotation is too high, the two tabs of the tab 2 and 4 need to be loosened ½ turn.

VERIFICATION OF CORRECT INSTALLATION

WARNING



IT IS THE DUTY OF THE INSTALLER TO ENSURE NO RISK OF THE COLLIMATOR FALLING EXISTS. ENS URE THE FOLLOWING, AS IN EACH SCENARIO BELOW SERIOUS RISK OF INJURY AND/OR PROPERTY DAMAGE MAY EXIST DUE TO NON-ADHERENCE.

1. The 4 tabs should overlap the flange outer ring, see **Fig. Correct Overlap**.
2. Ensure the mounting flange is flat against the collimator mounting plane, see **Fig. Correct Overlap**.
3. Ensure the 4 tabs are not in contact with only the mounting flange edge, see **Fig. Incorrect Overlap**.
4. Once the collimator is mounted, if not already, return the collimator/tube head to the intended use position. Rotate and/or gently pull the collimator to ensure correct coupling.
5. If the collimator is loose, something is incorrect. Repeat above mounting instructions, and if issues persist, please contact Ralco.

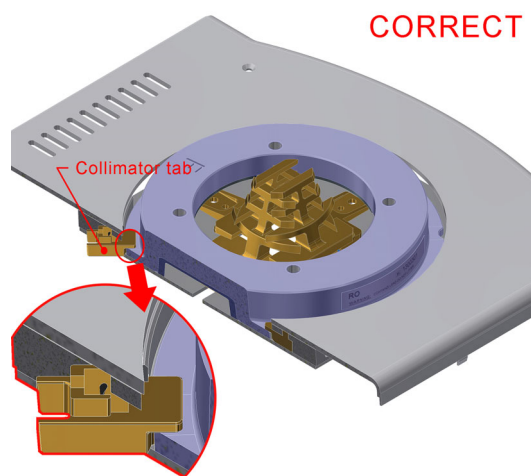


Fig. Correct Overlap



Fig. Incorrect Overlap

ELECTRICAL CONNECTION

NOTE



THE WIRING DIAGRAM INCLUDED IN THIS DOCUMENT REFERS TO THE STANDARD PRODUCT. IT IS THE RESPONSIBILITY OF THE CUSTOMER WHO HAS REQUESTED AN ELECTRIC CUSTOMIZATION, TO ENSURE THAT AN ELECTRIC DIAGRAM RELATING TO THE CUSTOMIZATION HAS BEEN PROVIDED WITH THE DOCUMENTATION.



THE DEVICE IS PROJECTED TO OPERATE WITH A PERMANENT POWER SUPPLY PRESENT SO THE PROCEDURE OF SWITCHING OFF THE COLLIMATOR IS NOT FORESEEN. NO RISK OR DEVICE DAMAGE WILL OCCUR IF THE MACHINE IS ACCIDENTLY SWITCHED OFF.

WARNING



THE DEVICE MUST BE EXCLUSIVELY CONNECTED TO POWER NETWORK, WITH EARTH PROTECTION, IN ORDER TO AVOID A RISK OF ELECTRICAL SHOCK.



SUPPLY AND SIGNALS TO THE COLLIMATOR MUST BE TO 2007/47/CE STANDARDS. DEVICES THAT SUPPLY THE COLLIMATOR MUST THEREFORE FEATURE DOUBLE OR REINFORCED INSULATION AS PROVIDED BY THE GENERAL STANDARD ON ELECTROMEDICAL EQUIPMENT CEI62-5 +A2 (EN 60601-1). THE DEVICE IS ELECTROSTATIC SENSITIVE, CONSEQUENTLY ALL THE RELATING SAFETY STANDARDS MUST BE COMPLIED WITH.



COLLIMATOR MUST BE SUPPLIED AS SPECIFIED, SEE CHAPTER **SPECIFICATIONS**. THE SUPPLY MUST COME FROM A SEPARATE SOURCE FROM THE POWER NETWORK THROUGH DOUBLE INSULATION OR REINFORCED INSULATION AND WITH LIMITED CURRENT. TRANSFORMER CHARACTERISTICS MUST CONFORM TO THE REQUIREMENTS OF STANDARD IEC 60601-1.




CABLES AND TERMINALS USED FOR THE INTERNAL CONNECTION OF THE COLLIMATOR MUST BE SUITABLE FOR OPERATION AT TEMPERATURES OF 80°C AND COLLIMATOR CURRENT ABSORPTION.



TO ENSURE THE SAFE USE OF THE COLLIMATOR, IT IS MANDATORY TO SECURELY AND FIRMLY INSERT THE POWER SUPPLY/DATA CABLE BETWEEN THE COLLIMATOR AND RADIOLOGICAL SYSTEM. IT IS ALSO MANDATORY THAT THE POWER SUPPLY/DATA CABLE IS CORRECTLY SECURED TO THE STRAIN RELIEF. IF THE STRAIN RELIEF IS MISSING IT IS MANDATORY ONE BE CORRECTLY AND SECURELY INSTALLED. RALCO IS NOT LIABLE FOR DAMAGES IN THE EVENT OF MISSING OR INCORRECT CABLE INSTALLATION.

Power Supply Connection

- Remove the part of cover to access the terminal, see Chapter- **COVER REMOVAL**.
- Connect supply cables to the relative collimator terminals and earth on the screw marked with the relating symbol:  . Use the protective earth cable with green/yellow insulation.
- In collimators with a free cable stop, remove the cable stop, connect the supply cables, remount and then secure the cable stop.
- Remount the cover.

WARNINGS




INCORRECT POWER SUPPLY COULD DAMAGE THE ELECTRONIC BOARDS AND/OR THE LIGHT SOURCE.


SUPPLY MAY BE EITHER IN ALTERNATE OR DIRECT CURRENT.
MAKE CERTAIN THAT POLARITY IS RESPECTED.

Wiring Diagram

MTR 221 ACS - MTR 221 ACS DHHS - Installation

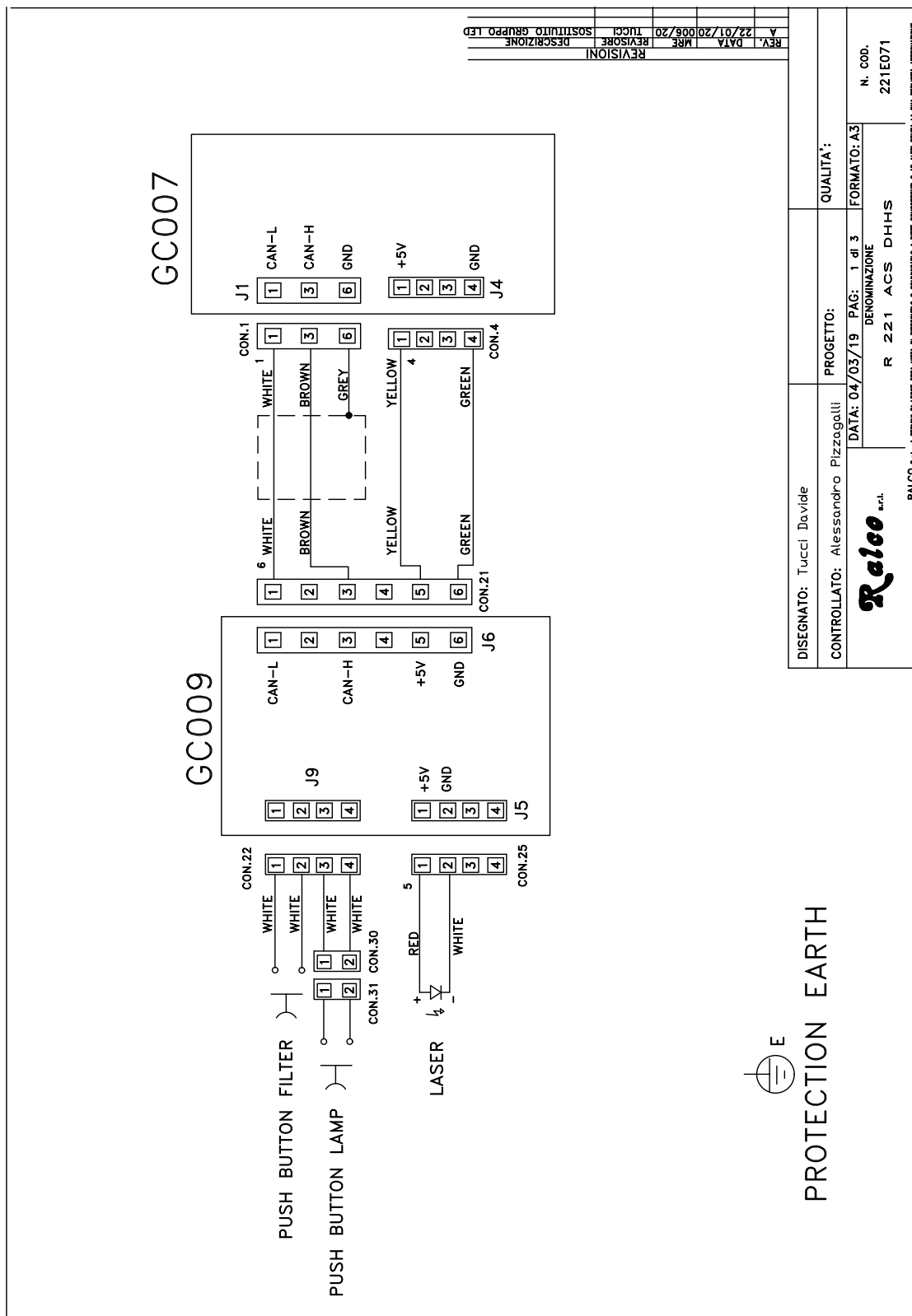

MTR 221 ACS - MTR 221 ACS DHHS - Installation

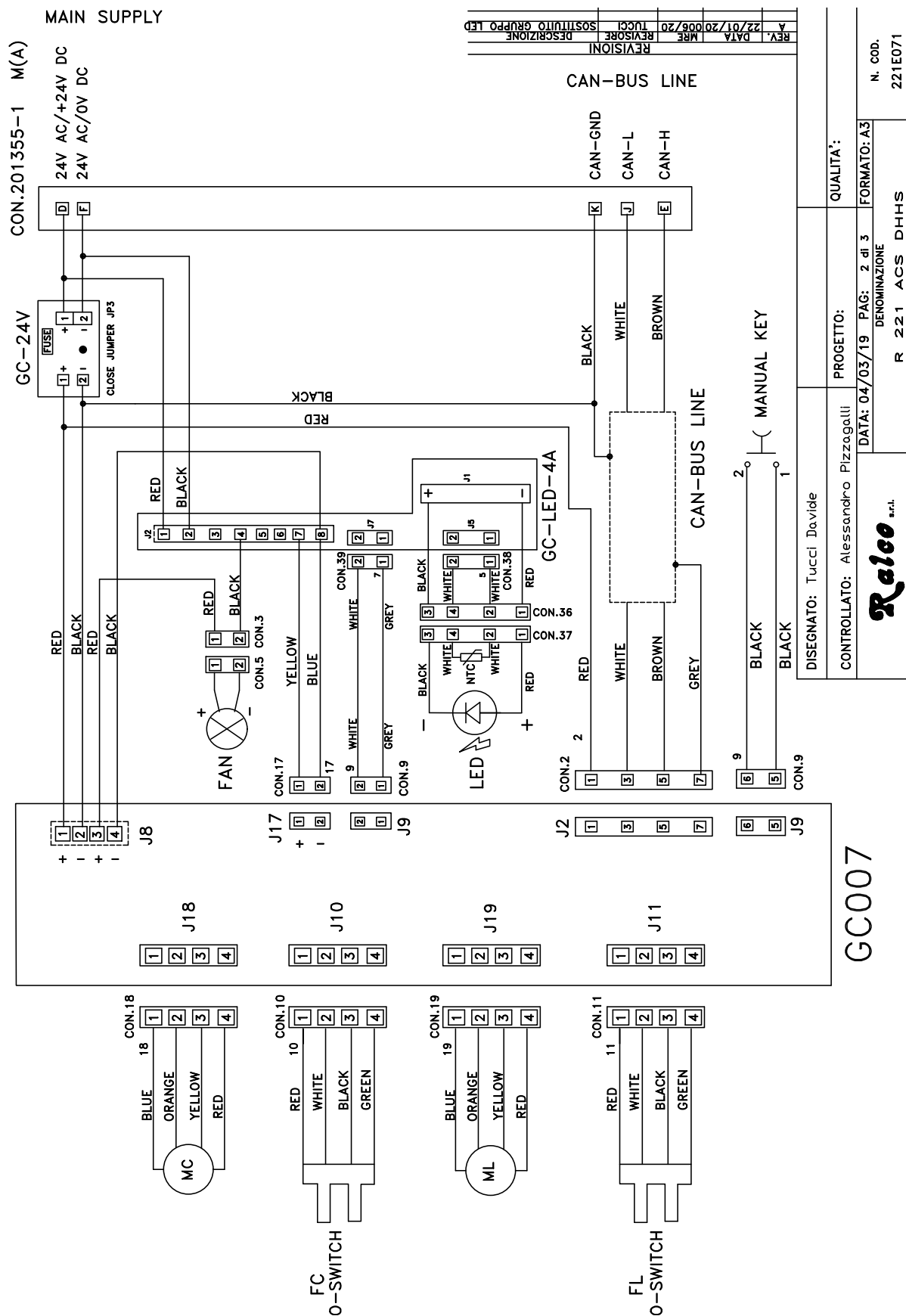

MTR 221 ACS - MTR 221 ACS DHHS - Installation

GC009

UP ↓ CROSS
LONG

DOWN ↑

Mov. Long.
UP
CROSS ↑ LONG
Front.

Mov. Cross.

INSERT JUMPER JP1 FOR END LINE CAN-BUS

INSERT JUMPER JP12-JP13 FOR SW.4.00

POWER SUPPLY
FAN
CONTACT LIGHT

GC-LED-4A

GC007

SW2: 1/3 ON CAN-SPEED 500Kbit/s

SW1

SW2

SW3

SW4

SW5

SW6

SW7

SW8

SW9

SW10

SW11

SW12

SW13

SW14

SW15

SW16

SW17

SW18

SW19

SW20

SW21

SW22

SW23

SW24

SW25

SW26

SW27

SW28

SW29

SW30

SW31

SW32

SW33

SW34

SW35

SW36

SW37

SW38

SW39

SW40

SW41

SW42

SW43

SW44

SW45

SW46

SW47

SW48

SW49

SW50

SW51

SW52

SW53

SW54

SW55

SW56

SW57

SW58

SW59

SW60

SW61

SW62

SW63

SW64

SW65

SW66

SW67

SW68

SW69

SW70

SW71

SW72

SW73

SW74

SW75

SW76

SW77

SW78

SW79

SW80

SW81

SW82

SW83

SW84

SW85

SW86

SW87

SW88

SW89

SW90

SW91

SW92

SW93

SW94

SW95

SW96

SW97

SW98

SW99

SW100

SW101

SW102

SW103

SW104

SW105

SW106

SW107

SW108

SW109

SW110

SW111

SW112

SW113

SW114

SW115

SW116

SW117

SW118

SW119

SW120

SW121

SW122

SW123

SW124

SW125

SW126

SW127

SW128

SW129

SW130

SW131

SW132

SW133

SW134

SW135

SW136

SW137

SW138

SW139

SW140

SW141

SW142

SW143

SW144

SW145

SW146

SW147

SW148

SW149

SW150

SW151

SW152

SW153

SW154

SW155

SW156

SW157

SW158

SW159

SW160

SW161

SW162

SW163

SW164

SW165

SW166

SW167

SW168

SW169

SW170

SW171

SW172

SW173

SW174

SW175

SW176

SW177

SW178

SW179

SW180

SW181

SW182

SW183

SW184

SW185

SW186

SW187

SW188

SW189

SW190

SW191

SW192

SW193

SW194

SW195

SW196

SW197

SW198

SW199

SW200

SW201

SW202

SW203

SW204

SW205

SW206

SW207

SW208

SW209

SW210

SW211

SW212

SW213

SW214

SW215

SW216

SW217

SW218

SW219

SW220

SW221

SW222

SW223

SW224

SW225

SW226

SW227

SW228

SW229

SW230

SW231

SW232

SW233

SW234

SW235

SW236

SW237

SW238

SW239

SW240

SW241

SW242

SW243

SW244

SW245

SW246

SW247

SW248

SW249

SW250

SW251

SW252

SW253

SW254

SW255

SW256

SW257

SW258

SW259

SW260

SW261

SW262

SW263

SW264

SW265

SW266

SW267

SW268

SW269

SW270

SW271

SW272

SW273

SW274

SW275

SW276

SW277

SW278

SW279

SW280

SW281

SW282

SW283

SW284

SW285

SW286

SW287

SW288

SW289

SW290

SW291

SW292

SW293

SW294

SW295

SW296

SW297

SW298

SW299

SW300

SW301

SW302

SW303

SW304

SW305

SW306

SW307

SW308

SW309

SW310

SW311

SW312

SW313

SW314

SW315

SW316

SW317

SW318

SW319

SW320

SW321

SW322

SW323

SW324

SW325

SW326

SW327

SW328

SW329

SW330

SW331

SW332

SW333

SW334

SW335

SW336

SW337

SW338

SW339

SW340

SW341

SW342

SW343

SW344

SW345

SW346

SW347

SW348

SW349

SW350

SW351

SW352

SW353

SW354

SW355

SW356

SW357

SW358

SW359

SW360

SW361

SW362

SW363

SW364

SW365

SW366

SW367

SW368

SW369

SW370

SW371

SW372

SW373

SW374

SW375

SW376

SW377

SW378

SW379

SW380

SW381

SW382

SW383

SW384

SW385

SW386

SW387

SW388

SW389

SW390

SW391

SW392

SW393

SW394

SW395

SW396

SW397

SW398

SW399

SW400

SW401

SW402

SW403

SW404

SW405

SW406

SW407

SW408

SW409

SW410

SW411

SW412

SW413

SW414

SW415

SW416

SW417

SW418

SW419

SW420

SW421

SW422

SW423

SW424

SW425

SW426

SW427

SW428

SW429

SW430

SW431

SW432

SW433

SW434

SW435

SW436

SW437

SW438

SW439

SW440

SW441

SW442

SW443

SW444

SW445

SW446

SW447

SW448

SW449

SW450

SW451

SW452

SW453

SW454

SW455

SW456

SW457

SW458

SW459

SW460

SW461

SW462

SW463

SW464

SW465

SW466

SW467

SW468

SW469

SW470

SW471

SW472

SW473

SW474

SW475

SW476

SW477

SW478

SW479

SW480

SW481

SW482

SW483

SW484

SW485

SW486

SW487

SW488

SW489

SW490

SW491

SW492

SW493

SW494

SW495

SW496

SW497

SW498

SW499

SW500

SW501

SW502

SW503

SW504

SW505

SW506

SW507

SW508

SW509

SW510

SW511

SW512

SW513

SW514

SW515

SW516

SW517

SW518

SW519

SW520

SW521

SW522

SW523

SW524

SW525

SW526

SW527

SW528

SW529

SW530

SW531

SW532

SW533

SW534

SW535

SW536

SW537

SW538

SW539

SW540

SW541

SW542

SW543

SW544

SW545

SW546

SW547

SW548

SW549

SW550

SW551

SW552

SW553

SW554

SW555

SW556

SW557

SW558

SW559

SW560

SW561

SW562

SW563

SW564

SW565

SW566

SW567

SW568

SW569

SW570

SW571

SW572

SW573

SW574

SW575

SW576

SW577

SW578

SW579

SW580

SW581

SW582

SW583

SW584

SW585

SW586

SW587

SW588

SW589

SW590

SW591

SW592

SW593

SW594

SW595

SW596

SW597

SW598

SW599

SW600

SW601

SW602

SW603

SW604

SW605

SW606

SW607

SW608

SW609

SW610

SW611

SW612

SW613

SW614

SW615

SW616

SW617

SW618

SW619

SW620

SW621

SW622

SW623

SW624

SW625

SW626

SW627

SW628

SW629

SW630

SW631

SW632

SW633

SW634

SW635

SW636

SW637

SW638

SW639

SW640

SW641

SW642

SW643

SW644

SW645

SW646

SW647

SW648

SW649

SW650

SW651

SW652

SW653

SW654

SW655

SW656

SW657

SW658

SW659

SW660

SW661

SW662

SW663

SW664

SW665

SW666

SW667

SW668

SW669

SW670

SW671

SW672

SW673

SW674

SW675

SW676

SW677

SW678

SW679

SW680

SW681

SW682

SW683

SW684

SW685

SW686

SW687

SW688

SW689

SW690

SW691

SW692

SW693

SW694

SW695

SW696

SW697

SW698

SW699

SW700

SW701

SW702

SW703

SW704

SW705

SW706

SW707

SW708

SW709

SW710

SW711

SW712

SW713

SW714

SW715

SW716

SW717

SW718

SW719

SW720

SW721

SW722

SW723

SW724

SW725

SW726

SW727

SW728

SW729

SW730

SW731

SW732

SW733

SW734

SW735

SW736

SW737

SW738

SW739

SW740

SW741

SW742

SW743

SW744

SW745

SW746

SW747

SW748

SW749

SW750

SW751

SW752

SW753

SW754

SW755

SW756

SW757

SW758

SW759

SW760

SW761

SW762

SW763

SW764

SW765

SW766

SW767

SW768

SW769

SW770

SW771

SW772

SW773

SW774

SW775

SW776

SW777

SW778

SW779

SW780

SW781

SW782

SW783

SW784

SW785

SW786

SW787

SW788

SW789

SW790

SW791

SW792

SW793

SW794

SW795

SW796

SW797

SW798

SW799

SW800

SW801

SW802

SW803

SW804

SW805

SW806

SW807

SW808

SW809

SW810

SW811

SW812

SW813

SW814

SW815

SW816

SW817

SW818

SW819

SW820

SW821

SW822

SW823

SW824

SW825

SW826

SW827

SW828

SW829

SW830

SW831

SW832

SW833

SW834

SW835

SW836

SW837

SW838

SW839

SW840

SW841

SW842

SW843

SW844

SW845

SW846

SW847

SW848

SW849

SW850

SW851

SW852

SW853

SW854

SW855

SW856

SW857

SW858

SW859

SW860

SW861

SW862

SW863

SW864

SW865

SW866

SW867

SW868

SW869

SW870

SW871

SW872

SW873

SW874

SW875

SW876

SW877

SW878

SW879

SW880

SW881

SW882

SW883

SW884

SW885

SW886

SW887

SW888

SW889

SW890

SW891

SW892

SW893

SW894

SW895

SW896

SW897

SW898

SW899

SW900

SW901

SW902

SW903

SW904

SW905

SW906

SW907

SW908

SW909

SW910

SW911

SW912

SW913

SW914

SW915

SW916

SW917

SW918

SW919

SW920

SW921

SW922

SW923

SW924

SW925

SW926

SW927

SW928

SW929

SW930

SW931

SW932

SW933

SW934

SW935

SW936

SW937

SW938

SW939

SW940

SW941

SW942

SW943

SW944

SW945

SW946

SW947

SW948

SW949

SW950

SW951

SW952

SW953

SW954

SW955

SW956

SW957

SW958

SW959

SW960

SW961

SW962

SW963

SW964

SW965

SW966

SW967

SW968

SW969

SW970

SW971

SW972

SW973

SW974

SW975

SW976

SW977

SW978

SW979

SW980

SW981

SW982

SW983

SW984

SW985

SW986

SW987

SW988

SW989

SW990

SW991

SW992

SW993

SW994

SW995

SW996

SW997

SW998

SW999

SW1000

REV. DATA MRE 22/01/2006/20 TUCC

REVISIONE

DESCRIZIONE

SOSTITUITO GRUPPO LED

QUALITA':

PROGETTO:

CONTROLLATO: Alessandro Pizzagalli

DATA: 04/03/19 PAG: 3 di 3

DENOMINAZIONE

R 221 ACS DHHS

N. COD.

221E071


RALCO S.p.A. A TERMINI DI LAVORO ESSE VERA DI IMPIANTO O COMPLESSO A TUTTE CONDIZIONI O AD ALTE SCELTA LA SUA ESPERTA APPROPRIAZIONE.

OSIZ.

DENOMINAZIONE

N° DISEGNO

PEZZI

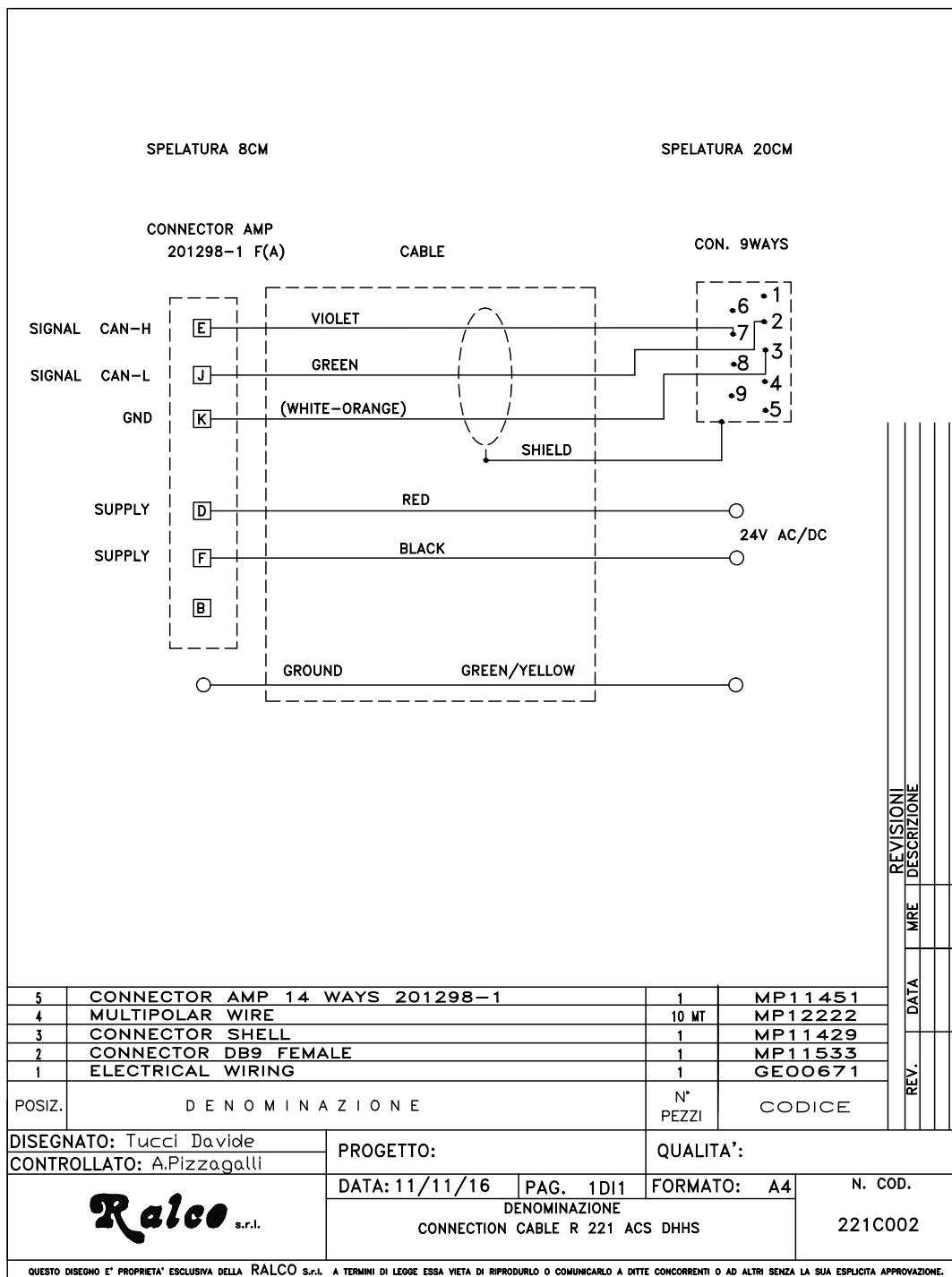

MTR 221 ACS - MTR 221 ACS DHHS - Installation

OPERATION INSTRUCTIONS

This collimator can be used in both Radiological and Tomographic applications. It can be operated in automatic mode or manual mode. The following section describes generally how each mode works. This section also describes how to operate the light field.

The three different colored LEDs (red, green, and yellow) tell the user in what mode the collimator is functioning and whether any errors/dangerous operating conditions are present.

When the system is turned on it will switch to one of the following modes:

Red LED Exposure HOLD	<ul style="list-style-type: none"> The SID in use is too short for the cassette in use; max. opening of shutters: 43x43 cm at 90 cm - i.e. the shutters are at end run but the motors are still running to attain a correct position. Increase the SID. Motors move to their correct position.
Increase SID	<ul style="list-style-type: none"> The collimator will automatically adjust the field, etc.
Green LED READY	<ul style="list-style-type: none"> The system is ready. All pre-set functions are allowed.
Yellow LED MANUAL	<ul style="list-style-type: none"> Cassette is not inserted. Cassette tray not inserted correctly. The system has no SID - or SID signal below or over the minimum and maximum set value. Collimator inclination exceeds 3°. Key switch (back of collimator) is turned to the Manual Mode. Manual Mode selected.
Insert a cassette	<ul style="list-style-type: none"> The collimator will automatically adjust the field to the size of the cassette inserted. The collimator will automatically read SID and cassette size inserted. The two knobs on the front panel reduce field size. Note: shutters cannot be opened to a field larger than that of the cassette size inserted.
Readings (Cassette sizes, FFD (SID):	<p>If the system is in the manual mode, the display will show the last memorised SID and the last memorised field size.</p>

Prior to operating the collimator, check the settings related to correct X-ray beam functions.

RADIOGRAPHY

Automatic Operation of Square Field (Long and Cross)

Collimator in the Vertical Position

- The collimator must be vertical, $\pm 3^\circ$.
- The cassette-IN microswitch must be closed.
- The Bucky supplies cassette dimensions.
- SID is forwarded to the PCB on the collimator via CanBus.
- The collimator automatically sets to cassette dimension.
- The display indicates the SID selected and Long and Cross dimensions.
- The green LED on the front panel serves to indicate that the unit is ready.

The operator can:

- Turn the field light ON by activating the related push-button on the front panel or via the external control.
- Reduce the set field using the knobs on the front panel or via CanBus. The field may be re-sized but always within the automatically set field size. If the shutters close completely, the output contact signalling "shutter closed" will be activated.
- Vary SID selected keeping the field size automatically constant.

Collimator in the Lateral Position

- The collimator must be positioned -90° left, $+90^\circ$ right; $\pm 3^\circ$.
- The cassette-IN microswitch must be closed.
- The Bucky supplies cassette dimensions.
- One of the 5 SID lateral switch contacts must be closed.
- The collimator automatically sets the cassette dimensions - field dimensions may be reduced.

The operator can:

- Turn the light field light ON by activating the related push-button or via CanBus.
- Vary the set field size.
- Vary SID in use (selecting one of the 5 fixed SIDs) and keep the set field size automatically constant.

Manual Mode

The collimator is in the manual mode if:

- The collimator is inclined at an angle exceeding 3° and less than 87° .
- Manual control is selected.
- The cassette-IN microswitch is open.
- The Bucky provides no information regarding cassette format.
- No SID signal is present.

When the system is in the Manual Mode, the yellow LED on the front panel is lit and the output contact is activated.

The X-ray contact is always closed.

The operator can:

- Turn the light field ON by pressing the related button on the front panel.
- Reduce field directly with the knobs or push-buttons related to shutter opening. Check on opening with the light field.
- Increase the set field with the knobs or the shutter-related knobs or push-buttons. The field may be opened to the maximum mechanical opening or, if CanBus controlled, to the setting fixed during the installation of the system.
- Check the opening with the light field or display reading.


Manual Mode - Key Set

The system is set to the Manual mode if the key switch on the rear of the collimator is turned to OFF.

- When the system is in the Manual Mode, the yellow LED on the front panel is lit and the output contact is activated.
- Display indicates the last cassette format memorised, SID.
- The X-ray contact is always closed.

The operator can:

- Turn the light field ON by pressing the related button on the front panel.
- Reduce the set field size directly with the knobs.
- Check on opening with the light field. Use the knobs on the front panel.

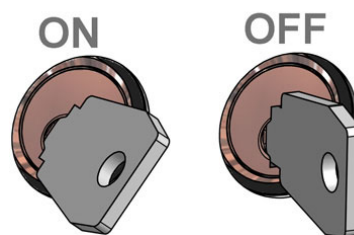

TOMOGRAPHY

Tomography Mode

- The systems sets in the manual mode when the Tomography mode is selected.

Exposure Hold Mode

The systems sets to EXPOSURE HOLD:

- in cases of incorrect operation conditions.
- When the system is in the automatic mode and the shutter motors are operating to reach the selected position.
- If the system fails to reach the selected position.
- The LED is lit and the output contact is activated.
- The exposure contact is open.

Inclinometer HOLD Mode

- The control to exclude the inclinometer deactivates the signals from the device for angulation detection. Consequently, the system could remain in the automatic mode even though it might not be in a vertical position. This forces the collimator to operate in a vertical position.
- The deactivation of this control causes the system not to calculate the signal from the angulation device on the PCB located within the collimator.

Light Field Operation

WARNING



PROLONGED LIGHTING WITHOUT ALLOWING THE LAMP TO COOL CAUSES THE COLLIMATOR TO OVERHEAT IN THE AREA NEAR THE LAMP.
FOR EMERGENCIES: MAXIMUM LIMIT ADVISE IS 5 SUCCESSIVE LAMP OPERATIONS.
ALLOW THE COLLIMATOR TO COOL (ABOUT 10 MIN.). THE OPERATOR MUST AVOID OVERHEATING THE COLLIMATOR AND CARE MUST BE TAKEN NOT TO SCORCH HIMSELF OR THE PATIENT.

The collimator is normally operated by activating the push-button on the front panel to switch the light field ON.

The collimator has been designed to operate as follows:

- Supply constantly connected during operation of the equipment.

- Lamp ON time: ON time is pre-set in factory to 30s (tol.20%).
- A normal lamp ON / OFF cycle is established at 2 sequences followed by 4 minutes to allow for cooling (i.e. 1 minute ON / 4 minutes OFF).
- The field is set automatically by external electronics. For field reduction use the knobs or the motor control push-buttons (if applicable) on the front panel and check the dimension on the display or using the light field.
- Exposure is driven by the electronics that control the collimator.

CALIBRATION

WARNINGS



THE FOLLOWING PROCEDURES REQUIRE THAT X-RADIATION BE PRODUCED. TAKE ADEQUATE PRECAUTIONS TO MAKE CERTAIN THAT NO PART OF THE HUMAN BODY IS EXPOSED TO X-RADIATION, DIRECT OR INDIRECT.

CENTERING X-RAY BEAM WITH COLLIMATOR LIGHT

- Select large focus, 60 kV and 5 mAs or equivalent for digital receptor.
- Place a loaded cassette 24 x 30 cm (9.5"x 12") on the table top and prepare exposure.
- Set SID = 100 cm (40") by using a ruler.

NOTE



IF THE DISTANCE OF 100 CM (40") SID CANNOT BE DETERMINED, USE THE SID VALUE CLOSEST TO 100 CM (40") AND CALCULATE THE MEASUREMENT TOLERANCES AS THE APPROPRIATE PERCENTAGES OF THE DISTANCE.



IF THE X-RAY BEAM CANNOT BE POSITIONED VERTICALLY USE CLAMPS, MASKING TAPE, OR OTHER MATERIAL AS REQUIRED TO POSITION THE X-RAY TUBE AND IMAGE RECEPTOR. THE X-RAY TUBE AND THE IMAGE RECEPTOR MUST BE SET OUT AT THE SPECIFIED SID DISTANCE PERPENDICULARLY TO THE X-RAY TUBE AS DESCRIBED IN THE FOLLOWING PROCEDURE.

NOTE



FIGURE CHECK LIGHT FIELD/X-RAY FIELD SHOWS AN EXAMPLE WITH FILM CASSETTE. THE SAME PRINCIPLE CAN BE USED FOR A DIGITAL SYSTEM.

- Set field to 24 x 30 cm.
- Centre the X-ray tube on the cassette.
- Switch ON the light and place four metal strips (c) on the light field edges (a).
- Place a washer (d) on the right hand-top of the light field.
- Make an exposure.
- Develop the film or print exposure scale 1:1.
- Measure the differences L1, L2, W1 and W2 between the outer edge of each copper strips image and the corresponding edge of the X-ray Field (b).

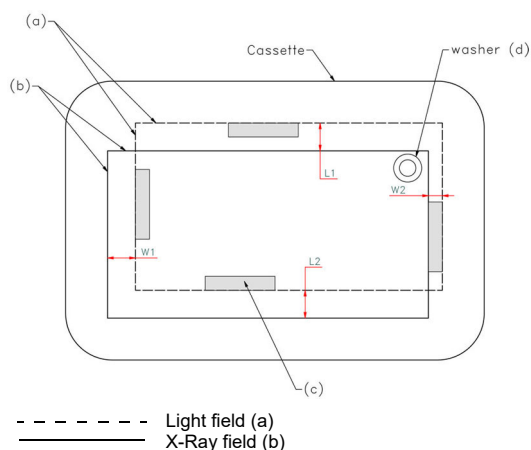


Fig: Check Light Field/X-ray Field

- For required values is referred to the System Reference Manual. However, the values L1 + L2 and W1 + W2 shall NEVER exceed 2 cm (representing 2% of the SID).
- Note the measured values in Chart below.
- Repeat preceding steps with small focus selected.

SID = 100 cm (40") 2.0% x 100 cm (40") =			
LARGE FOCUS	L1+L2 W1+W2	= + = = + =	= ≤ 2.0% SID = ≤ 2.0% SID
SMALL FOCUS	L1+L2 W1+W2	= + = = + =	= ≤ 2.0% SID = ≤ 2.0% SID

Collimator to Focal Spot Alignment (Primary Shutter Cut-Off)

Inspect the four images of the four collimator shutters which form the edges of the x-ray field.

A definitely indistinct edge indicates that the primary shutter, close to the focal spot, is the one forming the line, rather than the outermost shutter.

To correct the condition, use the four mounting/centering adjustment screws to shift the collimator in the direction of the indistinct line. Repeat the test film exposure after making the adjustment.

NOTE



THE HEEL EFFECT WILL CAUSE THE FIELD TOWARD THE CATHODE TO BE SLIGHTLY LESS SHARP THAN ON THE OTHER THREE SIDES. THIS IS NORMAL AND CANNOT BE CORRECTED BY ADJUSTMENT. IN ADDITION, AN X-RAY TUBE OF 12° OR LESS TARGET ANGLE WILL PRODUCE AN ASYMMETRICALLY SHAPED FIELD WHEN A LARGE FIELD SIZE IS USED AT SHORT SID, BECAUSE OF ANODE CUT-OFF EFFECT. THIS IS NORMAL AND MAY NOT BE CORRECTED BY ADJUSTMENT.

LIGHT FIELD TO X-RAY FIELD ALIGNMENT

Misalignment of the light field/ X-ray field in either the X (cross table) or Y (long table) direction must not exceed 2% of the SID. In this case, it would be less than 20 mm (0.80"). If the test film shows that the light field (shadows of the copper strips) matches the X-ray field (shadow of the collimator shutters) to within the diameter of one strips and, if the diameter is less than 20 mm (0.80"), then alignment complies with the regulations.

Greater precision than this is possible. Recommended maximum deviation is one fourth. It is important the greatest degree of congruency possible is achieved.

If misalignment is detected in both X and Y directions, check the spacing from the focal spot to the collimator mounting surface is 80 mm (3.14") +/- 1 mm. If spacing needs to be adjusted, repeat the test film exposure after the adjustment.

If the source flange distance (**SFD**) is correct, but adjustment is still necessary proceed as follows:

- Place the test film on the face of the cassette over the white paper or repeat exposure on the flat panel.
- Place the cassette in the position originally marked.
- Check the correct position of the film or the X-ray field on the flat panel by the shadows cast by the copper strips.
- Using the images of the collimator shutters as the references for the shape and size of the X-ray field, adjust the light field to match.
- If adjustment is necessary, adjust the travel of the light source. All procedures can be found in **LIGHT FIELD ADJUSTMENTS** in this chapter.

LIGHT FIELD ADJUSTMENT

Vertical Alignment

- Remove the part of the cover necessary to access the screws, see Chapter - **COVER REMOVAL**.
- If adjustment is required loosen the two screws **C** holding the light support.
- If the light-field is smaller than the X-ray field, move away the light source by adjusting screw **D**.
- If the light-field is bigger than the X-ray field, move the light source closer by adjusting screws **D**.
- Tighten the two screws **C**. See ***Fig. Light Field Adjustment***

Longitudinal Alignment (Long)


WARNING



DO NOT TOUCH THE DISSIPATER WITH YOUR HANDS; IT COULD BE HOT AND CAUSE SEVERE BURNS.

- Remove part of the cover, see Chapter - **COVER REMOVAL**.
- If the light-field needs to be moved laterally, loosen (not remove) the fixing screws **A**.
- Use screw **B** to adjust transversely.
- When calibration is terminated, lock the screws **A**. See ***Fig. Light Field Adjustment***

Light Field Adjustment

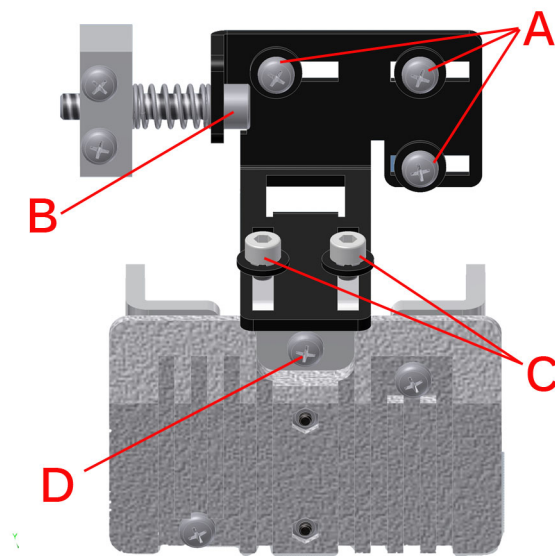

Transversal Calibration (CROSS)*

If the light-field needs calibration, the mirror needs to be adjusted as follows:

- Remove the part of the cover necessary to access the screws, see Chapter - **COVER REMOVAL**.
- Loosen the mirror fixing screw **A** (not remove) and rotate the cam **B** to adjust the position of the mirror, see **Fig. Transversal Calibration (CROSS)**.
- Once you have regulated the mirror tighten the screw **A** and remount the cover, see Chapter - **COVER REMOVAL**.

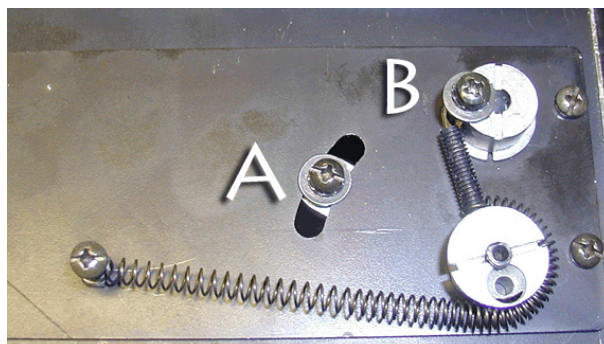


Fig. Transversal Calibration

NOTE




PLEASE CONSULT YOUR PERSONALIZATION PAGE TO PERFORM CORRECT TRANSVERSAL CALIBRATION IF THE MECHANICS DIFFER FROM THE STANDARD COLLIMATOR.

ELECTRONIC SYSTEM

This section describes the collimator control system which includes:

- **GC007 (Master)** - Electronic board which controls via CAN Bus (CAN Open on request) the two stepper motors of square field and one motor for filter positioning.
- **GC009 (Slave)** - Electronic board which controls the display, inclinometer and 2 encoders. The board receives the power (5 V) and commands from the GC 007.
- **GC-Led-4A** - Timer board for light source supply and operation.
- **GC 24 V** - AC/DC power supply converter.

GC 007 BOARD

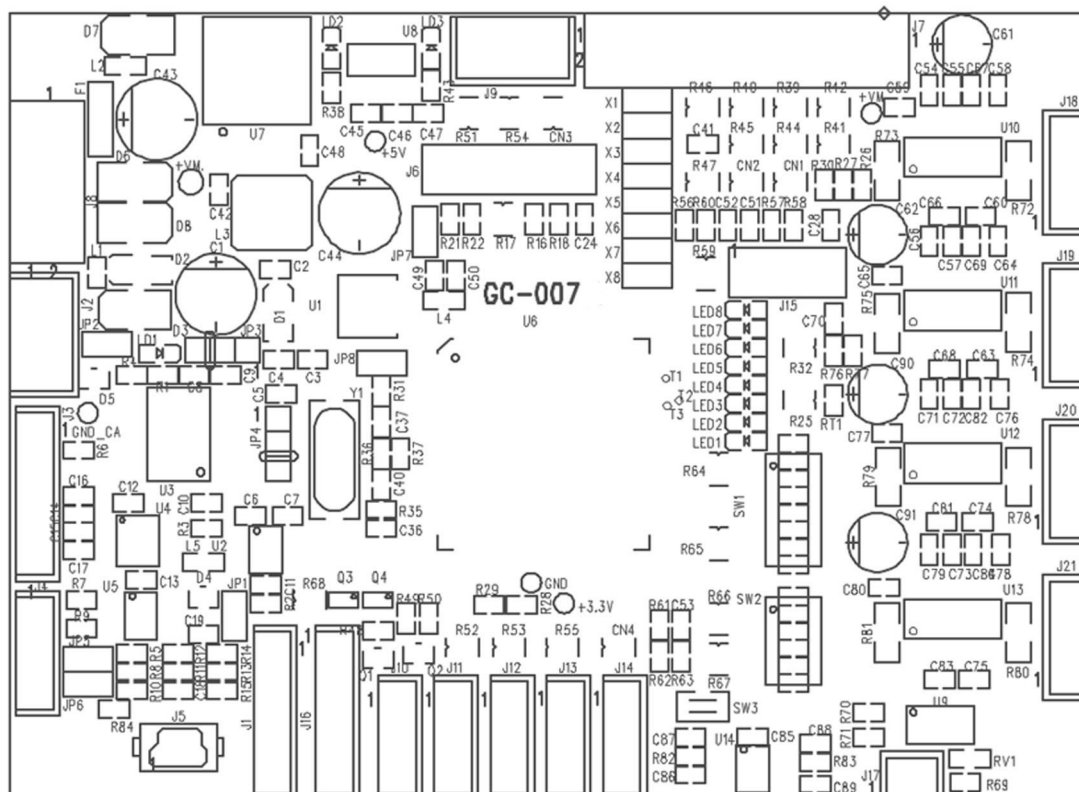

Characteristics

- CANBus Interface 500 Kbit/s default (from 125 Kbit/s to 1 Mbit/s).
- ID node 64 default (from 1 to 255).
- Supply 24 V DC.
- CANBus opto-isolated mode (OFF default).
- Connector for programming.

J2 - CAN Bus (+12/24 V DC)

1-2 +Vcan
3-4 CAN L
5-6 CAN H
7-8 GCAN

J8 - Supply

1. +24 V DC
2. 0V DC
3. +24 V DCr
4. 0V DC

J10 - Photocell A (IRIS)

1. +LED
2. IN
3. Gnd
4. Gnd

J11 - Photocell B (Shutter)

1. +LED
2. IN
3. Gnd
4. Gnd

J13 - Photocell C (Additional Filtration)

1. OAP
2. OAM
3. OBP
4. OBM

J18 - Motor A (IRIS)

1. OAP
2. OAM
3. OBP
4. OBM

J19 - Motor B (Shutter)

1. OAP
2. OAM
3. OBP
4. OBM


J20 - Motor C (Shutter rotation)

1. OAP
2. OAM
3. OBP
4. OBM

J21 - Motor D (Additional Filtration)

1. OAP
2. OAM
3. OBP
4. OBM

Switches

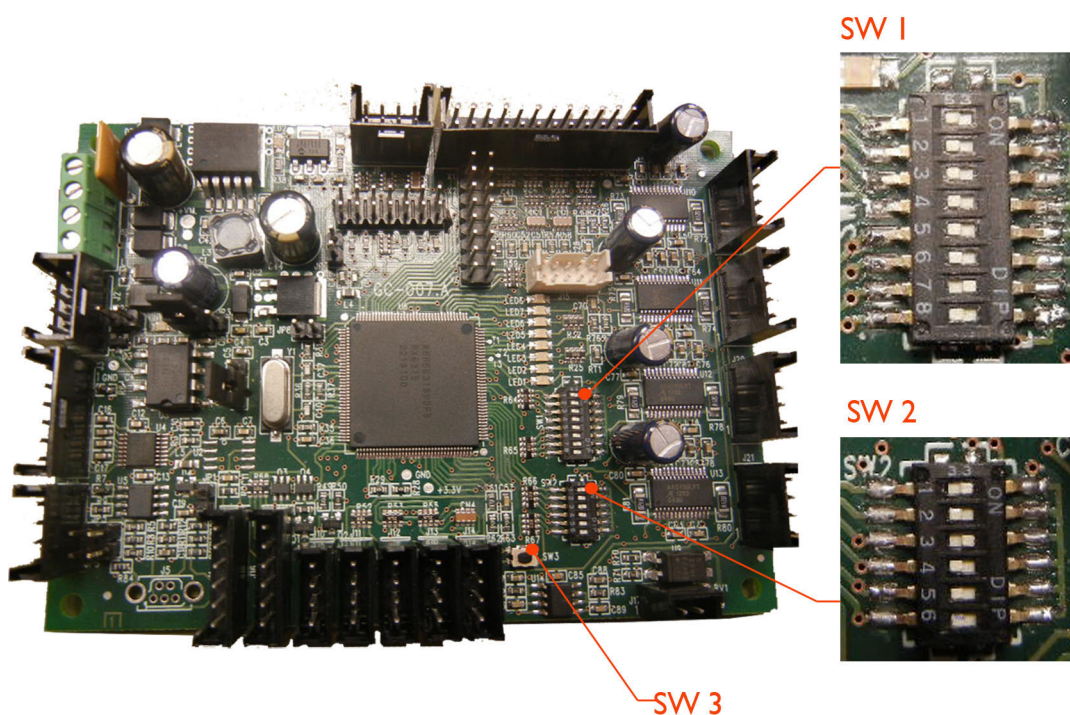

SW1: CAN Open

DIP 8 (MSbit)	DIP 7 (MSbit)	DIP 6 (MSbit)	DIP 5 (MSbit)	DIP 4 (MSbit)	DIP 3 (MSbit)	DIP 2 (MSbit)	DIP 1 (LSbit)	ID Node
0	0	0	0	0	0	0	0	0x00
0	0	0	0	0	0	0	1	0x01
0	0	0	0	0	0	1	0	0x02
...
0	1	0	0	0	0	0	0	0x40 Default
...

0- OFF

1 - ON

SW2: CAN Bus

DIP 3 (MSbit)	DIP 2 (MSbit)	DIP 1 (MSbit)	Bit Rate
0	0	0	20 kbit/s
0	0	1	50 kbit/s
0	1	0	100 kbit/s
0	1	1	125 kbit/s
1	0	0	250 kbit/s
1	0	1	500 kbit/s Default
1	1	0	800 kbit/s
1	1	1	1 Mbit/s

0 - OFF
1 - ON

SW 3 - BUTTON

This push button restores data to Ralco default settings.

J6 - Connector for programming

Jumpers:

- JP2 - CAN Bus
- JP3 - between 1/2 CANBus opto-isolated - externally supplied (+12/24V DC)
between 2/3 CANBus supplied by GC007 PC board (+5V default)
- JP4 - between 1/2 CANBus opto-isolated - externally supplied (+12/24V DC)
between 2/3 CANBus supplied by GC007 PC board (+5V default)

The following 9 LEDs on the PCB indicate:

LD1: +VCAN current power supply
LD2: +5V current power supply
LD3: +3,3V current power supply

LED1: Led LIFE flashes at 1Hz - program runs
LED2: Led Can RX - short blinking speed - CAN Bus message received

LED3: Led Memory - Flash memory communication:
• slow blinking speed (1sec): default data upload
• fast continuous blinking: memory error


LED4: Led Config - Configuration message received:
• very short blinking speed (0.05 sec): wrong or not existing data
• medium blinking speed (0.3 sec): temporary correct data
• slow blinking speed (1 sec): data correct and Flash Memory stored

LED5, LED6, LED7, LED8: HOME photocell status per each connected motor:


- ON: photocell engaged
- OFF: photocell free
- LED5 corresponds to photocell J10
- LED6 corresponds to photocell J11
- LED7 corresponds to photocell J12
- LED8 corresponds to photocell J13

GC 009 Board

FRONT SIDE

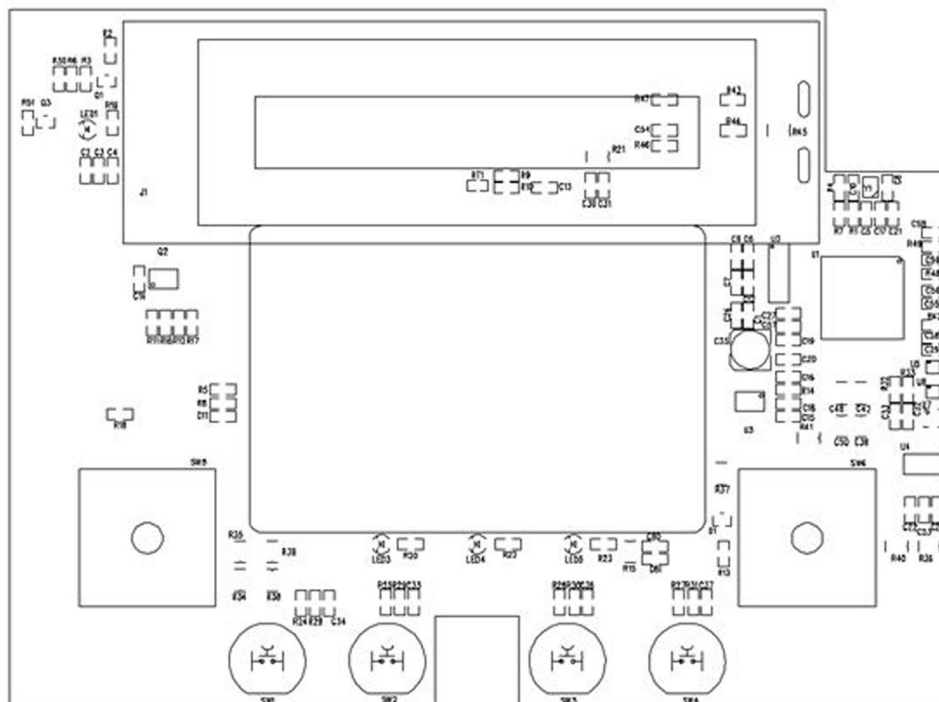

BACK SIDE

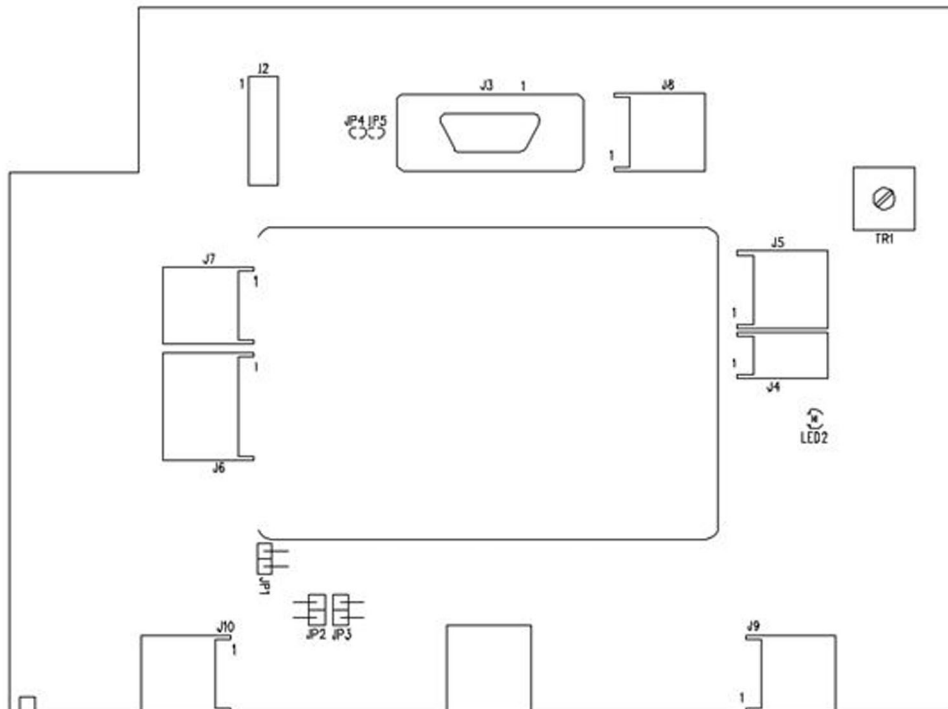

Characteristics

- Supply +5V from the GC007 board.
- CAN Bus line with a termination jumper
- Display LCD 20x2 Large letters
- 2 encoders per each front panel to control knobs
- Programming connector, compatible with Renesas E1
- Serial cable RS232, standard connector 9 pins
- 4 push buttons (in parallel to connectors for outside push buttons)
- 3 standard LEDs for collimator status and customized LED
- 2 Open Collector outputs (laser, ecc)
- 1 inclinometer 3 axes.

Front side

SW1	not used
SW2	Filter push button (default)
SW3	Light push button (default)
SW4	not used
SW5	Left shutter, Cross (default)
SW6	Right shutter, Long (default)
LED1	Custom LED, controlled by system
LED3	Green, collimator READY in automatic mode
LED4	Red, collimator in HOLD
LED5	Yellow, collimator in manual mode

Back Side

LED2	Supply + 5V present
TR1	Contrast adjustment of LCD
JP1	abilita la terminazione a 120 Ohm su CAN Bus
JP2	To be defined
JP3	To be defined
JP4	Enables pull-up on IN-1
JP5	Enables pull-up on IN-2
J2	Programming connector through E1 Renesas Programmer
J3 - Serial RS232, Standard	5. - nc 6. - TX 7. - RX 8. - nc 9. - GND 10. - nc 11. - CTS 12. - RTS 13. - nc
J8 - AUX Inputs	1. - IN-1 2. - GND 3. - IN-2 4. - GND

J4 - External NTC Inputs

1. - input
2. - GND

J5 - Open Collector Outputs

1. - +5V
2. - output 1 (closes GND)
3. - +5V
4. - output 2 (closes GND)

J6 - Supply and CAN Bus

1. - CAN-L
2. - CAN-L
3. - CAN-H
4. - CAN-H
5. - +5V
6. - GND

J7 - Supply

1. - +5V
2. - GND
3. - +5V
4. - GND

J9 - A Push Buttons Inputs

1. - SW2 push button input
2. - GND
3. - SW3 push button input
4. - GND

J10 - B Push Buttons Inputs

1. - SW1 push button input
2. - GND
3. - SW4 push button input
4. - GND

GC-LED-4A

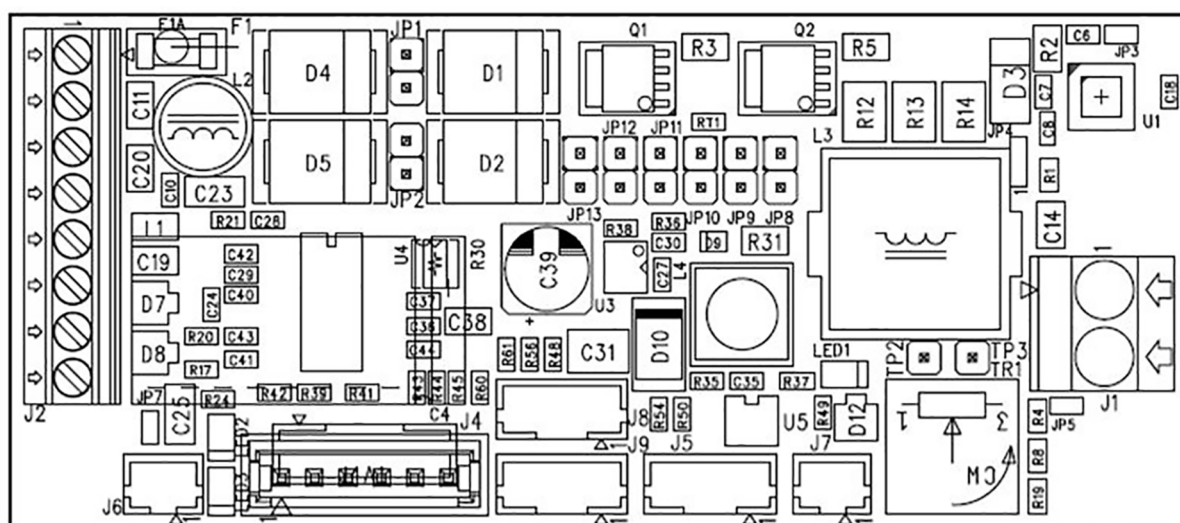


Fig. Timer Board Layout

CONNECTORS

J2 - SUPPLY AND OUTPUTS	1. 24 V AC/DC or 12 V DC 2. 24 V AC/DC or 12 V DC 3. +5 V Fan 4. GND Fan 5. +5 V Laser 6. GND Laser 7. + external light push-button 8. GND external light push-button
J1 - LED OUTPUT	1. + GC-LED-4A positive output 2. - GND GC-LED-4A
J3 - PROGRAMMING CONNECTORS	1. Vpp 2. +5 V 3. GND 4. PGD 5. PGC 6. NC
J4 - HOME SENSOR INPUT + EXTERNAL LED	1. + External LED 2. - External LED 3. - Photosensor LED Cathode 4. + Photosensor LED Anode 5. Input signal (Photosensor output collector) 6. GND (Photosensor output emitter)
J5 - PUSH BUTTON INPUT + NTC EXTERNAL LED	1. NTC LED input 2. NTC LED input 3. Laser input, pin 1 4. Laser input, pin 2
J6 - OPEN-COLLECTOR OUTPUT	1. +5 V 1. GND
J7 - OPEN-COLLECTOR (ABNORMAL OPERATION MANAGEMENT)	1. + 2. -
J8 - J9 - ENCODER (By connecting an opto-switch it is possible to switch ON the light when the shutter move) **The encoder reading is available for s.w. 2.00 and 5.00	1. +LED 2. IN 3. GND 4. GND
Jumpers	OFF: without jumper ON: with jumper
JP1, JP2 - ALTERNATE/DIRECT POWER SUPPLY	OFF: alternate power supply ON: direct power supply (pay attention to polarity)
JP5 - LED POWER SUPPLY UP TO 6.2 A (STANDARD SETTING 4 A)	OFF: LED power supply is adjustable through trimmer to max. 4.8 A ON: LED power supply is adjustable through trimmer to max. 6.2 A

JP14 - ADDITIONAL RESISTANCE	<p>OFF: standard input</p> <p>ON: used to power a retro-reflective photocell</p>
JP8, JP9, JP10, JP11, JP12, JP13 - Operation mode selection	
JP8 - TIME RENEW MODE	<p>OFF: Standard</p> <p>ON: the timer operates in the time renew mode. When the button is pressed the time is renewed without the LED OFF (if the LED is already ON, continues to stay ON but with renewed time). The same is applicable for the laser if is independent.</p>
JP9 - LASER CONTROLLED INDEPENDENTLY	<p>OFF: Standard</p> <p>ON: When the button, placed on J5, is pressed the laser switches ON independently from the light.</p>
<p>JP11, JP10* - TIME MANAGEMENT</p> <p>* Only in mode 4.00 JP10 takes the function of exclusion of auto-turning off LED (designed for temperature)</p> <p>JP10 - AUTO-SHUTDOWN EXCLUSION</p>	<p>OFF, OFF: 30 seconds</p> <p>ON, OFF: 45 seconds</p> <p>OFF, ON: 60 seconds</p> <p>ON, ON: power supply (maximum light ON time is 15 min. for safety reasons).</p> <p>JP10 - LED auto shutdown exclusion, available in s.w. 4.00 only</p>
JP12, JP13 - SOFTWARE CONFIGURATION	<p>OFF, OFF: s.w.2.00 (standard) - it is possible to use the encoder management to switch ON/OFF the laser and LED</p> <p>ON, OFF: s.w.3.01 - power supply without any safety and shutdown time.</p> <p>OFF, ON: s.w.5.00 with the LED off, pressing the button only turns on the LED and the fan.</p> <p>If the button is pressed again within 5 seconds, the laser also lights up. Pushing the button after the first 5 seconds, everything will switch off. The LED and eventually the laser switch off anyway when the time set with the trimmer has set</p> <p>(in this mode it is possible to use the encoder management to switch on light / laser).</p> <p>ON, ON: s.w.4.00 (following the button, the switching on of the board is controlled by closing the contact)</p> <p>If the board is configured as JP8 standard:</p> <p>OFF: (pushing the button the status will change) (if on will switch off and the other way around)</p> <p>ON: The board works in renewal time mode. In this mode, each time the button is pressed, the ignition time is renewed without switching off the light.</p> <p>If the light is already on, it continues to stay on but with renewed time. Also applies to the laser if independent</p>
FAN	<p>The fan switches ON together with the LED, and operates 15 seconds longer after the LED OFF.</p>
Trimmers	
TR1- CURRENT CONTROL	<p>Adjust the output current intensity to the LED.</p> <p>Step-less current adjustment from 0 to 4.8 A.</p> <p>-> Luxeon V LED - adjust current to ~ 2.2 A.</p>
LEDs	

GREEN LED 1	Power supply + 5 V
YELLOW LED 2	<p>Indicates software version:</p> <p>1 blink = standard sw</p> <p>2 blinks = 3.01 sw</p> <p>3 blinks = 5.00 sw</p> <p>4 blinks = 4.00</p> <p>LED 2 will stop blinking if LED 3 is ON.</p>
LED 3 - ALARMS	<p>Indicates an alarm type</p> <p>Switch OFF - OK</p> <p>1 blink = LED disconnected or faulty driver</p> <p>2 blinks = LED short-circuited</p> <p>3 blinks = Fan failure or disconnected</p> <p>4 blinks = Laser failure or disconnected</p> <p>5 blinks = Push button pressed longer than 5 seconds or short-circuited</p> <p>6 blinks = Timer temperature >105°</p> <p>7 blinks = LED temperature >110°</p>
ALARM DIAGNOSTIC	

The timer identifies anomalies/errors/malfunctions. The outgoing messages from J7 are managed and interpreted by "intelligent board"

bit = 10 ms

byte:

1 bit sync (1)

8 bit data - transmission bit L....H

1 bit stop (0)

Output status:

0 = output disabled

1 = output enabled

BYTE 1: firmware version

BYTE 2: firmware mode + jumper status

bit7: firmware mode bit 1

bit6: firmware mode bit 0

bit5: JP13

bit4: JP12

bit3: JP11

bit2: JP10

bit1: JP9

bit0: JP8

BYTE 3: input/output status

bit7: 1 = LED push button pressed

bit6: 1 = Laser push button pressed

bit5: 1 = ENC1 input activated

bit4: 1 = ENC2 input activated

bit3: 1 = O.C. output activated

bit2: 1 = Laser ON

bit1: 1 = Fan ON

bit0: 1 = LED ON

BYTE 4: alarms:

bit7:

bit6: High LED temperature

bit5: High timer temperature

bit4: Push button pressed more than 5 seconds


bit3: Laser error

bit2: Fan error

bit1: LED short-circuited

bit0: LED disconnected or faulty driver

GC 24 V Power Supply Converter

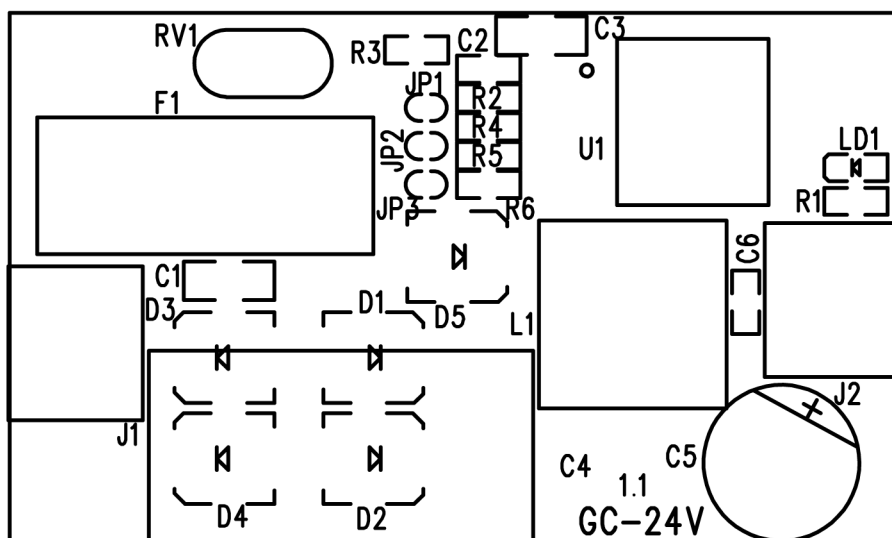

Characteristics:

- Input voltage V_{in} : 24-35 V AC $\pm 10\%$, 28-45 V DC $\pm 10\%$
- Fusible: 3.15 A T standard 5x20mm

Outputs:

- +24Vdc / 2A
- +5Vdc / 2A
- +48Vdc / 2A
- +12Vdc / 2A

SYSTEM START-UP

On system power-up, the GC007 board software performs a series of tests aimed at self-assessment. Depending on the position of the key at the back of the collimator, the automatic or manual control phase will initiate on receipt of CanBus controls.

The display will then read the following information:

- Bootloader version of the **GC 009** board

GC- 009 BOOTLOADER
v. 1.02

- Firmware version of the GC 009 (Slave) board

GC- 009 ver. 1.02

- collimator model and its serial number (see programming of serial number)

R 221 ACS Collimator
Serial Number 12345

Where 12345 stands for the serial number (see programming of serial number)

- collimator model, customer name and SW of the GC 007 (Master) board

R221 ACS - STANDARD
Sw. 1.10

- current collimator parameters:

CROSS 0 LONG 0
SID 0

- current filtering level will be shown if a four-position radiological filter (0 Al) is in place:

CROSS 0 LONG 0
SID 0 0AL

If the operator wishes to configure the system to different requirements, the configuration must be initiated by keeping the FILTER and the LIGHT push buttons pressed for approximately 5 seconds..



Current CanBus communication speed is displayed; this value may be changed with the use of the dipswitches placed on the **GC 007** board. See Chapter **ELECTRONIC SYSTEM**.

BOARD CONFIGURATION
Can Speed 500 kBits/s

NOTE



CAN BUS SPEED SHOWN ON THE DISPLAY HAS BEEN CONFIGURED ACCORDING TO THE DIPSWITCH SETTING ON THE GC007 PCB

followed by:

Wait Command.....

It is possible to perform the collimator configuration by sending the CAN Bus string, or by setting it manually. The configurable parameters and manual calibration are described in the following paragraph.

DESCRIPTION OF CONFIGURATION CONTROLS

The collimator configuration is performed by sending CAN Bus messages to ID 0x600, DLC=8.

Data Writing

Send the message using ID=0x600 as shown in the chart below.

Data Reading

Send the message using ID=0x601 and D0 with a number as shown in the chart below. The collimator will reply with the ID=0x602 message using formatted data according to the chart below.

Note: Byte D1 and D2 must be specified if the subcontrol 0x38 is required.

For each sent control the operator will get feedback on the display, therefore, it will be possible to check the correctness of the configuration.

A Warning will be displayed in the event of an error message.

NOTE



TO SET THE MAXIMUM CROSS AND LONG OPENING, SEND THE VALUE IN MM EXPRESSED IN A 16 BIT NUMBER, EG:
430 MM CORRESPOND TO: 0x01 0xAE
250 MM CORRESPOND TO: 0x00 0xFA

UPDATING OF FIRMWARE


NOTE



FIRMWARE VERSIONS ARE UPDATED WITH LOADER.EXE PROGRAM SUPPLIED BY RALCO SRL.

Functional Overview

The Loader_2.1 is a Hardware + Software tool used to directly connect the USB Serial Converter to the X-Ray collimator and to configure X-Ray Collimator parameters according to RALCO CAN communication protocol.

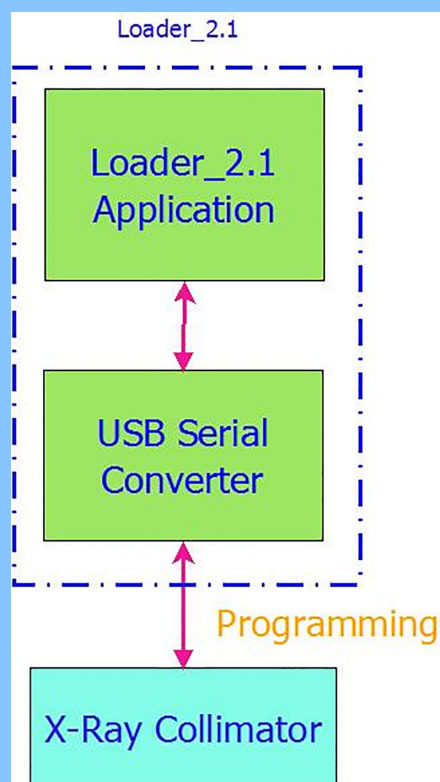

Bill of Material

The Loader_2.1 tool requires the following items:

- Loader_2.1 Application Program.
- USB Serial Converter.



Installation

USB Serial Converter


In order to install the USB Serial Converter, it's enough to connect it to the PC, and it will be automatically recognized as a serial port.

Loader_2.1 Application Program

In order to install the Loader_2.1 Application Program, simply unzip the .rar archive provided by RALCO into the selected folder.

Workflow

Following the detailed workflow.

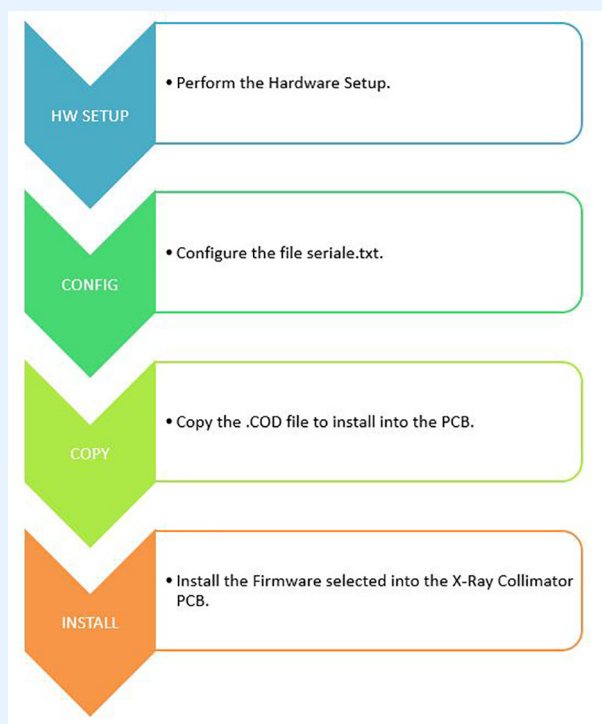

HW Setup

connect the USB Serial Converter adapter to the PC and to the Serial Port placed into the GC007 PCB (visible by unmounting the XRay Collimator front panel).

Connect the X-Ray Collimator to the power supply but keep it still powered OFF.

Config

Into the main folder of the Loader_2.1 Application program there is the file `seriale.txt`.

Such file is used by the program to set the following configurations:

1. COM Port Number : it's the number of the COM port to use. Such number shall match exactly the number of the COM port



2. detected by the Operating System.

For instance, if Windows configures the USB Serial converter as COM5 (see image below).

The file `seriale.txt` shall be modified as follows:

COM = 5

BAUD_RATE = 115200

3. COM Baud Rate : it's the baud rate used for the file transferring procedure. (that is equal to 115200kHz for the X-Ray Collimators).

Copy

Copy the .COD file into the same folder where are stored both `Loader_2.1.exe` and `seriale.txt` files.

Such file contains the Firmware to upgrade into the X-Ray Collimator.

Important note:

Use only .COD firmware files provided and approved by RALCO S.r.l. Any other file different from these ones can be responsible of a malfunctioning of the collimator.

Install

Firmware Upload Complete

Open the Loader_2.1.exe file and power ON the collimator.

If the configuration stage has been correctly completed, the upload procedure shall start automatically.

When the X-Ray Collimator upload is completed, the tool retrieves the proper status message.

If the firmware uploading procedure is completed successfully, the X-Ray Collimator shall reboot automatically. If not, reboot it.

Issues List

The Loader_2.1 application program can retrieve error messages in one of the following scenarios:

1. Fail to open COM port.

```
*****
BOOTLOADER ver 2.1
*****
OPENING SERIAL PORT COM3: FAILED!
UPLOAD FAILED.
Hit any key to exit.
```

Possible Root Causes of the problem:

- the serial.txt file has not been properly configured.
- the USB Serial Converter is not correctly connected both to the PC and / or to the X-Ray Collimator Serial Port.

2. Firmware Upload does not start.

```
*****
BOOTLOADER ver 2.1
*****
WAITING FOR COMMUNICATION. Hit \'Q\' to quit.
```

Possible Root Causes of the problem:

- The Bootloader is not installed on the GC007 PCB.
- The X-Ray Collimator is not powered on.
- The GC007 PCB is defective or broken.
- There are more than one COM devices active on the PC, and the Loader_2.1 application program is trying to upload the firmware on the wrong one.

CONFIGURATION VIA SERIAL INTERFACE

Configuration of the collimator via serial interface is performed in Ralco with updated software; the customer will be required to load the software as described in the preceding paragraph.

POTENTIOMETER POSITIONING ALARMS

The alarm may be enabled in message 0x100 (0x7 D0) through 0x600 entering control 0x33 in the configuration phase.

This message includes the control bytes for potentiometer positioning alarms.

When the collimator activates a movement alarm, the system will read the collimator alarm state through message 0x100 and the number of alarms activated from start-up.

The red LED, Exposure Hold, on the collimator front panel flashes to indicate an alarm state.

Alarm State Reset

Turn the collimator key from automatic to manual. This will reset the shutters and Iris if assembled.

CAN BUS MESSAGES

MESSAGE LIST OVERVIEW

Message Type	ID	DLC	Description
Tx_Config	0x600	8	Message sent to the collimator in order to set its configuration parameters. The collimator shall be in Configuration Mode.
Tx_Config	0x601	8	Message sent to the collimator in order to read the actual value of its configuration parameters. The collimator shall be in Configuration Mode.
Rx_Config	0x602	8	Message retrieved from the collimator with the actual value of its configuration parameters. The collimator shall be in Configuration Mode.
Tx_Command	0x7A0	8	Message sent to the collimator in order to command its functionalities.
Tx_Command	0x7A1	8	Message sent to the collimator in order to set its Left and Right Side Formats.
Tx_Command	0x7A2	8	Message sent to the collimator in order to set its Lateral Side SID value.
Tx_Command	0x7A3	8	Message sent to the collimator in order to command its functionalities.
Tx_Command	0x7A4	8	Message sent to the collimator in order to set its Operation Mode (Automatic / Manual).
Tx_Command	0x7A5	8	Message sent to the collimator in order to set its SID value without changing the actual shutters opening value.
Tx_Command	0x7A6	8	Message sent to the collimator in order to get its actual Hardware / Firmware version.
Tx_Command	0x7A7	8	Message sent to the collimator in order to set the output frequency of 0x7F0, 0x7F1, 0x7F9 and 0x7FC status messages.
Tx_Command	0x7A8	8	Message sent to the collimator in order to set the maximum and the minimum opening value for both Cross and Long shutters.
Tx_Command	0x7A9	8	Message sent to the collimator in order to set the Light Trigger Time.
Tx_Command	0x7AA	8	Message sent from the ASR003 External Board to the Collimator in order to communicate the actual status of the ASR003 digital inputs.
Tx_Command	0x7AB	8	Message sent from the ASR003 External Board to the Collimator in order to communicate the actual voltage values of the ASR003 potentiometers and vertical buckies.
Tx_Command	0x7AC	8	Message sent from the ASR003 External Board to the Collimator in order to communicate the actual voltage values of the ASR003 potentiometers and left and right buckies.
Tx_Command	0x7AD	8	Message sent to the collimator in order to set some of its configuration parameters without entering in Configuration Mode.
Tx_Command	0x7AE	8	Message sent from the ASR003 External Board to the Collimator in order to visualise on the LCD display its actual Firmware Version.
Tx_Command	0x7AF	8	Message sent to the collimator in order to set all configuration parameters without entering in Configuration Mode.
Rx_Status	0x7F0	8	Status Message retrieved from the Collimator.
Rx_Status	0x7F1	8	Status Message retrieved from the Collimator.
Rx_Status	0x7F2	8	Message retrieved from the Collimator with its actual Serial Number.
Rx_Status	0x7F3	8	Diagnostic Message retrieved from the GC-LED 4A PCB.
Rx_Status	0x7F4	2	Message retrieved from the Collimator with the actual Front Panel Buttons Statuses relatively to table movements.
Rx_Status	0x7F4	8	Event and Error Messages retrieved from the Collimator.

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Message Type	ID	DLC	Description
Rx_Status	0x7F5	8	Status Message retrieved from the Collimator.
Rx_Status	0x7F7	8	Status Message retrieved from the Collimator.
Rx_Acknowledge	0x7F8	8	Acknowledge Message retrieved from the Collimator.
Rx_Status	0x7F9	8	Status Message retrieved from the Collimator.
Rx_Status	0x7FC	8	Status Message retrieved from the Collimator.
Tx_DisplayConfig	0x7C0	8	Message sent to the collimator in order to set its actual LCD Display Visualization Type.
Tx_DisplayConfig	0x7C1	8	Message sent to the collimator in order to set its Welcome Message (Valid for LCD Free Mode Visualization Type only).
Rx_Status	0x7D0	2	Status Message retrieved from the Collimator.
Rx_Status	0x7D0	8	Status Message retrieved from the Collimator.
Rx_Status	0x100	2	Status Message retrieved from the Collimator.
Rx_Status	0x100	8	Status Message retrieved from the Collimator.

COLLIMATOR CONFIGURATION

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x00	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Requires to the Collimator to visualize on the LCD display the string "RS232COMM".	1.00
Tx_Config	0x600	8	0	0x01	MSB_SerialNr	CSB_SerialNr	CSB_SerialNr	CSB_SerialNr	LSB_SerialNr	0x00	0x00	Store into the GC007 Flash Memory the X-Ray Collimator serial number with the provided value.	1.00
Tx_Config	0x600	8	0	0x02	Coll_Language	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator language displayed on the LCD screen with the provided setting value. Coll_Language : 0x01 = Italian. Coll_Language : 0x02 = English. (default)	1.00
Tx_Config	0x600	8	0	0x03	Meas_Unit	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator measurement unit displayed on the LCD screen with the provided setting value. Meas_Unit : 0x01 = Metric System. (default) Meas_Unit : 0x02 = Imperial System.	1.00
Tx_Config	0x600	8	0	0x04	Show_SID	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator SID visualization displayed on the LCD screen with the provided setting value. Show_SID : 0x00 = The LCD screen does not displays the SID value. Show_SID : 0x01 = The LCD screen displays the SID value. (default)	1.13

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x05	SFW_Installed	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator spectral filter wheel with the provided setting value. SFW_Installed : 0x00 = The X-Ray Collimator does not have the spectral filter wheel installed. (default) SFW_Installed : 0x01 = The X-Ray Collimator includes the spectral filter wheel.	1.00
Tx_Config	0x600	8	0	0x06	SFW_TickType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator spectral filter wheel thickness with the provided setting value. SFW_TickType : 0x01 = The X-Ray Collimator spectral filter wheel thickness is 1mm Al. SFW_TickType : 0x02 = The X-Ray Collimator spectral filter wheel thickness is 2mm Al. (default) SFW_TickType : 0x03 = The X-Ray Collimator spectral filter wheel thickness is 0.3mm Cu. SFW_TickType : 0x04 = The X-Ray Collimator spectral filter wheel thickness is Ag. SFW_TickType : 0x05 = The X-Ray Collimator spectral filter wheel thickness is custom.	1.13
Tx_Config	0x600	8	0	0x07	AutoLight	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator light automatic ignition with the provided setting value. AutoLight : 0x00 = The X-Ray Collimator light is not automatically ignited while moving knobs. AutoLight : 0x01 = The X-Ray Collimator light is automatically ignited while moving knobs.	1.00
Tx_Config	0x600	8	0	0x08	Incl_Installed	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator inclinometer with the provided setting value. Incl_Installed : 0x00 = The X-Ray Collimator does not have the inclinometer installed. (default) Incl_Installed : 0x01 = The X-Ray Collimator includes the inclinometer.	1.00
Tx_Config	0x600	8	0	0x09	SquaredAutoLimits	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator automatic limits of the squared field with the provided setting value. SquaredAutoLimits : 0x00 = The X-Ray Collimator does not have the automatic limits of the squared field configured. (default) SquaredAutoLimits : 0x01 = The X-Ray Collimator has the automatic limits of the squared field configured. (default)	1.00
Tx_Config	0x600	8	0	0x0A	LeftSIDType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator such that the left side SID value is set according to the provided setting value. LeftSIDType : 0x01 = The X-Ray Collimator left side SID set with CAN messages. (default) LeftSIDType : 0x02 = The X-Ray Collimator left side SID set with discreet input values. LeftSIDType : 0x03 = The X-Ray Collimator left side SID set with values retrieved by the potentiometer. LeftSIDType : 0x04 = The X-Ray Collimator left side SID set with fixed values.	1.09

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x0B	LeftReceptorType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator such that the left receptor is set according to the provided setting value. LeftReceptorType : 0x00 = The X-Ray Collimator left receptor is not installed. LeftReceptorType : 0x01 = The X-Ray Collimator left receptor is set with CAN messages (default). LeftReceptorType : 0x02 = The X-Ray Collimator left receptor is set with values retrieved by the bucky. LeftReceptorType : 0x03 = The X-Ray Collimator left receptor is set with fixed values. Switch installed on the left bucky.	1.13
Tx_Config	0x600	8	0	0x0C	RightReceptorType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator such that the right receptor is set according to the provided setting value. RightReceptorType : 0x00 = The X-Ray Collimator right receptor is not installed. RightReceptorType : 0x01 = The X-Ray Collimator right receptor is set with CAN messages (default). RightReceptorType : 0x02 = The X-Ray Collimator right receptor is set with values retrieved by the bucky. RightReceptorType : 0x03 = The X-Ray Collimator left receptor is set with fixed values. Switch installed on the right bucky.	1.13
Tx_Config	0x600	8	0	0x0D	VertSIDType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator such that the vertical SID value is set according to the provided setting value. VertSIDType : 0x01 = The X-Ray Collimator left side SID set with CAN messages. (default) VertSIDType : 0x02 = The X-Ray Collimator left side SID set with a constant value. VertSIDType : 0x03 = The X-Ray Collimator left side SID set with a single input value. VertSIDType : 0x04 = The X-Ray Collimator left side SID set with a differential input value.	1.00
Tx_Config	0x600	8	0	0x0E	VertReceptorType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator such that the vertical receptor is set according to the provided setting value. VertReceptorType : 0x00 = The X-Ray Collimator right receptor is not installed. VertReceptorType : 0x01 = The X-Ray Collimator right receptor is set with CAN messages (default). VertReceptorType : 0x02 = The X-Ray Collimator right receptor is set according to ATS specifications. VertReceptorType : 0x03 = The X-Ray Collimator right receptor is set with values retrieved by the bucky. VertReceptorType : 0x04 = The X-Ray Collimator right receptor is set with fixed formats values retrieved by the firmware. VertReceptorType : 0x05 = The X-Ray Collimator right receptor is set with fixed formats values. Switch installed on the table bucky.	1.13

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x0F	LSB_FixedSIDValue	MSB_FixedSIDValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator fixed SID (Source Image Distance) with the provided setting value defined with bytes [MSB_FixedSIDValue; LSB_FixedSIDValue]. FixedSIDValue : 100 cm (default) FixedSIDValue eligible range of values : [50;300]cm	1.00
Tx_Config	0x600	8	0	0x10	LSB_MaxFixedSIDValue	MSB_MaxFixedSIDValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator maximum fixed SID (Source Image Distance) with the provided setting value defined with bytes [MSB_MaxFixedSIDValue; LSB_MaxFixedSIDValue]. MaxFixedSIDValue : 180 cm (default) MaxFixedSIDValue eligible range of values : [50;300]cm	1.00
Tx_Config	0x600	8	0	0x11	LSB_MinFixedSIDValue	MSB_MinFixedSIDValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator minimum fixed SID (Source Image Distance) with the provided setting value defined with bytes [MSB_MinFixedSIDValue; LSB_MinFixedSIDValue]. MinFixedSIDValue : 80 cm (default) MinFixedSIDValue eligible range of values : [50;300]cm	1.00
Tx_Config	0x600	8	0	0x12	VertSTD	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator vertical STD (Source Table Distance) with the provided setting value. VertSTD : 0 cm (default) VertSTD eligible range of values : [0;15]cm	1.00
Tx_Config	0x600	8	0	0x13	InclAlarmThreshold	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator inclinometer alarm threshold with the provided setting value. InclAlarmThreshold : 3° (default) InclAlarmThreshold eligible range of values : [0;15]°	1.00
Tx_Config	0x600	8	0	0x14	SFW_RetToZero	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator return to zero configuration of the spectral filter wheel with the provided setting value. SFW_RetToZero : 0x00 = The spectral filter wheel does not return to the zero position before reaching the target position. (default) SFW_RetToZero : 0x01 = The spectral filter wheel returns to the zero position before reaching the target position.	1.00
Tx_Config	0x600	8	0	0x15	Show_KeyLock	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator key lock icon visualization displayed on the LCD screen with the provided setting value. Show_KeyLock : 0x00 = The LCD screen does not displays the key lock icon. Show_KeyLock : 0x01 = The LCD screen displays the key lock icon. (default)	1.00
Tx_Config	0x600	8	0	0x16	Show_InclAngle	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator incline angle visualization displayed on the LCD screen with the provided setting value. Show_KeyLock : 0x00 = The LCD screen does not displays the incline angle value. Show_KeyLock : 0x01 = The LCD screen displays the key incline angle value. (default)	1.00
Tx_Config	0x600	8	0	0x17	SqrdField_RetToZero	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator return to zero configuration of the squared field with the provided setting value. SqrdField_RetToZero : 0x00 = Shutters do not return to the 0x0 position before reaching the target position. (default) SqrdField_RetToZero : 0x01 = Shutters return to the 0x0 position before reaching the target position..	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x18	0x01	MSB_XAxis	LSB_XAxis	MSB_YAxis	LSB_YAxis	0x00	0x00	Configure the X-Ray Collimator such that it will automatically retrieve offset inclinometer data values for the selected side. (the side is the same as the lexan one)	1.00
Tx_Config	0x600	8	0	0x19	CrossSteps	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Cross shutter manual regulation steps with the provided setting value. CrossSteps : 0x0A. (default) CrossSteps eligible range of values : [1;20]mm	1.00
Tx_Config	0x600	8	0	0x1A	LongSteps	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Long shutter manual regulation steps with the provided setting value. LongSteps : 0x0A. (default) LongSteps eligible range of values : [1;20]mm	1.00
Tx_Config	0x600	8	0	0x1B	CrossCorrection	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Cross shutter opening adjustment with the provided setting value. Such configuration depends on the 0x42 Configuration setting: If 0x42 config. Setting = 0x01 : Cross opening adjustment eligible range : [-25;25]cm on SID value. If 0x42 config. Setting = 0x02 : Cross opening adjustment eligible range : [-10.0;+10.0]% on Cross shutters actual opening value.	1.00
Tx_Config	0x600	8	0	0x1C	LongCorrection	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Long shutter opening adjustment with the provided setting value. Such configuration depends on the 0x42 Configuration setting: If 0x42 config. Setting = 0x01 : Long opening adjustment eligible range : [-25;25]cm on SID value. If 0x42 config. Setting = 0x02 : Long opening adjustment eligible range : [-10.0;+10.0]% on Long shutters actual opening value.	1.00
Tx_Config	0x600	8	0	0x1D	DisplayVisualType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator data organization displayed on the LCD screen with the provided setting value. DisplayVisualType : 0x01 = Data displayed on the LCD according to Standard configuration. (default) DisplayVisualType : 0x02 = Data displayed on the LCD according to Sedecal configuration. DisplayVisualType : 0x03 = Data displayed on the LCD according to Open configuration.	1.00
Tx_Config	0x600	8	0	0x1E	MSB_MaxCrossOpeningValue	LSB_MaxCrossOpeningValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator maximum opening value for Cross shutters with the provided setting value. MaxCrossOpeningValue : 430mm. (default) MaxCrossOpeningValue eligible range of values : [0;500]mm	1.00
Tx_Config	0x600	8	0	0x1F	MSB_MaxLongOpeningValue	LSB_MaxLongOpeningValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator maximum opening value for Long shutters with the provided setting value. MaxLongOpeningValue : 430mm. (default) MaxLongOpeningValue eligible range of values : [0;500]mm	1.00
Tx_Config	0x600	8	0	0x20	EncodersSensitivity	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator encoders sensitivity with the provided setting value. EncodersSensitivity : 25ms. (default) EncodersSensitivity eligible range of values : [0;250]ms	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x21	EncodersInversionSensitivity	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator encoders inversion sensitivity with the provided setting value. EncodersInversionSensitivity : 25ms. (default) EncodersInversionSensitivity eligible range of values : [0;250]ms	1.00
Tx_Config	0x600	8	0	0x22	ShuttersConfiguration	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator shutters configuration with the provided setting value. ShuttersConfiguration = 0x00 : Cross and Long shutters are configured according to technical drawing. (default) ShuttersConfiguration= 0x01 : Cross and Long shutters are configured swapped compared to technical drawing.	1.00
Tx_Config	0x600	8	0	0x23	CollCommProtocol	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator communication protocol with the provided setting value. CollCommProtocol= 0x01 : the X-Ray Collimator operates with the Standard Communication Protocol. (default) CollCommProtocol= 0x02 : the X-Ray Collimator operates with the Sedecal Communication Protocol. CollCommProtocol= 0x03 : the X-Ray Collimator operates with the GMM Communication Protocol. CollCommProtocol= 0x04 : the X-Ray Collimator operates with the CAN CARESTREAM Communication Protocol. CollCommProtocol= 0x05 : the X-Ray Collimator operates with the R302 v 2.09 R15 Communication Protocol.	1.15
Tx_Config	0x600	8	0	0x24	BaudRate	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Baud Rate with the provided setting value. BaudRate= 0x01 : the X-Ray Collimator operates with a Baud Rate of 1Mbit/s. BaudRate= 0x02 : the X-Ray Collimator operates with a Baud Rate of 500kbit/s. BaudRate= 0x03 : the X-Ray Collimator operates with a Baud Rate of 250kbit/s. BaudRate= 0x04 : the X-Ray Collimator operates with a Baud Rate of 125kbit/s. BaudRate= 0x05 : the X-Ray Collimator operates with a Baud Rate of 100kbit/s. BaudRate= 0x06 : the X-Ray Collimator operates with a Baud Rate of 50kbit/s. BaudRate= 0x07 : the X-Ray Collimator operates with a Baud Rate of 20kbit/s. BaudRate= 0x08 : the X-Ray Collimator operates with a Baud Rate of 10kbit/s.	1.00
Tx_Config	0x600	8	0	0x28	MSB_SFW_StepsPosition	LSB_SFW_StepsPosition	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator number of steps that the spectral filter wheel needs to perform in order to reach each position with the provided setting value. StepsPosition : 4262. (default) StepsPosition eligible range of values : [0;12000]	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x29	AnalogInputUsage	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator analog input with the provided setting value. AnalogInputUsage = 0x01 : the GC007 Analog Input is used for the proximity sensor. (default) AnalogInputUsage = 0x02 : the GC007 Analog Input is not used.	1.00
Tx_Config	0x600	8	0	0x2A	MSB_CommandID	LSB_CommandID	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator command message ID with the provided setting value. CommandID : 0x7A0. (default) CommandID eligible range of values : [0x000;0x7F0]	1.00
Tx_Config	0x600	8	0	0x2B	MSB_FAST_ShutMaxFreq	LSB_FAST_ShutMaxFreq	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator maximum frequency of shutters (when they operate in FAST mode) with the provided setting value. ShutMaxFreq : 6000 (Hz) ShutMaxFreq eligible range of values : [500;7000]Hz	1.00
Tx_Config	0x600	8	0	0x2C	MSB_MaxSteps	LSB_MaxSteps	MSB_RefSize	LSB_RefSize	MSB_RefSteps	LSB_RefSteps	0x00	Configure the X-Ray Collimator calibration values (Maximum number of steps, Reference Size, Reference Steps number) for the Cross shutter with the provided setting values. MaxSteps : 7050. RefSize : 495. RefStep : 7050.	1.00
Tx_Config	0x600	8	0	0x2D	MSB_MaxSteps	LSB_MaxSteps	MSB_RefSize	LSB_RefSize	MSB_RefSteps	LSB_RefSteps	0x00	Configure the X-Ray Collimator calibration values (Maximum number of steps, Reference Size, Reference Steps number) for the Long shutter with the provided setting values. MaxSteps : 5540. RefSize : 495. RefStep : 5540.	1.00
Tx_Config	0x600	8	0	0x2E	MSB_RotCommandID	LSB_RotCommandID	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator rotative board ID with the provided setting value. RotCommandID : 0x740. (default) CommandID eligible range of values : [0x000;0x7F0]	1.00
Tx_Config	0x600	8	0	0x2F	SpectralFilterType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator spectral filter wheel type with the provided setting value. SpectralFilterType = 0x01 : on the collimator is installed the 1-HOLE spectral filter wheel. SpectralFilterType = 0x02 : on the collimator is installed the 5-HOLES spectral filter wheel. SpectralFilterType = 0x02 : on the collimator is installed the 5-HOLES SHORT WAY spectral filter wheel.(default)	1.07
Tx_Config	0x600	8	0	0x30	PotInstalled	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator potentiometers with the provided setting value. PotInstalled = 0x00 : potentiometers not installed.(default) PotInstalled = 0x01 : potentiometers installed on shutters. PotInstalled = 0x01 : encoders installed on shutters.	1.14
Tx_Config	0x600	8	0	0x31	SettingsPotOpened	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator potentiometers with shutters totally opened. SettingsPotOpened = 0x01 : start potentiometers configuration with totally opened shutters.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x32	SettingsPotClosed	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator potentiometers with shutters totally closed. SettingsPotOpened = 0x01 : start potentiometers configuration with totally closed shutters.	1.00
Tx_Config	0x600	8	0	0x33	EnableAlarmMsg	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator 0x100 alarm message with the provided setting value. EnableAlarmMsg = 0x00 : the X-Ray Collimator does not retrieve the 0x100 Alarm Message.(default) EnableAlarmMsg = 0x01 : the X-Ray Collimator retrieves the 0x100 Alarm Message.	1.00
Tx_Config	0x600	8	0	0x34	EnableCollRot	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator rotation with the provided setting value. EnableCollRot = 0x00 : the X-Ray Collimator rotation is disabled.(default) EnableCollRot = 0x01 : the X-Ray Collimator rotation is enabled.	1.00
Tx_Config	0x600	8	0	0x37	ProxSensorType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator proximity sensor type with the provided setting value. ProxSensorType : 0x00 = The X-Ray Collimator includes the old PNP proximity sensor type. ProxSensorType : 0x01 = The X-Ray Collimator includes the new NPP proximity sensor type. (default)	1.00
Tx_Config	0x600	8	0	0x38	0x01	FormatsNumber	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator number of fixed formats for both Cross and Long shutters with the provided setting value. FormatsNumber : 0x00. (default) FormatsNumber eligible range of values : [0x00;0x05]	1.00
Tx_Config	0x600	8	0	0x38	0x02	FormatNumber	CrossFixedFormatAperture	LongFixedFormatAperture	0x00	0x00	0x00	Configure the aperture value (in cm) for the selected fixed format for both Cross and Long shutters with the provided setting values. CrossFixedFormatAperture = opening value for Cross shutters for the fixed format selected. LongFixedFormatAperture = opening value for Long shutters for the fixed format selected. CrossFixedFormatAperture, LongFixedFormatAperture eligible range of values : [0x00;0x2B]cm	1.00
Tx_Config	0x600	8	0	0x38	0x03	FormatNumber	CrossFixedFormatAperture	LongFixedFormatAperture	0x00	0x00	0x00	Configure the aperture value (in inches) for the selected fixed format for both Cross and Long shutters with the provided setting values. CrossFixedFormatAperture = opening value for Cross shutters for the fixed format selected. LongFixedFormatAperture = opening value for Long shutters for the fixed format selected. CrossFixedFormatAperture, LongFixedFormatAperture eligible range of values : [0x00;0x11]inch	1.00
Tx_Config	0x600	8	0	0x39	Filter1	Filter2	Filter3	Filter4	0x00	0x00	0x00	Configure the X-Ray Collimator spectral filter wheel filter sequence with the provided setting values. Default Filter Sequence = 0,1,2,3. Filter1,Filter2,Filter3,Filter4 eligible range of values : [0x00;0x03]	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x3A	SetFirmNameOnLCD	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator message that appears on the LCD at the power on. SetFirmNameOnLCD = 0x00 : At the power on, the LCD screen retrieves the string "RALCO" SetFirmNameOnLCD = 0x01 : At the power on, the LCD screen retrieves the string set with the 0x3B Configuration message.	1.00
Tx_Config	0x600	8	0	0x3B	Char1	Char2	Char3	Char4	Char5	Char6	Char7	Configure the X-Ray Collimator message that appears on the LCD at the power on with the provided characters. Note : such string appears only if the SetFirmNameOnLCD parameter of the 0x3A configuration message is set to 0x01.	1.00
Tx_Config	0x600	8	0	0x3C	MSB_ShutterFreqMovement	LSB_ShutterFreqMovement	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator shutters stepper motor frequency when moved by using proper CAN messages. ShutterFreqMovement: 1500 Hz. (default) ShutterFreqMovement eligible range of values : [0x1F4;0x1B58]Hz	1.00
Tx_Config	0x600	8	0	0x3F	EnableUltrasonicRuler	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator ultrasonic ruler with the provided setting value. EnableUltrasonicRuler = 0x00 : the X-Ray Collimator does not include the ultrasonic ruler.(default) EnableUltrasonicRuler = 0x01 : the X-Ray Collimator includes the ultrasonic ruler.	1.00
Tx_Config	0x600	8	0	0x40	MSB_OffsetUltrasonicRuler	LSB_OffsetUltrasonicRuler	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator ultrasonic ruler offset with the provided setting value. ShutterFreqMovement: 270mm. (default)	1.00
Tx_Config	0x600	8	0	0x41	SFW_StepperMotorType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator stepper motor type with the provided setting value. SFW_StepperMotorType = 0x01 : the X-Ray Collimator stepper motor type is the old model (round shape).(default) SFW_StepperMotorType = 0x02 : the X-Ray Collimator stepper motor type is the new model (squared shape). SFW_StepperMotorType = 0x03 : the X-Ray Collimator stepper motor type is the FAST (Dual Energy) model (squared shape).	1.07
Tx_Config	0x600	8	0	0x42	FormatCorrectionType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator format correction type with the provided setting value. FormatCorrectionType = 0x01 : the X-Ray Collimator shutters opening correction type is in cm respect to the actual SFD value. FormatCorrectionType = 0x02 : the X-Ray Collimator shutters opening correction type is in percentage respect to the shutters opening value. (default) NOTE : the changing of correction type selected sets the correction data to 0.	1.00
Tx_Config	0x600	8	0	0x43	MSB_FMaxKnobs	LSB_FMaxKnobs	SpeedLimit	Acceleration	0x00	0x00	0x00	Configure the X-Ray Collimator knobs movement parameters(maximum frequency sent to the stepper motor during movements, knob rotation speed that corresponds to FMax and knobs acceleration) with the provided setting value. FMaxKnobs = 7000Hz. SpeedLimit = 4 Acceleration = 3	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x44	ManMovSpeedType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator manual movement speed type with the provided setting value. ManMovSpeedType = 0x01 : Shutters are manually moved at a constant frequency value (default). ManMovSpeedType = 0x02 : Shutters are manually moved at a frequency value defined with the 0x45 configuration command message.	1.00
Tx_Config	0x600	8	0	0x45	MSB_ManMovFreqSpeed	LSB_ManMovFreqSpeed	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator manual movement speed frequency with the provided setting value. ManMovFreqSpeed : frequency value defined for shutters manual movement. ManMovFreqSpeed = 100 mm/s (default). ManMovFreqSpeed eligible range of values : [10;500] mm/s	1.00
Tx_Config	0x600	8	0	0x46	PotInputType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the GC007 potentiometer input with the provided setting value. PotInputType = 0x01 : the GC007 potentiometer input PIN is connected to the SINGLE STATIVE (Default). PotInputType = 0x02 : the GC007 potentiometer input PIN is connected to the Table.	1.00
Tx_Config	0x600	8	0	0x47	RightSIDType	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator such that the right side SID value is set according to the provided setting value. RightSIDType : 0x01 = The X-Ray Collimator right side SID set with CAN messages. (default) RightSIDType : 0x02 = The X-Ray Collimator right side SID set with discrete input values. RightSIDType : 0x03 = The X-Ray Collimator right side SID set with values retrieved by the potentiometer. RightSIDType : 0x03 = The X-Ray Collimator right side SID set with fixed values.	1.09
Tx_Config	0x600	8	0	0x48	LeftSTD	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator left STD (Source Table Distance) with the provided setting value. LeftSTD : 0 cm (default) LeftSTD eligible range of values : [0;15]cm	1.00
Tx_Config	0x600	8	0	0x49	RightSTD	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator right STD (Source Table Distance) with the provided setting value. RightSTD : 0 cm (default) RightSTD eligible range of values : [0;15]cm	1.00
Tx_Config	0x600	8	0	0x4A	EnableMinFormats	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator minimum formats with the provided setting value. EnableMinFormats = 0x00 : the X-Ray Collimator minimum formats are disabled (Default). EnableMinFormats = 0x01 : the X-Ray Collimator minimum formats are enabled.	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x4B	EnableManCollimation	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator manual collimation with the provided setting value. EnableManCollimation = 0x00 : the X-Ray Collimator manual collimation disabled. EnableManCollimation = 0x01 : the X-Ray Collimator manual collimation is enabled. (default)	1.07
Tx_Config	0x600	8	0	0x4D	FilterBtnEnabling	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Filter Button installed on the Front Panel. FilterBtnEnabling = 0x00 : the X-Ray Collimator front panel filter button is disabled. FilterBtnEnabling = 0x01 : the X-Ray Collimator front panel filter button is enabled.	1.07
Tx_Config	0x600	8	0	0x4E	LightBtnEnabling	0x00	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator Light Button installed on the Front Panel. LightBtnEnabling = 0x00 : the X-Ray Collimator front panel Light button is disabled. LightBtnEnabling = 0x01 : the X-Ray Collimator front panel Light button is enabled.	1.07
Tx_Config	0x600	8	0	0x4F	lightTrigTime	0x00	0x00	0x00	0x00	0x00	0x00	Configure the Light Trigger time when shutters are moved by using knobs. lightTrigTime [5;60] s: default : 6s.	1.09
Tx_Config	0x600	8	0	0x50	LSB_MaxFixedLeftSIDValue	MSB_MaxFixedLeftSIDValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator maximum fixed Left SID (Source Image Distance) with the provided setting value defined with bytes [MSB_MaxFixedLeftSIDValue; LSB_MaxMaxFixedLeftSIDValue]. MaxFixedLeftSIDValue : 100 cm (default) MaxFixedLeftSIDValue eligible range of values : [50;300]cm	1.10
Tx_Config	0x600	8	0	0x51	LSB_MaxFixedRightSIDValue	MSB_MaxFixedRightSIDValue	0x00	0x00	0x00	0x00	0x00	Configure the X-Ray Collimator maximum fixed Right SID (Source Image Distance) with the provided setting value defined with bytes [MSB_MaxFixedRightSIDValue; LSB_MaxFixedRightSIDValue]. MaxFixedRightSIDValue : 100 cm (default) MaxFixedRightSIDValue eligible range of values : [50;300]cm	1.10
Tx_Config	0x600	8	0	0x52	MSB_0x7F5	LSB_0x7F5	0x00	0x00	0x00	0x00	0x00	Command message used to set the 0x7F5 output frequency status messages (in msec). (default value : 0ms) MSB_0x7F5, LSB_0x7F5 : output frequency for the 0x7F5 status message. MSB_0x7F5, LSB_0x7F5 eligible range of values : [50;1000] ms Note : if the frequency output message set is equal to 0ms, the output message is not emitted .	1.13

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Config	0x600	8	0	0x53	filterNr	Ch1	Ch2	Ch3	Ch4	Ch5	Ch6	Configure the name of the spectral filter selected. filterNr : number of the filter for which to configure the name. Ch1...Ch6 : first bundle of Characters for setting the name of the spectral filter. Note : If the spectral filter name is shorter than 6 characters, use 0x00 as terminator. Otherwise, complete the name with the second bundle of characters defined with the configuration command 0x600 sub 0x53.	1.13
Tx_Config	0x600	8	0	0x54	filterNr	Ch7	Ch8	Ch9	Ch10	Ch11	Ch12	Configure the name of the spectral filter selected. filterNr : number of the filter for which to configure the name. Ch7...Ch12 : second bundle of Characters for setting the name of the spectral filter. Note : the last character of the spectral filter name shall be 0x00.	1.13
Tx_Config	0x600	8	0	0x55	0x01	M50	0x00	0x00	0x00	0x00	0x00	Configure the ASR003 M50 pin. D2 = 0x00 : Open = automatic, close = manual (default) D2 = 0x01 : Open = manual; close = automatic	1.13
Tx_Config	0x600	8	0	0x55	0x02	M38	0x00	0x00	0x00	0x00	0x00	Configure the ASR003 M38 pin. D2 = 0x00 : Open = automatic, close = manual (default) D2 = 0x01 : Open = manual; close = automatic	1.13
Tx_Config	0x600	8	0	0x55	0x03	M56	0x00	0x00	0x00	0x00	0x00	Configure the ASR003 M56 pin. D2 = 0x00 : If Open the collimator operates in Manual Mode (Default) D2 = 0x01 : If Open the collimator operates in Automatic Mode, and the LCD displays FTD instead of SID.	1.13
Tx_Config	0x600	8	0	0x56	MSB_OutputFreq	LSB_OutputFreq	0x7F3OutputEnabling	0x00	0x00	0x00	0x00	Configure the GC-LED 4A PCB Diagnostic message. MSB_OutputFreq, LSB_OutputFreq = output frequency for the 0x7F3 status message. Eligible range : [50; 1000]ms. If 0, 0x7F3 status message is not retrieved. 0x7F3OutputEnabling = if 1, 0x7F3 status message emitted from the collimator.	1.14
Tx_Config	0x600	8	0	0x64	SaveConfig	0x00	0x00	0x00	0x00	0x00	0x00	Save the X-ray Collimator configuration parameters with values set. SaveConfig = 0x00 : discard modifications. SaveConfig = 0x01 : save modifications.	1.00
Tx_Config	0x600	8	0	0xFD	0xAA	0x00	0x00	0x00	0x00	0x00	0x00	Restore the X-Ray Collimator configuration parameters to their default values.	1.09
Tx_Config	0x600	8	0	0xFE	0x55	0x00	0x00	0x00	0x00	0x00	0x00	Reboot the X-Ray Collimator.	1.00
Tx_Config	0x600	8	0	0xFF	EnStepperMotorModif	0x00	0x00	0x00	0x00	0x00	0x00	Enable or disable the X-Ray Collimator stepper motor configuration parameter. EnStepperMotorModif = 0x00 : Disable the stepper motor configuration parameter. EnStepperMotorModif = 0x01 : Enable the stepper motor configuration parameter.	1.00
Tx_Config	0x601	8	0	Mux	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Read the X-Ray Collimator Configuration Value for the parameter specified by Mux.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Config	0x602	8	0	Mux	Data	Data	Data	Data	Data	Data	Data	Message retrieved from the collimator as response to the 0x601 Configuration Message with the actual Value for the configuration parameter specified by Mux.	1.00
Tx_Command	0x7A0	8	0	b7 : 1 = The X-Ray Collimator operates in Manual Mode (for STANDARD and GMM Communication protocol only) b6 : 1 = The X-Ray Collimator operates in Automatic Mode (for STANDARD and GMM Communication protocol only) b5 : 1 = Totally Close Cross shutter. b4 : 1 = Totally Open Cross shutter. b3 : 1 = Totally Close Long shutter. b2 : 1 = Totally Open Long shutter.y. b1 : 1 = Not Used. b0 : 1 = Not Used.	b7 : 1 NON-GMM Communication Protocol disable format limits (configuration not saved in Flash Memory) GMM Communication Protocol Set the spectral filter wheel to position defined by bits b7 and b5) b6 : not used. b5: 1 (for GMM Communication Protocol only) Set the spectral filter wheel to position defined by bits b7 and b5) b4: 0 = Not Used b3 : 1 = while sending the 0x7A0 command, ignore data written in bytes D3,D4,D5,D6,D7. b2: 0 = the collimator operates in Radiology mode. b2 : 1 = the collimator operates in Fluoroscopy mode. b1 : 1 = move the spectral filter wheel to the adjacent position (for STANDARD and SEDECAL communication protocol only). Standard Communication Protocol b0 : 1 = revert the actual light status. b0 : 0 = don't change the actual light status. Sedecal Communication Protocol b0 : 1 = switch the light ON. b0 : 0 = don't change the actual light status. GMM Communication Protocol b0 : 1 = switch the light ON. b0 : 0 = switch the light OFF.	b7 : not used. b6 : 1 = MSB_SIDValue. b5: not used. b4: not used. b3 : not used. b2 : not used. b1 : not used. b0 : not used.	LSB_SID Value	MSB_CrossValue	LSB_CrossValue	MSB_LongValue	LSB_LongValue	Command Message.	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7A0	2	0	Cross Shutter Incline Value [0;70°]	Long Shutter Incline Value [0;70°]							Command Message.	1.00
Tx_Command	0x7A1	8	0	MSB_LeftCrossValue	LSB_LeftCrossValue	MSB_LeftLongValue	LSB_LeftLongValue	MSB_RightCrossValue	LSB_RightCrossValue	MSB_RightLongValue	LSB_RightLongValue	Command message used in order to set lateral side formats for both Cross and Long shutters.	1.00
Tx_Command	0x7A2	8	0	MSB_LateralSIDValue	LSB_LateralSIDValue	0x00	0x00	0x00	0x00	0x00	0x00	Command message used in order to set the lateral SID value.	1.00
Tx_Command	0x7A3	8	0	0x01	Filter_position	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to moves the spectral filter wheel to the provided position. Filter_position = [0x00; 0x03].	1.00
Tx_Command	0x7A3	8	0	0x02	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used for requiring the PCB serial number.	1.00
Tx_Command	0x7A3	8	0	0x03	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to change the actual LIGHT status. If light ON, after sending this command it is switched OFF. If light is OFF, after sending this command it is switched ON.	1.00
Tx_Command	0x7A3	8	0	0x04	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to switch ON the D21 led installed on the GC007 PCB.	1.00
Tx_Command	0x7A3	8	0	0x05	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to switch OFF the D21 led installed on the GC007 PCB.	1.00
Tx_Command	0x7A3	8	0	0x06	MSB_StepperMotorFreq	LSB_StepperMotorFreq	0x00	0x00	0x00	0x00	0x00	Command message used to change the actual speed of shutters while moving in automatic mode with the provided value. MSB_StepperMotorFreq, LSB_StepperMotorFreq = [500;6000] Hz	1.00
Tx_Command	0x7A3	8	0	0x07	MSB_StepperMotorFreq	LSB_StepperMotorFreq	0x00	0x00	0x00	0x00	0x00	Command message used to change the actual speed of shutters while moving in manual mode with the provided value. MSB_StepperMotorFreq, LSB_StepperMotorFreq = [500;6000] Hz	1.00
Tx_Command	0x7A3	8	0	0x08	LightStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to change the actual LIGHT status. LightStatus = 0x00 : the LIGHT is switched OFF. LightStatus = 0x01 : the LIGHT is switched ON.	1.00
Tx_Command	0x7A3	8	0	0x09	InclStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to change the actual Inclinator status. InclStatus = 0x00 : the INCLINOMETER is disabled. (collimator always considered as vertical) InclStatus = 0x01 : the INCLINOMETER is enabled. Note : this is a temporary setting : at the power on, the inclinometer status is the same as per defined in configuration. Note 2 : this command works only if the inclinometer is enabled in configuration.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7A3	8	0	0x0B	ShuttersMovingType	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the shutters moving type when movements are performed with message 0x7A0 sub D1. ShuttersMovingType = 0x01 : shutters are moved with constant frequency (as per defined with the 0x7A3 sub. 0x07 command) ShuttersMovingType = 0x02 : shutters are moved with constant speed, even if SID is changed.	1.00
Tx_Command	0x7A3	8	0	0x0C	Shutter	MSB_ShutterSpeed	LSB_ShutterSpeed	0x00	0x00	0x00	0x00	Command message used to set the shutters speed when movements are performed with message 0x7A0 sub D1. Shutter = 0x00 : Cross Shutter Selected. Shutter = 0x01 : Long Shutter Selected. MSB_ShutterSpeed, LSB_ShutterSpeed : shutter speed set [10; 500] mm/s. Default : 100mm/s	1.00
Tx_Command	0x7A3	8	0	0x0D	KnobsStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual KNOBS status. KnobsStatus = 0x00 : knobs are DISABLED. KnobsStatus = 0x01 : knobs are ENABLED.	1.00
Tx_Command	0x7A3	8	0	0x0E	FilterButtonStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual FILTER BUTTON status. FilterButtonStatus = 0x00 : Filter Button is DISABLED. FilterButtonStatus = 0x01 : Filter Button is ENABLED.	1.00
Tx_Command	0x7A3	8	0	0x0F	LightButtonStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual LIGHT BUTTON status. LightButtonStatus = 0x00 : Light Button is DISABLED. LightButtonStatus = 0x01 : Light Button is ENABLED.	1.00
Tx_Command	0x7A3	8	0	0x13	LightTriggerTime	CmdType	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual LIGHT trigger time. LightTriggerTime = [5;60]s. Default : 5s. CmdType = 0x00 : Temporary Setting. CmdType = 0x01 : Permanent Setting. (setting value is saved in flash memory)	1.00
Tx_Command	0x7A3	8	0	0x14	LightStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to ignite the LIGHT for the trigger time set with the 0x7A3 sub 0x13. LightStatus = 0x00 : the LIGHT is switched OFF. LightStatus = 0x01 : the LIGHT is switched ON for the trigger time set, then OFF.	1.00
Tx_Command	0x7A3	8	0	0x15	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used for requesting the GC009 PCB Firmware Version.	1.09
Rx_Command	0x7A3	8	0	0x15	0x95	MSB_GC009FWVersion	CSB_GC009FWVersion	CSB_GC009FWVersion	CSB_GC009FWVersion	CSB_GC009FWVersion	LSB_GC009FWVersion	Message retrieved from the collimator with the actual GC009 PCB Firmware Version.	1.09
Tx_Command	0x7A3	8	0	0xF0	EventStatus	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual 0x7F4 event generation status. EventStatus = 0x00 : 0x7F4 event messages not retrieved by the collimator. EventStatus = 0x01 : 0x7F4 event messages retrieved by the collimator.	1.13

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Co mmand	0x7A3	8	0	0xF1	OptForkStepTest	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual optical fork step test status. OptForkStepTest = 0x00 : optical fork test step disabled. OptForkStepTest = 0x01 : optical fork test step enabled.	1.13
Tx_Co mmand	0x7A3	8	0	0xFD	InclSimStatus	InclValue	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual inclinometer simulation status. InclSimStatus = 0x00 : the inclinometer simulation is disabled. InclSimStatus = 0x01 : the inclinometer simulation is enabled. InclValue = 0x00 : Error simulation. InclValue = 0x01 : Vertical Inclination simulation. InclValue = 0x02 : Left Inclination simulation. InclValue = 0x03 : Right Inclination simulation.	1.00
Tx_Co mmand	0x7A3	8	0	0xFE	RebootType	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to perform the X-Ray Collimator Reboot. RebootType = 0x55 : physical reboot the collimator. RebootType = 0x56 : at the next reboot, the collimator enters in Configuration mode.	1.09
Tx_Co mmand	0x7A4	8	0	CollOperMode	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the X-Ray Collimator Actual Operating Mode. CollOperMode = 0x80 : the collimator is in Manual Mode. CollOperMode = 0x40 : the collimator is in Automatic Mode.	1.00
Tx_Co mmand	0x7A5	8	0	LSB_SID	MSB_SID	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the X-Ray Collimator Actual SID value. MSB_SID, LSB_SID : actual SID value, whose eligible range of values is the one set with Configuration Message.	1.00
Tx_Co mmand	0x7A6	8	1	0x00	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to get the X-Ray Collimator GC007 PCB Actual HW/FW Version.	1.00
Rx_Co mmand	0x7A6	8	1	MSB_HW	CSB_HW	CSB_HW	LSB_HW	0x53	MSB_FW	CSB_FW	LSB_F W	Command message used to get the X-Ray Collimator GC007 PCB Actual HW/FW Version.	1.00
Tx_Co mmand	0x7A7	8	0	MSB_0x7F0	LSB_0x7F0	MSB_0x7F1	LSB_0x7 F1	MSB_0x7 F9	LSB_0x7 F9	LSB_0x7 FC	LSB_0 x7FC	Command message used to set the output frequency status messages (in msec). (default value : 0ms) MSB_0x7F0, LSB_0x7F0 : output frequency for the 0x7F0 status message. MSB_0x7F1, LSB_0x7F1 : output frequency for the 0x7F1 status message. MSB_0x7F9, LSB_0x7F9 : output frequency for the 0x7F9 status message. MSB_0x7FC, LSB_0x7FC : output frequency for the 0x7FC status message. Note : if the frequency output message set is lower than 100ms, the output message is retrieved only once.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7A8	8	0	MSB_MaxCross	LSB_MaxCross	MSB_MaxLong	LSB_MaxLong	MSB_MinCross	LSB_MinCross	MSB_MinLong	LSB_MinLong	Command message used to set the output frequency status messages (in msec). (default value : 0ms) MSB_MaxCross, LSB_MaxCross : Maximum Cross Opening Value. (default : 430mm) MSB_MaxLong, LSB_MaxLong : Minimum Long Opening Value. (default : 430mm) MSB_MinCross, LSB_MinCross : Maximum Cross Opening Value. (default : 0mm) MSB_MinLong, LSB_MinLong : Minimum Long Opening Value. (default : 0mm) Note : it's possible to set Minimum Cross and Long opening values only if they are enabled in configuration.	1.00
Tx_Command	0x7A9	8	0	LightTriggerTime	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to set the actual LIGHT trigger time. LightTriggerTime = [5;60]s. Default : 5s.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7AA	8	0	b0 : 1 = left side panoramic enabled. b1 : 1 = right side bucky enabled. b2 : 1 = right side panoramic enabled. b3 : 1 = open Long Shutter. b4 : 1 = close Long Shutter. b5 : 1 = open Cross Shutter. b6 : 1 = close Cross Shutter. b7 : Not Used	b0 : = not used. b1 : = not used. b2 : = not used. b3 : = not used. b4 : = not used. b5 : = not used. b6 : = not used. b7 : 1 = left side bucky enabled.	b0 : = not used. b1 : = not used. b2 : 1 = Set Fixed Format #5. b3 : 1 = Set Fixed Format #4. b4 : 1 = Set Fixed Format #3. b5 : 1 = Set Fixed Format #2. b6 : 1 = Set Fixed Format #1. b7 : 1 = The collimator operates in Fluoroscopy Mode. b7 : 0 = The collimator operates in Radiology Mode.	b0 : Not Used. b1 : 1 = Set Spectral Filter Wheel in position 1. b2 : = not used. b3 : 1 = Set Lateral SID to position 1. b4 : 1 = Set Spectral Filter Wheel in position 2. b5 : 1 = Set Spectral Filter Wheel in position 3. b6 : 1 = Set Lateral SID to position 5. b7 : 1 = Set Lateral SID to position 4.	b0 : 1 = Set Spectral Filter Wheel in position 2. b2 : 1 = Set Lateral SID to position 3. b3 : 1 = Set Lateral Panoramic SID to position 1. b4 : 1 = Set Lateral Panoramic SID to position 2. b5 : = not used. b6 : 1 = Inclinator Disabled Temporarily. b7 : = not used.	b0 : = not used. b1 : = not used. b2 : = not used. b3 : 1 = Set Spectral Filter Wheel in position 4. b4 : 1 = Switch the Light ON. b4 : 0 = Switch the Light OFF. b5 : = not used. b6 : = not used. b7 : = not used.	b0 : = not used. b1 : = not used. b2 : = not used. b3 : = not used. b4 : 1 = Read Calibration. b5 : = not used. b6 : = not used. b7 : = not used.	b0 : = not used. b1 : = not used. b2 : = not used. b3 : 1 = Set Tomography Mode. b4 : 1 = Set Manual Operation Mode. b5 : = not used. b6 : = not used. b7 : 1 =	Command message used from the ASR003 External Board in order to set the actual external board digital input status.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7AB	8	0	MSB_VPotStat	LSB_VPotStat	MSB_VPotTable	LSB_VPotTable	MSB_VPotBuckyCrossTable	LSB_VPotBuckyCrossTable	MSB_VPotBuckyLongTable	LSB_VPotBuckyLongTable	Command message used from the ASR003 External Board in order to set the actual potentiometers voltage value. MSB_VPotStat,LSB_VPotStat = actual stative potentiometer voltage value. MSB_VPotTable,LSB_VPotTable = actual table potentiometer voltage value. MSB_VPotBuckyCrossTable,LSB_VPotBuckyCrossTable = actual Cross Bucky Table potentiometer voltage value. MSB_VPotBuckyLongTable,LSB_VPotBuckyLongTable = actual Long Bucky Table potentiometer voltage value.	1.00
Tx_Command	0x7AC	8	0	MSB_VPotBuckyCrossLeft	LSB_VPotBuckyCrossLeft	MSB_VPotBuckyLongLeft	LSB_VPotBuckyLongLeft	MSB_VPotBuckyCrossRight	LSB_VPotBuckyCrossRight	MSB_VPotBuckyLongRight	LSB_VPotBuckyLongRight	Command message used from the ASR003 External Board in order to set the actual potentiometers voltage value. MSB_VPotBuckyCrossLeft,LSB_VPotBuckyCrossLeft = actual Cross Bucky Left potentiometer voltage value. MSB_VPotBuckyLongLeft,LSB_VPotBuckyLongLeft = actual Long Bucky Left potentiometer voltage value. MSB_VPotBuckyCrossRight,LSB_VPotBuckyCrossRight = actual Cross Bucky Right potentiometer voltage value. MSB_VPotBuckyLongRight,LSB_VPotBuckyLongRight = actual Long Bucky Right potentiometer voltage value.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7AD	8	0	b0 : 1 = Set English Language. b1 : 1 = Set Italian Language. b2 : 1 = Set Metric Measurement Unit. b3 : 1 = Set Imperial Measurement Unit. b4 : 1 = Cross SID correction as per defined in D2. b5 : 1 = Long SID correction as per defined in D3. b6 : = not used. b7 : 1 = Sub D1 Used.	Not Used.	If D0 b7 = 0 If Format Correction Type (0x600 sub 0x42) set is in cm respect to the actual SFD value. D2 = Cross Correction Value [-25;+25]cm If Format Correction Type (0x600 sub 0x42) set in percentage respect to the shutters opening value. D2 = Cross Correction Value [-10;+10]%	If D0 b7 = 0 If Format Correction Type (0x600 sub 0x42) set is in cm respect to the actual SFD value. D2 = Long Correction Value [-25;+25]cm If Format Correction Type (0x600 sub 0x42) set in percentage respect to the shutters opening value. D2 = Long Correction Value [-10;+10]% If D0 b7 = 1 D3 = 0x00 : Save Settings Temporarily. D3 = 0x01 : Save Settings Permanently	Not Used	Not Used	Not Used	Not Used	Command message used to set some configuration parameters without entering in Configuration Mode.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_Command	0x7AE	8	0	0x00	0x00	0x00	0x00	0x00	0x00	0x00	0x00	Command message used to get the actual ARS003 External Board FW Version.	1.00
Tx_Command	0x7AF	8	0	Data	Data	Data	Data	Data	Data	Data	Data	Command message used exactly as the 0x600 Configuration message in order to set X-Ray Collimator Configuration Parameters without entering in Configuration mode. Note : in order to make changes effective, it's mandatory to save configuration values and to reboot the collimator by using the following messages: 0x7AF 8 0x64 0x01 0x7AF 8 0xFE 0x55	1.00
Rx_Status	0x7F0	8	0	b0 : 1 = the collimator is rebooting. b1 : = not used. b2 : = not used. b3 : = not used. b4 : = not used. b5 : = not used. b6 : 1 = LIGHT switched ON. b7 : = not used.	b0 : = not used. b1 : = not used. b2 : = not used. b3 : = not used. b4 : = not used. b5 : 1 = the collimator is at 0°. b6 : 0 = the collimator operates in manual mode. b6 : 1 = the collimator operates in automatic or in ExpHold mode. b7 : 1 = proximity sensor is enabled.	b0 : 1 = spectral filter wheel actual position = 1. b1 : 1 = spectral filter wheel actual position = 2. b2 : = MSB for SID > 255cm. b3 : = not used. b4 : Not Used. b5 : 1 = spectral filter wheel stepper motor is moving. b6 : 1 = cross shutter stepper motor is moving. b7 : 1 = long shutter stepper motor is moving. Note : if both b0 and b1 = 1, the spectral filter wheel actual position is 3.	LSB SID Value	MSB Actual Vertical Cross Value	LSB Actual Vertical Cross Value	MSB Actual Vertical Long Value	LSB Actual Vertical Long Value	Status Message that retrieves information as per described in detail.	1.00
Rx_Status	0x7F1	8	0	Actual Inclination Value [-128;+127°]	Not Used	Not Used	Actual Spectral Filter Wheel Position [0;3]	LSB Lateral SID Value	MSB Lateral SID Value	Not Used	Not Used	Status Message that retrieves information as per described in detail.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Status	0x7F2	8	0	0x53	0x2F	0x4E	MSB_SN	CSB_SN	CSB_SN	CSB_SN	LSB_SN	Status Message that retrieves the actual X-Ray Collimator Serial Number Set. Note : the first 3 digits are constant (0x53 0x2F 0x43 = S/N)	1.00
Rx_Status	0x7F3	8	0	0x00 = GC-LED Not Connected 0xFF = GC-LED in short Circuit Otherwise : number of chars of the 0x7F3 Status Message	GC LED FW Version	GC LED FW Mode and Jumpers status	GC LED input / output actual status	GC LED Alarm	GC LED Internal Temperature	GC LED External Temperature	Not Used	Status Message that retrieves from the GC-LED PCB.	1.14
Rx_Status	0x7F4	1	0	b0 : 1 = left button pressed. b1 : 1 = right button pressed. b2 : 1 = down button pressed. b3 : 1 = up button pressed. b4 : 1 = left rotation button pressed. b5 : 1 = right rotation button pressed. b6 : = not used. b7 : = not used.								If enabled, this status Message retrieves the actual status of buttons installed on the X-Ray Collimator Front Panel. If button not pressed : Message retrieved each 1s. If button pressed : Message retrieved each 80ms.	1.13
Rx_Event	0x7F4	8	0	0x01	0x00	0x00	0x00	0x00	0x00	0x00	0x00	EV_POWER_ON : Event message retrieved from the collimator at the power on.	1.13
Rx_Event	0x7F4	8	0	0x02	StepperMotorNr	0x00	0x00	0x00	0x00	0x00	0x00	EV_MOT_STARTED : Event message retrieved from the collimator when the selected motor starts its movement.	1.13
Rx_Event	0x7F4	8	0	0x03	StepperMotorNr	0x00	0x00	0x00	0x00	0x00	0x00	EV_MOT_STOPPED : Event message retrieved from the collimator when the selected motor stops its movement.	1.13
Rx_Event	0x7F4	8	0	0x04	StepperMotorNr	0x00	0x00	0x00	0x00	0x00	0x00	EV_DSC_STARTED : Event message retrieved from the collimator when both the selected motor and the corresponding knob start their movement.	1.13

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Event	0x7F4	8	0	0x05	StepperMotorNr	0x00	0x00	0x00	0x00	0x00	0x00	EV_DSC_STOPPED : Event message retrieved from the collimator when both the selected motor and the corresponding knob stop their movement.	1.13
Rx_Event	0x7F4	8	0	0x06	StepperMotorNr	0x00	0x00	0x00	0x00	0x00	0x00	EV_DONE : Event message retrieved from the collimator when the selected motor complete its movement.	1.13
Rx_Event	0x7F4	8	0	0x07	0x00	AlarmTemp	0x00	MSB_Temp	CSB_Temp	CSB_Temp	LSB_Temp	EV_INFO : Event message retrieved from the collimator when its temperature value is again in eligible range of values. MSB_Temp,LSB_Temp = Actual X-Ray Collimator Temperature Value.	1.13
Rx_Event	0x7F4	8	0	0x07	0x00	AlarmLedPCB	0x00	0x00	0x00	0x00	0x00	EV_INFO : Event message retrieved from the collimator when the LED board is restored after a fault.	1.13
Rx_Event	0x7F4	8	0	0x08	0x00	0x00	0x00	MSB_LightStatus	CSB_LightStatus	CSB_LightStatus	LSB_LightStatus	EV_LIGHT : Event message retrieved from the collimator when its light status changes. MSB_LightStatus,LSB_LightStatus = Actual X-Ray Collimator Light Status: 0x00 = Light OFF 0x01 : Light ON	1.13
Rx_Event	0x7F4	8	0	0x09	ChNumber	CanBusStatus	0x00	0x00	0x00	0x00	0x00	EV_CANBUS : Event message retrieved from the collimator when the CAN Bus status changes on the selected channel. ChNumber = Channel Number related to the EV_CANBUS CanBusStatus = 0x01 = BusOff CanBusStatus = 0x01 = BusOff Recovery CanBusStatus = 0x01 = Can Bus Passive	1.13
Rx_Event	0x7F4	8	0	0x0A	0x00	0x00	0x00	0x00	0x00	0x00	0x00	EV_INIT_START : Event message retrieved from the collimator when it starts its initialization.	1.13
Rx_Event	0x7F4	8	0	0x0B	0x00	0x00	0x00	0x00	0x00	0x00	0x00	EV_MASTER_READY : Event message retrieved from the collimator when it finishes its initialization and it is in ready status.	1.13
Rx_Event	0x7F4	8	0	0x0C	0x00	0x00	0x00	MSB_KeyStatus	CSB_KeyStatus	CSB_KeyStatus	LSB_KeyStatus	EV_CHIAVE : Event message retrieved from the collimator when its key status changes. MSB_KeyStatus,LSB_KeyStatus = Actual X-Ray Collimator Key Status: 0x00 = Automatic 0x01 : Manual	1.13
Rx_Event	0x7F4	8	0	0x0D	StepperMotorNr	0x00	0x00	MSB_FilterPos	CSB_FilterPos	CSB_FilterPos	LSB_FilterPos	EV_FILTER_READY : Event message retrieved from the collimator when its spectral filter wheel changes its position. StepperMotorNr = number of the stepper motor connected to the spectral filter wheel. MSB_FilterPos,LSB_FilterPos = Actual X-Ray Collimator Spectral Filter Wheel Position [0;3]	1.13
Rx_Event	0x7F4	8	0	0x0E	StepperMotorNr	0x00	0x00	MSB_StepsNr	CSB_StepsNr	CSB_StepsNr	LSB_StepsNr	EV_FILTER_READY : Event message retrieved from the collimator when the selected shutter complete its opening movement in order to indicate the number of steps missed for obscuring the optical fork. StepperMotorNr = number of the stepper motor that has completed its opening motion. MSB_StepsNr,LSB_StepsNr = number of missing steps for obscuring the optical fork.	1.13

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Event	0x7F4	8	0	0x0F	StepperMotorNr	0x00	0x00	MSB_AnalogicPotValue	CSB_AnalogicPotValue	CSB_AnalogicPotValue	LSB_AnalogicPotValue	EV_POTENZIOMETRO : Event message retrieved from the collimator that contains the actual analogic value of the potentiometer connected to the indicated stepper motor. StepperMotorNr = number of the stepper motor. MSB_AnalogicPotValue,LSB_AnalogicPotValue = analogic value of the potentiometer connected to StepperMotorNr.	1.13
Rx_Event	0x7F4	8	0	0x10	FrontButtonNr	0x00	0x00	MSB_ButtonStatusValue	CSB_ButtonStatusValue	CSB_ButtonStatusValue	LSB_ButtonStatusValue	EV_TASTO_FRONTALE : Event message retrieved from the collimator that contains the actual status value of the indicated front panel button. FrontButtonNr = number of the front panel button pressed. FrontButtonNr = 0x01 : LIGHT front panel button. FrontButtonNr = 0x01 : SPECTRAL FILTER WHEEL front panel button. MSB_ButtonStatusValue,LSB_ButtonStatusValue = STATUS value of the front panel button pressed. MSB_ButtonStatusValue,LSB_ButtonStatusValue = 0x00 : Front Panel Button Released. MSB_ButtonStatusValue,LSB_ButtonStatusValue = 0x01 : Front Panel Button Pressed.	1.13
Rx_Event	0x7F4	8	0	0x40	StepperMotorNr	0x01	0x00	MSB_WrongPosValue	CSB_WrongPosValue	CSB_WrongPosValue	LSB_WrongPosValue	EV_ERROR : Error message retrieved from the collimator when it receives a movement value for the stepper motor indicated out of its eligible range of values. StepperMotorNr = stepper motor number for which the collimator has received a value out of the eligible range. MSB_WrongPosValue,LSB_WrongPosValue = wrong position value received by the collimator for the selected stepper motor.	1.13
Rx_Event	0x7F4	8	0	0x40	StepperMotorNr	0x02	0x00	MSB_AttemptsFilterPositioning	CSB_AttemptsFilterPositioning	CSB_AttemptsFilterPositioning	LSB_AttemptsFilterPositioning	EV_ERROR : Error message retrieved from the collimator that contains the number of attempts performed by the spectral filter wheel in order to reach the requested position. StepperMotorNr = stepper motor number related to the spectral filter wheel. MSB_AttemptsFilterPositioning,LSB_AttemptsFilterPositioning = number of attempts performed by the spectral filter wheel in order to reach the requested position.	1.13
Rx_Event	0x7F4	8	0	0x40	0x00	0x03	0x00	MSB_ActualTempValue	CSB_ActualTempValue	CSB_ActualTempValue	LSB_ActualTempValue	EV_ERROR : Error message retrieved from the collimator when its actual temperature is higher than the maximum threshold value. MSB_ActualTempValue,LSB_ActualTempValue = actual X-Ray Collimator temperature value.	1.13
Rx_Event	0x7F4	8	0	0x40	0x00	0x04	0x00	MSB_ActualTempValue	CSB_ActualTempValue	CSB_ActualTempValue	LSB_ActualTempValue	EV_ERROR : Error message retrieved from the collimator when its actual temperature is lower than the maximum threshold value. MSB_ActualTempValue,LSB_ActualTempValue = actual X-Ray Collimator temperature value.	1.13
Rx_Event	0x7F4	8	0	0x40	StepperMotorNr	0x05	0x00	0x00	0x00	0x00	0x00	EV_ERROR : Error message retrieved from the collimator when the external board received a message from the indicated stepper motor later than the timeout set. StepperMotorNr = stepper motor number that sends messages to the external board later than the timeout set.	1.13

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Event	0x7F4	8	0	0x40	StepperMotorNr	0x06	0x00	MSB_DeltaPotValue	CSB_DeltaPotValue	CSB_DeltaPotValue	LSB_DeltaPotValue	EV_ERROR : Error message retrieved from the collimator that contains the difference between the actual value retrieved by the potentiometer connected to the selected stepper motor and its theoretical value. StepperMotorNr = stepper motor number connected to the potentiometer. MSB_DeltaPotValue,LSB_DeltaPotValue = difference between the actual value retrieved by the potentiometer connected to the selected stepper motor and its theoretical value.	1.13
Rx_Event	0x7F4	8	0	0x40	StepperMotorNr	0x07	0x00	0x00	0x00	0x00	0x00	EV_ERROR : Error message retrieved from the collimator if the indicated stepper motor generates a timeout error during the X-Ray Collimator boot-up procedure. StepperMotorNr = number of the first stepper motor that generates a timeout error at the boot up.	1.13
Rx_Event	0x7F4	8	0	0x40	0x00	0x08	0x00	0x00	0x00	0x00	0x00	EV_ERROR : Error message retrieved from the collimator that indicated that a general failure error occurred.	1.13
Rx_Event	0x7F4	8	0	0x40	StepperMotorNr	0x09	0x00	MSB_DeltaPotValue	CSB_DeltaPotValue	CSB_DeltaPotValue	LSB_DeltaPotValue	EV_ERROR : Error message retrieved from the collimator that contains the difference between the actual value retrieved by the potentiometer connected to the selected stepper motor and its theoretical value while performing its calibration. StepperMotorNr = stepper motor number connected to the potentiometer. MSB_DeltaPotValue,LSB_DeltaPotValue = difference between the actual value retrieved by the potentiometer connected to the selected stepper motor and its theoretical value while performing its calibration.	1.13
Rx_Event	0x7F4	8	0	0x40	0x00	0x0A	0x00	MSB_LEDPCBErrorCode	CSB_LEDPCBErrorCode	CSB_LEDPCBErrorCode	LSB_LEDPCBErrorCode	EV_ERROR : Error message retrieved from the collimator that contains the error code generated by the LED PCB. LEDPCBErrorCode = Error Code generated by the LED PCB. LEDPCBErrorCode = 0x00 : LAMP_ALARM_NONE LEDPCBErrorCode = 0x01 : LAMP_ALARM_OUT_ZERO LEDPCBErrorCode = 0x02 : LAMP_ALARM_LED_SCROLL LEDPCBErrorCode = 0x03 : LAMP_ALARM_KEY_PRESSED LEDPCBErrorCode = 0x04 : LAMP_ALARM_INT_TEMP LEDPCBErrorCode = 0x05 : LAMP_ALARM_EXT_TEMP LEDPCBErrorCode = 0x06 : LAMP_ALARM_SHUT_OFF_INT LEDPCBErrorCode = 0x07 : LAMP_ALARM_SHUT_OFF_EXT LEDPCBErrorCode = 0x0A : LAMP_ALARM_ERROR LEDPCBErrorCode = 0x64 : LAMP_ALARM_ERROR_DIAG_0 LEDPCBErrorCode = 0x65 : LAMP_ALARM_ERROR_DIAG_1	1.13
Rx_Event	0x7F4	8	0	0x40	0x00	0x0B	0x00	MSB_MemoryErrorCode	CSB_MemoryErrorCode	CSB_MemoryErrorCode	LSB_MemoryErrorCode	EV_ERROR : Error message retrieved from the collimator that contains the error code generated by the PCB Data Flash Memory. MemoryErrorCode = Error Code generated by the PCB Data Flash Memory. MemoryErrorCode = 0x00 : MEM_ERR_CALIBR MemoryErrorCode = 0x01 : MEM_ERR_SN MemoryErrorCode = 0x02 : MEM_ERR_CONFIG	1.13

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Event	0x7F4	8	0	0x40	0x00	0x0C	0x00	MSB_CANFiFoOverRun	CSB_CANFiFoOverRunNr	CSB_CANFiFoOverRunNr	LSB_CANFiFoOverRunNr	EV_ERROR : Error message retrieved from the collimator that contains the error code generated by the indicated CAN FiFo while it is in OverRun status. CANFiFoOverRunNr = CAN FiFo that generates the OverRun error. CANFiFoOverRunNr = 0x00 : Command Message CAN FiFo CANFiFoOverRunNr = 0x01 : Configuration Message CAN FiFo CANFiFoOverRunNr = 0x02 : Remote Message CAN FiFo	1.13
Rx_Status	0x7F5	8	0	MSB_CrossPotValue	LSB_CrossPotValue	MSB_LongPotValue	LSB_LongPotValue	Not Used	Not Used	Not Used	Not Used	Status Message that retrieves Cross and Long actual potentiometer value at the output frequency set in configuration.	1.13
Rx_Status	0x7F7	8	0	D0 = 0x01 - Actual Inclinator Axes Values If D0 = 0x01 MSB_X axis value	If D0 = 0x01 LSB_X axis value	If D0 = 0x01 MSB_Y axis value	If D0 = 0x01 LSB_Y axis value	Not Used	Not Used	Not Used	Not Used	Status Message that retrieves information as per described in detail at the output frequency set in configuration.	1.00
Rx_Acknowledge	0x7F8	8	0	Data	Data	Data	Data	Data	Data	Data	Data	Acknowledge Message that contains the same data as per sent by means 0x7A3 Command Message. Note : such message is retrieved only if enabled with the 0x600 sub 0x53 configuration message.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Status	0x7F9	8	0	b0 : 1 = LIGHT switched ON. b1 : = not used. b2 : = not used. b3 : = not used. b4 : = not used. b5 : 1 = The collimator operated in Automatic or in Manual mode (front panel red LED switched OFF). b6 : = not used. b7 : 1 = at least one shutter (Cross or Long) is totally closed.	b0 : 1 = The collimator operated in Manual mode. (Yellow Front Panel LED switched ON) b1 : 1 = The collimator is NOT operational. (Red Front Panel LED switched ON) b2 : 1 = The collimator operated in Automatic mode. (Green Front Panel LED switched ON) b3 : 1 = The key is turned such that the collimator operates in Manual mode. (Yellow Front Panel LED switched ON) b4 : = not used. b5 : = not used. b6 : = not used. b7 : = not used.	b0 : 1 = Cross Shutter Configured. b1 : 1 = Long Shutter Configured. b2 : 1 = Not Used. b3 : 1 = Spectral Filter Wheel Configured. b4 : 1 = Vertical Fixed SID Selected. b5 : 1 = English Language active. b6 : = not used. b7 : 1 = Imperial Measurement Unit active.	b0 : 1 = Differential Vertical SID active. b1 : 1 = Vertical Receptor Bucky active. b2 : 1 = Right Side Receptor Bucky active. b3 : 1 = Left Side Receptor Bucky active. b4 : = not used. b5 : = not used. b6 : = not used. b7 : = not used.	b0 : 1 = 2mm Filter Width Selected. b1 : 1 = Vertical CAN or ATS receptor enabled. b2 : 1 = Right Side CAN or ATS receptor enabled. b3 : 1 = Left Side CAN or ATS receptor enabled. b4 : 1 = Left Side CAN SID enabled. b5 : 1 = Right Side CAN SID enabled. b6 : 1 = SID value visualized on LCD. b7 : = not used.	Not Used.	b0 : 1 = Error while updating the Flash Memory with Configuration Values. b1 : 1 = Flash Memory restored to its default configuration values. b2 : 1 = Flash Memory updated with the new configuration values. b3 : = not used. b4 : = not used. b5 : = not used. b6 : = not used. b7 : = not used.	Not Used.	Status Message that retrieves information as per described in detail at the output frequency set.	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Status	0x7FC	8	0	MSB Actual Cross Opening Value (measurement unit : number of steps)	LSB Actual Cross Opening Value (measurement unit : number of steps)	MSB Actual Long Opening Value (measurement unit : number of steps)	LSB Actual Long Opening Value (measurement unit : number of steps)	MSB Actual Spectral Filter Wheel Position Value (measurement unit : number of steps)	LSB Actual Spectral Filter Wheel Position Value (measurement unit : number of steps)	b0 : 1 = Optical Fork Obscured while Cross Shutter is totally closed. b1 : 1 = Optical Fork Obscured while Long Shutter is totally closed. b2 : 1 = Spectral Filter Wheel Optical Fork Obscured . b3 : 1 = Optical Fork Obscured while Cross Shutter is totally opened. b4 : 1 = Optical Fork Obscured while Long Shutter is totally opened. b5 : = not used. b6 : = not used. b7 := not used.	Not Used	Status Message that retrieves information as per described in detail at the output frequency set.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Tx_DisplayConfig	0x7C0	8	0	D0 = 0x01 - LCD Display Visualization Type Standard D0 = 0x01 - LCD Display Visualization Type Sedecal D0 = 0x01 - LCD Display Visualization Type Free	Not Used	Not Used	Not Used	Not Used	Not Used	Not Used	Not Used	Configuration Message to set-up the LCD Display Visualization Type.	1.00
Tx_DisplayConfig	0x7C1	8	0	Data	Data	Data	Data	Data	Data	Data	Data	Configuration Message to set-up the LCD Display Message. Note : such configuration is possible only if the LCD Visualization Type is configured as Free by means the 0x7C0 subD1 message. See Quick Start Guide for an example of LCD Configuration example.	1.00

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Status	0x7D0	2	0	b0 : 1 = Ready relay Output activation required b1 : 1 = ExpHold relay Output activation required b2 : 1 = Manual relay Output activation required. b3 : 1 = Closed Shutters relay Output activation required. b4 : 1 = X-Ray relay Output activation required. b5 : 1 = internal use. b6 : 1 = internal use. b7 : 1 = The Collimator is Booting Up.	Not Used							Status Message that retrieves information as per described in detail at the output frequency set.	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Status	0x7D0	8	0	b0 : 1 = Ready relay Output activation required b1 : 1 = ExpHold relay Output activation required b2 : 1 = Manual relay Output activation required. b3 : 1 = Closed Shutters relay Output activation required. b4 : 1 = X-Ray relay Output activation required. b5 : 1 = internal use. b6 : 1 = internal use. b7 : 1 = The Collimator is Booting Up.	Not Used	b0 : 1 = Isb_Number of Consecutive Spectral Filter Positioning Attempts. b1 : 1 = msb_Number of Consecutive Spectral Filter Positioning Attempts. b2 : = not used. b3 : 1 = Cross or Long Shutter Reset Alarm. b4 : Not Used. b5 : 1 = Long Shutter Alarm. (Wrong Positioning) b6 : 1 = Cross Shutter Alarm. (Wrong Positioning) b7 : 1 = Spectral Filter Wheel Alarm. (Wrong Positioning)	Total Number of Alarms generated for the Spectral Filter Wheel from the X-Ray Collimator Power ON.	Total Number of Alarms generated for the Cross Shutter from the X-Ray Collimator Power ON.	Total Number of Alarms generated for the Long Shutter from the X-Ray Collimator Power ON.	Not Used.	Not Used	Status Message that retrieves information as per described in detail at the output frequency set. Note : such message is retrieved only if configured with the 0x600 sub 0x33 Configuration Message.	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

Type	ID	DLC	RTR	D0	D1	D2	D3	D4	D5	D6	D7	Description	From FW Version
Rx_Stat us	0x100	2	0	b0 : 1 = Ready relay Output activation required b1 : 1 = ExpHold relay Output activation required b2 : 1 = Manual relay Output activation required. b3 : 1 = Closed Shutters relay Output activation required. b4 : 1 = X-Ray relay Output activation required. b5 : 1 = internal use. b6 : 1 = internal use. b7 : 1 = The Collimator is Booting Up.	Not Used							Status Message that retrieves information as per described in detail at the output frequency set.	1.00

MTR 221 ACS - MTR 221 ACS DHHS - Can Bus Messages

QUICK START GUIDE

BOARD GC 007

Description

This collimator features an open communication CanBus connected to board GC007.

CanBus messages used by the collimator are described in this Chapter **CAN BUS MESSAGES - COLLIMATOR CONFIGURATION**; the addresses not taken by the following messages may be used by other communications nodes without interfering with normal collimator operation.

NOTE



COLLIMATOR CANBUS COMMUNICATION SPEED IS SET AT 500 KBIT/S

Communication Protocol	X-Ray Collimator Model	Scenario Description	Messages	Message Description
GC007B_R221 ACS_CAN_RA LCO	R 221/ACS DHHS	Retrieves the X-Ray Collimator general information (Firmware Version, Serial Number) and activates its main functionalities.	0x7A6 8 RTR 0x00 0x00 0x00 0x00 0x00 0x00 0x00 0x00	Get the X-Ray Collimator Hardware and Firmware version. Note : Remote Message .
			0x7A6 8 RTR 0x52 0x32 0x32 0x35 0x53 0x32 0x32 0x32	The Collimator replies with its Hardware and Firmware version. (R221S222)
			0x7A3 8 0x02 0x00 0x00 0x00 0x00 0x00 0x00 0x00	Get the X-Ray Collimator Serial Number.
			0x7F2 8 0x53 0x2F 0x4E 0x31 0x32 0x33 0x34 0x35	The Collimator replies with its Serial Number. (S/N12345)
			0x7A0 8 0x40 0x00 0x00 0x64 0x-- 0x-- 0x01 0xAE	Moves the Long Shutter to 430 [mm] at a SID of 100cm in Automatic Mode.
			0x7F0 8 0x-- 0x-- 0x-- 0x-- 0x-- 0x-- 0x01 0xAE	The Collimator replies with the Long Shutter Actual Position.
			0x7A0 8 0x40 0x00 0x00 0x64 0x-- 0x-- 0x00 0x00	Moves the Long Shutter to 0 [mm] at a SID of 100cm in Automatic Mode.
			0x7F0 8 0x-- 0x-- 0x-- 0x-- 0x-- 0x-- 0x00 0x00	The Collimator replies with the Long Shutter Actual Position.
			0x7A0 8 0x40 0x00 0x00 0x64 0x01 0xAE 0x-- 0x--	Moves the Cross Shutter to 430 [mm] at a SID of 100cm in Automatic Mode.
			0x7F0 8 0x-- 0x-- 0x-- 0x-- 0x01 0xAE 0x-- 0x--	The Collimator replies with the Cross Shutter Actual Position.
			0x7A0 8 0x40 0x00 0x00 0x64 0x00 0x00 0x-- 0x--	Moves the Cross Shutter to 0 [mm] at a SID of 100cm in Automatic Mode.
			0x7F0 8 0x-- 0x-- 0x-- 0x-- 0x00 0x00 0x-- 0x--	The Collimator replies with the Cross Shutter Actual Position.
			0x7A3 8 0x01 0x01 0x00 0x00 0x00 0x00 0x00 0x00	Rotates the Pediatric Filter to position 1 [Adim].
			0x7F1 8 0x-- 0x-- 0x-- 0x01 0x-- 0x-- 0x-- 0x--	The Collimator replies with the Pediatric Filter Actual Position.
			0x7A3 8 0x01 0x02 0x00 0x00 0x00 0x00 0x00 0x00	Rotates the Pediatric Filter to position 2 [Adim].
			0x7F1 8 0x-- 0x-- 0x-- 0x02 0x-- 0x-- 0x-- 0x--	The Collimator replies with the Pediatric Filter Actual Position.
			0x7A3 8 0x01 0x03 0x00 0x00 0x00 0x00 0x00 0x00	Rotates the Pediatric Filter to position 3 [Adim].
			0x7F1 8 0x-- 0x-- 0x-- 0x03 0x-- 0x-- 0x-- 0x--	The Collimator replies with the Pediatric Filter Actual Position.

Communication Protocol	X-Ray Collimator Model	Scenario Description	Messages	Message Description
			0x7A3 8 0x01 0x00 0x00 0x00 0x00 0x00 0x00	Rotates the Pediatric Filter to position 0 [Adim].
			0x7F1 8 0x-- 0x-- 0x-- 0x00 0x-- 0x-- 0x-- 0x--	The Collimator replies with the Pediatric Filter Actual Position.
			0x7A3 8 0x14 0x1E 0x00 0x00 0x00 0x00 0x00	Switch the Light ON for 30s.
			0x7F0 8 0x40 0x-- 0x-- 0x-- 0x-- 0x-- 0x-- 0x--	The Collimator replies with the Light Actual Status.
			0x7A3 8 0x14 0x00 0x00 0x00 0x00 0x00 0x00	Switch the Light OFF.
			0x7F0 8 0x00 0x-- 0x-- 0x-- 0x-- 0x-- 0x-- 0x--	The Collimator replies with the Light Actual Status.

COMPLIANCE VERIFICATION

MINIMUM FILTRATION REQUIREMENT

To indicate compliance with 21 CFR, sub-chapter J, part 1020 of Performance Standard it is necessary for the assembler to perform a series of tests.

Description of test methods are illustrated in this chapter but factors, such as experience, availability of equipment and tolerance on compliance are referred directly to the Safety Standards covering Electro-medical equipment.

WARNINGS



THE FOLLOWING PROCEDURES REQUIRE THAT X-RADIATION BE PRODUCED. TAKE ADEQUATE PRECAUTIONS THAT NO PART OF HUMAN BEING IS EXPOSED TO X-RADIATION, DIRECT OR INDIRECT.

The above HVL requirements can be met if it is demonstrated that the aluminium equivalent in the primary beam is not less than that shown in the following table:

Minimum Filtration Requirement - Beam Quality (HVL)				
X-Ray Tube Voltage (kilovolt peak)		Minimum HVL (mm of aluminum)		
Designed Operating Range (kVp)	Measured Operating Potential (kVp)	Specified Dental System ¹	I-Other X-Ray Systems ²	II-Other X-Ray Systems ³
Below 51	30	1.5	0.3	0.3
	40	1.5	0.4	0.4
	50	1.5	0.5	0.5
From 51 a70	51	1.5	1.2	1.3
	60	1.5	1.3	1.5
	70	1.5	1.5	1.8
Above 70	71	2.1	2.1	2.5
	80	2.3	2.3	2.9
	90	2.5	2.5	3.2
	100	2.7	2.7	3.6
	110	3.0	3.0	3.9
	120	3.2	3.2	4.3
	130	3.5	3.5	4.7
	140	3.8	3.8	5.0
	150	4.1	4.1	5.4

¹ Dental X-ray Systems designed for use with intraoral image receptors and manufactured after December 1, 1980.

² Dental X-ray Systems designed for use with intraoral image receptors and manufactured before or on December 1, 1980, and all other X-ray systems subjected to this section and manufactured before June 10, 2006.

³ All X-ray systems, except dental X-ray systems designed for use with intraoral image receptors subjected to this section and manufactured on or after June 10, 2006.

The information contained in the above table was extracted from the Code of Federal Regulations FDA 21 1020.30 (m).

Type 100 Aluminium Alloy (as given in "ALUMINUM STANDARDS AND DATA" verification of compliance).

Visual Determination of Half-Value Layer (HVL)

The above HVL requirements can be met if it is demonstrated that the aluminium equivalent in the primary beam is not less than that shown in the following Total Filtration table:

Total Filtration Of Primary Beam In Aluminium Equivalence	
Operating Voltage (kVp)	Total Filtration (mm Al Equivalent)
Below 50	0.5
From 51 to 70	1.5
Over 70	2.2

The Aluminium equivalence of each component in the primary beam (X-ray tube and housing, beam limiting device and any additional filtration in the system) is specified on the component, in the technical data attached to the component or can be measured. Determine the total aluminium equivalence in the primary beam and make sure that it is equal or greater than those specified in the above **Table Total Filtration of Primary Beam in Aluminium Equivalence**.

Quick-Check of Minimum Filtration Requirement at a Particular kVp

If the total inherent filtration cannot be seen, then the HVL must be obtained with the following procedures:

The HVL in millimetres of aluminium in the system under test must be compared with those specified in **Table Minimum Filtration Requirement - Beam Quality (HVL)** and must be greater than or equal to the values shown in the table.


- Direct the central X-ray beam perpendicular and in the center of a RAD-Check instrument. Determine the exact distance from the X-ray tube focal spot to the window of the collimator (273 mm - 10.75"). Place the input area of the RAD-CHECK at an equal distance from the collimator window. Collimate the beam to an area slightly larger than the detector.
- Make an exposure at a pre-selected technique factor of 90 kVp and appropriate mA and time values with no added filtration in the beam; record the reading.
Using the type 1100 Aluminium Alloy, tape a total of 2.5 mm of Aluminium to the window of the collimator. Make an exposure using the same technique factors; record the reading.
- Verify that the radiation read with the 2.5 mm Al in the beam is greater or equal to 50% of the radiation read with no filtration in the beam.

Standard Absorber Method

The HVL determination obtained from the following procedures are to be compared with those illustrated in the **Table - Minimum Filtration Requirement - Beam Quality (HVL)**. The HVL

in millimetres of aluminium obtained during the test must be greater or equal than the values listed in the above mentioned table.

- Direct the central X-ray beam perpendicular and in the center of a RAD-Check instrument. Determine the exact distance from the X-ray tube focal spot to the window of the collimator (273 mm - 10.75"). Place the input area of the RAD-CHECK at an equal distance from the collimator window. Collimate the beam to an area slightly larger than the detector.
- Select a tube potential of 100 kVp and appropriate mA and seconds, with no added filtration in the beam make an exposure and record the reading. Using a set of several sheets of 1100 Aluminium Alloy, each having a thickness of 0.5 or 1.0 mm, tape the filtration to the window of the collimator. Make an exposure for each increments of filtration and record the reading.
- Plot the exposure readings (log scale) versus the total added filtration thickness on semi-log paper; see the sample hereunder.
- Verify that HVL values in the useful beam for the above specific tube potential is not less than the values shown in **Table - Minimum Filtration Requirement - Beam Quality (HVL)**.

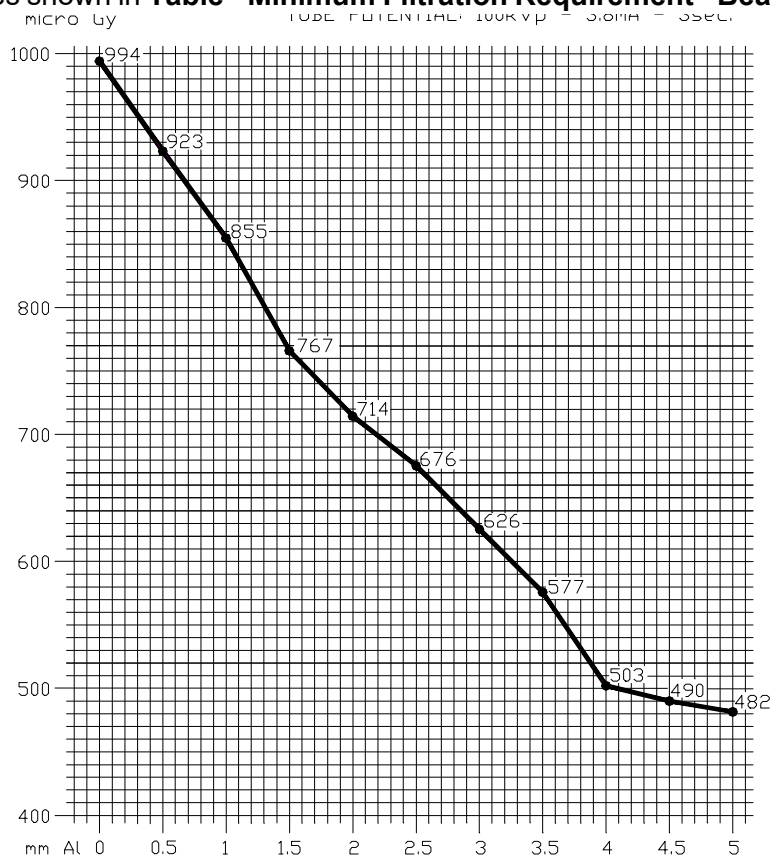

VISUAL DEFINITION OF X-RAY VERSUS LIGHT FIELD

Chapter - **COLLIMATOR CALIBRATION** - paragraph - **Light-Field to X-ray Field Alignment**.

FIELD SIZE INDICATION

Chapter- **ADJUSTMENTS**, paragraph **Field Size Indication**.

CROSSHAIR ALIGNMENT

CHAPTER- ADJUSTMENTS, PARAGRAPH CROSSHAIR ALIGNMENT.

LIGHT FIELD ILLUMINATION INTENSITY

- a) When a light field simulating the X-Ray field is used the illumination provided at 100 cm. cannot be less than: 160 lux [(21 CFR 1020.31 (d) (2) (ii)].
- b) Place the Focus of the X-Ray tube at 100 cm. from the table top were the light field as been projected. Open the collimator's shutters to assure that each quadrant of the light field is larger than the measuring area of the photometer.
- c) Check that the voltage specified by the manufacturer is applied to the lamp, make certain that all surfaces in the light beam are clean and unobstructed.
- d) Place a photometer capable of reading up to 160 lux in the centre of each of the four quadrants of the light field.
- e) Turn on the light beam and read the light intensity, subtract to it the ambient lighting, previously determined.
- f) Verify that the average illumination is higher than 160 lux.
- g) Verify that the contrast ratio is performed between two points:
 - The first point at 3 mm outside the edge of the light field.
 - The second point at 3 mm inside the edge of the light field.
 These measurements are to be performed with the probe of the lux metre set at 1 mm aperture.
- h) Record the measured values including all data regarding the instrument and voltage employed.

EMC COMPLIANCE

The ME Equipment is intended to be used in the PROFESSIONAL EM ENVIRONMENTS.

WARNING



TO ASSURE THAT ACCESSORIES, TRANSDUCERS AND CABLES THAT CAN AFFECT THE EMISSIONS OR IMMUNITY OF THE ME EQUIPMENT, ACCESSORIES, TRANSDUCERS AND CABLES ARE CHOSEN THAT WILL ALLOW THE ME EQUIPMENT TO CONTINUE TO MEET THE EMISSIONS AND IMMUNITY REQUIREMENTS OF THIS COLLATERAL STANDARD.

Electromagnetic Emissions

The R 221 ACS - R 221 ACS DHHS collimator is suitable for use in the specified electromagnetic environment. The purchaser or user of the R 221 ACS - R 221 ACS DHHS should assure that it is used in an electromagnetic environment as described below:

Emissions Test	Compliance	Electromagnetic Environment- Guidance
RF emissions CISPR 11	Group 1	The Collimator R 221 ACS - R 221 ACS DHHS needs special precautions regarding EMC and needs to be installed and put into service according to the EMC report. Portable and mobile RF communications equipment can affect the collimator R 221 ACS - R 221 ACS DHHS.
RF emissions CISPR 11	Class [A]	This R 221 ACS - R 221 ACS DHHS is suitable for use in all establishments other than domestic and those directly connected to the low voltage power supply network which supplies buildings used for domestic purposes. The EMISSIONS characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment
Harmonic emissions IEC 61000-3-2	[Not applicable]	
Voltage fluctuations/flicker emissions IEC 61000-3-3	[Not applicable]	

Electromagnetic Immunity for All Equipment and Systems

Collimator R 221 ACS - R 221 ACS DHHS is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
Electrostatic discharge (ESD) IEC 61000-4-2	8 kV contact 2/4/8/15 kV air	EN 60601-1-2 test level	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Radiated electromagnetic field IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz	IEC 60601-1-2 Test level	Portable and mobile RF communications equipment should be used no closer to any part of the collimator R 221 ACS - R 221 ACS DHHS, including cables. Minimum distance 30 cm
Electrical fast transient/burst IEC 61000-4-4	2 kV for power supply lines 1 kV for input/output lines >3m	EN 60601-1-2 test level	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	0.5/1 kV differential mode 0.5/1/2 kV common mode	EN 60601-1-2 test level	Mains power quality should be that of a typical commercial or hospital environment.
Conducted disturbances induced by RF fields IEC 61000-4-6	3 V 150 kHz to 80 MHz 6V ISM frequencies	IEC 60601-1-2 Test level	Portable and mobile RF communications equipment should be used no closer to any part of the collimator R 221 ACS - R 221 ACS DHHS, including cables. Minimum distance 30 cm
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	10 ms – 0% a 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 20 ms – 0% a 0° 500 ms – 70% a 0° 5 s – 0%	EN 60601-1-2 test level	Mains power quality should be that of a typical commercial or hospital environment. If the user of the collimator R 221 ACS - R 221 ACS DHHS requires continued operation during power mains interruptions, it is recommended that the collimator R 221 ACS - R 221 ACS DHHS be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	EN 60601-1-2 test level	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Frequency Range and Level: RF wireless communication EQUIPMENT

Collimator R 221 ACS - R 221 ACS DHHS is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.

Test Frequency (MHz)	Modulation	Minimum IMMUNITY Level (V/m)	IMMUNITY Level Applied (V/M)
385	**Pulse Modulation: 18 Hz	27	27
450	*FM \pm 5 Hz deviation: 1 kHz sine **Pulse Modulation: 18 Hz	28	28

Frequency Range and Level: RF wireless communication EQUIPMENT

710 745 780	**Pulse Modulation: 217 Hz	9	9
810 870 930	**Pulse Modulation: 18 Hz	28	28
1720 1845 1970	**Pulse Modulation: 217 Hz	28	28
2450	**Pulse Modulation: 217 Hz	28	28
5240 5500 5785	**Pulse Modulation: 217 Hz	9	9

COVER REMOVAL


1.

REMOVE THE SNAP-FIT FRONT PANEL BY PRISING IT OFF AS SHOW IN THE PHOTOGRAPH.




2.

DETACH THE FRONT PANEL CONNECTOR.

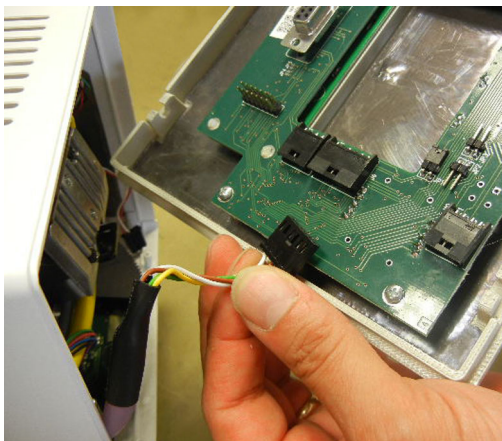

3.

TURN THE COLLIMATOR OVER AND REMOVE THE GUIDE/SPACER FIXING SCREWS.

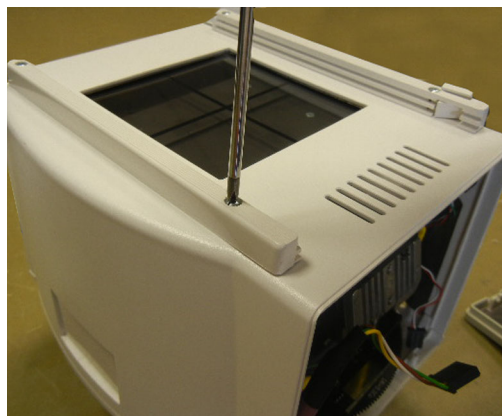


4.

**GENTLY EASE THE TAPE INTO THE
COLLIMATOR.**




5.

**REMOVE THE SCREW FROM THE SNAP-
FIT PLATE. REMOVE THE REAR PLATE.**

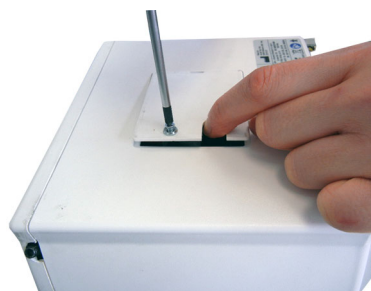


6.

**REMOVE THE SNAP-FIT LATERAL
PANEL BY PRISING IT OFF AS SHOW IN
THE PHOTOGRAPH.**

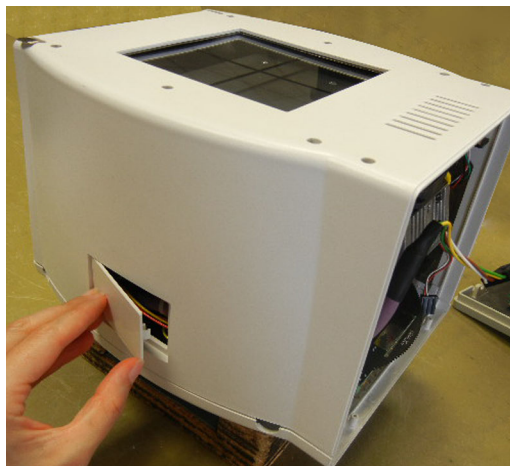

7.

**REMOVE THE COVER BY LIFTING IT
OFF THE COLLIMATOR.**

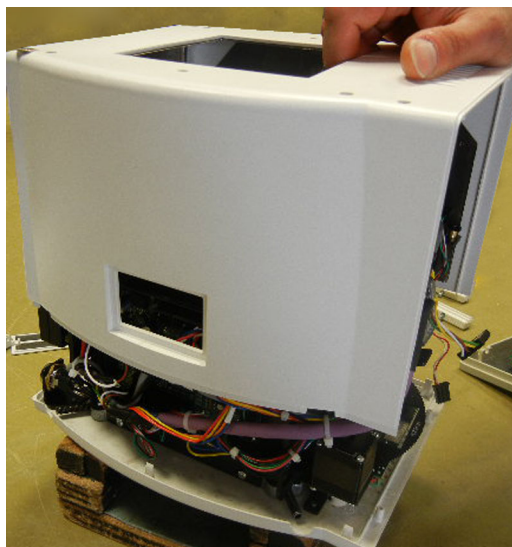

ADJUSTMENTS

WARNINGS



THIS IS PROVIDED AS AN AID TO THE END USER. RALCO IS NOT LIABLE FOR ANY DAMAGES RESULTING FROM THE ALTERING OF PRESET FACTORY CONFIGURATIONS.

ADJUSTMENTS HAVE BEEN PRESET AT THE FACTORY PRIOR TO SHIPMENT OF THE COLLIMATOR. SHOULD ANY PROBLEM ARISE REQUIRING THE NEED TO RECALIBRATE, PLEASE CONTACT RALCO FOR ASSISTANCE BEFORE PROCEEDING WITH THE ADJUSTMENT.

CROSSHAIR ADJUSTMENT

- Activate the light field.
- Adjust the light field to a narrow line for each pair of shutters by turning the two knobs alternately.
- Check that the project cross line is exact halfway between the edges of the shutters, see **Fig. Cross Line**.
- If adjustment is required, remove the cover from the sides and bottom of the collimator, see Chapter - **COVER REMOVAL**.
- Loosen the four screws securing the plastic panel and adjust the cross lines to coincide with the light lines, see Chapter - **COVER REMOVAL**.
- Tighten the screws.



Fig. Cross Line

LASER ADJUSTMENT

- Remove part of the cover to access the point of adjustment, see Chapter- **COVER REMOVAL**.
- The line is to fall on a perpendicular cross-line on the plastic anti-dust panel near the collimator controls, see **Fig. Laser Line**.
- Adjust the position of the line by rotating the laser or moving the base of the laser system.
 - To rotate the laser system, loosen the Allen screw **A**, see **Fig. Laser Adjustment**.
 - Tighten the Allen screw when the laser beam falls on or is parallel to the bisector line drawn on the anti-dust panel.
- Tilt the laser system by loosening the two **B** screws holding the laser system base to the beam limiting device front plate.

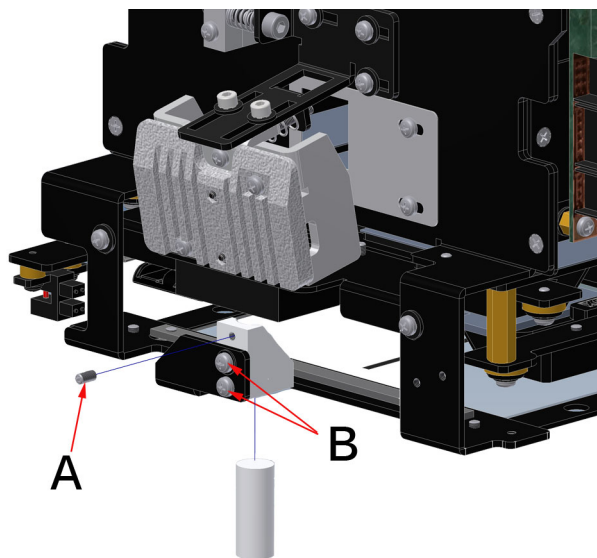


Fig. Laser Adjustment

- Tilt the base until the laser beam falls over the perpendicular bisector line on the anti-dust panel, see **Fig. Laser Alignment**.
- Tighten the two **B** screws.

WARNING



DO NOT APPLY EXCESSIVE FORCE TO THE SCREW. THE LASER SHELL IS IN PLASTIC AND EXCESSIVE PRESSURE COULD CRACK THE PLASTIC AND POSSIBLY SHORT-CIRCUIT THE LASER.

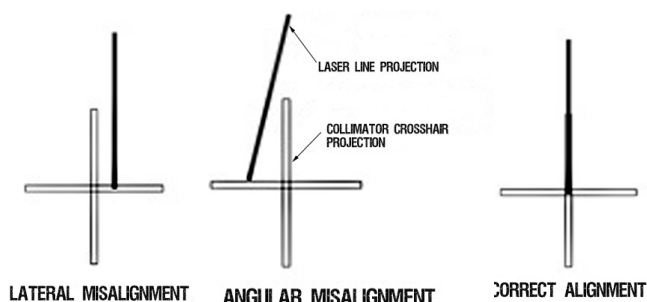


Fig. Laser Line

MOTOR ADJUSTMENT

- Disconnect supply.
- Remove the cover, see Chapter - **COVER REMOVAL**.
- Loosen the 2 screws **C** on the motor support, see **Fig. Motor Adjustment**.
- The plastic gears have a slightly oval shape. Find the critical point and adjust play on this point.
- Tighten the screws **C**.
- Remount the cover.

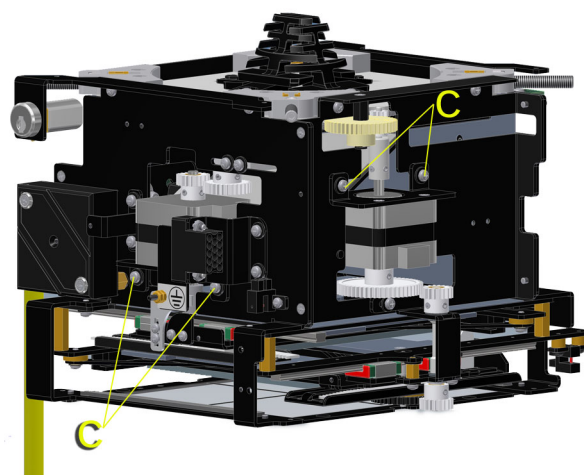


Fig. Motor Adjustment

Photocell Adjustment

Remove the cover, see Chapter - **COVER REMOVAL**.

Loosen the 2 screws **D** of the photocell to adjust it, see **Fig. Photocell Adjustment**.

Move the photocell until a return signal of photocell cut-off is received (related LED lights up on PC board).

Once the photocell is adjusted tighten the screws **D**.

Remount the cover.

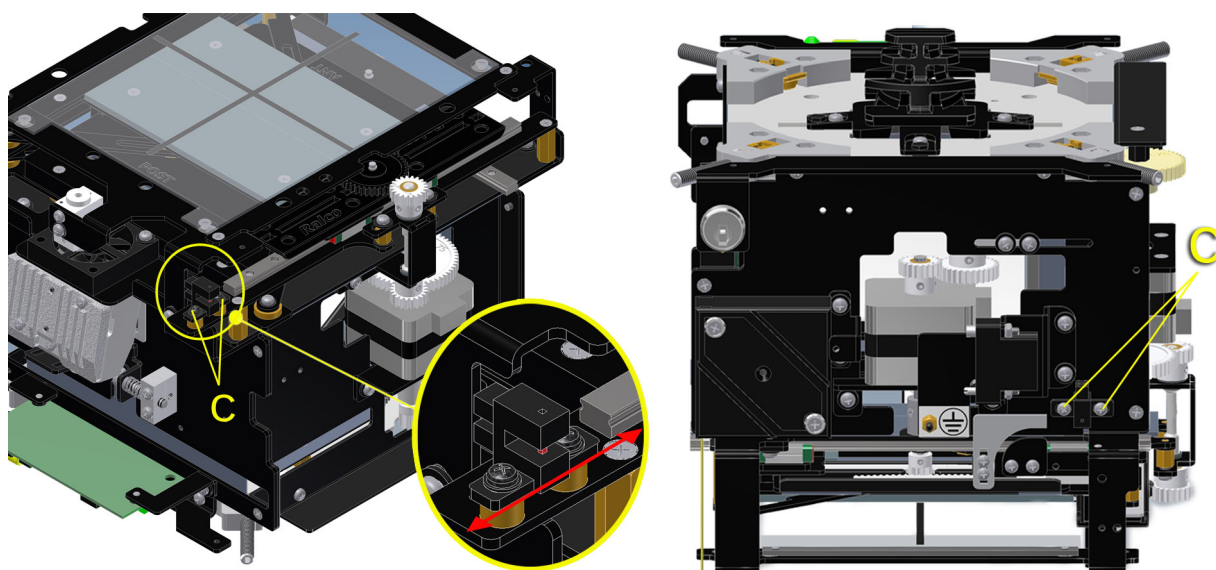


Fig. Photocell Adjustment

TROUBLESHOOTING

A faulty Collimator must not be used until it is repaired and checked.

The use of a faulty collimator might impair the safety of the operator and patient.

Before returning the collimator to Ralco for repair, please make sure that the fault is not caused by one of the problems listed below. If the indications provided fail to solve your fault, **please make sure that you obtain a Return number (RMA) from Ralco for the collimator**, see Chapter - **GENERAL**, paragraph **Repairs**.

GENERAL FAULT FINDING

Function	Fault Description	Fault finding and solutions
Light	Faulty light source	<ul style="list-style-type: none"> Check if power supply is OK (see chapter SPECIFICATION) Check if LED power supply is + 3 V If still NOK, replace LED (see chapter SUBSTITUTIONS)
	Faulty timer	<ul style="list-style-type: none"> Check if power supply is OK (see chapter SPECIFICATION) Check if timer board is supplied Check if fan operates when pressing the push button on the front panel Check if green diode place on the timer is ON when pressing the push button on the front panel. If NOK, replace the timer board (see chapter SUBSTITUTIONS)
	Area defined by Light Field not (completely) on film	<ul style="list-style-type: none"> Check if source flange distance is correct (see chapter INSTALLATION) Check if mirror or light source are positioned correctly (see chapter CALIBRATION) If still NOK, exchange the collimator.
	Faulty ON/OFF push-button	<ul style="list-style-type: none"> Check contact If NOK, replace the front panel

Function	Fault Description	Fault finding and solutions
Shutters	Shutters do not hold position.	<ul style="list-style-type: none"> Check if potentiometer are correctly calibrated (see chapter REPLACEMENTS) If still NOK, replace.
	Shutters fail to move	<ul style="list-style-type: none"> Check motor cabling If still NOK, replace motor
	Shutter close	<ul style="list-style-type: none"> Check motor cabling If still NOK, replace motor

INDICATORS GC007

Number	Color	Description	Fault finding and solutions
LD1	Green	+ V CAN present	<p>In case LD1 is OFF:</p> <ul style="list-style-type: none"> check if system CAN power supply is OK is still NOK, check if system cables are connected correctly if still NOK, replace GC007 (see chapter SUBSTITUTIONS) if still NOK, replace collimator (see chapter SUBSTITUTIONS)
LD2	Green	+5 V present	<p>In case LD2 is OFF:</p> <ul style="list-style-type: none"> check if +24 V is present; LD1 should be ON if still NOK, replace GC007 (see chapter SUBSTITUTIONS) if still NOK, replace collimator (see chapter SUBSTITUTIONS)

LD3	Green	+3.3 V present	<p>In case LD3 is OFF:</p> <ul style="list-style-type: none"> • check if system power supply is OK • is still NOK, check if system cable is connected correctly • if still NOK, replace GC007 (see chapter SUBSTITUTIONS) • if still NOK, replace collimator (see chapter SUBSTITUTIONS)
-----	-------	----------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

LED INDICATORS GC007

Number	LED	Color	Function	Fault Indication	Action
LD1	CAN power supply	Green	ON	• OK	• Not applicable
			OFF	<ul style="list-style-type: none"> • CAN is not connected / connection lost 	<ul style="list-style-type: none"> • Check CAN cabling; perform a power OFF/ON cycle. • Check if JP3 and JP4 are present. • If NOK, replace PCB, see chapter SUBSTITUTIONS
LD2	power supply	Green	ON	• OK	• Not applicable
			OFF	<ul style="list-style-type: none"> • Power supply error 	<ul style="list-style-type: none"> • Check value of power supply. • Perform a power OFF/ON cycle. • If NOK, replace PCB, see chapter SUBSTITUTIONS.
LD3	power supply	Green	ON	• OK	• Not applicable
			OFF	<ul style="list-style-type: none"> • Power supply error 	<ul style="list-style-type: none"> • Perform a power OFF/ON cycle. • If NOK, replace PCB, see chapter SUBSTITUTIONS.
LED1	LED LIFE	Green	Blinking at 1 Hz	• OK, program runs	• Not applicable
			OFF	• CPU error if LD2 and LD3 are ON	• Reset or replace PCB
LED2	LED CAN RX	Green	Short blinking speed	• OK, CAN Bus message received	• Not applicable
			OFF	• Waiting for CAN Bus control	• Not applicable
LED3	LED MEMORY Flash memory communication	Green	Slow blink speed (1 sec)	• OK, default data upload	• Not applicable
			Fast continuous blinking	• Memory error	• Replace PCB
			ON	• Boot loader is working	• Wait for process to finish
			OFF	• Collimator is not in Boot loader state.	•

MTR 221 ACS - MTR 221 ACS DHHS - Troubleshooting

Number	LED	Color	Function	Fault Indication	Action
LED4	LED CONFIG Configuration message received	Green	Very slow blink speed (0.05 sec)	• Incorrect or not existing data	• Used by Ralco
			Medium blink speed (0.3 sec)	• Temporary correct data	• Used by Ralco
			Slow blink speed (1 sec)	• OK, data correct and Flash Memory stored	• Used by Ralco
LED5	HOME photocell status per each connected motor	Green	ON	• Photocell J10 (Cross shutters) engaged	• Not applicable
			OFF	• Photocell J10 (Cross shutters) free	• Not applicable
LED6	HOME photocell status per each connected motor	Green	ON	• Photocell J11 (Long shutters) engaged	• Not applicable
			OFF	• Photocell J11 (Long shutters) free	• Not applicable
LED7	HOME photocell status per each connected motor	Green	ON	• Photocell J12 (Filter) engaged	• Not applicable
			OFF	• Photocell J12 (Filter) free	• Not applicable
LED8	HOME photocell status per each connected motor	Green	ON	• Photocell J13 not used	• Not applicable
			OFF	• Photocell J13 not used	• Not applicable

INDICATORS GC-LED-4A

Number	Color	Description	Fault finding and solutions
LED1	Green	+12 V DC or +24 V DC	<p>In case LED1 is OFF:</p> <ul style="list-style-type: none"> • check if +12/+24 V is present; LED1 should be ON • if still NOK, check if system cables are connected correctly to GC-LED-4A • if still NOK, replace GC-LED-4A (see chapter SUBSTITUTIONS) • if still NOK, replace collimator.
LED2	Yellow	Software version	<ul style="list-style-type: none"> • check if LED2 blinks 4 times • if NOT, wrong software version, replace collimator (see chapter SUBSTITUTIONS)

LED3	Red	Alarms	<p>1 blink: Disconnected LED or Driver failure</p> <ul style="list-style-type: none"> • check if LED is connected correctly • if still NOK, replace LED • if still NOK, replace GC-LED-4A (see chapter SUBSTITUTIONS) • if still NOK, replace collimator. <p>2 blinks: LED short circuit</p> <ul style="list-style-type: none"> • remove the short circuit • if still NOK, replace LED • if still NOK, replace GC-LED-4A (see chapter SUBSTITUTIONS) • if still NOK, replace collimator. <p>3 blinks: Fan failure or disconnected</p> <ul style="list-style-type: none"> • check if fan is connected correctly • if still NOK, replace fan (see chapter SUBSTITUTIONS) • if still NOK, replace GC-LED-4A (see chapter SUBSTITUTIONS) • if still NOK, replace collimator. <p>4 blinks: Laser failure or disconnected</p> <ul style="list-style-type: none"> • check if laser is connected correctly • if still NOK, replace laser (see chapter SUBSTITUTIONS) • if still NOK, replace GC-LED-4A (see chapter SUBSTITUTIONS) • if still NOK, replace collimator. <p>5 blinks: push button pressed longer than 5 seconds or short-circuited</p> <ul style="list-style-type: none"> • check if push button works properly • remove short circuit if present • if still NOK, replace GC-LED-4A (see chapter SUBSTITUTIONS) • if still NOK, replace collimator. <p>6 blinks: PCB temperature > 105°</p> <ul style="list-style-type: none"> • switch OFF the collimator for 10 minutes
------	-----	--------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

SUBSTITUTIONS

LED SUBSTITUTION

WARNING



NOT IMMEDIATELY TOUCH THE DISSIPATER WITH YOUR FINGERS IT COULD BE HOT AND CAUSE SEVERE BURNS.



DO NOT TOUCH THE LIGHT SOURCE, THE SOCKET, OR THE LIGHT BRACKET WITH YOUR FINGER. THEY CAN BE VERY HOT AND CAUSE SEVERE BURNS.



DO NOT TOUCH THE LED LENS WITH YOUR FINGERS DIRECTLY, EVEN WHEN IT IS COLD. OIL FROM YOUR SKIN WILL LOWER THE LED PERFORMANCE. IF YOU HAVE TOUCHED THE LED, WIPE THE LED SURFACE WITH A SOFT CLOTH IN CASE OF CONTACT.

The LED Substitution Kit is composed of the following components: the LED (**A**), and an extension cable (**B**), pictured in **Fig. LED Substitution Kit**. This Kit is designed to substitute any LED Ralco provides.

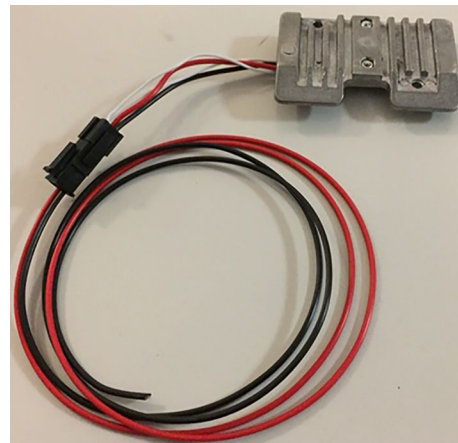
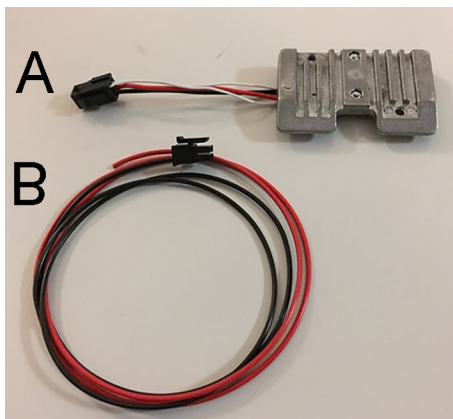


Fig. LED kit

Prior to replacing the LED, confirm which version of LED is installed on your collimator which will ensure the substitution is performed correctly.

- The previous version LED is connected to the board via 2 cables (red and black), see **Fig. Previous Version LED** below.

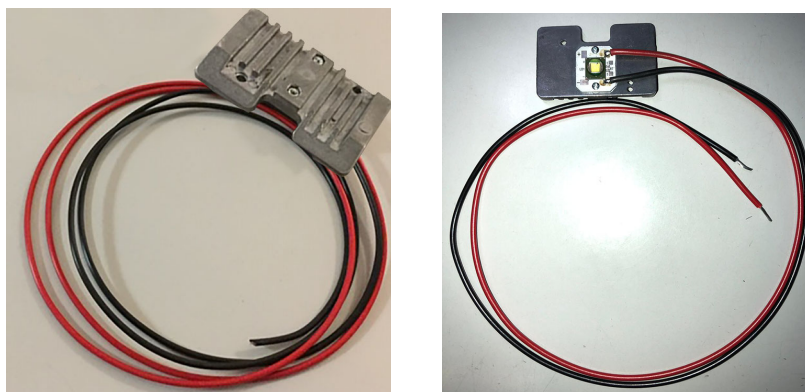


Fig. Previous Version LED

- The new LED is connected to the board via the connector, see **Fig. New Version LED** below.

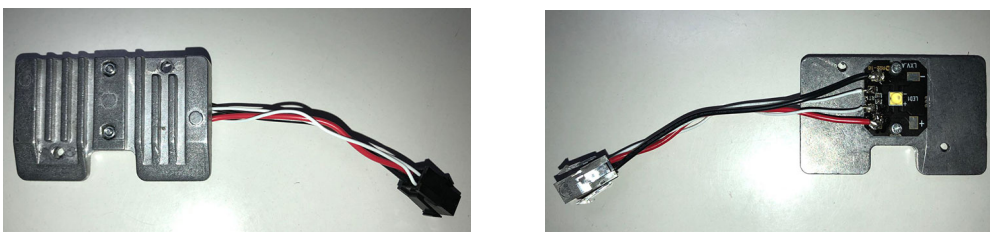


Fig. New Version LED

Replacing the Previous Version LED

To replace the previous version of LED, both components of the LED Substitution Kit (A and B) must be used.

To substitute the LED, proceed as follows:


- Disconnect the collimator supply.
- Remove the cover, see Chapter - **COVER REMOVAL** in the Instruction Manual for your specific model.
- Remove the LED heat sink protection by unscrewing the 2 screws **C**, see **Fig. LED Substitution** below.
- Prior to disconnecting the LED cables from the board, identify the cables and their position on the terminal board, see Chapter **INSTALLATION**, paragraph **Wiring Diagram** in the Instruction Manual for your specific model.
- Carefully remove the LED and the extension cable from their packaging.
- Make sure that the extension cable (**B**) is firmly connected to the LED (**A**), see **Fig. LED kit**.
- Adjust the length of the extension (**B**) cable by cutting the 2 cables (red/black).
- Connect both cables to the board.
- Remount the heat sink protection by tightening the two fixing screws (**C**), see **Fig. LED Substitution** below.
- Remount the covers in reverse order, see Chapter **COVER REMOVAL**.
- Verify the Light Field/X-Ray field correspondence, see Chapter **CALIBRATION**.

Replacing the New Version LED

To replace the new version LED, only part A of the Kit must be used.

To substitute the LED, proceed as follows:

- Disconnect supply.
- Remove the cover, see Chapter **COVER REMOVAL**.
- Remove the LED heat sink protection by unscrewing the 2 screws **C**, see **Fig. LED Substitution** below.
- Detach the connector of the faulty LED from the collimator wiring.
- Substitute the LED with an identical item by using the part **A** of the LED Substitution Kit.
- Remount the heat sink protection by tightening the 2 fixing screws **C**, see **Fig. LED Substitution** below.
- Remount the covers in reverse order, see Chapter **COVER REMOVAL**.
- Verify the Light Field/X-Ray field correspondence, see Chapter **CALIBRATION**.

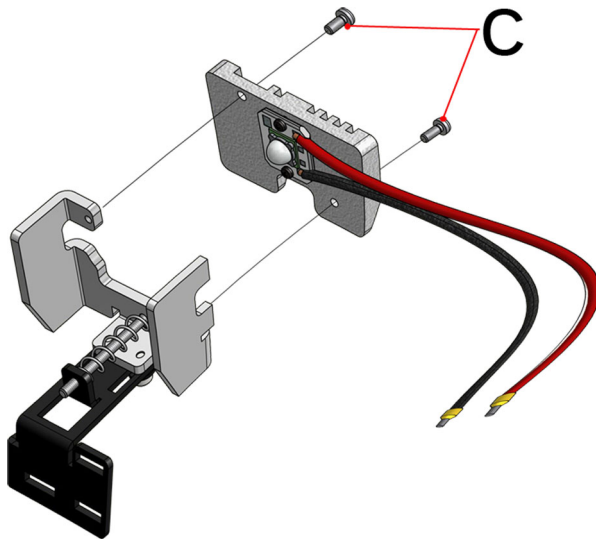

C - LED protection
heatsink screws

SUBSTITUTION OF THE ELECTRONIC BOARD FOR LED

The PCB Substitution Kit is composed of the following components: the board (A), an extension cable (B)* and 8 jumpers (C), pictured in opposite **Fig. PCB Substitution Kit**. This Kit is designed to substitute the previous version of the timer board for LED.

(B)* - The extension cable is added only if the collimator model is equipped with the additional filtration and with the indicator diode place on the front panel. In other cases, is useless and it will not be added to the PCB Substitution Kit.

To substitute the timer board, proceed as follows:

1. Disconnect the collimator supply.
2. Remove the cover, see Chapter **COVER REMOVAL** in the Instruction Manual for your specific model.
3. Prior to disconnecting the cables from the board, identify the cables and their position on the terminal board, see Chapter **INSTALLATION**, paragraph Wiring Diagram in the Instruction Manual for your specific model
4. Carefully remove the timer board, the extension cable and the jumpers from their packaging.

The next 3 steps are applicable only to the collimator equipped with the additional filtration and the indicator diode placed on the front panel.

- Insert the extension cable (B) in the J4 connector, placed on the new timer board, see **Fig. Extension Cable**.
- Detach the connector from the faulty timer board, see **Fig. Connector Removal** below.
- Connect the unplugged connector to the extension cable (B), see **Fig. Extension Cable Connection** below.

5. Remove the 2 screws holding the timer board.
6. Install the new timer board by proceeding in a reverse order; pay particular attention to the connection of the cables on the 2 or 8-way electric terminal.
7. Correctly set up the board by using the 8 jumpers (C) according to the Wiring Diagram. See Chapter **INSTALLATION**, paragraph Wiring Diagram and **GC-LED-4A** in the Instruction Manual for your specific model for further information.
8. Remount the covers in reverse order, see Chapter **COVER REMOVAL**.
9. Verify the Light Field/X-Ray field correspondence, see Chapter **CALIBRATION**.

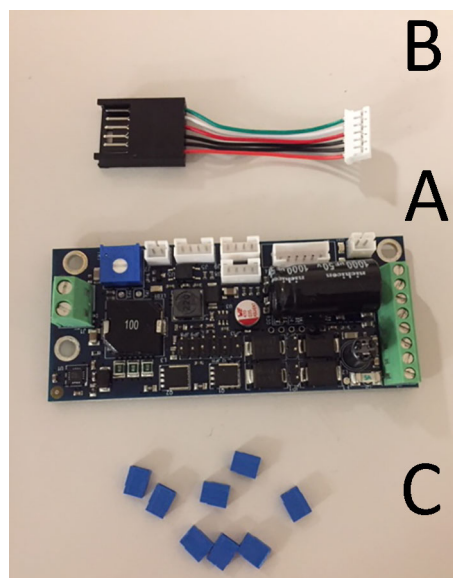


Fig. PCB Substitution Kit

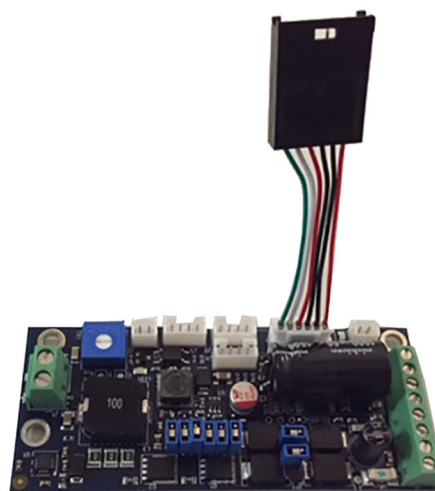


Fig. Extension Cable

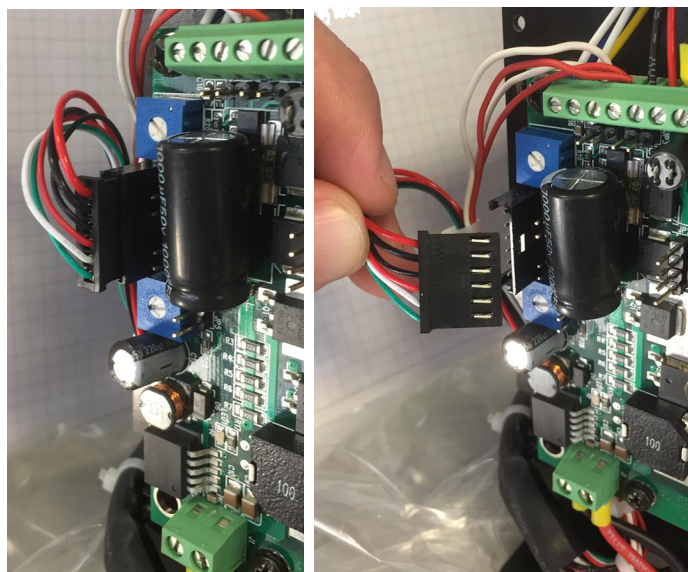


Fig. Connector Removal

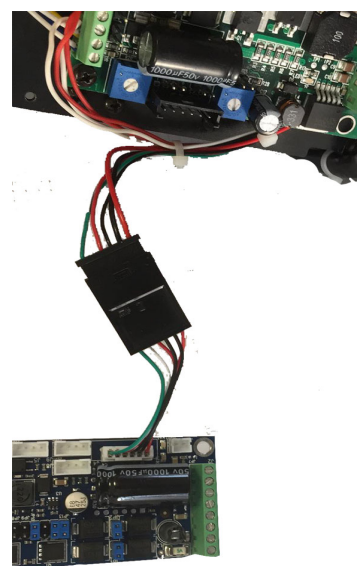


Fig. Extension Cable Connection

SUBSTITUTION OF THE ELECTRONIC BOARD GC 007

- Disconnect Supply.
- Remove the cover to access the component, see Chapter- **COVER REMOVAL**.
- Remove the GC LED 4A and CAN Step 4 boards by unscrewing the screws **F** and **G**. Do not detach the connectors of the boards. See **Fig. Electronic Board GC 007**.
- Identify the cables and their position on the board, see Chapter **INSTALLATION**.
- Disconnect the cables from the electronic board GC007.
- Install the new board and proceed a reverse order; pay particular attention to the connection of the cables.
- Power up the collimator and make sure that LD1 and LD2 are ON, whereas LED 1 is blinking.

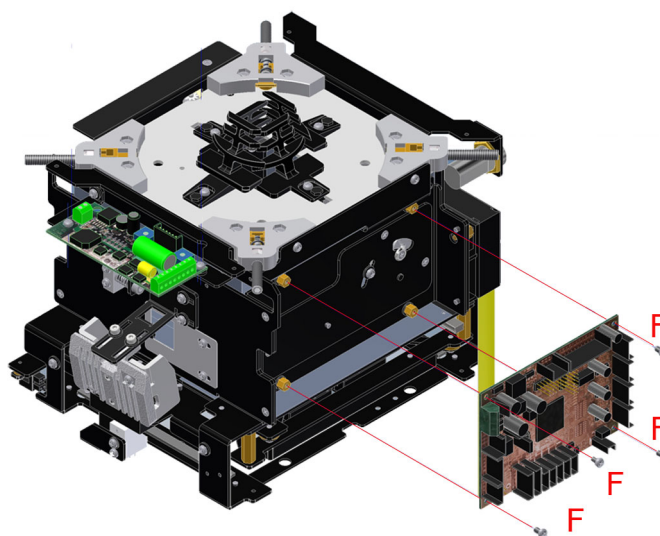


Fig. Electronic Board GC 007

Substitution of the Electronic Board GC 009

- Disconnect Supply.
- Remove part of the cover to access the component, see Chapter- **COVER REMOVAL**.
- Remove the 4 screws holding the electronic board. See **Fig. GC009 PCB**.
- Identify the cables and their position on the board, see Chapter **INSTALLATION**.
- Disconnect the cables from the electronic board.
- Install the new board by proceeding in a reverse order; pay particular attention to the connection of the cables and connectors.
- Power up the collimator and make sure that LED 2 is ON.

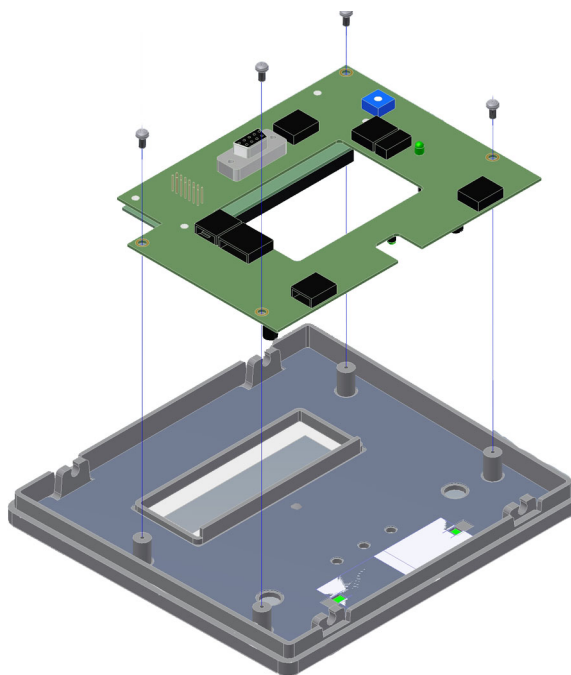


Fig. Electronic Board GC009

SUBSTITUTION OF RETRACTABLE TAPE MEASURE

- Disconnect power supply.
- Remove the parts of the cover to access the component, see Chapter **COVER REMOVAL**.
- Unscrew the two screws **A**, see **Fig. Retractable Tape Measure**.
- Carefully remove the retractable tape measure.
- Substitute with an identical item.
- Tighten the two screws **A**.
- Remount the cover.

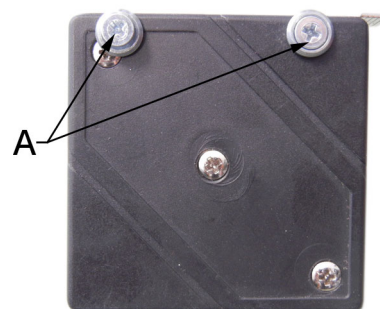


Fig. Retractable Tape Measure

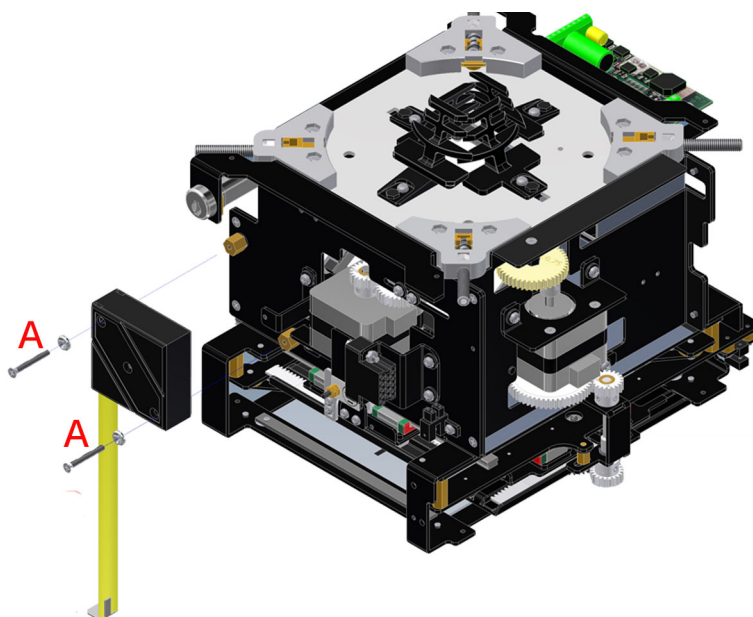


Fig. Substitution of Retractable Tape Measure

SUBSTITUTION OF THE LASER

- Disconnect power supply.
- Remove a part of the cover, see Chapter- **COVER REMOVAL**.
- Unscrew the fixing Allen screws on the upper part of the support.
- Detach the laser cables from the terminal board - white 0 V, red 5 V.
- Remove the laser and substitute with an identical item.
- Connect the laser.
- Tighten the screws.

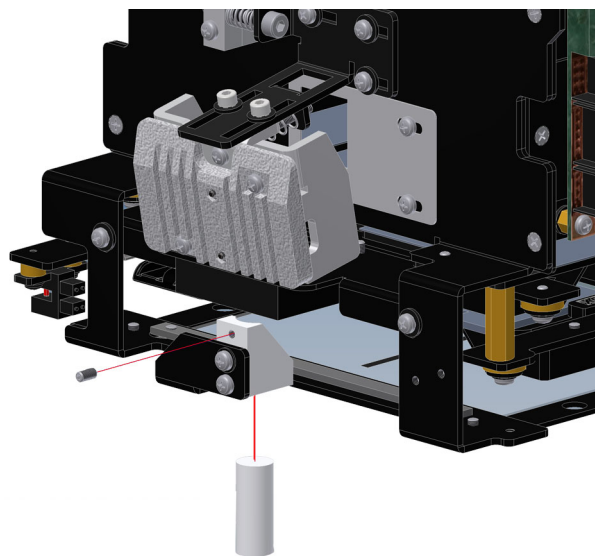


Fig. Laser Substitution

WARNING



DO NOT APPLY EXCESSIVE FORCE TO THE SCREW. THE LASER SHELL IS IN PLASTIC AND EXCESSIVE PRESSURE COULD CRACK THE PLASTIC AND POSSIBLY SHORT-CIRCUIT THE LASER.

- Check the laser alignment, see Chapter- **ADJUSTMENTS**.
- Remount the cover in reverse order.

SUBSTITUTION OF THE MIRROR

- Remove the collimator cover, see Chapter - **COVER REMOVAL**.
- Disconnect power supply.
- Remove the 4 screws holding the electronic board.
- Identify the cables and their position on the board, see Chapter **INSTALLATION**.
- Disconnect the cables from the electronic board.
- Remove the three screws on the upper partition, see **Fig. 1**.
- Take the mirror group out, see **Fig. 2**.
- Unscrew the 2 screw, see **Figure 3**.
- Substitute the mirror with an identical item and remount in reverse order.

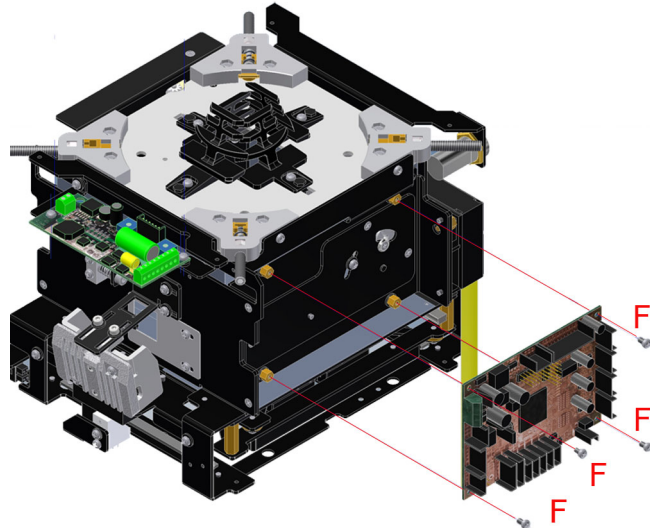


Fig. 1

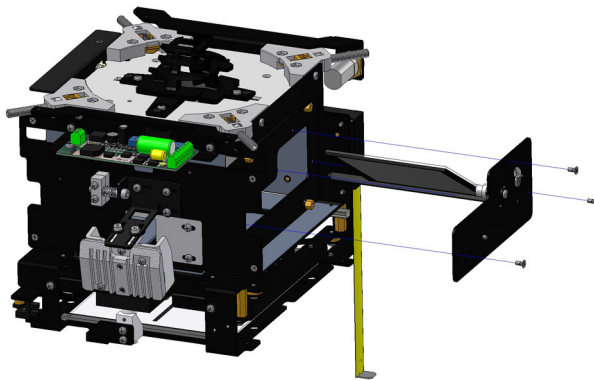


Fig. 2



Fig. 3

PHOTOCELL SUBSTITUTIONS

Cross photocell

- Disconnect supply.
- Remove the cover, see Chapter - **COVER REMOVAL**.
- Close the shutters completely.
- Detach the photocell connector.
- Remove the photocell by unscrewing the 2 screws **K**. See **Fig. Cross Photocell**.
- Substitute the photocell with an identical item.
- Remount the new photocell, do not tighten the screws.
- Connect the photocell connector.
- Power up the collimator.
- Calibrate the photocell by moving it until a return signal of photocell cut-off is received (related LED lights up on board GC 007).
- Tighten the 2 screws **K**.
- Remount in a reverse order.

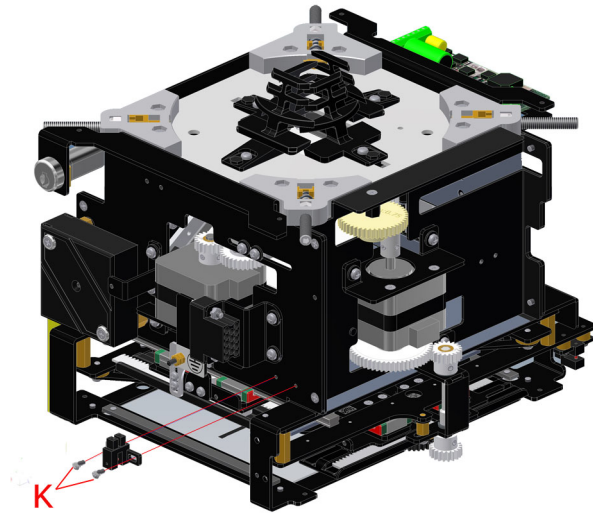


Fig. Cross Photocell

Long photocell

- Disconnect supply.
- Remove the cover, see Chapter - **COVER REMOVAL**.
- Close the shutters completely.
- Detach the photocell connector.
- Remove the photocell by unscrewing the 2 screws **L**. See **Fig. Cross Photocell**.



THE PHOTOCELL IS MOUNTED ON THE 2 BRASS SPACERS. PAY ATTENTION NOT TO LOSE THE SPACERS ONCE THE 2 SCREWS FROM THE PHOTOCELL ARE REMOVED.

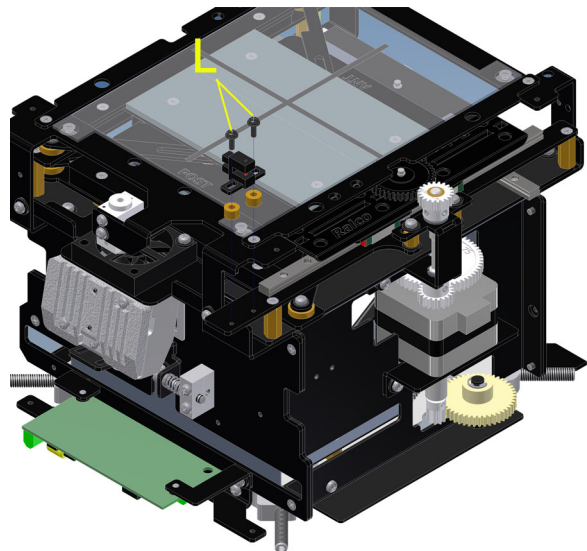


Fig. Long Photocell

- Substitute the photocell with an identical item.
- Remount the new photocell, do not tighten the screws.
- Connect the photocell connector.
- Power up the collimator.
- Calibrate the photocell by moving it until a return signal of photocell cut-off is received. (related LED lights up on board GC 007).
- Tighten the 2 screws **L**.
- Remount in a reverse order.

SPARE PARTS

NOTE



WHEN ORDERING SPARE PARTS, THE CUSTOMER IS REQUESTED TO SPECIFY THE COLLIMATOR MODEL AND SERIAL NUMBER.




THE SPARE PARTS LISTED BELOW REFER TO THE STANDARD MODEL ONLY.



PLEASE CONSULT YOUR PERSONALIZATION PAGE TO VERIFY WHICH SPARE PARTS MAKE UP YOUR CUSTOMIZED MODEL.

LABELS

LABEL 1A

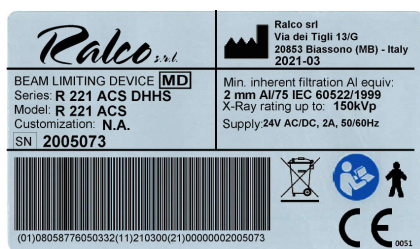

LABEL 1B




LABEL 2



MTR 221 ACS - MTR 221 ACS DHHS - Spare Parts

LABEL 3A

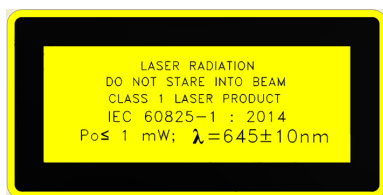

LABEL 4



LABEL 5



LABEL 6



LABEL 7

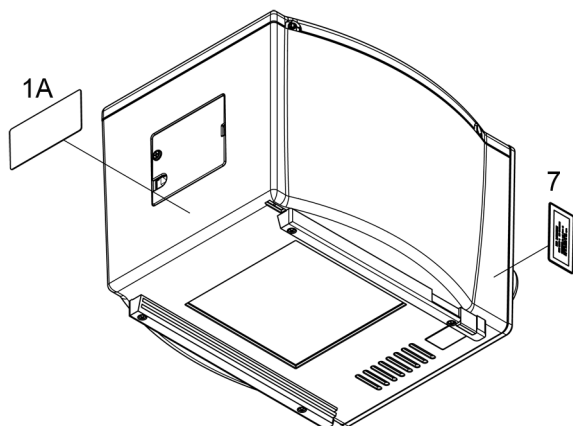
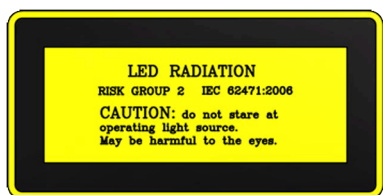


Fig. Standard Label

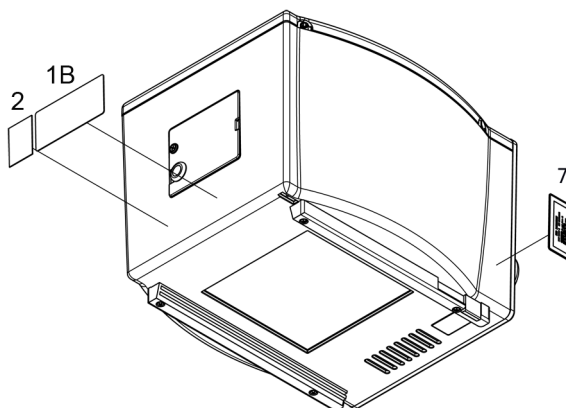


Fig. DHHS Labels


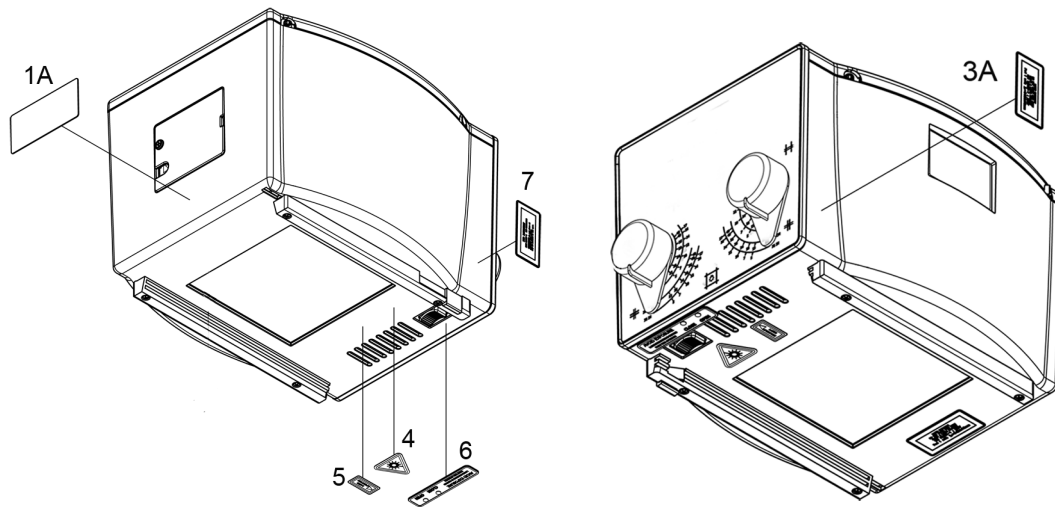


Fig. Standard and DHHS Model Labels, LED and Laser (Red Laser Line, Class 1)

R 221 ACS - R 221 ACS DHHS

PN	DESCRIPTION
RS 596	Screw, TC M3x25mm
RS 754	Right accessory guide
RS 755	Left accessory guide
RS 781	Key switch
RS 907	Fan
RS 1012	Mounting flange
RS 1120	Knob
RS 1164	Antidust panel
RS 1265	GC 24 V board
RS 1625	Retractable tape measure
RS 1988	Upper cover
RS 1992	GC009 board
RS 2181	Rear access panel
RS 2305	Flange spacer
RS 2521	Front panel
RS 2539	LED board
RS 2548	GC007 board
RS 2556	Lateral access panel
RS 2589	Laser
RS 2736	Mirror
RS 2748	LED
RS 2764	Lower cover

Parts Breakdown

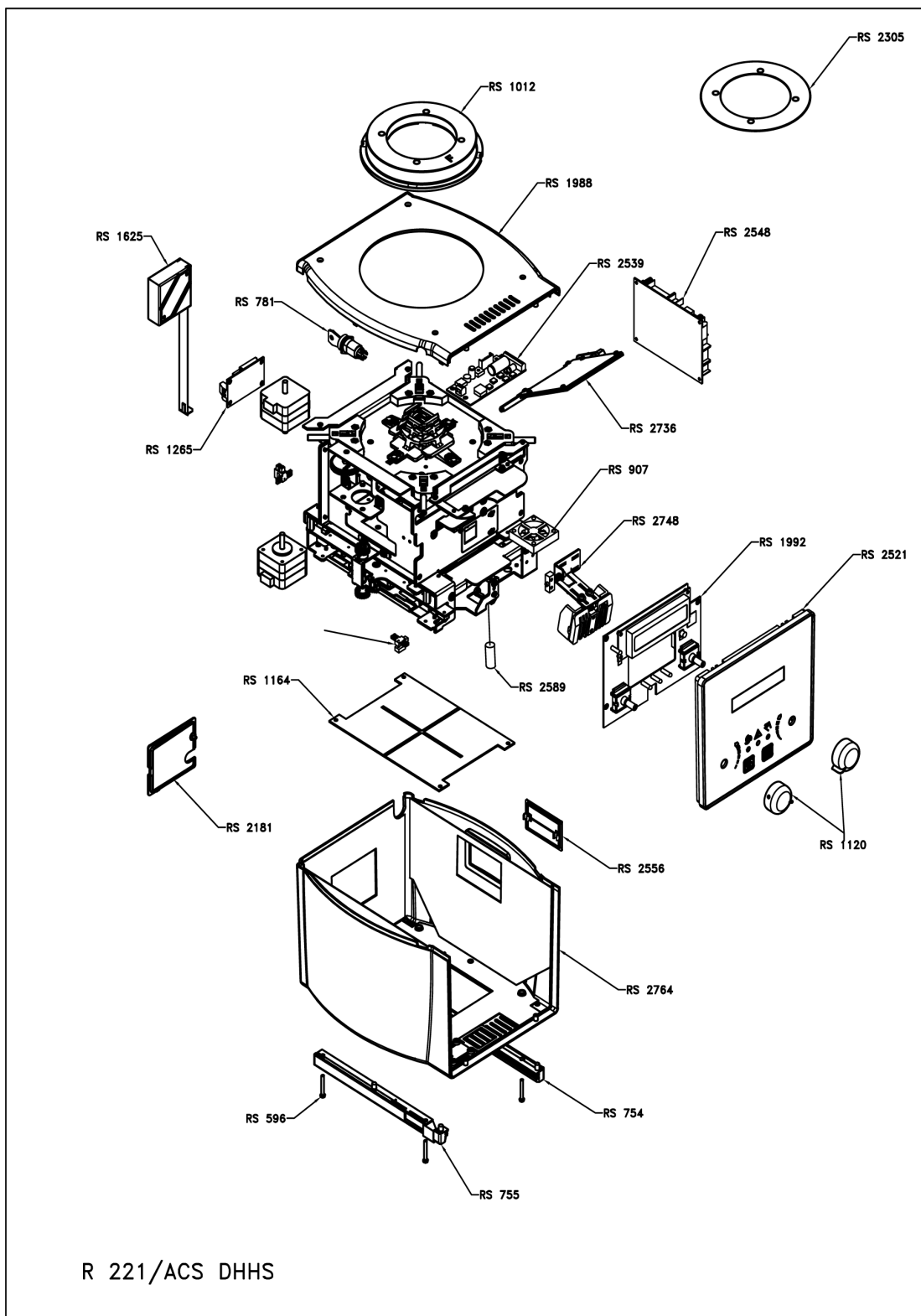


MTR 221 ACS - MTR 221 ACS DHHS - Spare Parts

OPTIONAL ITEMS

This collimator may have the following optional items; a detailed description is provided in this chapter.


RO 002 Iron Mounting Flange Spacer

This accessory is used for the mounting flange; 1.5 mm thickness.

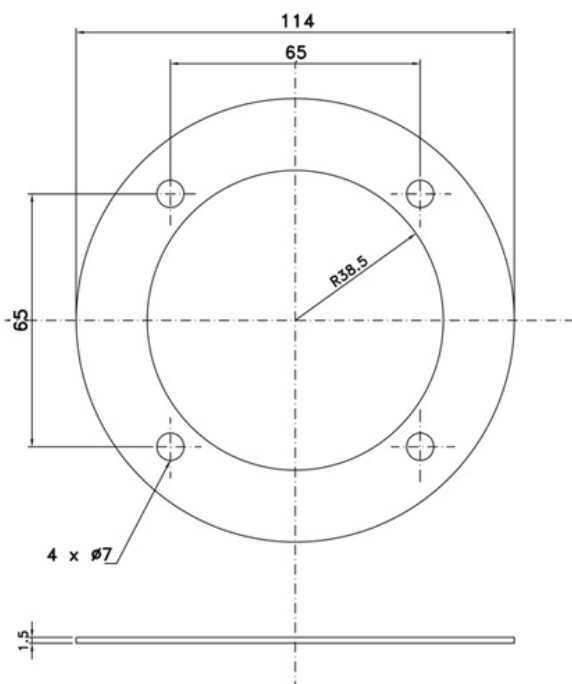

RO 054 Square Iron Mounting Flange Spacer

This accessory is used for the mounting flange; 1.5 mm thickness.

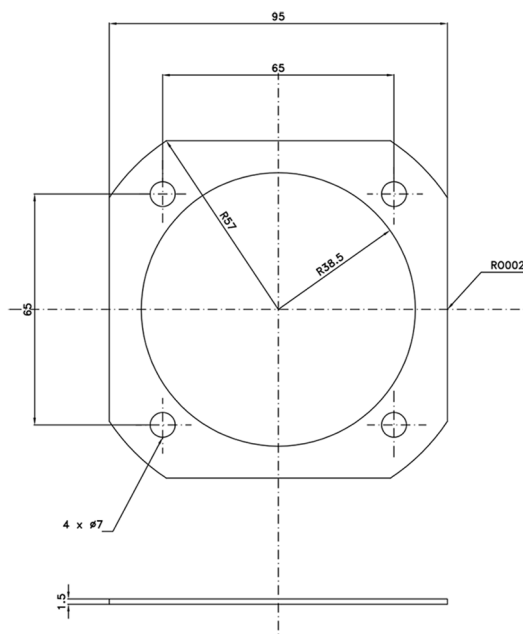

RO 063 Final Quality Test Report Documentation

This test provides higher control to check 100% of the production lot to verify product quality from beginning to end of the production cycle.

Final tests include:

- Light field illumination intensity (if applicable),
- Light field to X-ray field alignment (if applicable),
- X-ray leakage test,
- Edge contrast (if applicable),
- Operation and electronic check up.

RO 074 External housing and guide rails in customized color

The customer can specify a collimator cover color. Ralco's standard color is RAL9003.

RO 082 Glass Mirror

Glass mirror, 0.8 mm thickness, inherent filtration 1 mm Al equivalent. With this mirror, the collimator equivalent filtration value of 2 mm Al is reduced to 1 mm Al.

RO 096 Wiring Customization

Customized electrical wiring. If ordered please refer to the **PERSONALIZATIONS**, annex included with this Instruction Manual, Chapter – Installation, paragraph **Wiring Diagram**.

RO 107 Knob Color Customization

The customer can specify the required knob color.

RO 109 Front panel frame color customization


The customer can specify the required frame color.

RO 111 Front panel customization

The customer can customize the front panel adding, for example, the Company Logo and/or the Company colors.

RO 203 Self-centering top-cover bracket for Siemens X-Ray tube

This bracket is designed for Siemens X-Ray tube only (flange provided by Siemens only).

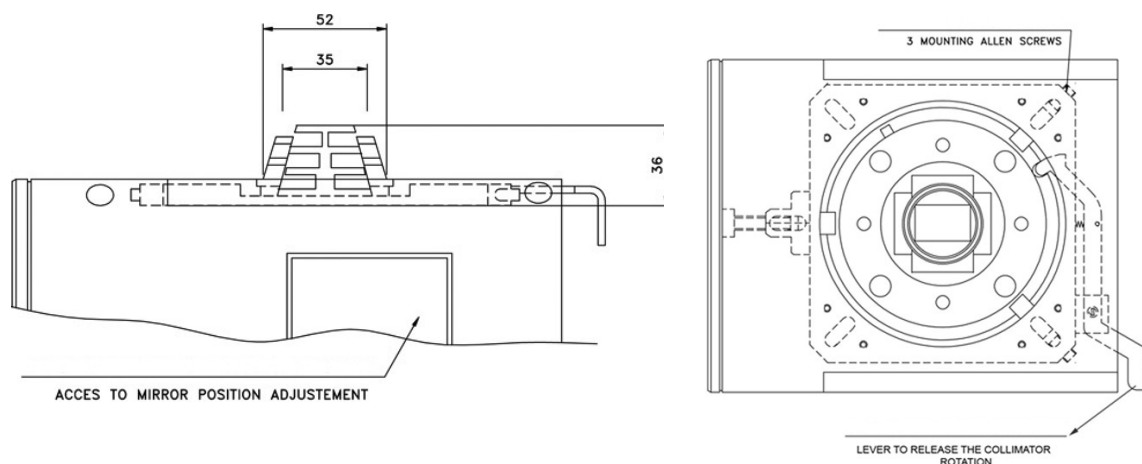

NOTE



RALCO RECOMMENDS FOLLOWING THE BELOW PROCEDURE, DURING THE INSTALLATION OF THE COLLIMATOR EQUIPPED WITH THE SIEMENS TOP COVER TO THE X-RAY TUBE. THE CORRECTLY PERFORMED PROCEDURE ALLOWS THE COLLIMATOR TO ROTATE FREELY.

PROCEDURE:

1. REMOVE THE COLLIMATOR FROM ITS PACKAGING AND PLACE IT ON A FLAT SURFACE,
2. UNSCREW ALL 3 FIXING SCREWS OF THE COLLIMATOR, SEE **FIG. 1**,
3. INSTALL THE COLLIMATOR ONTO THE FLANGE PREVIOUSLY MOUNTED TO THE X-RAY TUBE.
4. TIGHTEN ALL 3 SCREWS COMPLETELY, SO THAT THE TABS WILL TOUCH THE FLANGE SLIGHTLY,
5. ONCE THE SCREWS ARE COMPLETELY TIGHTENED LOOSEN ONE BY ONE ABOUT 1/2 TURN (180°) TO ALLOW THE COLLIMATOR TO ROTATE FREELY. SEE **FIG. 2**.

Fig. 1


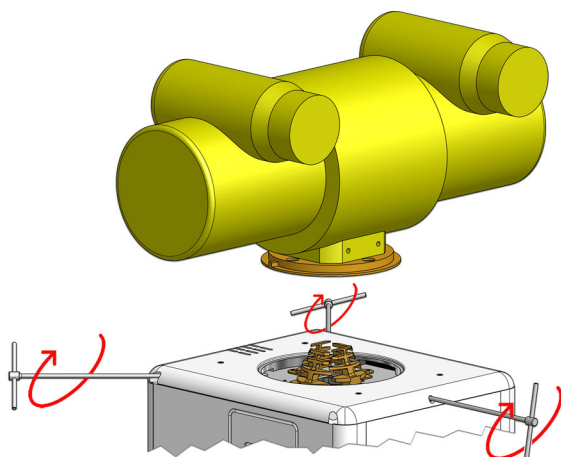


Fig. 2

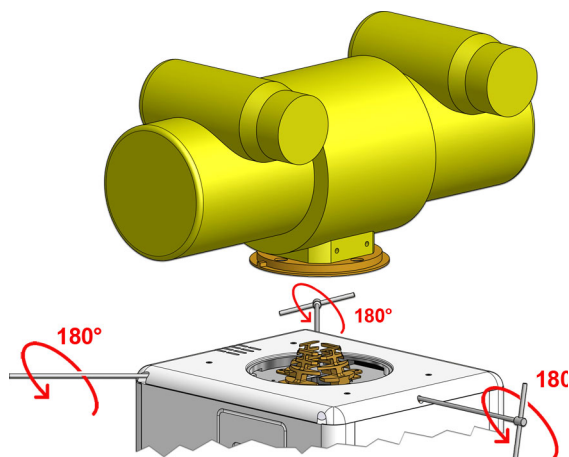

RO 210 Aluminium Accessory Guides


Aluminium accessory guides in RAL 9003 used to input filters or ionization chamber.

RO 240 Focal-spot to skin spacer

This accessory ensures a minimum safety distance (309.5 mm) between the X-ray focus and the patient.



THIS ACCESSORY DEALS WITH PREVENTING THE USE OF INAPPROPRIATE SMALL FOCUS/SKIN DISTANCES IN ORDER THAT THE DOSE EQUIVALENT TO THE PATIENT BE KEPT AS LOW AS REASONABLY POSSIBLE.

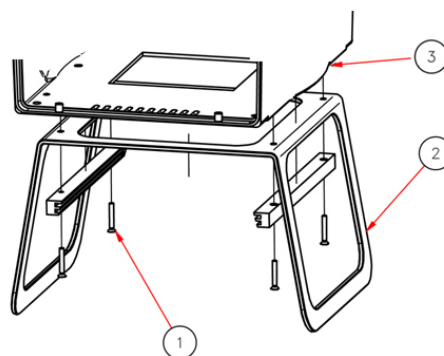


1 - Countersunk Screws, 2 - Focus-Skin Distance Spacers, 3 - Collimator Cover.

RO 242/1 Single laser line to align collimator and detector center: Class 2

The collimator laser is classified as Class 2 (1 m W - wavelength = 645 nm, +/- 10 nm); used for collimator/image receptor center alignment, see **Fig. Laser Line**.

Fig. Laser Line

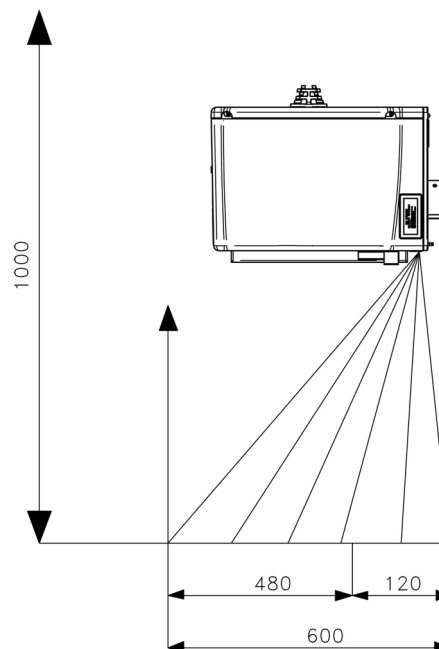

WARNING



CAUTION: CLASS 2 LASER SYSTEM DO NOT STARE INTO THE BEAM

WARNING



DO NOT APPLY EXCESSIVE FORCE TO THE SCREW. THE LASER SHELL IS IN PLASTIC AND EXCESSIVE PRESSURE COULD CRACK THE PLASTIC AND POSSIBLY SHORT-CIRCUIT THE LASER.

Adjustment

- Remove part of the cover to access the point of adjustment. See Chapter **COVER REMOVAL**.
- The line is to fall on a perpendicular cross-line on the plastic anti dust panel near the collimator controls. See **Fig. Laser Alignment**.
- Adjust the position of the line by rotating or moving the base of the laser system.
- To rotate the laser system, loosen the Allen screw **A**. See **Fig. Laser Adjustment**.
- Tighten the Allen screw when the laser beam falls on or is parallel to the bisector line drawn on the anti-dust panel.

Fig. Laser Adjustment


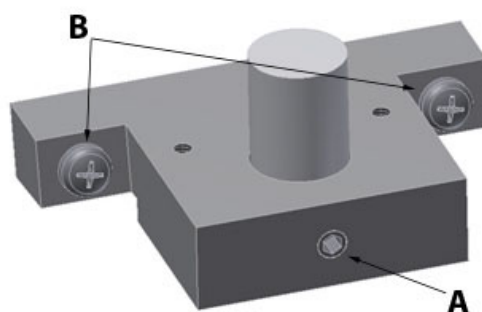


Fig. Laser Alignment

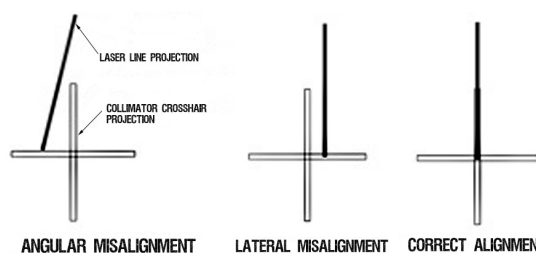


Substitution

- Disconnect supply.
- Remove the cover, see Chapter- **COVER REMOVAL**.
- Unscrew the fixing Allen screws **A**, see **Fig. Laser Adjustment**.
- Disconnect the timer cables from the terminal board - white 0 V, red 5 V.
- Remove the laser and substitute with an identical item.


- Tighten the screws.
- Check the laser alignment, see Chapter- **ADJUSTMENTS**.
- Remount the cover, see Chapter- **COVER REMOVAL**.

Classification EN 60825-1 par. 1 -5: Class 2 laser product => Red laser line.

LABEL 3A

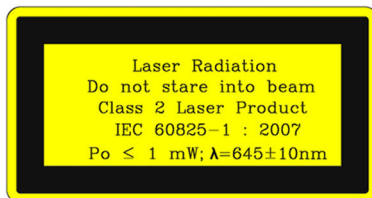

LABEL 3B

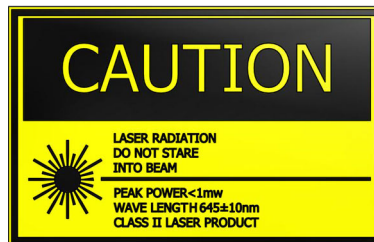

LABEL 4



LABEL 6



LABEL 5

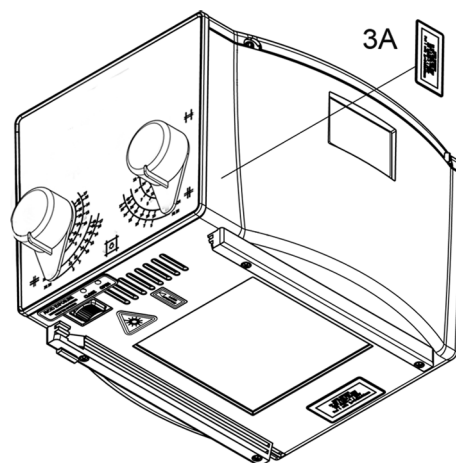
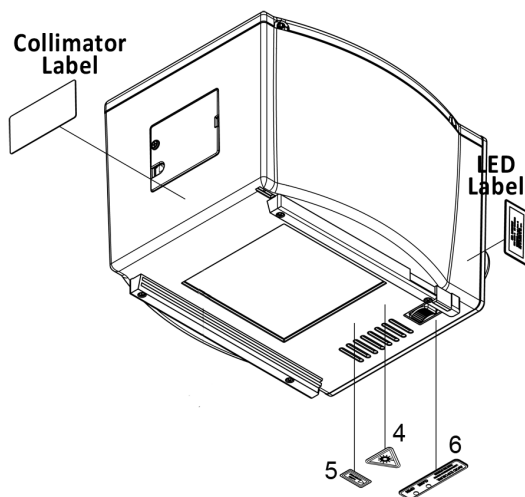


Fig. Standard Model Labels, LED and Laser (Red Laser Line - optional RO 242/1)

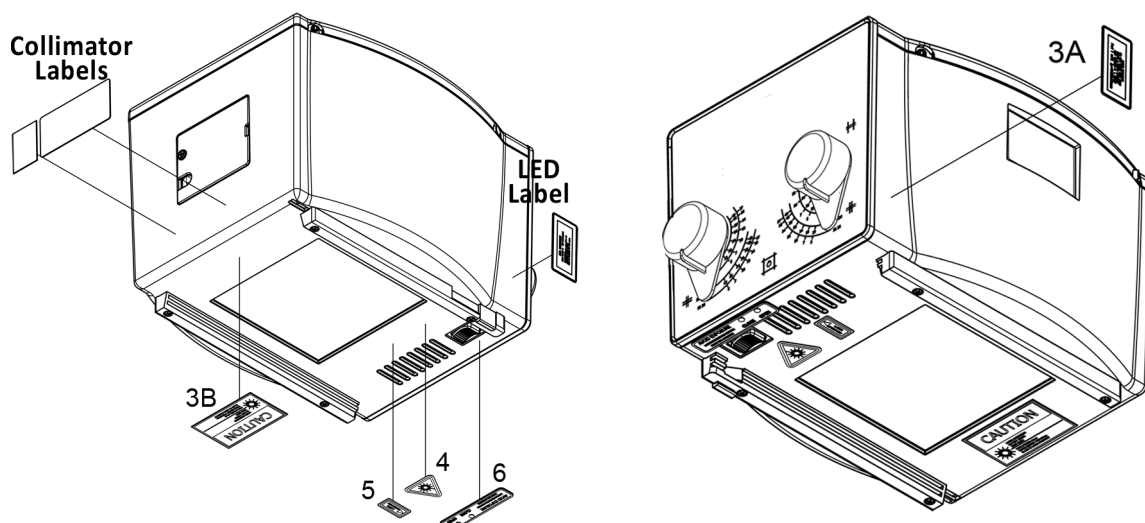


Fig. DHHS Model Labels, LED and Laser (Red Laser Line - optional RO 242/1)

RO 242/2 Two lasers (one mounted externally) forming a crosshair to center the patient to the collimator: Class 2


Second laser which serves to center the patient using the cross projection.

The collimator lasers are classified as Class 2 (1 m W - wavelength = 645 nm, +/- 10 nm) and are used for collimator/image receptor center alignment.

WARNING



CAUTION: CLASS 2 LASER SYSTEM DO NOT STARE INTO THE BEAM

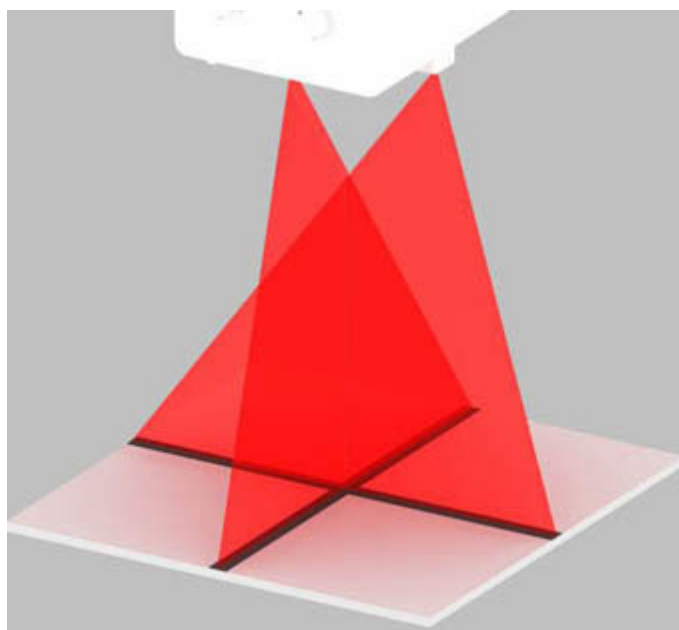

CALIBRATION CONTROL

- Draw a cross (two lines at 90°) on a sheet of paper and use this as reference.
- Switch the light ON and set the sheet of paper at 1 m from the focus.
- Make sure the cross on the paper coincides exactly with the two lines silk-screened on the plastic window.
- Examine the laser projections on the paper: the laser is correctly calibrated when the projections coincide with the lines drawn on the paper, otherwise an adjustment will be necessary.

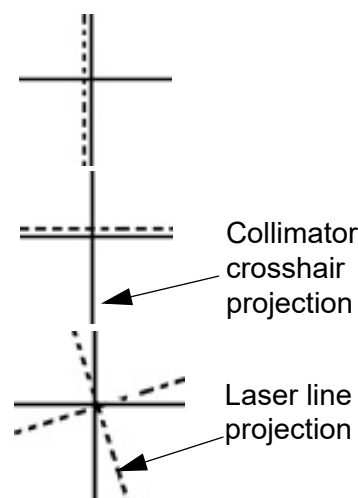


Fig. Laser projections

ADJUSTMENT

Horizontal

- Disconnect power supply.
- Remove the laser cover located on the rail by unscrewing the two screws **A**, see **Fig. Horizontal Laser**.
- To correct horizontal misalignment, loosen the screws **B**, see **Fig. Horizontal Laser** and move the laser support until the projected laser line coincides with the cross on the sheet of paper. Tighten screws **B**.

Vertical

- Disconnect power supply.
- Remove the knobs and front panel, see Chapter- **COVER REMOVAL**.
- To correct vertical misalignment, loosen the Allen screw **A**, see **Fig. Vertical Laser** on the laser support; rotate the laser and align the laser line over the cross on the sheet of paper. Tighten **B** screw s.

WARNING



DO NOT APPLY EXCESSIVE FORCE TO THE SCREW. THE LASER SHELL IS IN PLASTIC AND EXCESSIVE PRESSURE COULD CRACK THE PLASTIC AND POSSIBLY SHORT-CIRCUIT THE LASER.

SUBSTITUTION

Horizontal

- Disconnect supply.
- Remove the laser cover located on the rail by unscrewing the two screws **A**, see **Fig. Horizontal Laser**.
- Disconnect the timer cables from the terminal board - white 0 V, red 5 V.
- Remove the laser and substitute with an identical item.
- Tighten the screws.
- Check the laser alignment, see paragraph **ADJUSTMENT**.
- Remount the laser cover.

Vertical

- Disconnect supply.
- Remove the knobs and front panel, see Chapter- **COVER REMOVAL**.
- Loosen the Allen screw **A**, see **Fig. Vertical Laser** on the laser support.
- Disconnect the timer cables from the terminal board - white 0 V, red 5 V.
- Remove the laser and substitute with an identical item.

- Tighten the Allen screw.
- Check the laser alignment, see paragraph **ADJUSTMENT**.
- Remount the laser cover.

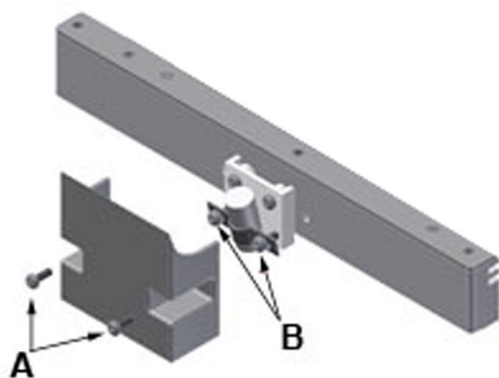


Fig. Horizontal Laser


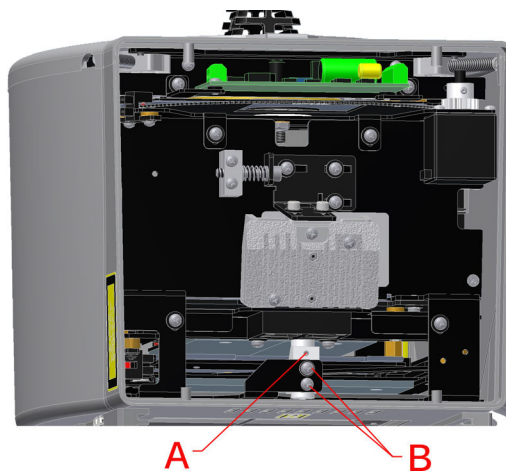



Fig. Vertical laser

Classification EN 60825-1 par. 1 -5: Class 2 laser product => Red laser line.

LABEL 3A

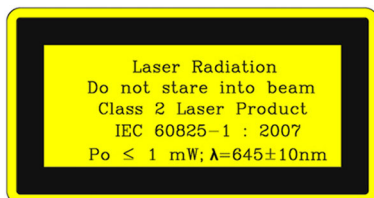

LABEL 3B

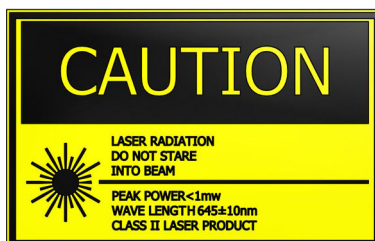

LABEL 4



LABEL 6



LABEL 5



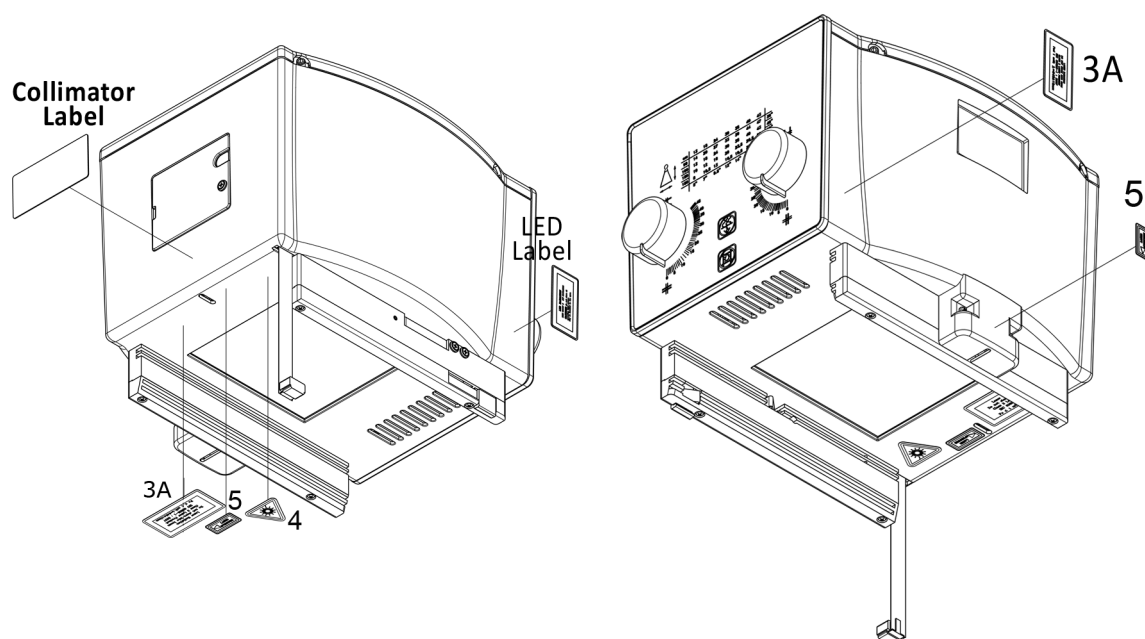


Fig. Standard Model Labels, LED and Laser (Red Laser Line - optional RO 242/2)


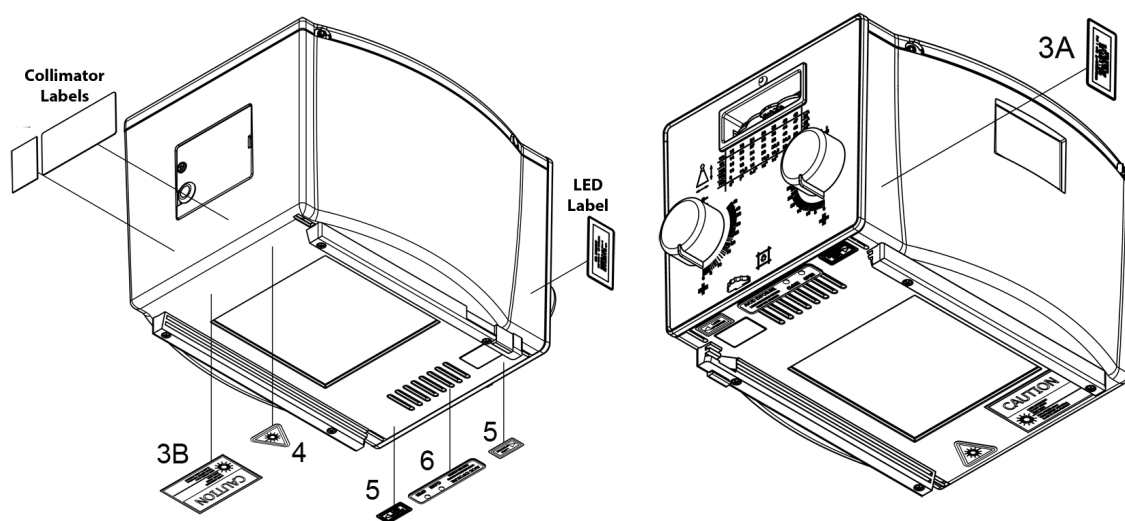


Fig. DHHS Model Labels, LED and Laser (Two Red Laser Lines - optional RO 242/2)

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items


RO 253 Accessory guide rail spacers in standard housing color

A pair of plastic rail guides complete (RAL 9003) with support (unless RO 074 is selected) designed to accommodate the ionization chamber and ensure its lateral protection.

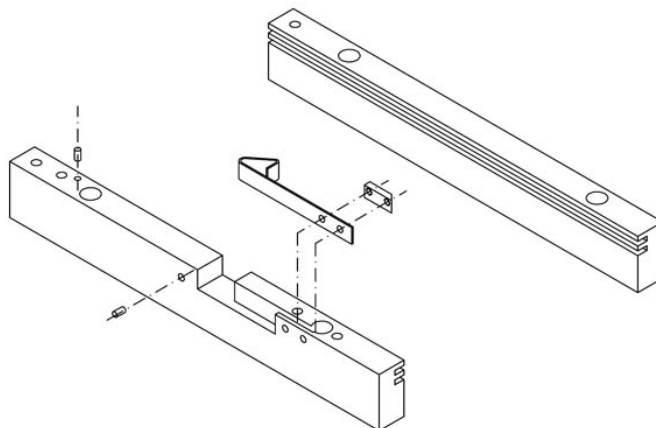

RO 278 Fixed Near Focal Shutters

Upper cover with extra-focal shutters (single layer collimation) for use with a mounting plane of 80mm from the X-Ray focus.

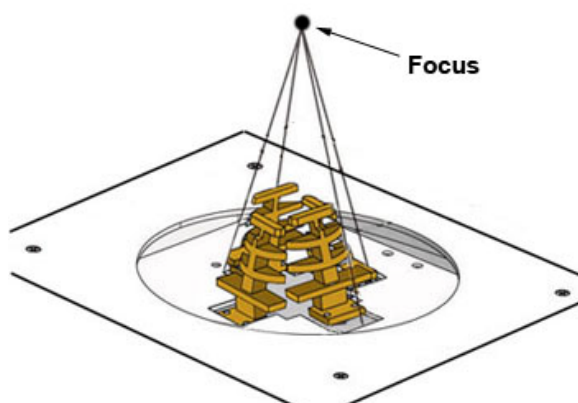

RO 305 Additional Variable Filtration - Automatic Selection

Additional variable filtration may be added to the minimum collimator filtration in the form of an aluminium disk, 1 mm thickness, with automatic control via the electronic PCB.

The disk features a hole allowing the passage of X-rays and includes three filters with the following filtration values:

- 0: no filtration
- 0,1 mm Cu + 1 mm Al. (Al eq. 3.5 mm)
- 0,2 mm Cu + 1 mm Al (Al eq. 6.0 mm)
- 1 mm Al + 1 mm Al support (Al eq. 2.0 mm).

Adjustments

PHOTOCELL ADJUSTMENT

- Remove the front panel - See Chapter- **COVER REMOVAL**.
- Disconnect the photocell connector from the GC 007 PCB.
- Unscrew the 2 screws **C** holding the filter system on the front plate, see **Fig. 1**
- Remove the support together with the filter from the collimator.
- Adjust the photocell by loosening the 2 screws **B** located on the support, see **Fig. 2**
- Move the photocell until a return signal of photocell cut-off is received (related LED lights up on PC board).
- When adjusted is terminated, tighten the **B** screws, see **Fig. 2**
- Remount the support and the front plate.

Substitutions

SUBSTITUTION OF THE FILTER MOTOR

- Disconnect supply.
- Remove the front panel, see Chapter **COVER REMOVAL**.
- Disconnect the motor connector from the electronic board.
- Unscrew the 2 screws **C** holding the filter system on the front plate, see **Fig. 1**.
- Take the support together with the filter out from the collimator.
- Remove the motor from its support by unscrewing the 2 screws **A**, see **Fig. 2**
- Substitute the motor with an identical item.
- Remount the support with the new motor.
- Connect the connector removed previously.
- Remount the front panel.


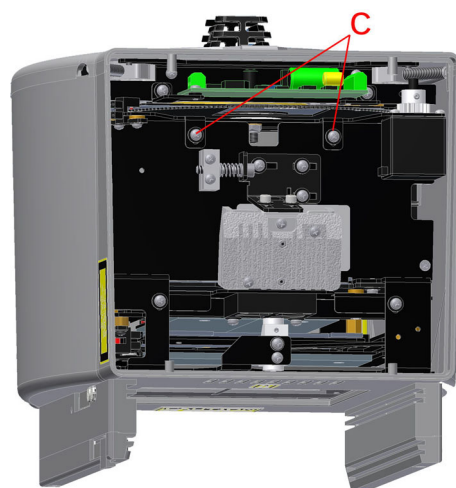


Fig. 1


Fig. 2

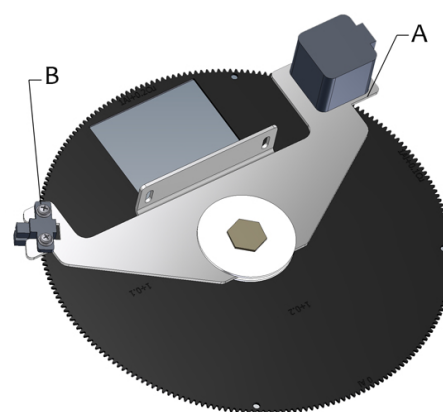

SUBSTITUTION OF FILTER MOVEMENT PHOTOCELL

- Disconnect supply.
- Remove the front panel, see Chapter **COVER**
- **REMOVAL..**
- Disconnect the photocell connector from the electronic board.
- Unscrew the 2 screws **C** holding the filter system on the front plate, see **fig. 1**
- Take the support together with the filter out from the collimator.
- Remove the photocell by unscrewing the 2 screws **B**, see **Fig. 3**
- Substitute the photocell with an identical item.
- Remount the support with the photocell.
- Connect the four photocell removed previously.
- Remount the cover and front panel.

Fig. 3

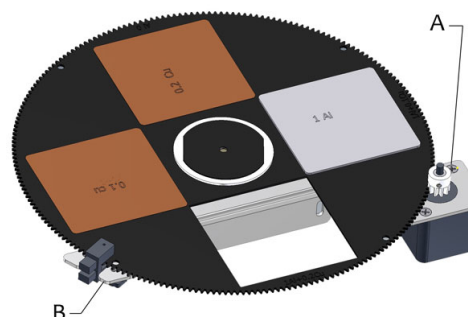

RO 305/1 Additional Variable Filtration

Additional variable filtration may be added to the minimum collimator filtration in the form of an aluminium disk, 1 mm thickness, with automatic control via the electronic PCB.

The disk features a hole allowing the passage of X-rays and includes three filters with the following filtration values:

- 0: no filtration.
- 0,1 mm Cu (Al eq. 2.5 mm)
- 0,2 mm Cu (Al eq. 5.0 mm)
- 0,3 mm Cu (Al eq. 7.5 mm)

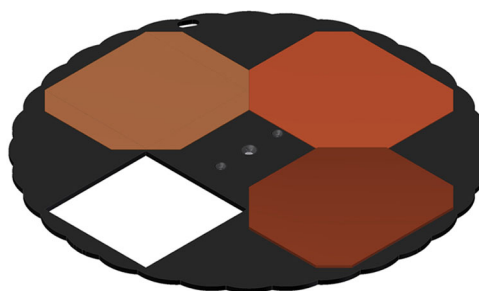


Fig. RO 258/1

To adjust and/or to change the filter see **RO 305 Additional Variable Filtration** description in this Chapter.

RO 308 External Board ASR003

ASR003 Board

Characteristics

- Analogical/ Digital interface.
- Vertical SID interface with potentiometers for table and stand, Potter Bucky, (vertical, right, left) inputs.
- Interface for 5 fixed lateral distances.
- Interface for general controls; e.g., open/close for shutters, tomography.
- CanBus interface.


Description

- Board ASR003 is an extension of the radiological system, providing an interface for the collimator to other system devices with analogical signals.
- It operates via CAN BUS with a Ralco proprietary protocol.
- The user end features 32 digital inputs, 8 analogue inputs and 10 relay outputs.

- Digital inputs are optoisolated to allow connection to devices that supply output control voltages (typical range 12-32 V DC).
- Supply: 24V DC.
- Fuse: n.a. (not supplied by Ralco).

Digital and Analogue Inputs

Digital Input

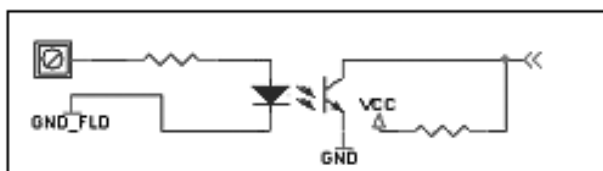


Since the input signal sometimes comes from a voltage-free contact, the board features a number of pins to which an external supply may be connected. As an alternative, these pins may be connect to a voltage incoming from board ASR003 itself by simply connecting pin M79 to pin M80 and, pin M81 to pin M82.

The available voltage is not taken upstream of the voltage regulator but downstream of the fuse and rectifier bridge.

The type of software applied determines the function of each input.

Analogue Input

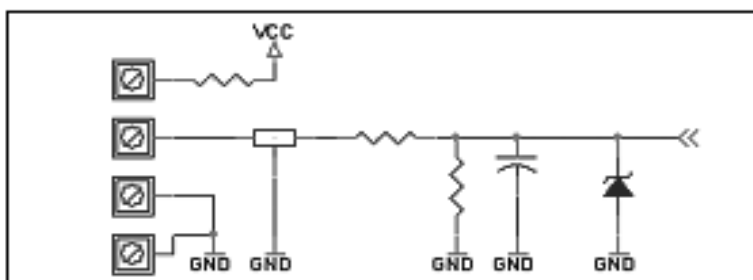

In this case the input range will be reduced with respect to the digitalization capacity of the microprocessor digital/analogue convertor. However, the board will have better protection against wrong Potentiometer connections.

The analogue inputs are designed primarily for connection to 1 Kohm potentiometers.

Additional safeguards have been added because the user could supply an analogue signal directly without using potentiometers.

The main purpose of these inputs is to furnish the collimator with the cassette size data and information related to focus-film/focus-floor distances (depending on the configuration).

In general, non linearity introduced by the safeguards is contained. However, should a greater degree of precision be required (such as in the case of focus-film/focus-floor signals) we suggest that a voltage between 0 and 6 V DC be supplied so as to make the most of the convertor range.

Higher voltages up to 12 V, which could be caused by incorrect connections, will not damage the input.

The 10 output relays each have an NO/NC contact and may consequently be adapted to a number of different configurations. Their use depends on the software applied.

This board also supplies the go-ahead to program some of the system parameters.

Mechanical Installation of the External Interface Unit


The external box is to be installed in such a way as to ensure connection of all the electric devices present in the radiological system.

We suggest that the box be installed at a safe distance from electromagnetic sources such as high voltage transformers.



Use the holes on the sides of the box to perform the installation.

Should it be necessary to install the board within a rack of the general system, remove the electronic board from the box prior to proceeding with the installation of the box itself.


Make sure that each portion of the rack that is to accommodate the box is adequately earthed.

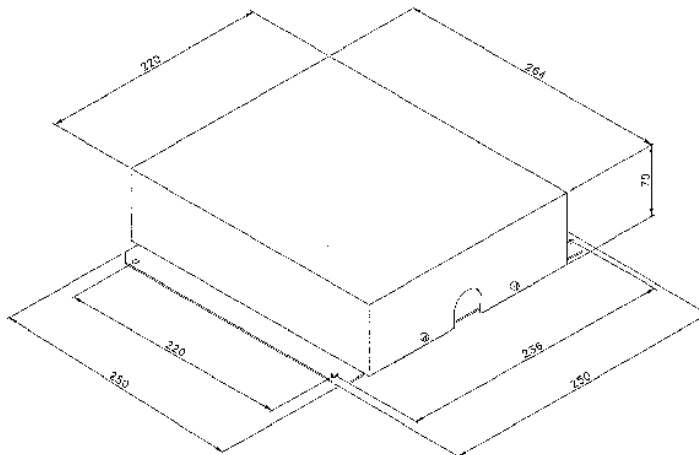

External Unit Labels

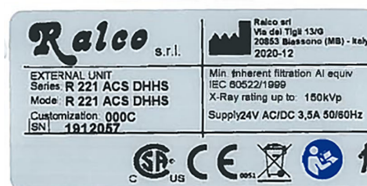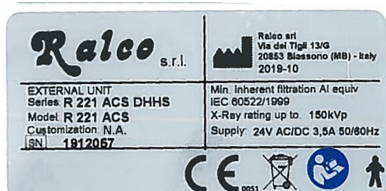

Electrical Connection

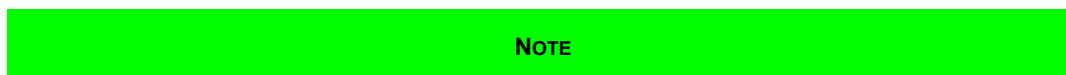


POWER MUST BE SUPPLIED BY A SAFETY TRANSFORMER COMPLIANT WITH CURRENT NORMS AND PROTECTED BY AN FUSE 4 A.

- Connect the collimator using the connection cable. The cable end with the AMP 201298-1 connector is to be inserted into the connector at the back of the collimator.

- The other end of the cable, with the DB9 connector and 3 wires, must be connected to the CanBus interface and power supply, see **Fig. 1**.
- Consult the electric drawing to perform a correct connection of inputs.

Fig. 1

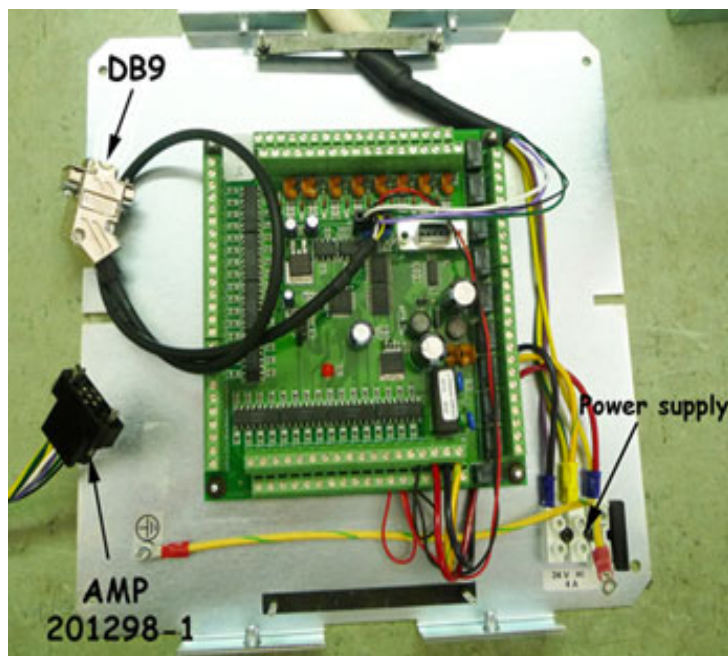

WARNINGS



COLLIMATOR SUPPLY IS NOT PROTECTED BY A FUSE. PRIOR TO CONNECTING THE COLLIMATOR, CHECK THAT SUPPLY IS PROTECTED BY AN EXTERNAL FUSE 4A, NOT SUPPLIED BY RALCO, SEE CHAPTER- **SPECIFICATIONS**.



SUPPLY CABLES (24V DC) USED FOR COLLIMATOR CONNECTION MUST BE SUITABLE FOR THE PURPOSE AND COLLIMATOR CURRENT ABSORPTION.(MIN. SECTION 1.5 MM²).



THE SYSTEM REQUIRES SCREENED CABLES WITH THE ADDITION OF FERRITE CORES (NOT SUPPLIED) TO AVOID ELECTROMAGNETIC INTERFERENCE WITH THE SERVER ELECTRONICS.

CABLES FROM THE SYSTEM TO THE EXTERNAL BOARD MUST BE SCREENED. THE FERRITE CORES ARE TO BE MOUNTED ON THE CABLES INSIDE THE BOX AS SHOWN IN THE DRAWING BELOW. FERRITE CORES ARE USED TO ATTENUATE INDUCTION AND IRRADIATION OF ELECTROMAGNETIC INTERFERENCE. WE SUGGEST A 279 FERRITE, ATTENUATION RANGE 100 MHZ TO 200 MHZ, SUITED TO YOUR CABLES; IE: MSFC-10 FERRITE BY RICHCO.

NOTE



THE WIRING DIAGRAM INCLUDED IN THIS DOCUMENT REFERS TO THE STANDARD PRODUCT. IT IS THE RESPONSIBILITY OF THE CUSTOMER WHO HAS REQUESTED AN ELECTRIC CUSTOMIZATION, TO ENSURE THAT AN ELECTRIC DIAGRAM RELATING TO THE CUSTOMIZATION HAS BEEN PROVIDED WITH THE DOCUMENTATION.



THE DEVICE IS PROJECTED TO OPERATE WITH A PERMANENT POWER SUPPLY PRESENT SO THE PROCEDURE OF SWITCHING OFF THE COLLIMATOR IS NOT FORESEEN. NO RISK OR DEVICE DAMAGE WILL OCCUR IF THE MACHINE IS ACCIDENTLY SWITCHED OFF.

WARNING



THE DEVICE MUST BE EXCLUSIVELY CONNECTED TO POWER NETWORK, WITH EARTH PROTECTION, IN ORDER TO AVOID A RISK OF ELECTRICAL SHOCK.



SUPPLY AND SIGNALS TO THE COLLIMATOR MUST BE TO 2007/47/CE STANDARDS. DEVICES THAT SUPPLY THE COLLIMATOR MUST THEREFORE FEATURE DOUBLE OR REINFORCED INSULATION AS PROVIDED BY THE GENERAL STANDARD ON ELECTROMEDICAL EQUIPMENT CEI62-5 +A2 (EN 60601-1). THE DEVICE IS ELECTROSTATIC SENSITIVE, CONSEQUENTLY ALL THE RELATING SAFETY STANDARDS MUST BE COMPLIED WITH.



COLLIMATOR MUST BE SUPPLIED AS SPECIFIED, SEE CHAPTER **SPECIFICATIONS**. THE SUPPLY MUST COME FROM A SEPARATE SOURCE FROM THE POWER NETWORK THROUGH DOUBLE INSULATION OR REINFORCED INSULATION AND WITH LIMITED CURRENT. TRANSFORMER CHARACTERISTICS MUST CONFORM TO THE REQUIREMENTS OF STANDARD IEC 60601-1.



CABLES AND TERMINALS USED FOR THE INTERNAL CONNECTION OF THE COLLIMATOR MUST BE SUITABLE FOR OPERATION AT TEMPERATURES OF 80°C AND COLLIMATOR CURRENT ABSORPTION.



TO ENSURE THE SAFE USE OF THE COLLIMATOR, IT IS MANDATORY TO SECURELY AND FIRMLY INSERT THE POWER SUPPLY/DATA CABLE BETWEEN THE COLLIMATOR AND RADIOLOGICAL SYSTEM. IT IS ALSO MANDATORY THAT THE POWER SUPPLY/DATA CABLE IS CORRECTLY SECURED TO THE STRAIN RELIEF. IF THE STRAIN RELIEF IS MISSING IT IS MANDATORY ONE BE CORRECTLY AND SECURELY INSTALLED. RALCO IS NOT LIABLE FOR DAMAGES IN THE EVENT OF MISSING OR INCORRECT CABLE INSTALLATION.

Connection of System to ASR003 Board

All the system devices listed in the following tables (variable SID table, variable SID stand,

fixed lateral SID, vertical Bucky, lateral Bucky, etc.) are to be connected to the connectors of the external board. **Correct operation of the system is guaranteed if the return signals from the Bucky (if mounted) and SID Potentiometers are clean and devoid of electrical disturbances. Use tri-polar screened cables with the addition of ferrite cores near the connections to the PC board (external unit).**

The SID variation control signal must be analogue DC, 5 V DC max.

WARNINGS



TAKE CARE THAT ELECTRICAL RESIDUES HOWEVER SMALL (BITS OF COPPER LEADS FROM WIRES, ETC.) ARE NOT LEFT TO CLUTTER THE CIRCUIT - THESE COULD CAUSE BOARD MALFUNCTION OR ANOMALIES.

Earthing of the collimator circuit is used to disperse noise or extra voltage coming from the collimator supply system.


The device that disperses noise is reversible, consequently noise instead of being eliminated is fed into the system power supply circuit. Therefore, it is essential that the earth outlet to which the system is connect be effectively the unipotential protection point of the system or the room.

POWER SUPPLY




IN-AN	Supply 24V DC	M83
IN-AN	Supply 24V DC	M84
IN/OUT	Mains Ground Contact	M85

IN =	Digital Input Signal
IN-AN =	Analogical Input Signal
OUT=	Output Signal

Fig. ASR003 Board

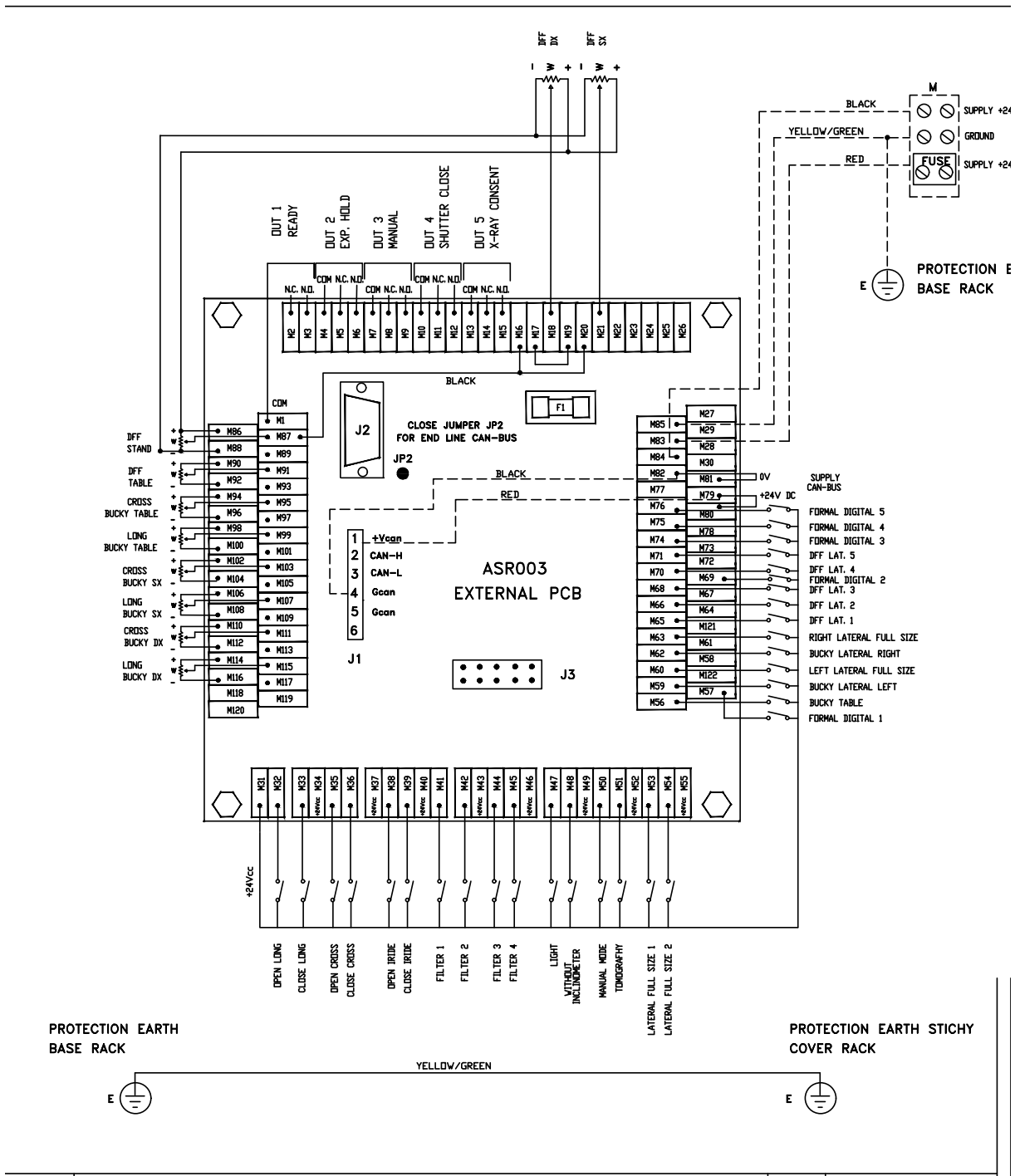

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

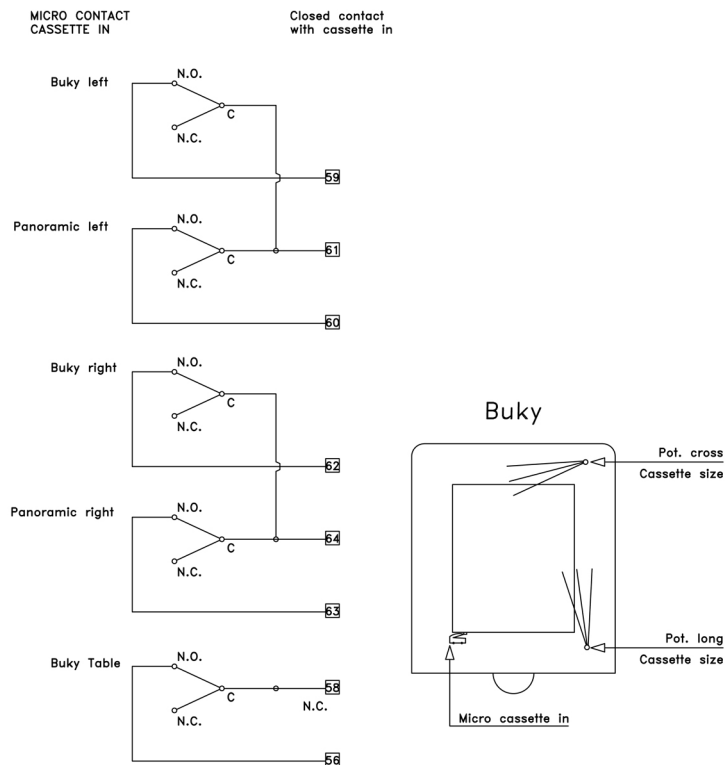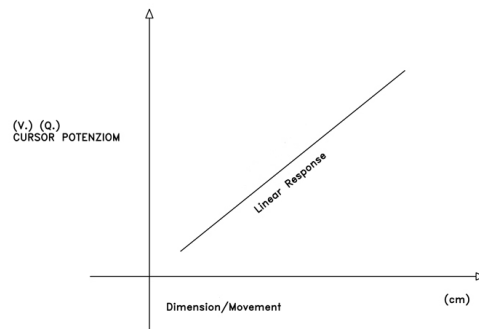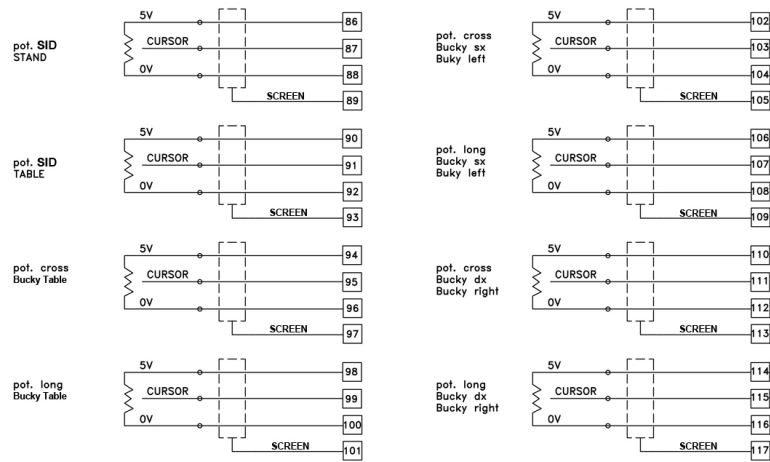

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items



OUTPUTS

All outputs are simple relay contacts (1 A 24 V DC) configured as normally open. Output relays can switch 1 A to 110 V AC or 30 V DC; do not exceed these values and provide fuses on load supply circuits.

If relays are used to switch inductive loads (coils of other relays), provide these loads with appropriate snubber R-C circuits.

Should an electric signal be produced by a multi-contact relay, ground the contacts surrounding the signal or set them at minimum potential (max. 24 V) by using a snubber circuit for the relating load.

Signal	Description	Contacts	
OUT1	With closed contact the system is in "Ready " mode	COM	M1
		N.C.	M2
		N.O.	M3
OUT2	With closed contact the system is in "Exp.Hold" mode	COM	M4
		N.C.	M5
		N.O.	M6
OUT3	With closed contact system is in "Manual"	COM	M7
		N.C.	M8
		N.O.	M9
OUT4	Closed contact: "Closed Shutters"	COM	M10
		N.C.	M11
		N.O.	M12
OUT5	With closed contact the system allows the X-ray "Exposure"	COM	M13
		N.C.	M14
		N.O.	M15

Signal	Contact	Description
IN	M31	Common +V DC
IN	M32	Long shutters open
IN	M33	Long shutters closed
IN	M34	Common +V DC
IN	M35	Cross shutters open
IN	M36	Cross shutters closed
IN	M37	Common +V DC
IN	M38	Iris shutters open
IN	M39	Iris shutters closed
IN	M40	Common +V DC
IN	M41	Filter selection 1
IN	M42	Filter selection 2

Signal	Contact	Description
IN	M43	Common +V DC
IN	M44	Filter selection 3
IN	M45	Filter selection 4
IN	M46	Common +V DC
IN	M47	Collimator Light ON
IN	M48	Exclusion of collimator tilt device
IN	M49	Common +V DC
IN	M50	Manual Mode selection
IN	M51	Tomography Mode selection
IN	M52	Common contacts selection (+V DC)
IN	M53	Contact for the fixed lateral full size SID - 1
IN	M54	Contact for the fixed lateral full size SID - 2
IN	M55	Common contacts selection (+V DC)
IN	M56	Cassette IN contact, table Bucky
IN	M57	Fluoro 1
IN	M58	Common contact table Bucky (+V DC)
IN	M59	Cassette IN contact, left lateral Bucky
IN	M60	Cassette IN contact, full size left lateral
IN	M61	Common contact and full size left lateral (24 V DC)
IN	M62	Cassette IN contact, right lateral Bucky
IN	M63	Cassette IN contact, full size right lateral
IN	M64	Common contacts Bucky and full size right lateral (+V DC)
IN	M65	Contact for the fixed lateral SID -1
IN	M66	Contact for the fixed lateral SID - 2
IN	M67	Common contacts Bucky and full size right lateral (+V DC)
IN	M68	Contact for the fixed lateral SID - 3
IN	M69	Fluoro 2
IN	M70	Contact for the fixed lateral SID - 4
IN	M71	Contact for the fixed lateral SID - 5
IN	M72	Common contacts of fixed lateral SIDs (+V DC)
IN	M73	Common contacts of fixed lateral SIDs (+V DC)
IN	M74	Fluoro 3
IN	M75	Fluoro 4
IN	M76	Fluoro 5
IN	M78	Common contacts of fixed lateral SIDs (+V DC)
Signal	Contact	Description
IN-AN	M86	Potentiometer SID/tube-stand terminal (high-level 5 V DC)
IN-AN	M87	Potentiometer SID/ tube-stand slider
IN-AN	M88	Potentiometer SID/ tube-stand terminal (low-level 0 V DC)

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

Signal	Contact	Description
IN-AN	M89	Cable shielding of potentiometer SID/ table
IN-AN	M90	Potentiometer SID/Table terminal (high-level 5 V DC)
IN-AN	M91	Potentiometer SID/Table slider
IN-AN	M92	Potentiometer SID/Table terminal (low-level 0 V DC)
IN-AN	M93	Cable shielding of potentiometer SID/Table
IN-AN	M94	Cross potentiometer/table Bucky terminal (high-level 5 V DC)
IN-AN	M95	Cross potentiometer/table Bucky slider
IN-AN	M96	Cross potentiometer/table Bucky terminal (low-level 0 V DC)
IN-AN	M97	Cable Shielding of Cross potentiometer/table Bucky
IN-AN	M98	Long pot./table Bucky terminal (high-level 5 V DC)
IN-AN	M99	Long potentiometer/table Bucky slider
IN-AN	M100	Long potentiometer/table Bucky terminal (low-level 0 V DC)
IN-AN	M101	Cable Shielding of Long potentiometer/table
IN-AN	M102	Cross potentiometer/left lateral Bucky terminal (high-level 5 V DC)
IN-AN	M103	Cross potentiometer/ left lateral Bucky slider
IN-AN	M104	Cross potentiometer/ left lateral Bucky terminal (low-level 0 V DC)
IN-AN	M105	Cable Shielding of Cross potentiometer/ left lateral Bucky
IN-AN	M106	Long potentiometer/left lateral Bucky terminal (high-level 5 V DC)
IN-AN	M107	Long potentiometer/left lateral Bucky slider
IN-AN	M108	Long potentiometer/left lateral Bucky terminal (low-level 0 V DC)
IN-AN	M109	Cable Shielding of Long potentiometer/ left lateral Bucky
IN-AN	M110	Cross pot./right lateral Bucky terminal (high-level 5 V DC)
IN-AN	M111	Cross potentiometer/right lateral Bucky slider
IN-AN	M112	Cross potentiometer/ right lateral Bucky terminal (low-level 0 V DC)
IN-AN	M113	Cable Shielding of Cross potentiometer/right lateral Bucky
IN-AN	M114	Long pot./right lateral Bucky terminal (high-level 5V DC)
IN-AN	M115	Long potentiometer/ right lateral Bucky slider
IN-AN	M116	Long potentiometer/right lateral Bucky terminal (low-level 0VDC)
IN-AN	M117	Cable Shielding of Long potentiometer/right lateral Bucky


Check the Following Whenever the Potentiometer Operates Irregularly:

- The analogue signal (V DC) from the table Bucky potentiometer sliders must increase with the increase of cassette format. Use a voltmeter to check this condition between contacts M99 (+) and M100 (-) for the Long function and, between M95 (+) and M96 (-) for the Cross function.
- The analogue signal (V DC) from the right lateral Bucky potentiometer sliders must increase with the increase of cassette format. Use a voltmeter to check this condition between contacts M115 (+) and M116 (-) for the Long function and between M111 (+) and, M112 (-) for the Cross function.
- The analogue signal (V DC) from the left lateral Bucky potentiometer sliders must increase with the increase of cassette format. Use a voltmeter to check this condition between contacts M107 (+) and M108 (-) for the Long function and between M103 (+) and M104 (-) for the Cross function.


- The analogue signal (V DC) from the table translator potentiometer slider must increase with the decrease of the SID (as the table rises). Use a voltmeter to check this condition between contacts M91 (+) and M92 (-).
- The analogue signal (V DC) from the stand potentiometer slider must increase with the increase of the SID (as the stand rises). Use a voltmeter to check this condition between contacts M87(+) and M88 (-) as the stand rises.

Using the ASR003 board, the operator is able to calibrate the collimator to the components, see **Fig. Radiology System**.


Fig. Radiology System.

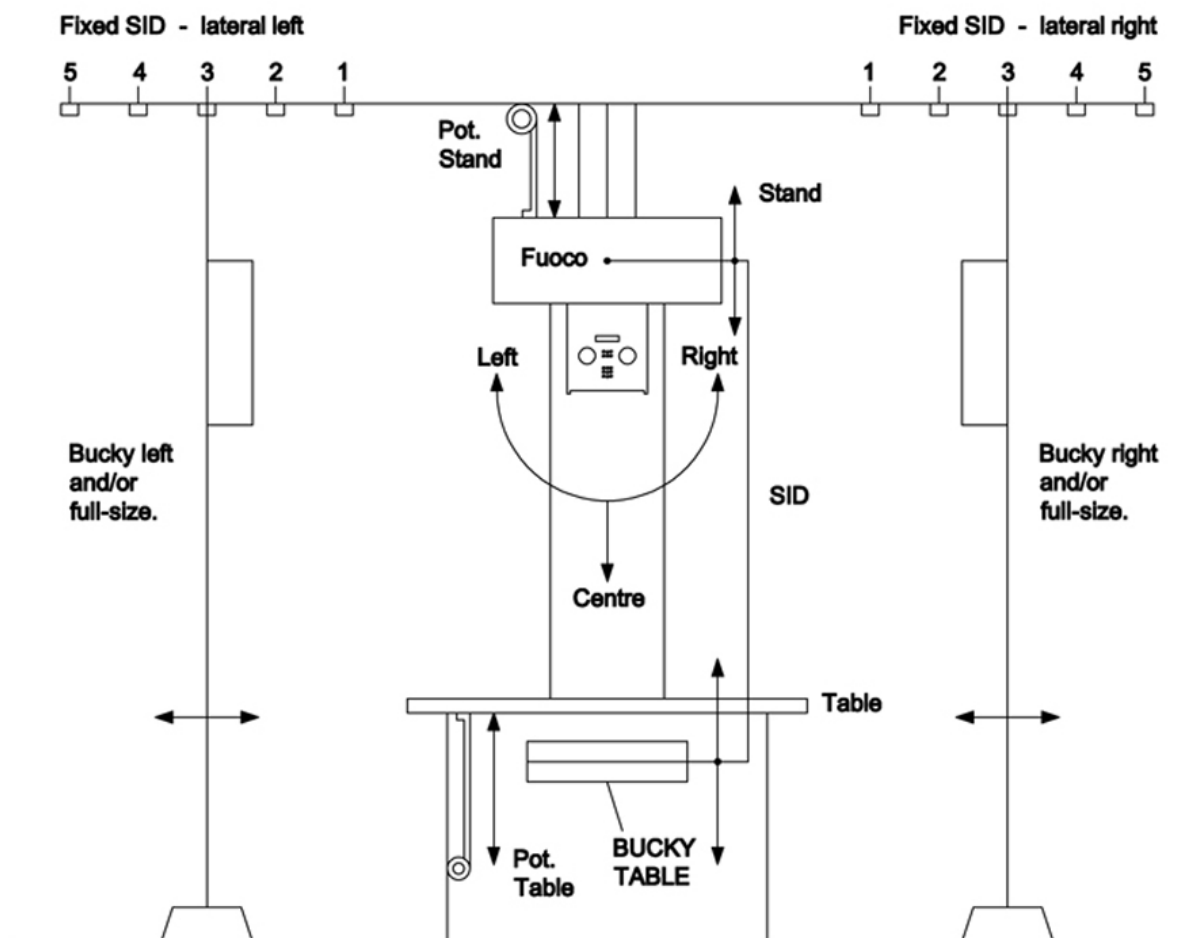

X RAY CASSETTE

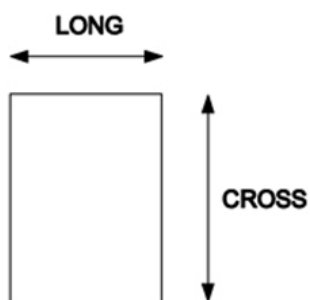

RADIOLOGY SYSTEM

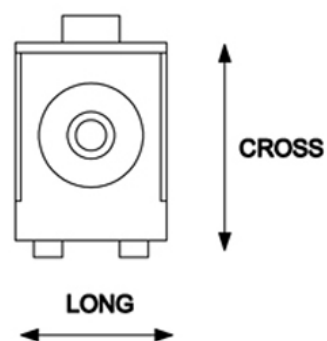


Calibration Start-Up

It is possible to start calibration in two manners:

- Press the LIGHT (C) push button and keep pressed while switching the collimator ON, see **Fig. 1**.
- jumper J4 on the external board, see **Fig. 1**. The systems sees the insertion and displays the following:


Remove Jumper to start

Fig. 1

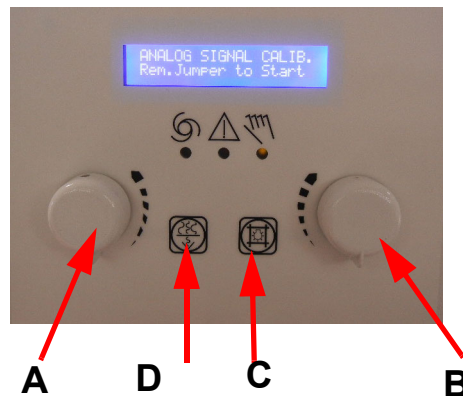

- A - Encoder Cross changes option and set up the Cross cassette size.
B - Encoder Long sets up Long cassette size.
C - to confirm
D - to skip without saving

Fig. 2

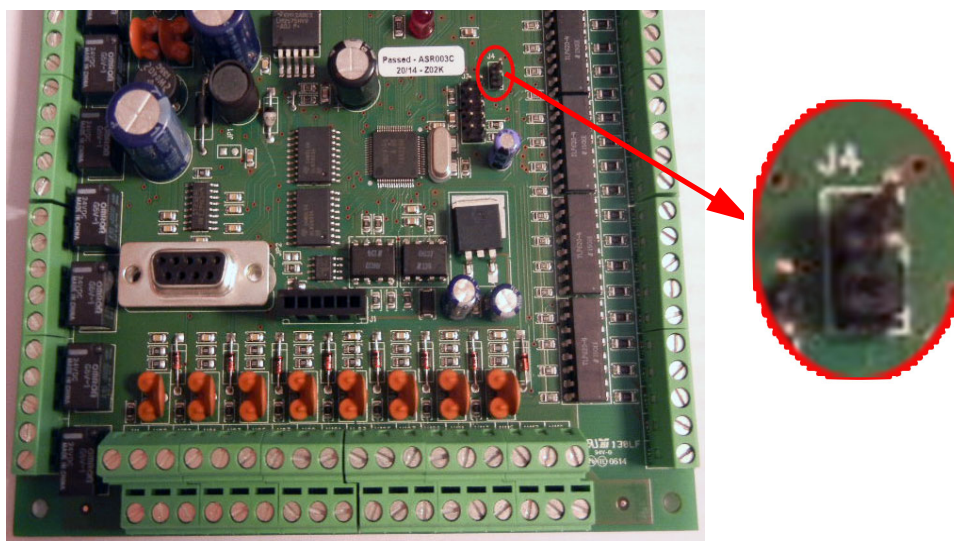

By removing the jumper, the procedure starts with the calibration of the SID potentiometers.
Calibration Procedure

1. Language Selection

The display will read one of the two following messages, depending on the current language setting.

Language
English

2. Measuring Unit - Centimetres

The Encoder Cross (A) will allow to switch from CENTIMETRES to INCHES.

*Measuring Units
centimetres*

Press LIGHT to confirm the setting; the next selection will be displayed:

3. Inclinometer Present

*Inclinom. Present?
YES/NO*

Please Note: When wall Bucky is present Inclinometer must be enabled.

The Encoder Cross (A) switches from Yes to NO.

Press LIGHT (C) to confirm the setting.

If Inclinometer is disabled go to **Vertical SID Measure section 4.**
If Inclinometer is enabled, the inclinometer offset may be calibrated:

3.1. Offset Inclinometer - Sets 0° position



WARNING

MAKE SURE THAT THE COLLIMATOR IS IN A VERTICAL POSITION (0°) WHILE ACQUIRING DATA, OTHERWISE THE CALIBRATION WILL NOT BE VALID.

*Inclinometer Offset 0°
X: xx Y: xx*

xx: value read by inclinometer
yy: value read by inclinometer

Press LIGHT (C) to confirm the setting.

3.2. Offset Inclinometer - 90° CCW



WARNING

MAKE SURE THAT THE COLLIMATOR IS ROTATED 90° TOWARD RIGHT (CCW) WHILE ACQUIRING DATA, OTHERWISE THE CALIBRATION WILL NOT BE VALID.

*Inclinometer Offset 90 R
X: xx Y: xx*

xx: value read by inclinometer
yy: value read by inclinometer

Press LIGHT (C) to confirm the setting.

3.3. Offset Inclinometer - 90° CW



WARNING

MAKE SURE THAT THE COLLIMATOR IS ROTATED 90° TOWARD LEFT (CW) WHILE ACQUIRING DATA, OTHERWISE THE CALIBRATION WILL NOT BE VALID.

Inclinometer Offset 90 L
X: xx Y: xx

xx: value read by inclinometer

yy: value read by inclinometer

Press LIGHT (C) to confirm the setting.

3.4. Degrees Tolerance

Degrees Tolerance
XX degrees

The encoder Cross (A) will allow to enter values from 1 to 15 degrees. Press LIGHT (C) to confirm the setting.

3.5. Collimator Angle Visualization

Angle Visualization
Yes

The Encoder Cross (A) activates or deactivates the display of the collimator tilt angle in degrees.

Press LIGHT (C) to confirm the setting.

4. Vertical SID Measure

The Encoder Cross (A) will allow to switch between SID: SINGLE, DIFFERENTIAL, FIXED and CAN.

- **SINGLE:** only one potentiometer is calibrated for SID calculation:
 - Table (go to the section 4.1 and 4.1.1)
 - Stand (go to the section 4.1 and 4.1.2)
- **FIXED:** the fixed value is set during calibration (go to the section 4.2).
- **CAN:** SID value is transmitted to the collimator via CANBus (go to **Right Lateral Measure section 5**)
- **DIFFERENTIAL:** both potentiometers are calibrated, one for the table and one for the stand (go to the section 4.1.1 and 4.1.2).

Vertical SID
Meas.

4.1. Potentiometer Input

*Potentiometer Input
Stand Pot/Table Pot*

Indicate the potentiometer to be calibrated (table or stand)

Press LIGHT (C) to confirm the setting.

4.1.1. Calibration of Vertical SID - Table Down

Indications displayed for the calibration of the table analogue signal require the table to be lowered:

*Move Table Down
xxx pts*

The second line of the display shows the digitized voltage value generated by the table potentiometer. The operator will lower the table. When reading is stabilized, press LIGHT (C) to complete the setting. Should the operator wish to keep the value previously stored and maintain the last calibration, this setting may be avoided by pressing FILTER (D).

Both instances will lead to the next measurement.

Calibration of Vertical SID - Raise Table 25 cm

*Set Table at + 25 cm
xxx pts*

The operator is required to place the table at the stated height. When the setting is stabilized, press LIGHT (C) to confirm the setting. Should the operator wish to keep the value previously stored and maintain the last calibration, this setting may be avoided by pressing FILTER (D).

Calibration of Vertical SID - Focus Table Distance

*Focus Table Distance
xxx pts*

The operator is required to place the table at the stated height. When the setting is stabilized, press LIGHT (C) to confirm the setting. Should the operator wish to keep the value previously stored and maintain the last calibration, this setting may be avoided by pressing FILTER (D).

4.1.2. Calibration of Vertical SID - Stand All Up

*Move Stand Full Up
xxx pts*

The display will provide calibration indications for the stand analogue signal, requiring the stand be raised to a higher position.

The second line of the display shows the digitized voltage value generated by the stand potentiometer.

The operator is required to raise the stand. When the value is stabilized, press LIGHT (C) to confirm the setting.

Should the operator wish to keep the value previously stored and maintain the last calibration, this setting may be avoided by pressing FILTER (D).

Calibration of Vertical SID - Lower Stand 50 cm

*Lower 50 cm
xxx pts*

The operator is required to place the table at the assigned height. When the value is stabilized, press LIGHT (C) to complete the setting.

Should the operator wish to keep the value previously stored and maintain the last calibration, this setting may be avoided by pressing FILTER (D).

Calibration of Vertical SID - SID at 100 cm

*Set stand to 100cm
xxx pts*

The operator is required to set the stand at 1 m (100 cm) from the table top. When the value is stabilized press LIGHT (C) to complete the setting.

Should the operator wish to keep the value previously stored and maintain the last calibration, this setting may be avoided by pressing FILTER (D).

4.2. Fixed Vertical SID Value

The Encoder Cross (A) will allow to increase or decrease the SID value from 50 cm min to 300 cm max.

*Vertical Fixed SID
xxx cm*

Press LIGHT to confirm the setting; the next selection will be displayed (go to section 5):

5. Right Lateral SID Measure

The Encoder Cross (A) will allow to switch between DISCRETE SID, CAN or POTENTIOMETER.

Press LIGHT (C) to confirm the setting; the next selection will be displayed:

Right Lateral SID
DISCRETE/CAN/POTENTIOMETER

- **DISCRETE:** lateral SID is selected via contacts on terminals M65, M66, M68, M70, M71.
- **CAN:** Lateral SID is transmitted to the collimator via CANBus.
- **POTENTIOMETER:** if SID is measured through potentiometer, the operator tilts the collimator first and then calibrates the potentiometer.

- **MIN RIGHT POT. SID**

Minimum distance from a surface of image receptor to the film is to be set up.

Min Right Pot. SID: nnn
... cm

nnn - (value read by potentiometer).

- **MAX. RIGHT POT. SID**

Maximum distance from a surface of image receptor to the film is to be set up.

Max. Right Pot. SID: nnn
... cm

nnn - (value read by potentiometer).

6. Left Lateral SID Measure

The Encoder Cross will allow to switch between DISCRETE SID, CAN or POTENTIOMETER.

Press LIGHT (C) to confirm the setting; the next selection will be displayed:

Left Lateral SID
DISCRETE/CAN/POTENTIOMETER

- **DISCRETE:** lateral SID is selected via contacts on terminals M65, M66, M68, M70, M71.
- **CAN:** Lateral SID is transmitted to the collimator via CANBus.
- **POTENTIOMETER:** if SID is measured using potentiometer, the operator tilts the collimator first and then calibrates the potentiometer.

- **MIN LEFT POT. SID**

Minimum distance from a surface of image receptor to the film is to be set up.

*Min.Left Pot. SID: nnn
... cm*

nnn - (value read by potentiometer).

- **MAX LEFT POT. SID**

Maximum distance from a surface of image receptor to the film is to be set up.

*Max. Left Pot.SID: nnn
... cm*

nnn - (value read by potentiometer).

7. Vertical Receptor

The Encoder Cross (A) will allow to switch between NO, BUCKY, CAN, ATS or FIXED FORMATS.

*Vertical Receptor
NO/BUCKY/CAN/ATS/FIXED FORMATS*

- **NO:** no receptor is connected.
- **BUCKY:** The receptor is a classical Bucky.
 - **Film Table Distance**
This requests the distance from the table top to the image receptor.
The display will read the following information:

*Film Table Distance
xxx cm*

The Encoder Cross (A) will allow to enter values from 0 min. to 15 cm max.
Press LIGHT (C) to confirm the setting.

- **CAN:** formats are CanBus related to the collimator.
- **ATS:** formats are CanBus related to the collimator (ATS detector mode)

- **FIXED FORMATS:** 5 fixed formats are selected.
 - **DIGITAL FORMAT # n**
The display will read indications to enter the fixed measures of lateral SID. It is possible to connect, to the external board ASR003, the inputs (max 5) that allow to enter the fixed formats in the automatic mode.

Formato Digitale # n
AA x BB cm

AA - Cross format. The Encoder Cross (A) will allow to increase or decrease the SID value.

BB - Long format. The Encoder Long (B) will allow to increase or decrease the SID value.

Values will be given in inches if this was the initial setting.

Press LIGHT (C) to confirm the setting; the next selection will be displayed.

8. Right Lateral Receptor

The type of lateral (right) receptor installed with the system must be selected. The Encoder Cross (A) will allow to switch between BUCKY, CAN, or NO.

Right Lateral Receptor
NO/BUCKY/CAN

- **NO:** no receptor is connected.
- **BUCKY:** classical Bucky receptor.
 - **Right Film-Plane Distance**
This requests distance from the receptor top to the image receptor. The display will read the following information

Right FilmPlane Distance
xxx cm

The Encoder Cross (A) will allow to enter values from 0 min to 15 cm max. Press LIGHT (C) to confirm the setting; the next selection will be displayed.

- **CAN:** formats are transmitted to the collimator via CanBus. Press LIGHT (C) to confirm the setting; the next selection will be displayed:

9. Left Lateral Receptor

The type of lateral receptor (left) installed with the system must be selected. The Encoder Cross (A) will allow to switch between BUCKY, CAN, NO.

*Left Lateral Receptor
NO/BUCKY/CAN*

- **NO:** no receptor is connected.
- **BUCKY:** classical Bucky receptor.
 - **Left Film-Plane Distance**
This requests the distance from the receptor top to the image receptor.
The display will read the following information

*Left FilmPlane Distance
xxx cm*

The Encoder Cross (A) will allow to enter values from 0 min to 15 cm max.
Press LIGHT (C) to confirm the setting; the next selection will be displayed.

- **CAN:** formats are transmitted to the collimator via CanBus.
Press LIGHT(C) to confirm the setting; depending on the vertical SID value entered.

10. Setting of SID Horizontal # 1

The display will require the fixed values for the lateral SID to be entered.

*Horiz. SID Meas. #1
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max.
Values will be given in inches if this was the initial setting.

Press LIGHT (C) to confirm the setting and move forward to the next Lateral SID value.

11. Setting of SID Horizontal # 2

*Horiz. SID Meas. #2
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max.
Press LIGHT (C) to confirm the setting.

12. Setting of SID Horizontal # 3

*Horiz. SID Meas. #3
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max.
Press LIGHT (C) to confirm the setting.

13. Setting of SID Horizontal # 4

*Horiz. SID Meas. # 4
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max. Press LIGHT (C) to confirm the setting.

14. Setting of SID Horizontal # 5

*Horiz. SID Meas. # 5
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max. Press LIGHT (C) to confirm the setting.

15. Setting of FULL-SIZE SID # 1

*Panoramic SID #1
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max. LIGHT (C) confirms the selection and passes to the next SID Panoramic value.

16. Setting of FULL-SIZE SID # 2

*Panoramic SID #2
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 25 min to 300 cm max. LIGHT (C) confirms the selection and passes to the next SID Panoramic value.

17. Calibration of Vertical SID Limits - minimum

The display will show indications of the minimum Vertical SID value accepted by the collimator.

*Vertical SID Min.
xxx cm*

The Encoder Cross (A) will allow to enter the measurements from 50 cm min to 300 cm max. Press LIGHT (C) to confirm the setting.

18. Calibration of Vertical SID Limits - maximum

The display will show indications of the maximum Vertical SID value accepted by the collimator.

Vertical SID Max.
xxx cm

The Encoder Cross (A) will allow to enter the measurements from 50 cm min to 300 cm max. Press LIGHT (C) to confirm the setting; maximum Vertical SID data will be displayed.

19. SID Visualisation

SID Visualization
Yes

The Encoder Cross (A) activates or deactivates the display of the SID value on the main window.

Press LIGHT (C) to confirm the setting.

20. Key Status Visualization

Key Status Visual.
Yes

The Encoder Cross (A) activates or deactivates the display of the key status on the main window (open or closed padlock). Press LIGHT(C) to confirm the setting.

Calibrate the vertical Bucky cassettes.

The system will, otherwise, analyze the settings related to the two lateral receptors: if at least one of the two has been programmed as analogue Bucky, the lateral left and/or right cassettes will be calibrated. If no Bucky was selected, the system will display the end of calibration. Cassette Calibration Information:

The first thing the system requires is to select a quick calibration procedure or a full calibration procedure for all types of cassettes (vertical, lateral left and lateral right).

The differences between the two calibrations are:

- **Quick Cassette Calibration:** the system requires to enter two cassette sizes only (one small and one large).
- **Full Cassette Calibration:** the system requires to enter all possible cassette values (max. 10 Buckys).
IE: If 3 cassette sizes are stored as 13x18, 24x30 and 30x40 cassettes, the system will also be able to identify cassette sizes such as 13x30, 13x40, 24x18, 24x40, 30x18, 30x30.

21. Vertical Bucky Cassettes, Calibration

*Table Bucky Cal.
Quick*

The system starts with the calibration type. The Encoder Cross (A) will allow to switch from Quick to Full calibration. Press LIGHT (C) to confirm the setting.

To cancel the procedure press FILTER (D). This keeps the last calibration values and moves forward to the next calibration phase. The number of cassette sizes to be processed must be entered in the event of a Full Calibration.

This step is skipped for the Quick Calibration procedure and 2 cassette values will be processed.

Full Calibration

*Set Number of
Table Buckies: xxx*

xx - cassette number from 0 to 10.

Once the number is selected the cassettes need to be calibrated one by one.

*Cross 1 T cm 13: XXX
Long 1 T cm 13: XXX*

*Cross 2 T cm 43: XXX
Long 2 T cm 43: XXX*

At this point the operator may:

- Set the Cross cassette from 13 cm (") min to 43 cm (17") max with the Encoder Cross (A).
- Set the Long cassette from 13 cm (") min to 43 cm (17") max with the Encoder Cross (A).

When values are stabilized, confirm the setting by pressing LIGHT (C).

22. Calibration of Left Lateral Bucky Cassette

*Left Bucky Cal.
Quick*

The system starts with the calibration type. The Encoder Cross (A) will allow to switch from Quick to Full calibration. Press LIGHT (C) to confirm the setting.

To cancel the procedure press FILTER (D). This keeps the last calibration values and moves forward to the next calibration phase. The number of cassette sizes to be processed must be entered in the event of a Full Calibration.

This step is skipped for the Quick Calibration procedure and 2 cassette values will be processed.

Full Calibration

*Set Number of
Left Side Buckie xxx*

xx - cassette number from 0 to 10.

Once the number is selected the cassettes need to be calibrated one by one.

*Cross 1 S cm 13: XXX
Long 1 S cm 13: XXX*

*Cross 2 S cm 43: XXX
Long 2 S cm 43: XXX*

At this point the operator may:

- Set the Cross cassette from 13 cm (") min to 43 cm (17") max with the Encoder Cross (A).
- Set the Long cassette from 13 cm (") min to 43 cm (17") max with the Encoder Cross (A).

When values are stabilized, confirm the setting by pressing LIGHT (C).

23. Calibration of Right Lateral Bucky Cassette

*Right Bucky Cal.
Quick*

The system starts with the calibration type. The Encoder Cross (A) will allow to switch from Quick to Full calibration. Press LIGHT to confirm the setting.

To cancel the procedure press FILTER (D). This keeps the last calibration values and moves forward to the next calibration phase. The number of cassette sizes to be processed must be entered in the event of a Full Calibration.

This step is skipped for the Quick Calibration procedure and 2 cassette values will be processed.

Full Calibration

*Set Number of
Right Side Buckies xxx*

xx - cassette number from 0 to 10.

Once the number is selected the cassettes need to be calibrated one by one.

*Cross 1 D cm 13: XXX
Long 1 D cm 13: XXX*

*Cross 2 D cm 43: XXX
Long 2 D cm 43: XXX*

At this point the operator may:

- Set the Cross cassette from 13 cm (") min to 43 cm (17") max with the Encoder Cross (A).
- Set the Long cassette from 13 cm (") min to 43 cm (17") max with the Encoder Cross (A).

When values are stabilized, confirm the setting by pressing LIGHT (C).

24. Saving Calibration Data

The system informs the operator that calibration has terminated.

*End of Calibration
LIGHT per RESET*

Save the setting by restarting up the collimator or pressing the push-button LIGHT (C)

RO 329 Internal Proximity Sensor

A proximity sensor is a sensor able to detect the minimum distance to a patient without any physical contact, thereby avoiding the risk of collision.

The proximity sensor emits an inferred beam and register a return signal if utilized correctly.

The minimum distance that this sensor detects is 30 cm. A shorter distance will generate alarms to be managed by the X-ray system and end user.

WARNING



THE LIABILITY FOR THE CORRECT SENSOR FUNCTIONING DEVOLVES UPON THE END USER AND/OR THE X-RAY EQUIPMENT MANUFACTURER.

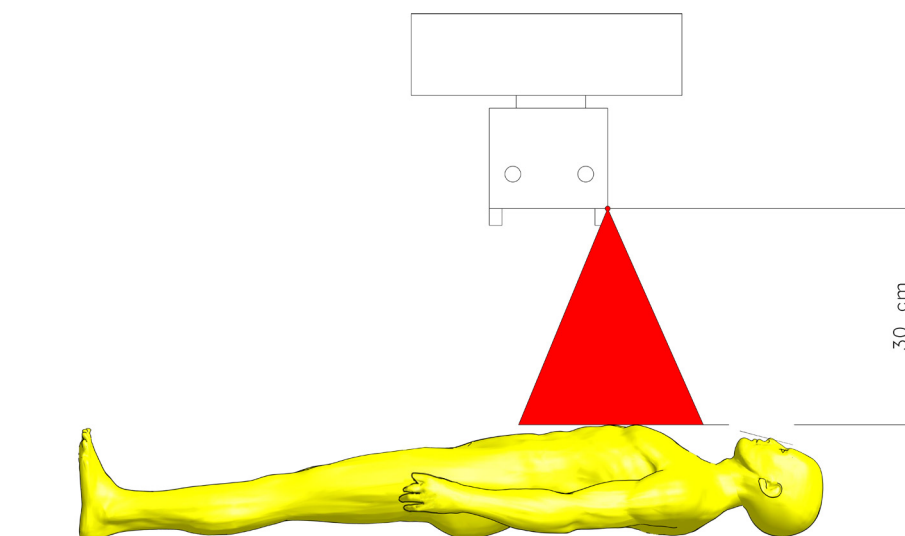



Fig. Minimum distance patient/X-ray focus

Connection

The 12 V supply is to be drawn from connector J10; the output signal is to be connected to pin 2 of connector J12. A resistor, 10 Kohm 1/4 W, is to be mounted between pins 2 and 3 on connector J12. See Fig. 1 for the wiring diagram of the sensor.

Calibration

The two switches on the sensor front serve as reference

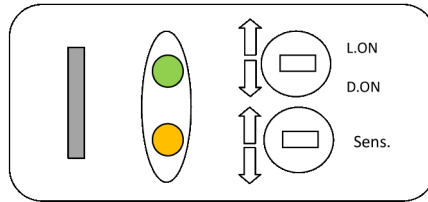

Sensor Calibration

- Rotate the Light/Dark switch to LIGHT ON to activate the sensor when the object is in measuring range.
- Set the object at the required detection distance and adjustment the sensitivity switch by increasing the potentiometer until the orange LED light is lit.
- Remove the object and set sensitivity half-way between the detected point and maximum potentiometer value.

WARNING




MAKE SURE THAT THE PROXIMITY SENSOR IS WORKING PROPERLY.
AN IMPROPER USE OF THE PROXIMITY SENSOR MIGHT CAUSE
HARM TO THE PATIENT.

WARNING



RALCO REFUSES TO ACCEPT ANY LIABILITY FOR DAMAGES IN
THE EVENT OF FAILED OR IMPROPER OPERATION OF THE DEVICE.

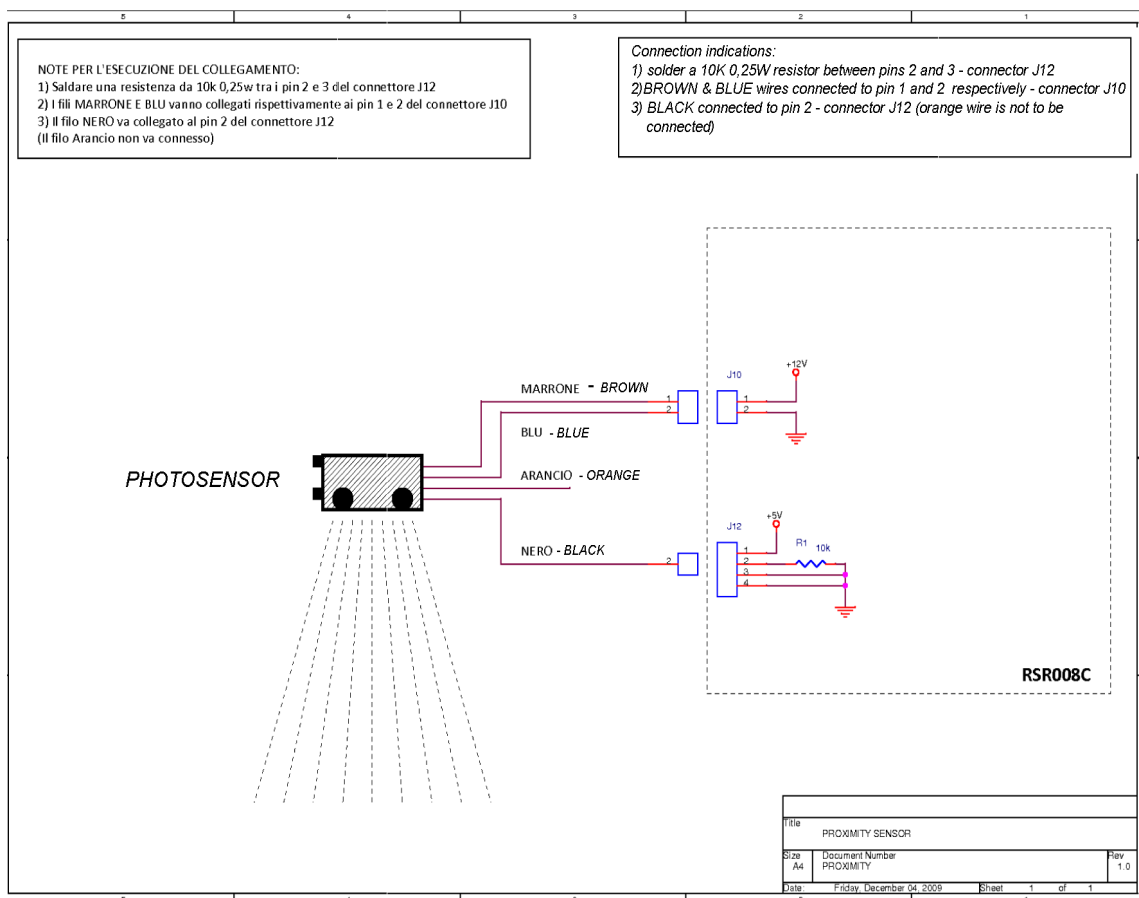

RO 330 Connecting cable extension


Connecting cable extension; Longer cable is provided for your system.

RO 332/A Resin Rotating Mounting Flange

Resin rotating mounting flange with fixing screws:


- 15 mm thickness
- +/- 90° detent
- 140 mm diameter

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/collimator mounting instructions.

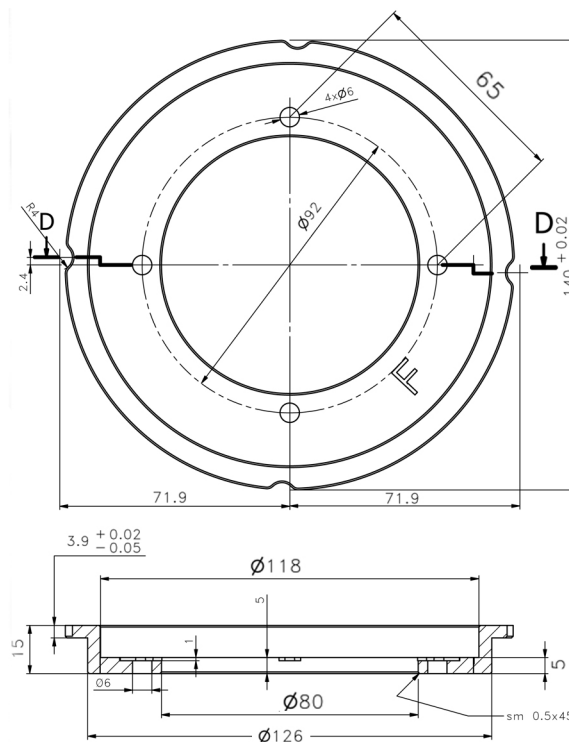


RO 336 Aluminium accessory guide rails

Pair of aluminium rail guides designed for an ionisation chamber with different dimensions to standard chamber and to allow the insertion of other accessories.

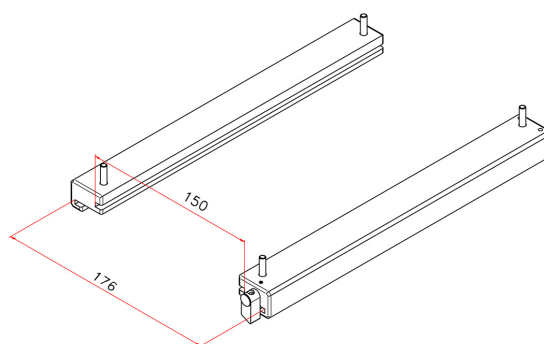


RO 344 Accessory guide rail spacers with reinforced support

Pair of plastic rail guides (RAL 9003) on metal support designed to accommodate the ionization chamber and allow the insertion of relative filter (153 mm \pm 0.5).

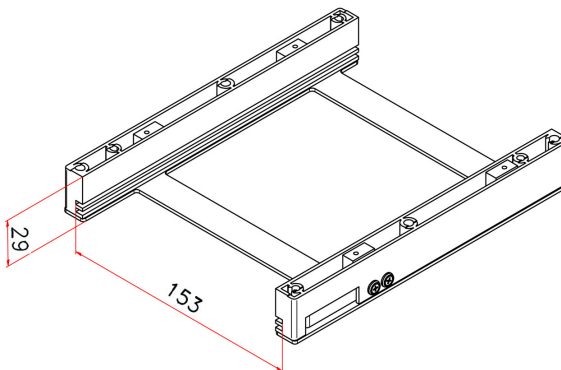

RO 356 Shutter position verified by potentiometer

Supplementary safety device to ensure correct positioning of shutters. It consists of two potentiometers mounted on the collimator: One is mounted on the back of the collimator to control the Long shutters and the other potentiometer is mounted on the side to control the Cross shutters. The output signal is managed by a customer.

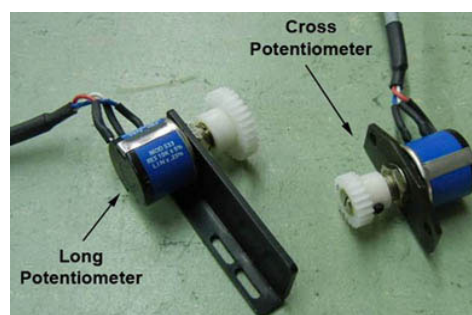

RO 405 Housing Customization

Customized covers in aluminium according to customer design.


RO 409 Resin Rotating Mounting Flange

Resin rotating mounting flange milled to form a square:

- 20 mm thickness
- $\pm 0^\circ$ detent
- 140 mm diameter

Option includes three (3) iron mounting flange spacers also milled to form a square: 1.5 mm thickness.

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/collimator mounting instructions.

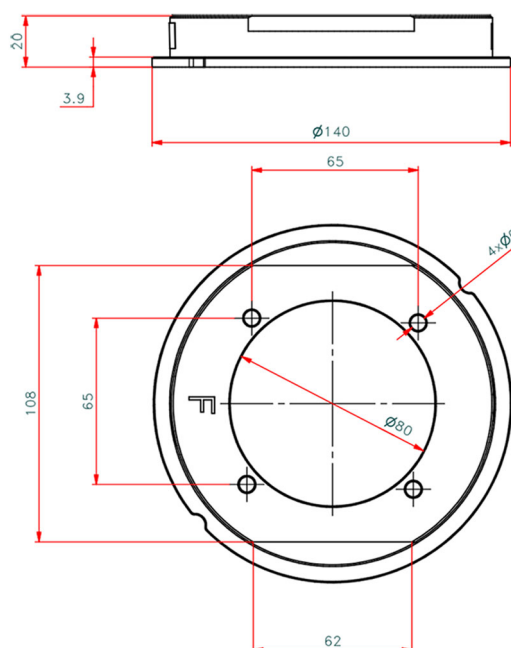


RO 441 Resin Rotating Mounting Flange

Resin rotating mounting flange:

- 20 mm thickness
- $\pm 50^\circ$ detent
- 140 mm diameter

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/collimator mounting instructions.

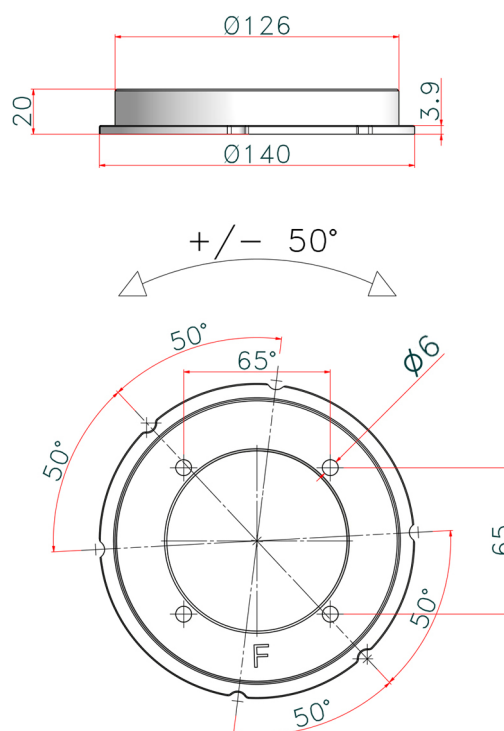


RO 442 Resin Rotating Mounting Flange

Resin rotating mounting flange:

- 20 mm thickness
- +/- 90° detent
- 140 mm diameter

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/collimator mounting instructions.

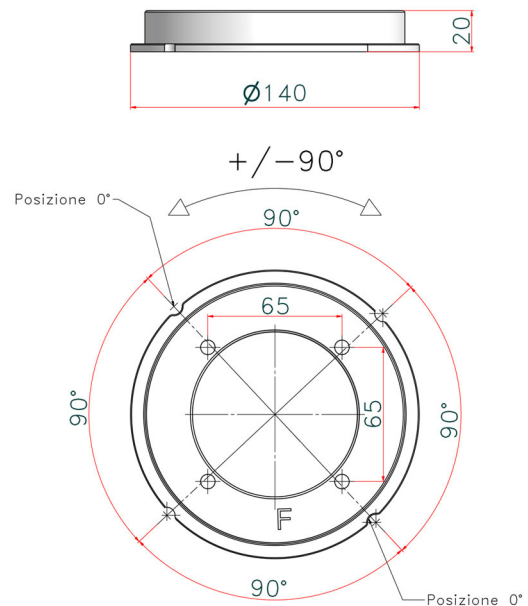

RO 444 Can Open Protocol


Software that controls the collimator functions via CanOpen protocol.

RO 445 Metal Rotating Mounting Flange

Metal rotating mounting flange:

- 20 mm thickness
- +/- 90° mechanical stop
- 140 mm diameter

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/ collimator mounting instructions.

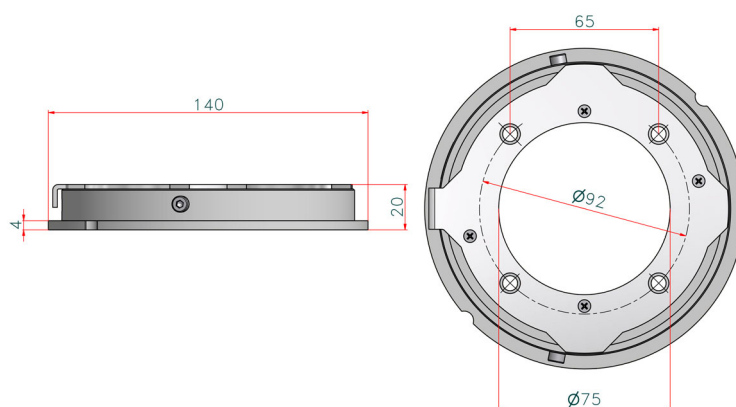

When the collimator is assembled check the collimator to Focal Spot Alignment (Primary Shutter Cut-Off).

Inspect the four images of the four collimator shutters which form the edges of the X-ray field. A definitely indistinct edge indicates that the primary shutter, close to the focal spot, is the one forming the line, rather than the outermost shutter.

To correct the condition, use the four mounting/centering adjustment screws to shift the collimator in the direction of the indistinct line. Repeat the test film exposure after making the adjustment.

NOTE




THE HEEL EFFECT WILL CAUSE THE FIELD TOWARD THE CATHODE TO BE SLIGHTLY LESS SHARP THAN ON THE OTHER THREE SIDES. THIS IS NORMAL AND CANNOT BE CORRECTED BY ADJUSTMENT. IN ADDITION, AN X-RAY TUBE OF 12° OR LESS TARGET ANGLE WILL PRODUCE AN ASYMMETRICALLY SHAPED FIELD WHEN A LARGE FIELD SIZE IS USED AT SHORT SID, BECAUSE OF ANODE CUT-OFF EFFECT. THIS IS NORMAL AND MAY NOT BE CORRECTED BY ADJUSTMENT.

RO 456 Metal Rotating Mounting Flange

Metal rotating mounting flange:

- 20 mm thickness
- +/- 90° with mechanical stop
- 136 mm diameter

Please refer to section **RO 492 Adjustable Top-Cover Bracket** in this chapter of the Instruction Manual for the correct flange/collimator mounting instructions..

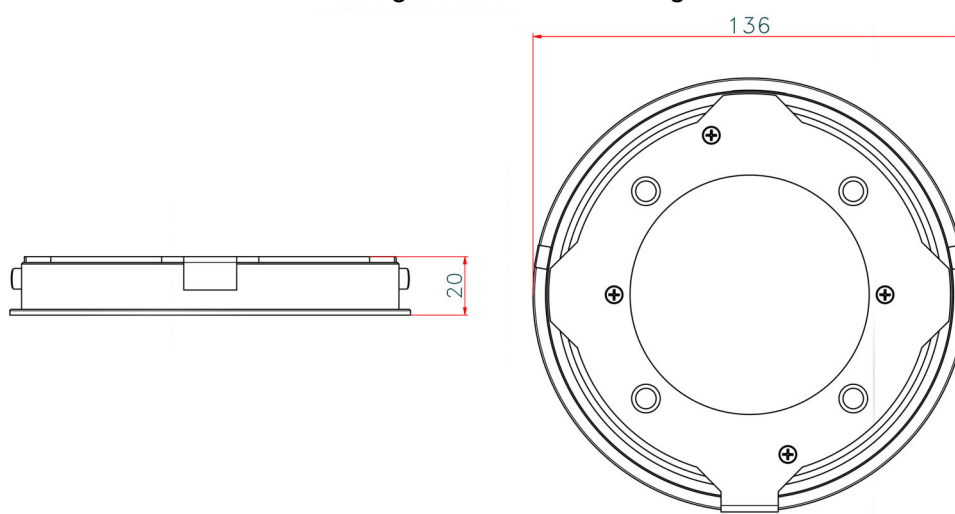

RO 476 Asymmetric longitudinal and transversal shutters


This option has been designed to drive the shutters independently (Stitching mode)
Increased housing dimension: 244x282x216 cm - R 225 ACS housing.

WARNINGS




TO AVOID THE SHUTTER DAMAGE THE FOUR INDEPENDENT SHUTTERS ARE IMMOBILISED DURING TRANSPORT. REMOVE THE LOCKING SCREWS BEFORE STARTING TO USE THE COLLIMATOR.


- Remove the rear cover by unscrewing the four screws **A**.

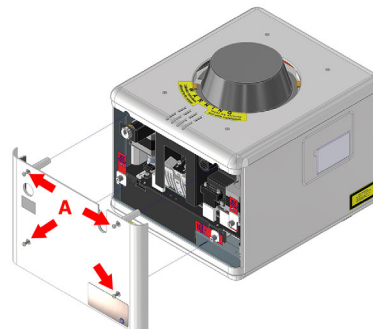

- The locking screws are marked by a red label.

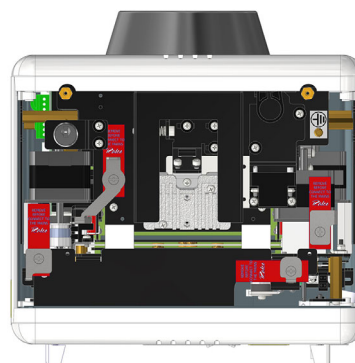

- Unscrew the four locking screws **B** to release the shutter movement.
- Remount the rear cover.

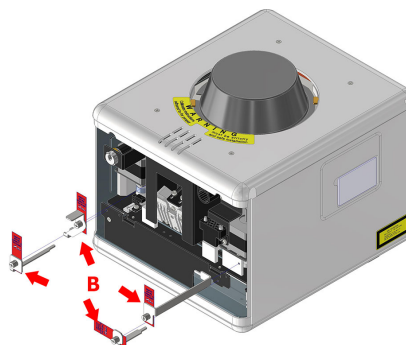


RO 484 Infrared Remote Control Box

This accessory runs two collimators at the same time (Coll.A and Coll.B)

Collimator features:

- Light push button, the light is automatically set for 30 seconds (1).
- Laser push button, the laser is automatically set for 60 seconds (2).
- 8 push buttons (3) for manual opening and closing of each blade, identified by drawing.
- 4 push buttons (4) for selecting the filter (0,1,2,3).
- 4 push buttons (5) for choosing 4 different formats (A, B, C, D). See CAN Bus commands below.

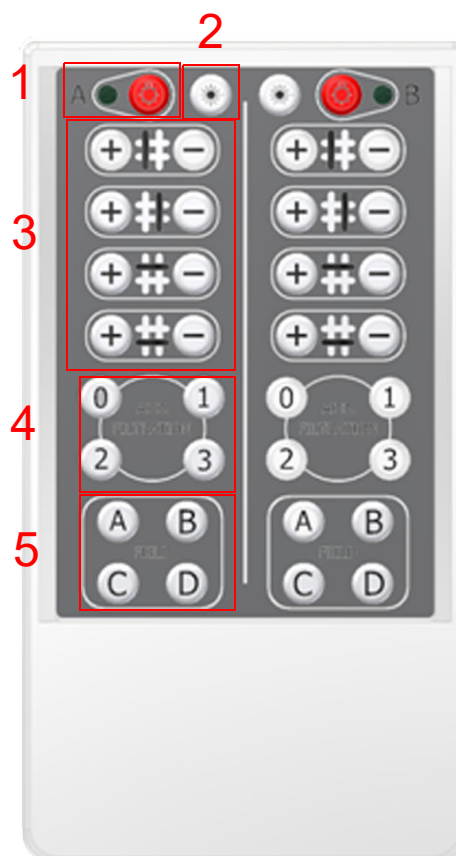

The four formats can be changed in the following way:

- Set the 4 shutters to the desired position.
- Press and do not release the format push button (eg: C) for the setting to be stored. The collimator stores the new format after about 5 seconds, informing the user by lighting the red LED (remote reception) on the front panel for a few seconds.
- Release the format push button on the remote control. The new format is automatically stored in permanent memory.

CAN Bus messages:

The controls are standard commands of Ralco's protocol:

0x7A0 controls the 4 shutters simultaneously in 1/10 mm.

0x7A4 controls one shutter at a time (1-4), or the filter motor (5), etc.

0x7A8 manual handling.



WARNING

THE ANTI-COLLISION CONTROL OF A PAIR OF SHUTTERS IS DISABLED FOR THE MANUAL HANDLING.

Customized commands:

id=7AE dlc=8 D0=0x01 D1 sets the type of collimator (A or B) remote control code

D1 = 0x01 collimator A

D1 = 0x02 collimator B

Data is stored in the permanent memory

id=7AE dlc=8 D0=0x02 D1 powers up laser

D1 = 0x00 switches the laser OFF


D1 = 0x01 switches the laser ON (maximum 60 seconds)

RO 489 Resin Rotating Mounting Flange

Resin rotating mounting flange:

- 20 mm thickness
- +/- 45° detent
- 140 mm diameter

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/collimator mounting instructions.

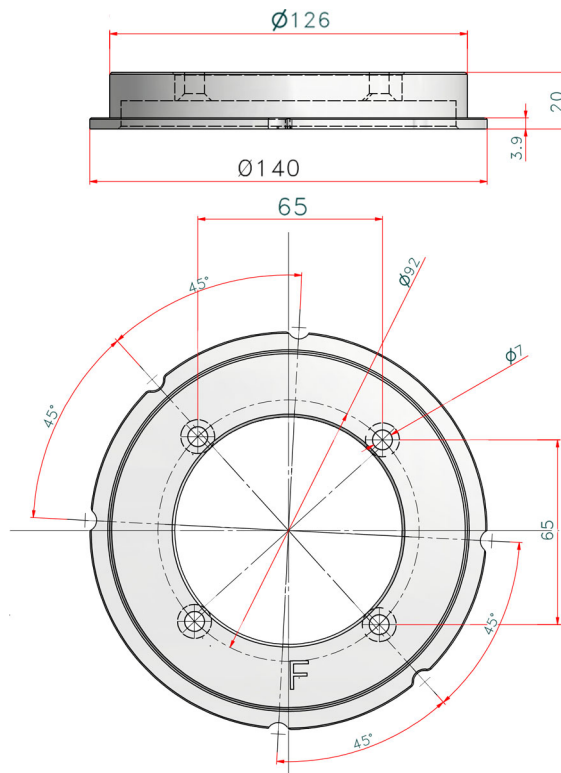


RO 490 Metal Rotating Mounting Flange

Metal rotating mounting flange with countersunk mounts:

- 18 mm thickness
- +/- 90° with mechanical stop
- 140 mm diameter

Please refer to section **RO 492 Adjustable Top-Cover Bracket** in this chapter of the Instruction Manual for the correct flange/ collimator mounting instructions.

This flange is **ONLY** available with **RO 492**.

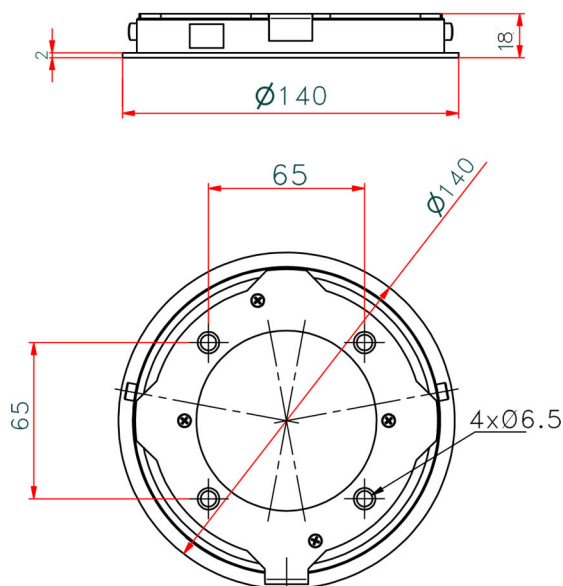

RO 492 Adjustable Top-Cover Bracket

This accessory has been designed to precisely install the collimator with a 136mm diameter metal flange (not included) to the X-ray tube.

WARNINGS



THE COLLIMATOR MUST BE INSTALLED TO THE X-RAY TUBE THROUGH A MOUNTING FLANGE. RALCO PROVIDES VARIOUS FLANGE OPTIONS WHICH MAY NOT BE INTER-CHANGEABLE. ONLY FLANGES PROVIDED WITH THE COLLIMATOR MAY BE UTILIZED. THE END-USER MAY INSTALL THEIR OWN FLANGE, HOWEVER RALCO CANNOT GUARANTEE COMPATIBILITY. ANY PREEXISTING FLANGE ON THE END-USER SYSTEM MUST NOT BE USED.



WHEN A FLANGE IS PROVIDED WITH THE COLLIMATOR BEARING A MATCHING SERIAL NUMBER, ENSURE THEY ALWAYS REMAIN COUPLED. IT IS MANDATORY THEY REMAIN TOGETHER AND THE CORRECT FLANGE PART NUMBER IS USED.



THE CONTENTS OF THE INSTRUCTIONS BELOW SHOULD BE STRICTLY ADHERED TO. RALCO IS NOT LIABLE FOR ANY PROPERTY DAMAGE OR RESULTING HARM IF NON-RALCO COMPONENTS OR NON-COMPATIBLE RALCO COMPONENTS ARE USED DURING THE INSTALLATION PROCESS.

Installation

TUBE COMPATIBILITY

1. Using the dimensions in **Fig. Tube Compatibility** below, ensure the near port shutters of the collimator are placed in the X-Ray tube port without interference.
2. The distance between the X-Ray tube focus and the flange mounting plane (collimator upper plate) must be: 80 mm (3.14"), tolerance ± 1 mm (0.04 ").

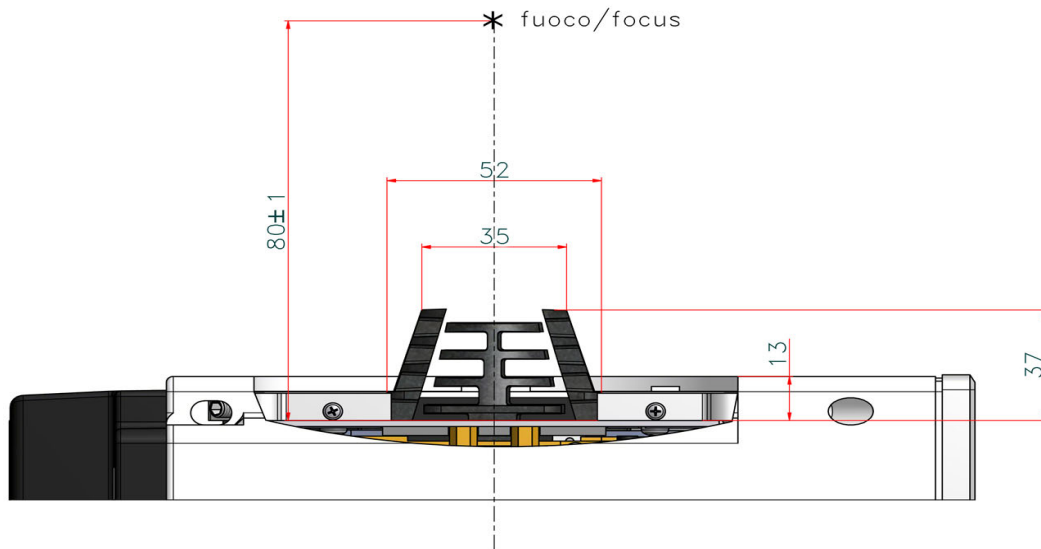


Fig. Tube Compatibility

3. Carefully remove the collimator and the mounting flange (if purchased) from their packaging.
4. Use the X-Ray tube housing datasheet to determine the distance (A) from the focal spot to the X-Ray tube port, see **Fig. Collimator Installation**.
5. Subtract the resulting distance from the source flange distance (B) and determine the number of spacers (1.5 mm) which, combined with the thickness of the mounting flange, will make up the difference (C). Allowable tolerance is 1 mm. (0.04"), see **Fig. Collimator Installation**.
6. Once the mounting plane distance has been confirmed, continue with the mounting flange installation to the X-Ray tube.

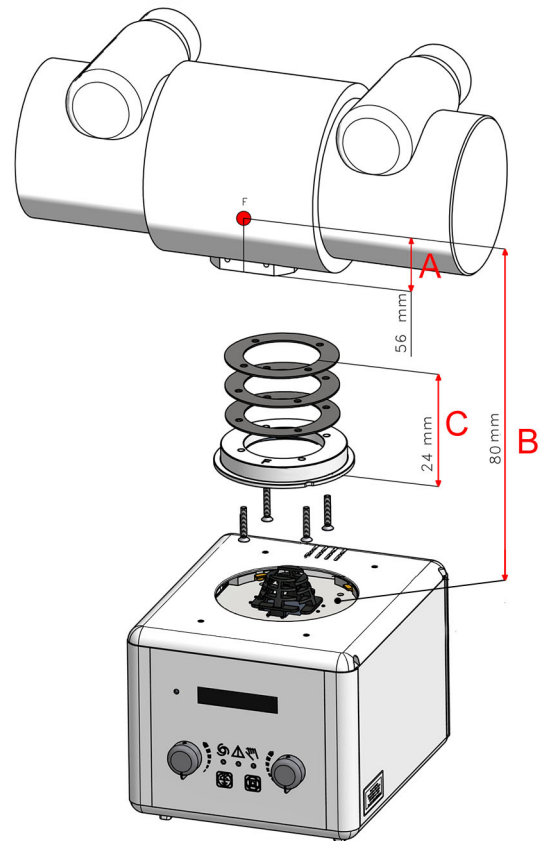


Fig. Collimator Installation
*illustrative purpose only

**The flange fixing screws and the spacers of the previous flange may be reused if the flange thickness is the same.*

Mounting the Flange to the X-ray Tube

NOTE



THE FOLLOWING MOUNTING INSTRUCTIONS ARE ONLY APPLICABLE FOR COMPATIBLE FLANGES WITH THIS OPTIONAL ITEM. IF YOU ARE UNSURE THE MOUNTING FLANGE YOUR COLLIMATOR IS EQUIPPED WITH IS COMPATIBLE, PLEASE CONSULT YOUR PERSONALIZATION PAGE PROVIDED WITH THIS MANUAL TO LOCATE THE FLANGE PART NUMBER (RO REFERENCE). FOLLOW THE MOUNTING INSTRUCTIONS SET FORTH UNDER THE SPECIFIC RO REFERENCE IN THIS MANUAL.

WARNING



FLANGES MAY BE PROVIDED BY RALCO OR BY THE SYSTEM MANUFACTURER. FLANGES MAY NOT BE INTERCHANGEABLE. ONLY THE FLANGE PROVIDED WITH THE COLLIMATOR BEING INSTALLED WITH A SPECIFIC PART CODE MAY BE UTILIZED. ANY PRE-EXISTING FLANGES MAY NOT TO BE USED. IF THERE ARE ANY QUESTIONS REGARDING COMPATIBILITY, PLEASE CONTACT RALCO.

CAUTION



RALCO CANNOT GUARANTEE COMPLIANCE WITH RADIATION STANDARDS CONCERNING SAFETY IF THIS CONTROL HAS BEEN OMITTED.

1. Place the flange on the X-Ray tube port, see **Fig. Flange Installation**.
2. Mount the mounting flange and spacers (optional) to the X-Ray tube port using 4 screws.**

****Please ensure no conflicting information nor dangerous conditions exist due to adhering to these instructions or those provided by the X-Ray tube manufacturer. When in doubt please contact X-ray tube manufacturer and/or Ralco.**

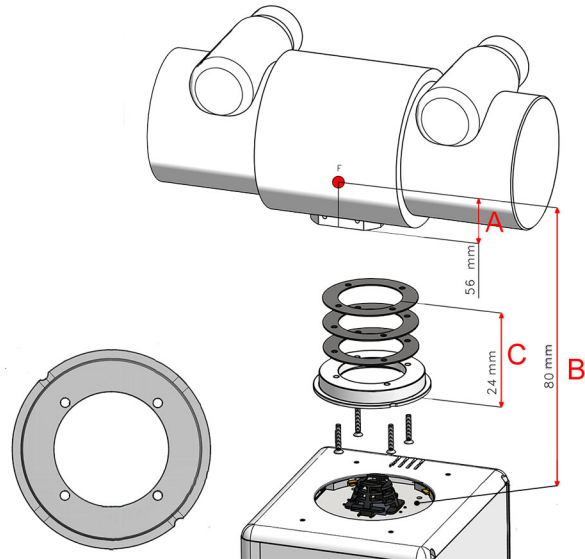


Fig. Flange Installation
**illustrative purpose only*


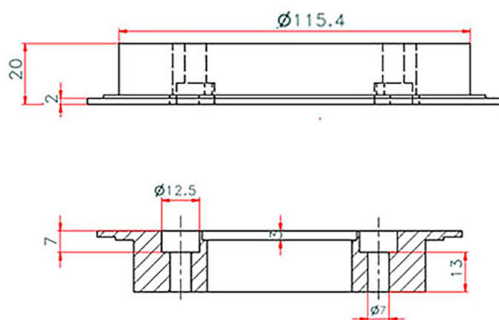


Fig. Mounting Flange
**illustrative purpose only*



CAUTION

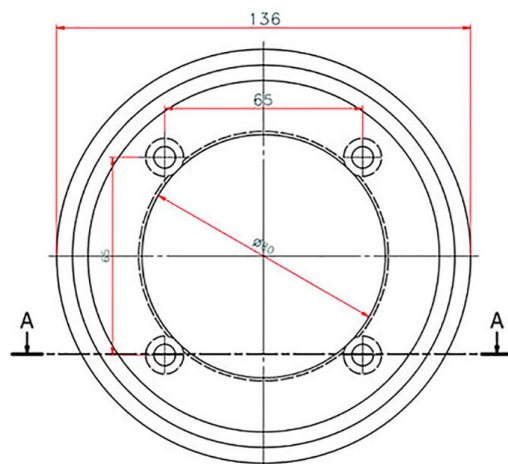

ENSURE THE SCREW HEAD IS INDEED CORRECT FOR THE FLANGE SELECTED. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE ALL SAFETY MEASURES ARE IMPLEMENTED TO ENSURE THE SCREWS ARE OPTIMALLY TIGHTENED INCLUDING THE USE OF APPROPRIATE LOCKTITE.



TIGHTEN THE 4 SCREWS TO THE X-RAY TUBE HEAD SECURELY, STRICTLY ACCORDING TO THE INSTRUCTIONS OF THE X-RAY TUBE MANUFACTURER. DO NOT EXCEED 0.45 NM OF FORCE.

Alignment of X-Ray Tube Focus and Collimator

Ralco guarantees the correct collimator functionality, format compliance and light/X-Ray field alignment only if the mounting flange and the collimator have been installed exactly in the centre of the X-Ray beam.

All Ralco collimators are aligned on our test bench utilizing specific references/values for our X-Ray tube focus, detector and Source to Image Detector Distance (SID). The customer must know and verify all known variables which may influence the X-Ray tube focus and collimator alignment. These may include, the X-Ray tube focus position tolerance, distance from X-Ray tube focus to collimator mounting plane, or the SID.


Alignment Device

This device is used to ensure the collimator mounting flange is correctly aligned to the X-ray beam. The X-ray tube manufacturer provides a tolerance for the placement of X-ray tube focus.

Ralco recommends to use the Focal Alignment Device (jig) to ensure the correct flange alignment with the centre of the X-Ray beam, see **Fig. Focal Alignment Device**. By making an exposure, it is possible to verify the perpendicularity and concentricity using fixed references on the X-Ray image.

Once the mounting flange is aligned the collimator light/X-ray field should also be aligned (within specific tolerances).

Please consult the technical specifications of your X-Ray tube to find the maximum tolerance for the position of the focus. Should the use of an alignment device not be possible, Ralco collimators allow for the regulation of the light field.

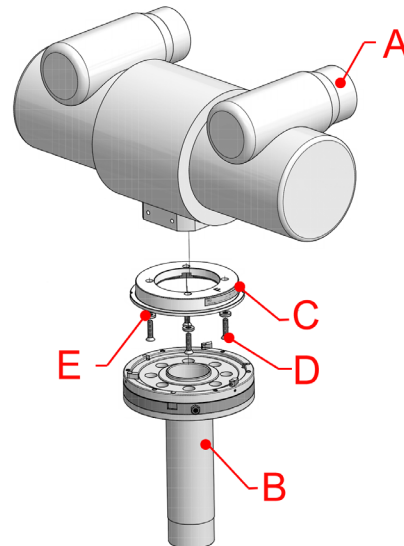

A - X-Ray Tube, B - Focal Adjustment Device
C - Mounting Flange, D - Screw, E - Washer

Fig. Focal Alignment Device

Mounting the Collimator to the Flange

NOTE



THE MOUNTING FLANGE PROVIDED (IF PURCHASED) WITH THE COLLIMATOR IS SUBJECTED TO TESTING PURSUANT TO ALL APPLICABLE STANDARDS.



MOUNTING BRACKET TABS CONFORM TO EN60601.

WARNINGS



PURSUANT TO APPLICABLE STANDARDS, RALCO HAS TESTED THE COLLIMATOR AND FLANGE APPLYING STATIC LOADS. RALCO IS NOT IN A POSITION TO KNOW THE DYNAMIC FORCES OF ALL END-USER SYSTEMS. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE DYNAMIC FORCES OF THE SYSTEM DO NOT CREATE A DANGEROUS CONDITION.



IT IS THE RESPONSIBILITY OF THE SYSTEM MANUFACTURER TO ENSURE AND MITIGATE ANY DANGEROUS CONDITIONS WHICH MAY OCCUR DUE TO THE DYNAMIC FORCES CREATED BY THE SYSTEM. THE END-USER MUST PERFORM A SYSTEMATIC AND STRUCTURAL ANALYSIS DURING THE INSTALLATION AND USUAL MAINTENANCE.



SHOULD ANY DAMAGE TO THE COLLIMATOR OR FLANGE OCCUR A RISK ANALYSIS AND DAMAGE ASSESSMENT NEEDS TO BE CONDUCTED IMMEDIATELY. CONTACT RALCO IMMEDIATELY SHOULD THIS OCCUR. RALCO IS NOT LIABLE FOR RESULTING PROPERTY DAMAGE AND/OR HARM DUE TO AN UNREPORTED INCIDENT.



RALCO HAS DESIGNED AND TESTED THE COLLIMATOR FOR A LIFETIME OF 10 YEARS. AFTER THIS TIME PERIOD, IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE THE PROPER FUNCTIONING OF THE COLLIMATOR AND FLANGE. LIABILITY FOR ANY DANGEROUS CONDITIONS WHICH MAY BE PRESENT AFTER THE 10YEAR LIFETIME OF THE COLLIMATOR AND FLANGE RESTS WITH THE END-USER.



TO ENSURE THE SAFETY OF THE COLLIMATOR AND FLANGE AFTER 10 YEARS OF USE, RALCO HAS INSTITUTED A PROGRAM TO ASSESS THE SAFETY OF THE COLLIMATOR AND FLANGE. AFTER APPLYING A CHECK LIST OF QUALITY CONTROLS AND REFURBISHMENT ACTIVITIES (AT END-USER EXPENSE), RALCO MAY CERTIFY THE COLLIMATOR AND FLANGE FOR ADDITIONAL YEARS OF USE.

1. Prepare the collimator to be installed by unscrewing the 4 hexagonal socket screws until the four tabs are completely withdrawn from the collimator top mounting plane, see **Fig. 4 Mounting Screws**.
2. If installing a manual collimator, adjust the collimator shutters to the fully open position using both knobs.

CAUTION



WHEN UNSCREWING THE HEXAGONAL SOCKET SCREW WHICH CONTROL THE TABS, DO NOT USE FORCE EXCEEDING 0,45 NM. UNSCREW WITH CARE SO AS NOT TO DAMAGE THE HEXAGONAL SCREW HEAD AND TABS.

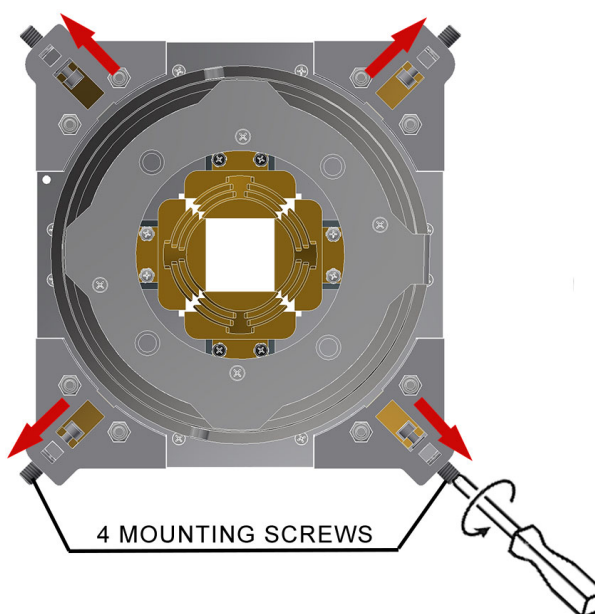


Fig. 4 Mounting Screws

3. With the tabs fully retracted, the hexagonal socket screws of the mounting bracket must be tightening equally up to the end stroke with 0.45 Nm torque (at least 7 turns), see **Fig. Mounting Bracket**.
4. The 4 tabs of the hexagonal socket screws overlap on the flange outer ring in the same manner.
5. The collimator tabs adhere to the flange outer ring. Depending on the optional flange purchased, the collimator may rotate or be fixed (no rotation).
6. Once the collimator is coupled to the flange in the method described above, verify the distance between the collimator housing and the mounting flange is equal in all directions and the collimator face is parallel to the axis of the table. Loosen the screws and adjust as necessary.
7. The collimator should be coupled to the flange firmly. If the collimator is loose, please repeat the above mounting instructions, and if issues persist, please contact Ralco.

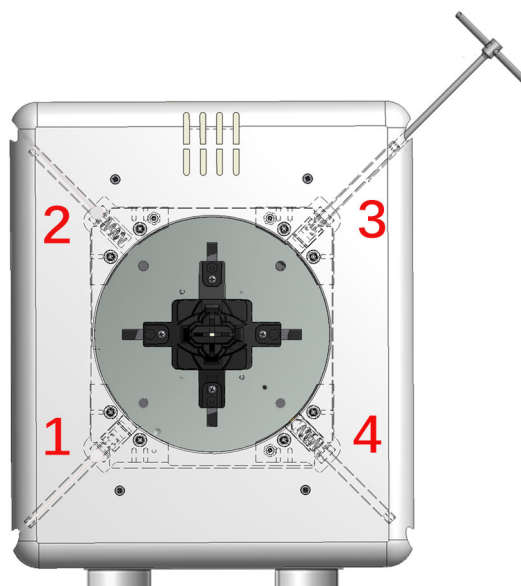


Fig. Mounting Bracket
*illustrative purpose only

Verification of Correct Installation

WARNING



IT IS THE DUTY OF THE INSTALLER TO ENSURE NO RISK OF THE COLLIMATOR FALLING EXISTS. ENSURE THE FOLLOWING, AS IN EACH SCENARIO BELOW SERIOUS RISK OF INJURY AND/OR PROPERTY DAMAGE MAY EXIST DUE TO NON-ADHERENCE.

1. The 4 tabs should overlap the flange outer ring, see **Fig. Correct Overlap**.
2. Ensure the mounting flange is flat against the collimator mounting plane, see **Fig. Correct Overlap**.
3. Ensure the 4 tabs are not in contact with only the mounting flange edge, see **Fig. Incorrect Overlap**.
4. Once the collimator is mounted, if not already, return the collimator/tube head to the intended use position. Rotate and/or gently pull the collimator to ensure correct coupling.
5. If the collimator is loose, something is incorrect. Repeat above mounting instructions, and if issues persist, please contact Ralco.

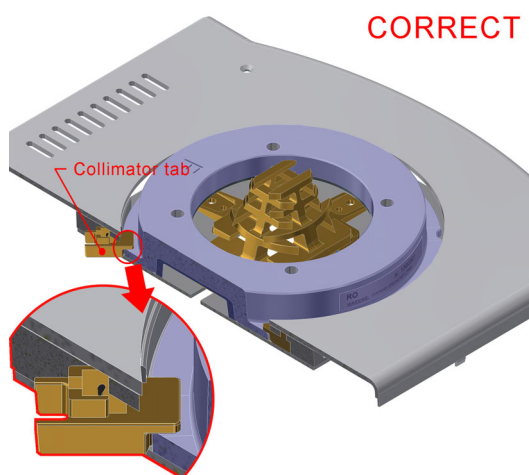


Fig. Correct Overlap



Fig. Incorrect Overlap

RO 493 Dual Energy Additional Variable Filtration

This accessory allows extremely fast filter switching (200 ms) per 90° movement and minimal vibration (filters not included- only available with RO 494).

RO 494 Customized Additional Variable Filtration


This option allows our customer to specify a type of filtration. It is possible to assembled up to 3 different filters.

RO 495 Camera assembled internally for patient monitoring


An IP ethernet interface is a type of digital camera commonly employed for surveillance, and which, unlike analog closed circuit television (CCTV) cameras, can send and receive data via a computer network and the Internet.

Substitution of the Video Camera


- Remove the knobs by unscrewing the 2 screws per knob.

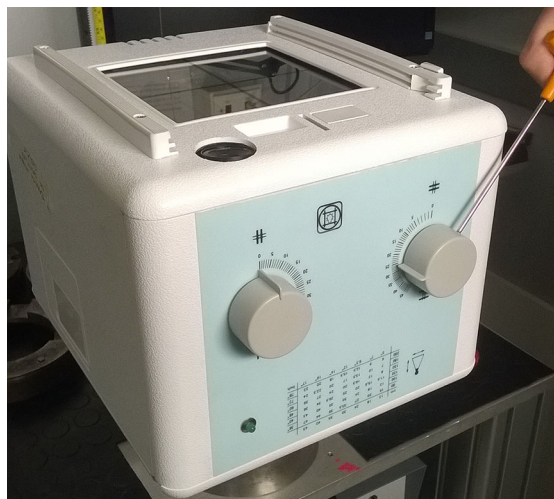


- Unscrew the 4 screws placed on the rails.

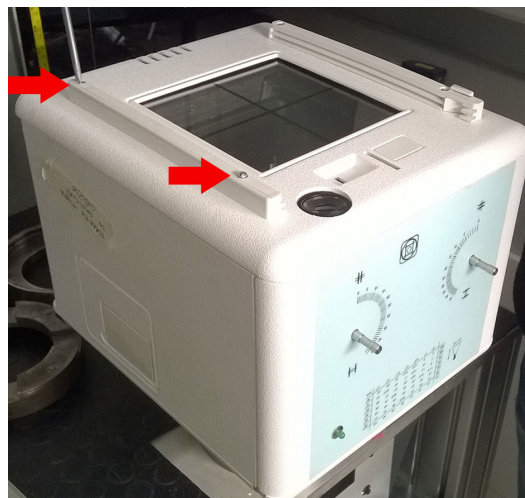



- Remove carefully the snap-on front panel and detach the connectors:
A for the push button
B for the front panel LED (if present).

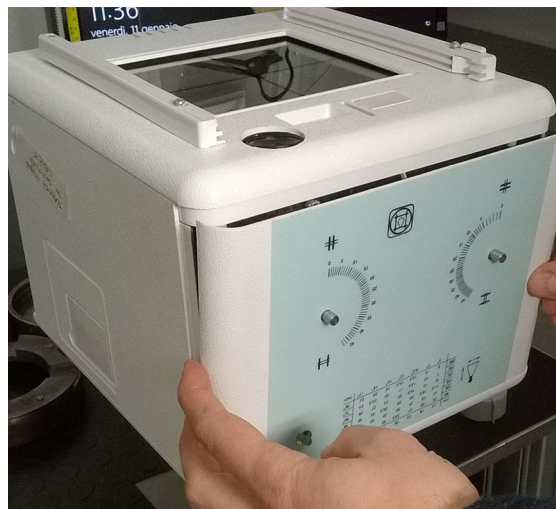


- Remove the lower cover.
- Remove the 2 lateral covers, if necessary.
- Remove the video camera with its support by unscrewing the fixing screws (C).

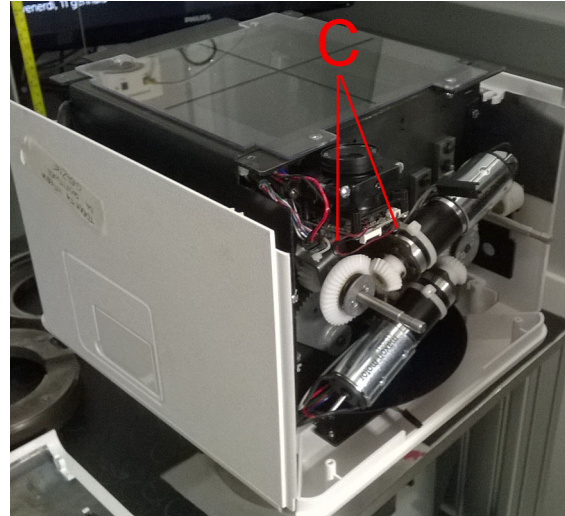


- Detach the connectors from the video camera, as shown in the photos.

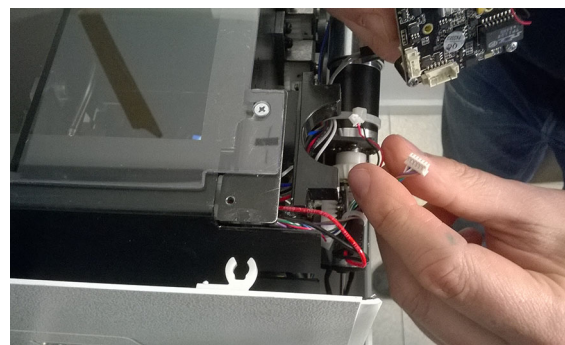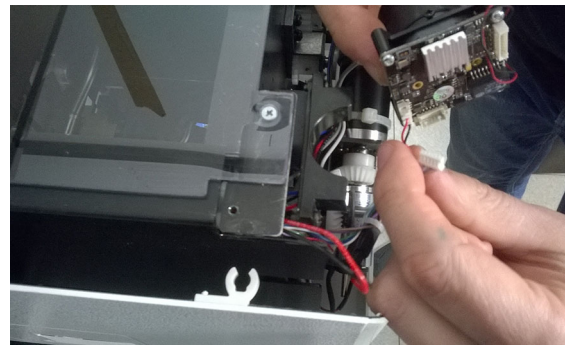


- Remount the video camera with its support by connecting the connectors and then fixing the support to the collimator housing.

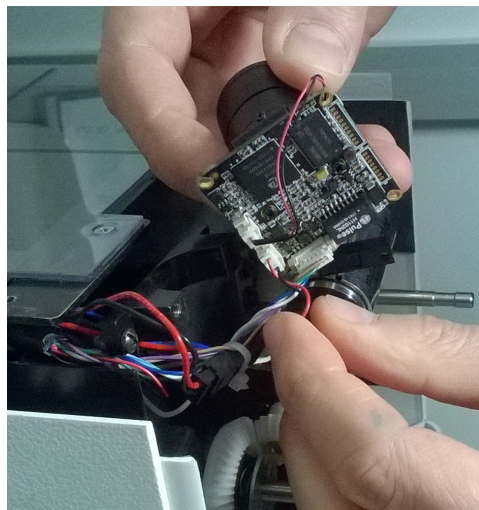

- The video camera can be adjusted vertically (**C.1**) or be tilted (**C.2**) whenever it is necessary.

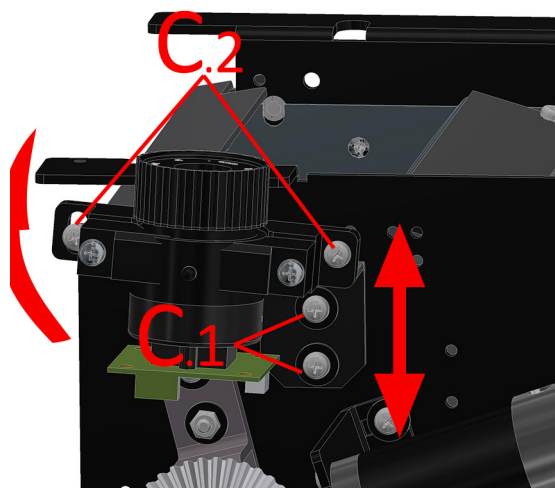


- Power up the collimator and check if the video camera is calibrated.

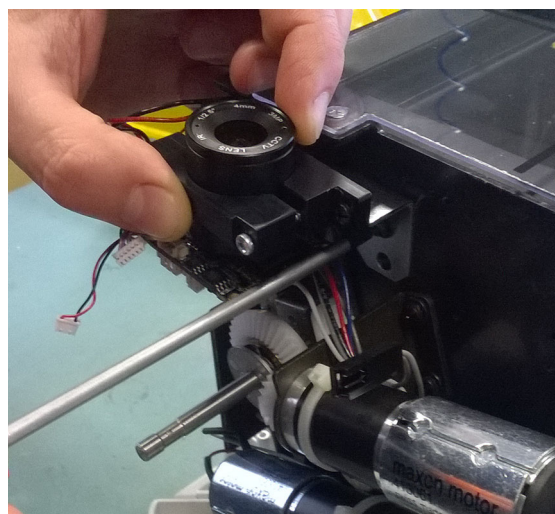




THE VIDEO CAMERA IS ALREADY WELL-FOCUSED IN THE FACTORY. PLEASE FOLLOW THE BELOW PROCEDURE TO READJUST THE FOCUS.


- If necessary, loosen the 2 screws **(D)** and rotate the whole video camera to calibrate it.

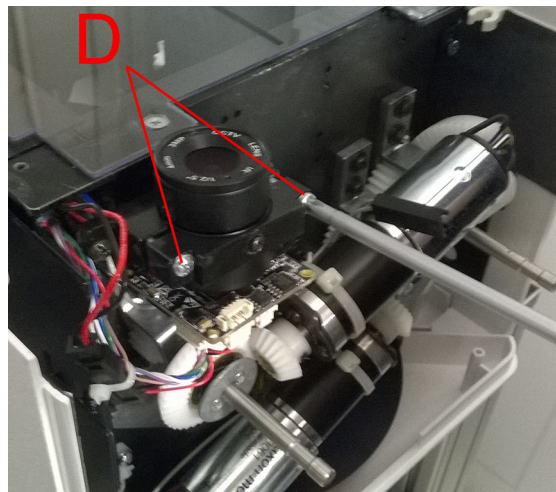

- Loosen the screw **E** placed on the video camera support.
- Adjust the focus by rotating the lens.

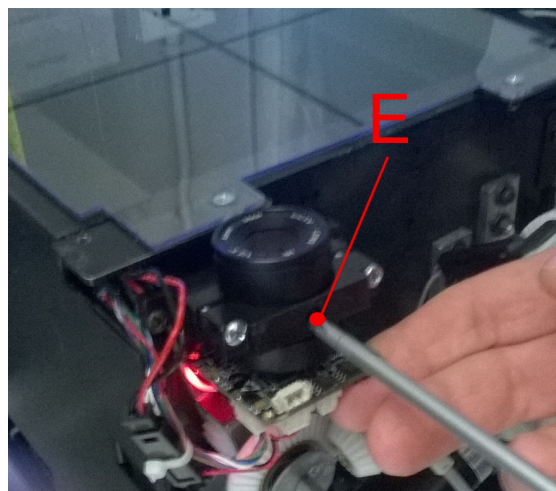

- Loosen the screw **F** and adjust the video camera inclination, if necessary.
- Tighten the screws **E** and **F**.
- Remount the collimator cover in reverse order.

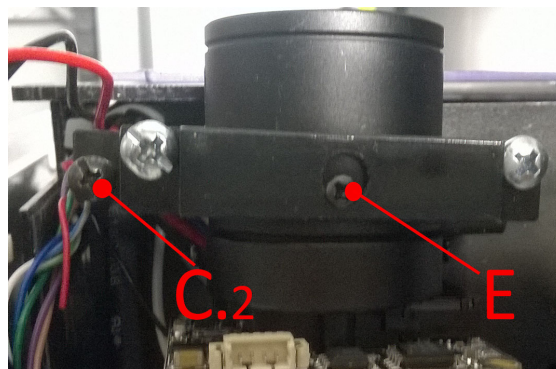

RO 496 Software Customization


Software that is developed according to the customer particular preferences, expectations and needs. The price is variable depending on the specific requests.

RO 510/A Resin Rotating Mounting Flange

This accessory is used to rotate the collimator assembled to the X-ray tube.

- Flange thickness: 18 mm,
- +/- 90° detent,
- Flange diameter: 140 mm.

See Chapter **INSTALLATION** for the correct flange/collimator mounting.

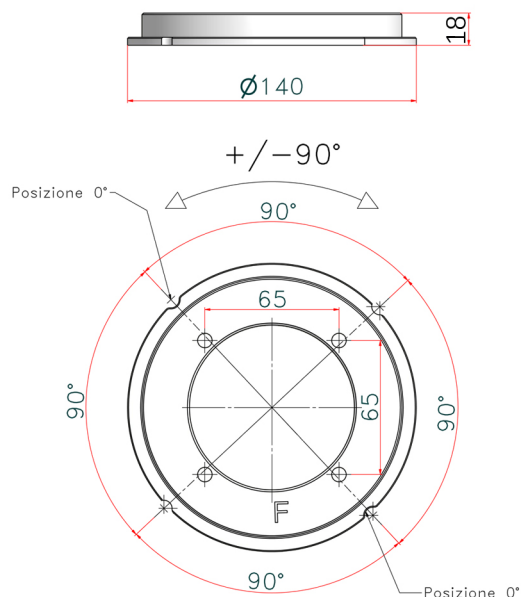

RO 525 Customized Knob design

The customer can specify the knob design.

RO 526 Semiautomatic mode (front panel with push buttons included)

This option allows the customer to control the collimator in the semi-automatic mode through customized software and the front panel.

MANUAL CONFIGURATION

Calibration Start-Up


The collimator can be manually configured with the use of the display and the control panel.

To start the collimator configuration:

- Switch the collimator ON. The display will show a series of self-tests.
- Once the self-test is finished press both the LIGHT (C) and FILTER (D) push buttons and keep them pressed for approximately 5 seconds.
- Release them when the 3 LEDs light up on the front panel. See **Fig. 1**.
- The display will read:

Wait command

Fig.

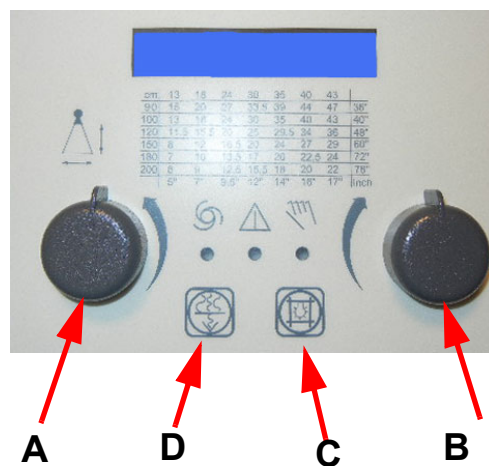

- A - left knob (DSC) changes menus and set up the Cross cassette size.
B - right knob (DSC) sets up Long cassette size.
C - light push button to confirm
D - filter push button to exit without saving

- Turn the left knob counterclockwise to skip from one menu to the next. There are the four menus available on the collimator:

**Collimator Config
Confirm? --> NO**

--> In this menu the operator sets all collimator functions

**Analog Calibration
Confirm? --> NO**

--> In this menu the operator calibrates all potentiometers present on the X-ray system (Bucky, Stand, Table, Lateral pots)

**Shutter Calibration
Confirm? --> NO**

--> In this menu the operator calibrates the apertures of LONG and CROSS shutters

**Serial Number
Confirm? --> NO**

--> In this menu the operator sets the collimator serial number

- Choose the menu which must be configured.
- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the submenu.
 - Press LIGHT (C) push button to change the configuration in the submenu.

Language Set
Italian *

- An asterisk (*) which appears in the bottom right corner of the display allows the operator to change the configuration.
- Turn the right knob to switch from one option to another.

Language Set
English *

- Press the LAMP (C) push button to confirm the choice or FILTER (D) push button to exit.
- Press LAMP (C) and FILTER (D) push button and keep them pressed to return to normal operation.

Configuration Procedure

The following section depicts a flow chart of the configuration process:

Collimator Config
Confirm? --> NO

- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the submenu.

Language Set
English/Italian

Filter Enable
Disable/Enable

Filter Button Enable
No/Yes

Lamp Time
30 sec

**from 5 to 120 sec*

Automatic Light
Yes/No

Lamp Button Enable
Enable/Disable

Knob Lamp Time
10 sec

**from 10 to 120 sec*

0x7F0 Sending Time
200 ms

**from 0 to 2000 ms*

Unit measure
cm/inch

Show SID
Yes/No

Filter Type
1 mm Al/2 mm Al/0.3 mm Cu/ 0.5 mm Ag

Inclinometer
Yes/No

SID Left Side
CAN/Discrete/Potentiometer/Fixed

Fixed Left SID
100 cm (default)

**from 25 to 300 cm*

SID Right Side
CAN/Discrete/Potentiometer/Fixed

Fixed Right SID
100 cm (default)

Left Side Receptor
CAN/Bucky/No

Right Side Receptor
CAN/Bucky/No

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

Vertical SID CAN/Fixed/Differential/Single	
Vert. SID Poten. Input Stand/Table/No	
DFT Sx 0 cm	<i>*from 0 to 15 cm</i>
DFT Dx 0 cm	<i>*from 0 to 15 cm</i>
Vertical Receptor CAN/Bucky/No/ATS/Fixed	
Fixed SID 100 cm	<i>*from 25 to 300 cm</i>
Min SID 50 cm	<i>*from 25 to 300 cm</i>
Max SID 300 cm	<i>*from 25 to 300 cm</i>
Dist. Film Ravolo DFT 0 cm	<i>*from 0 to 15 cm</i>
Inclinometer Alarm 3 cm	<i>*from 1 to 15</i>
Show Lock Yes/No	
Show Angle Yes/No	

Correction type
% on APERT/CM on SID

Cross Correction
+0.0 %

**from -10.0% to +10.0%*

Long Correction
+0.0 %

**from -10.0% to +10.0%*

Display Selection
CARESTREAM/FREE/SEDECAL/STANDARD

CAN Protocol
GMM/CARESTREAM/SEDECAL/STANDARD

Command Address
0x7A0

Potent. on Shutter
Yes/No

Alarm on 0x100
Enable/Disable

Motor Type Filter
FAST/NEW

Min Format on 0x7A8
Disable/Enable

Collimation in Man.
Disable/Enable

Dig. Fixed Format
LAMP = START

Manual Movem. Type
Fixed Frequency / mm/sec

Manual mm/sec
100 mm/sec

**from 10 to 500 mm/sec*

Potentiometer open
LAMP = START

Potentiometer close
LAMP = START

Display ASR003 val.
LAMP = START

Analog Calibration
Confirm? --> NO

- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the submenu.



THE FLOW CHART BELOW DEPICTS ALL OPTIONS ENABLE
IN THE MENU "COLLIMATOR CONFIG".

R 221 ACS Calibration

Inclinom. Offset 0
X: 2 Y: -54

- Confirm with LIGHT (C) push button to get to the next display.

Inclinom. Offset 90R
X: 2 Y: -54

Inclinom. Offset 90L
X: 2 Y: -54

Move Table Down
xxx pts

Set Table at +25 cm
xxx pts

Move Stand Full Up
xxx pts

Lower 50 cm
xxx pts

Lower 50 cm
POT. Value Error!

Set stand to 100 cm
Focus-Table xxx pts

Min. Right Pot SID: xxx
80 cm

**from 25 to 300 cm*

Max. Right Pot SID: xxx
250 cm

**from 25 to 300 cm*

Min. Left Pot SID: xxx
80 cm

**from 25 to 300 cm*

Max. Left Pot SID: xxx
250 cm

**from 25 to 300 cm*

Table Bucky Cal.
Full/Quick

Set Number of
Table Buckies xxx

Cross 1T Cm 13: xxx
Long 1T Cm 13: xxx

Left Bucky Cal.
Full/Quick

Set Number of
Left cassette xxx

Cross 1S Cm 13: xxx
Long 1S Cm 13: xxx

Right Bucky Cal.
Full/Quick

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

Set Number of
Right cassette xxx

Cross 1D Cm 13: xxx
Long 1D Cm 13: xxx

End of Calibration
Press LAMP to RESET

Shutter Calibration
Confirm? --> NO

- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the submenu.

NOTE



DURING THE SHUTTER CALIBRATION THE LIGHT IS ON FOR 10 SECONDS, AFTER THE LAST SHUTTER MOVEMENT.

CROSS Manual Calibr.
Confirm? --> NO

- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the CROSS Manual Calibration.

CROSS 480 mm P: xxx

- Rotate the left knob to adjust the CROSS shutter to the desired aperture.
- Confirm with LIGHT (C) push button.
- Press FILTER (D) to go to the next aperture.

CROSS 400 mm P: xxx

NOTE



THE POTENTIOMETER VALUE DIFFERENCE MUST BE AT LEAST 300 STEPS OTHERWISE, THE SOFTWARE GIVES THE ERROR MESSAGE.

CROSS 300 mm P: xxx

CROSS 240 mm P: xxx

CROSS 130 mm P: xxx

CROSS 000 mm P: xxx

LONG Manual Calibr.

- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the LONG Manual Calibration.

LONG 480 mm P: xxx

- Rotate the left knob to adjust the LONG shutter to the desired aperture.
- Confirm with LIGHT (C) push button.
- Press FILTER (D) to go to the next aperture.

LONG 400 mm P: xxx

LONG 300 mm P: xxx

LONG 240 mm P: xxx

LONG 130 mm P: xxx

LONG 000 mm P: xxx

**Serial Number
Confirm? --> NO**

- Turn the right knob clockwise to switch from NO to YES.
- Confirm with LIGHT (C) push button to get into the submenu.

**Serial Number
45555**

- Confirm with LIGHT (C) to change the collimator serial number.

**Password
00000**

- Insert the five digits number by rotating the right knob and confirming the set number. each time.
- Press LIGHT (C) to confirm or FILTER (D) to abort.


**Insert serial num.
00000**

- If the password was correct, the software will allow the operator to insert a new serial number.

**Numero Seriale
00000**

- Press LIGHT (C) to confirm.
- Press both the LIGHT (C) and FILTER (D) push buttons to exit the collimator configuration. Data will be automatically saved.


RO 530 Resin Rotating Mounting Flange

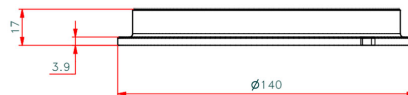

Resin rotating mounting flange:

- 17 mm thickness
- 0° detent
- 140 mm diameter

Please refer to the chapter **INSTALLATION** in this Instruction Manual for the correct flange/collimator mounting instructions.

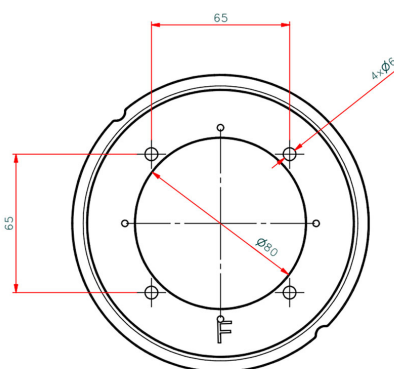

RO 533 Glass Mirror

Glass mirror, 1.7 mm thickness with the additional Aluminium filter, 0.5mm thickness, inherent filtration 2.3 mm aluminium equivalent.

RO 537 Addition of electronics and motorized shutter movement to render automatic collimator into manual collimator

This option allows the customer to control the collimator in the manual mode by excluding collimator electronics and motorized shutters of the square field.

RO 544 7" Touchscreen Display

A full colour display which allows an operator to configure the collimator parameters, control the X-ray system and preview an examination via Ethernet in real time (not available with RO 436)


Characteristics:

- Full colour display
- Diagonal Screen Size 7" or 4.3"
- Format 16:9
- Resolution 800x480 pixel
- Custom graphic and colours to meet all needs.

Functionality

All touch screen applications for a normal use are included in a single page. The display, however, has been designed to add additional pages for future requests. The icon view on the display adapts accordingly to the collimator inclination (vertical/lateral). This change occurs when the collimator rotates by 45 degrees and is measured by a built-in tilt sensor.


The touch screen menu:

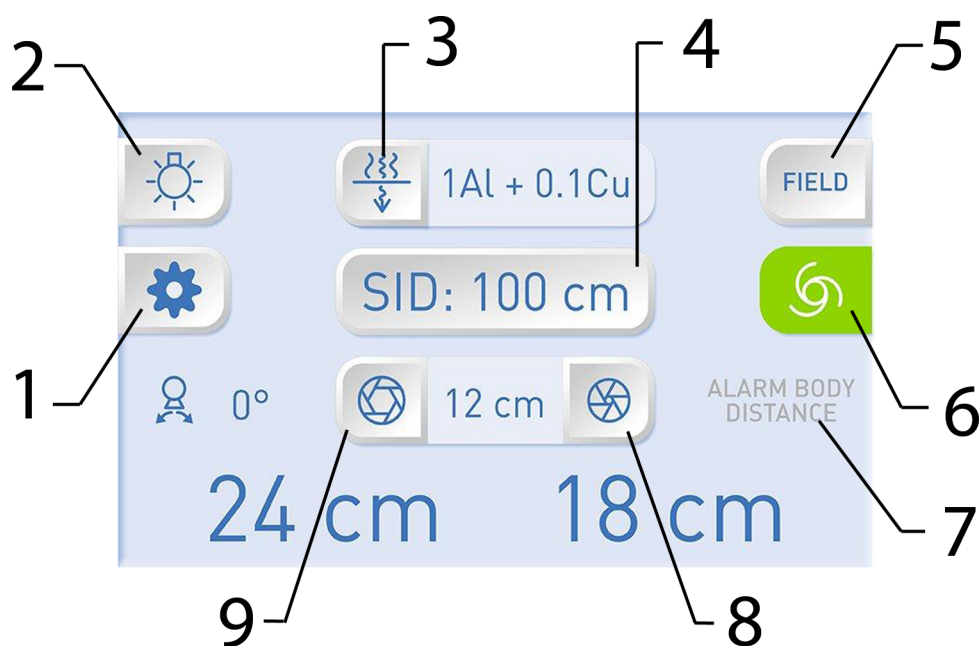


- 1 - System configuration
- 2 - ON/OFF
- 3 - Filter change
- 4 - SID
- 5 - Pre-set Format
- 6 - Collimator Status
- 7 - Proximity Sensor
- 8 - Iris closed
- 9 - Iris opened

Electrical connection


MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

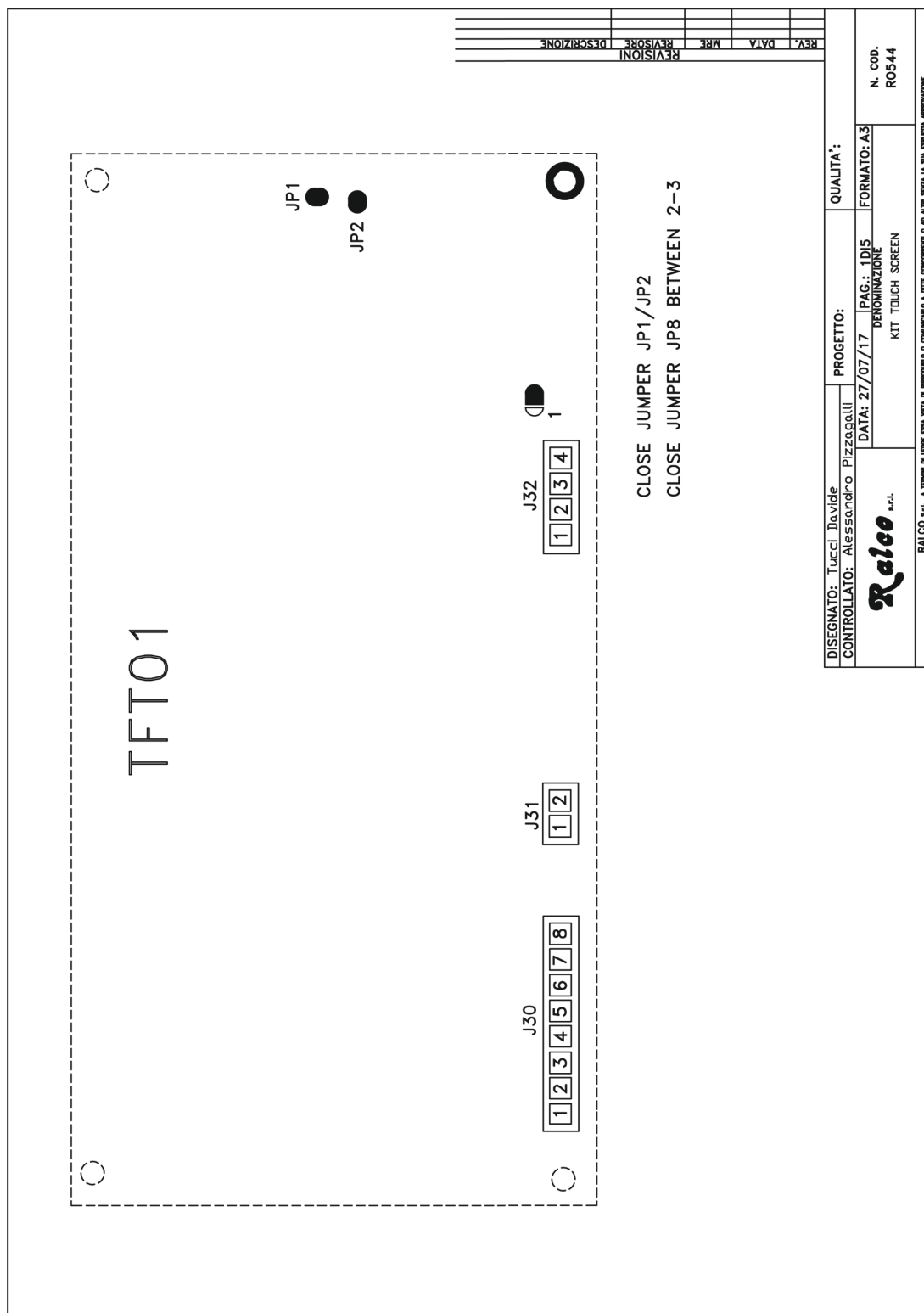


MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

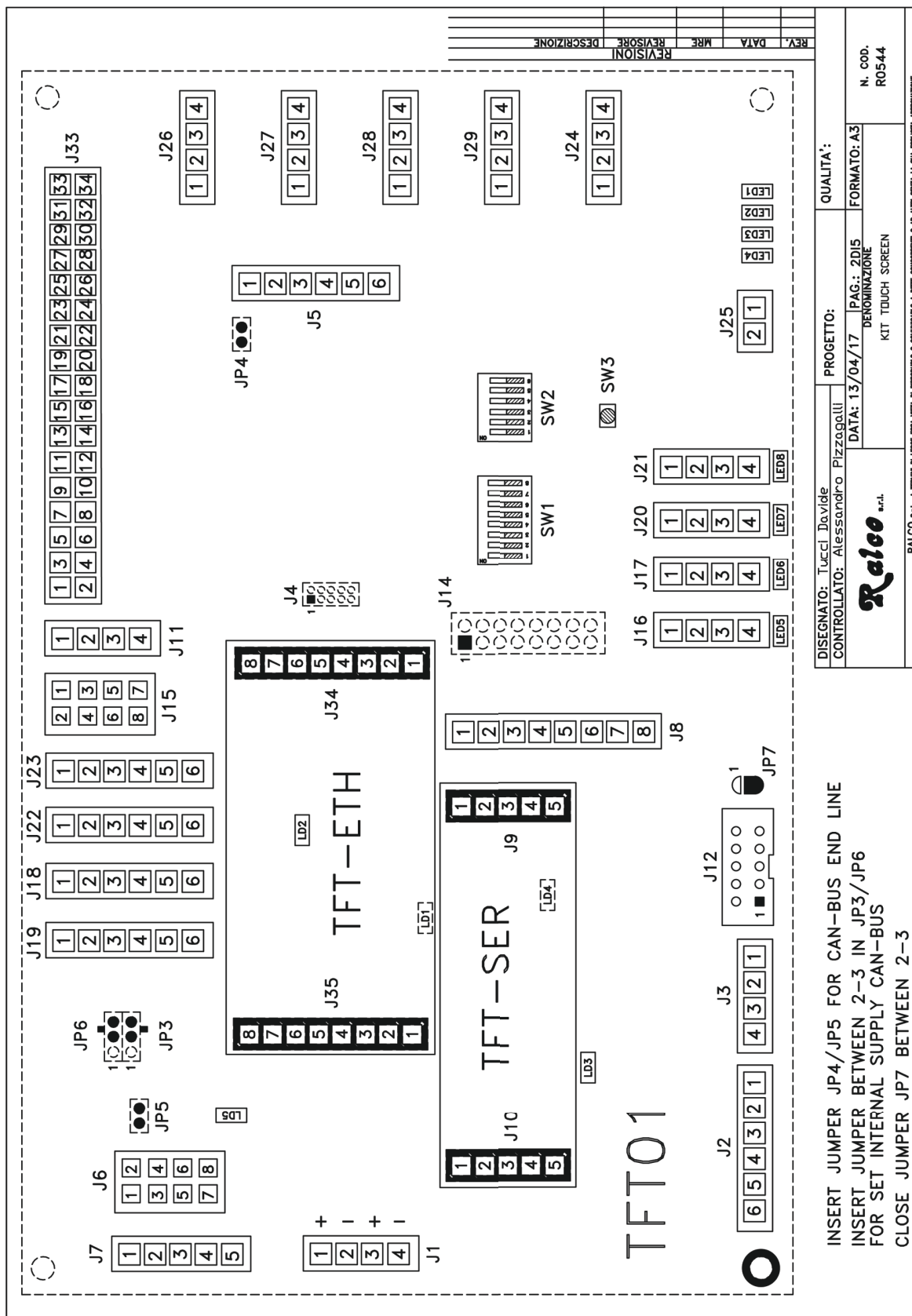


MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

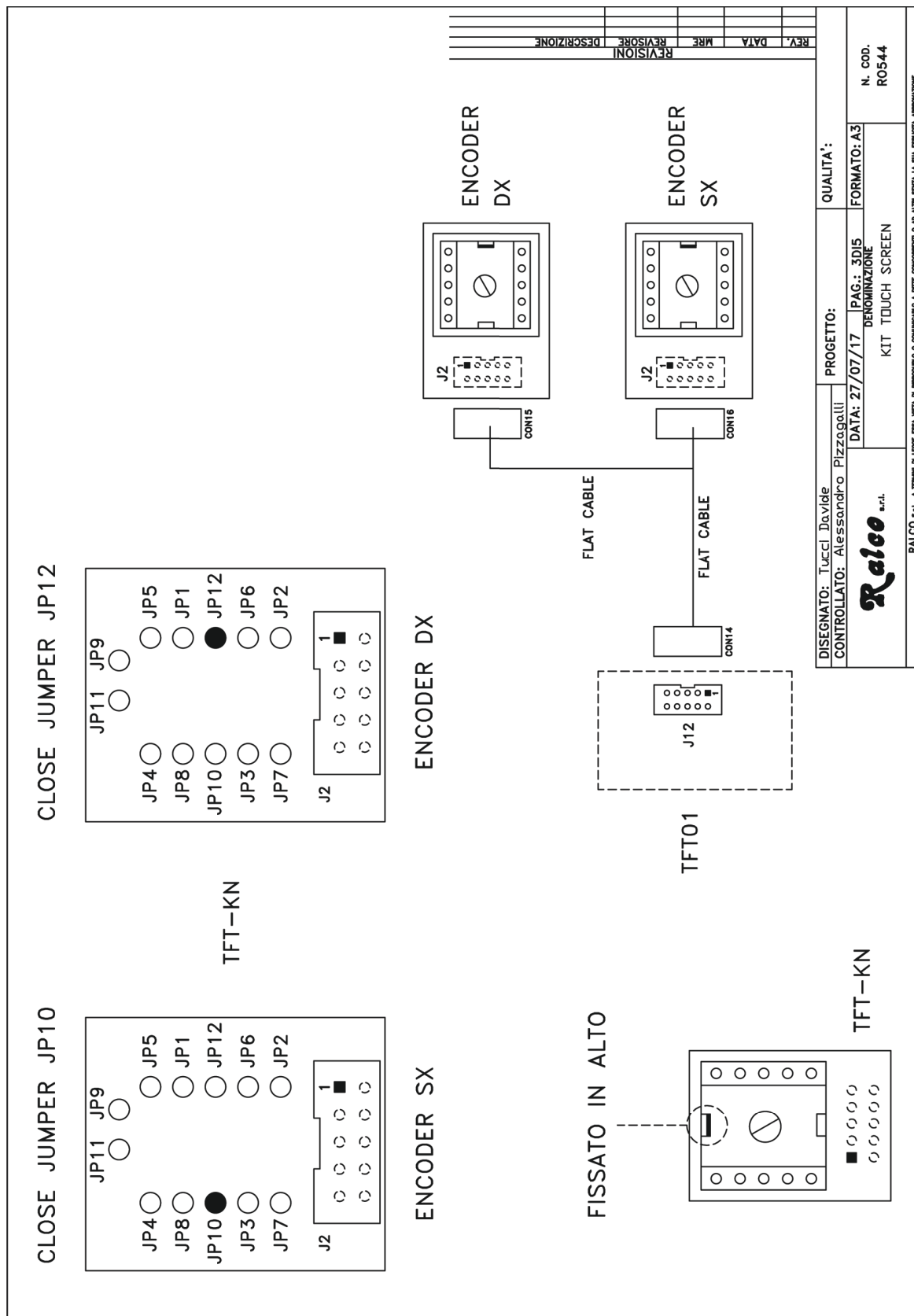

MTR 221 ACS - MTR 221 ACS DHHS - Optional Items

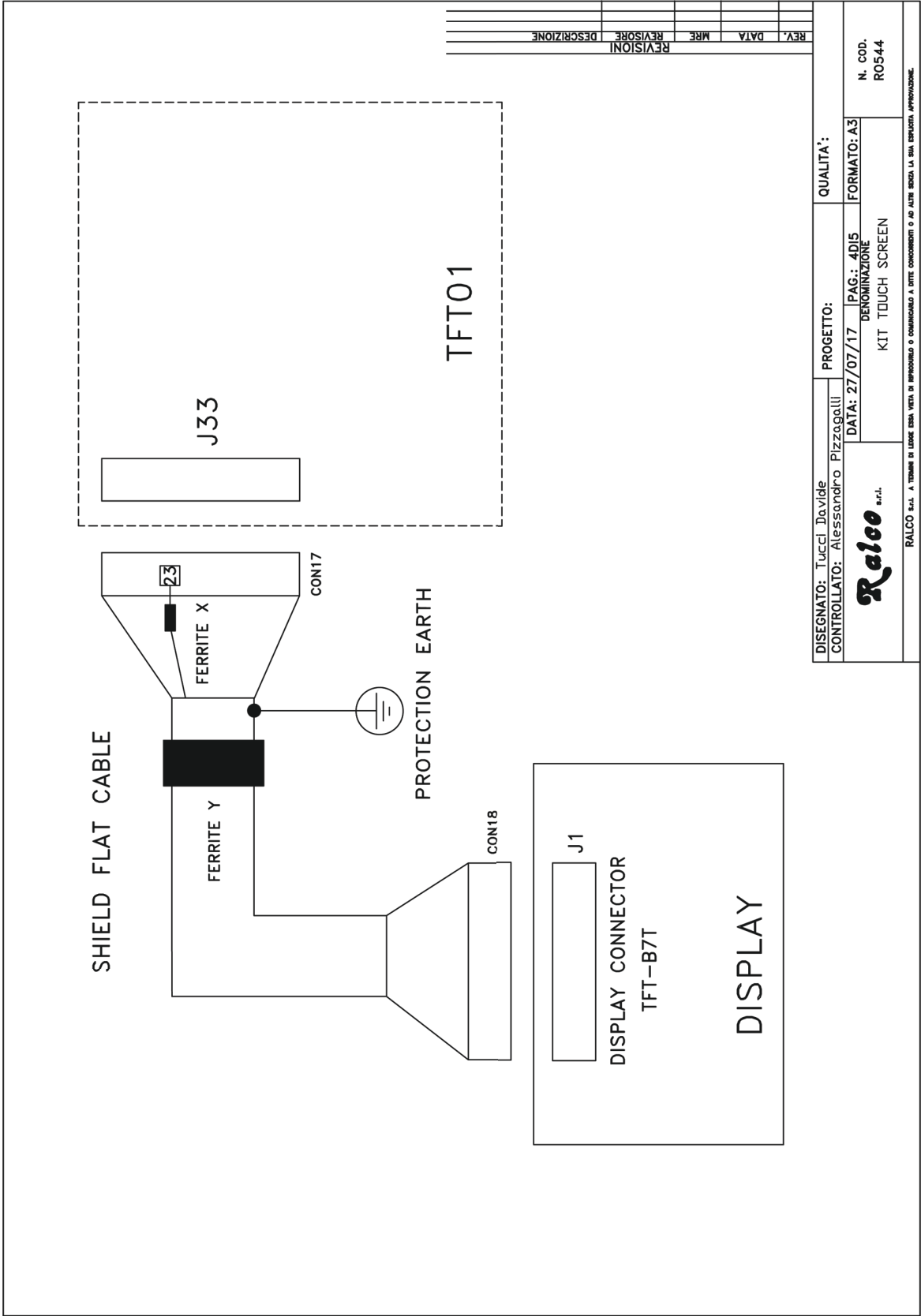

MTR 221 ACS - MTR 221 ACS DHHS
MANUALE ISTRUZIONI / INSTRUCTION MANUAL

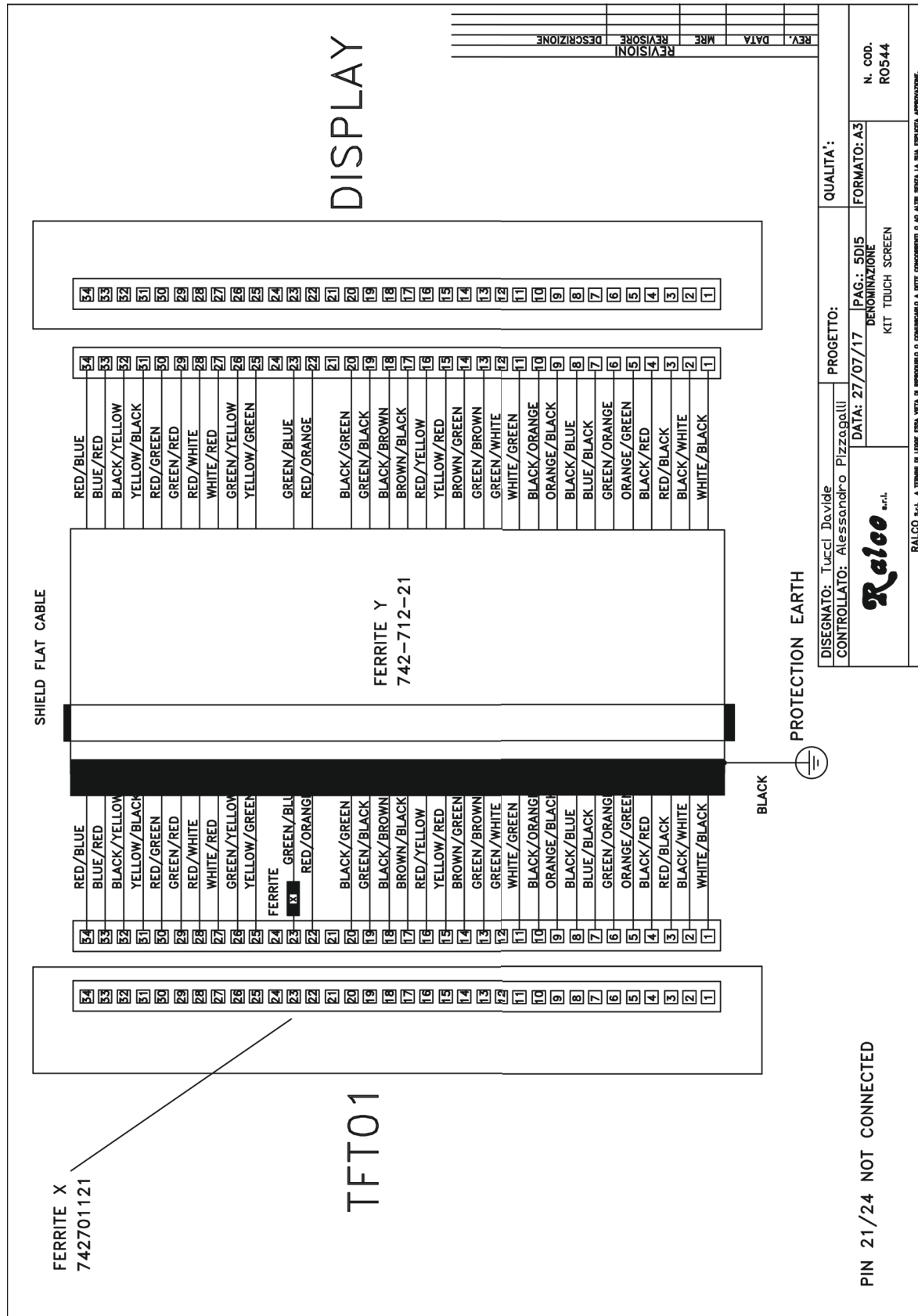

EMC COMPLIANCE
Electromagnetic Emissions

The R 221 ACS - R 221 ACS DHHS collimator is suitable for use in the specified electromagnetic environment. The purchaser or user of the R 221 ACS - R 221 ACS DHHS should assure that it is used in an electromagnetic environment as described below:

Emissions Test	Compliance	Electromagnetic Environment- Guidance
RF emissions CISPR 11	Group 1	The Collimator R 221 ACS - R 221 ACS DHHS needs special precautions regarding EMC and needs to be installed and put into service according to the EMC report. Portable and mobile RF communications equipment can affect the collimator R 221 ACS - R 221 ACS DHHS.
RF emissions CISPR 11	Class [A]	This R 221 ACS - R 221 ACS DHHS is suitable for use in all establishments other than domestic and those directly connected to the low voltage power supply network which supplies buildings used for domestic purposes.
Harmonic distortion IEC 61000-3-2	[Not applicable]	
Limitation of Voltage Fluctuation and Flicker IEC 61000-3-3	[Not applicable]	This R 221 ACS - R 221 ACS DHHS is not suitable to be connected to other equipments.

Electromagnetic Immunity for All Equipment and Systems

Collimator R 221 ACS - R 221 ACS DHHS is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
Electrostatic discharge (ESD) IEC 61000-4-2	8 kV contact 15 kV air	EN 60601-1-2 test level	Hospital

Electromagnetic Immunity

The R 221 ACS - R 221 ACS DHHS collimator is intended for use in the electromagnetic environment specified below. The customer or the user of the collimator should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
			Portable and mobile RF communications equipment should be used no closer to any part of the R 221 ACS - R 221 ACS DHHS, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended Separation Distance
Radiated RF IEC 61000-4-3	3 V 80 MHz to 2,5 GHz	3 V/m	$d=1.2 \sqrt{P}$ 80 MHz to 800MHz $d=2.3 \sqrt{P}$ 800 MHz to 2.5GHz
			Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).
			Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:



Recommended Separation Distances for Non-Life Supporting Equipment

Collimator R 221 ACS - R 221 ACS DHHS is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the collimator can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the collimator as recommended below, according to the maximum output power of the communications equipment.

Rated Maximum Output Power of Transmitter W	Separation Distance According to Frequency of Transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \times \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \times \sqrt{P}$	800 MHz to 2,5 GHz $d = 2.3 \times \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

RO 556 Measuring Tape Removal

This option excludes the tape measure presence inside the collimator.

RO 570 Shutter position verified by encoders

Supplementary safety device to ensure correct positioning of shutters. It consists of two encoders mounted on the collimator.


RO 582 Metal Fixed Mounting Flange

Metal fixed mounting flange:

- 16 mm thickness
- 136 mm diameter

Please refer to section **RO 492 Adjustable Top-Cover Bracket** in this chapter of the Instruction Manual for the correct flange/ collimator mounting instructions.

This flange is **ONLY** available with **RO 492**.

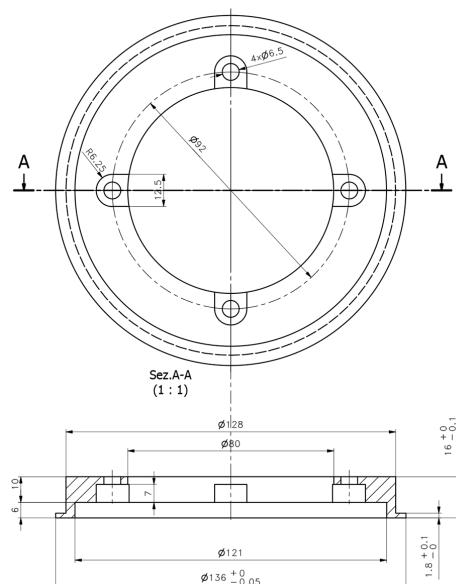

RO 587/1 Two lasers forming a single line at 1-meter SID: Class 1

The collimator has two lasers which serve for the optical definition of the prefixed SID. The lasers are mounted behind the front panel and they are classified as Class 1 (1 m W - wavelength = 645 nm, +/- 10 nm). Two laser lines must measure 60 cm (+/-1) at 1 m (+/- 0.5 cm) and projected at 10 cm (+/-1) from the centre of the light/X-ray field. The projection of a single line signifies that the two lines overlap and consequently the lasers are correctly focussed at the set SID, which can be adjusted from 90 to 200 cm. The projection of two laser lines signifies the SID value has not been entered correctly. See fig.

Laser Line.



CAUTION: CLASS I LASER SYSTEM DO NOT STARE INTO THE BEAM.

Laser Adjustment

- Remove part of the cover to access the point of adjustment, see Chapter- **COVER REMOVAL**.
- The line is to fall on a perpendicular cross-line on the plastic anti-dust panel near the collimator controls, see **Fig. Laser Line**.
- Adjust the position of the line by rotating or moving the base of the laser system.
 - To rotate the laser system, loosen the Allen screw **A**, see **Fig. Laser Adjustment**.
 - Tighten the Allen screw when the laser beam falls on or is parallel to the bisector line drawn on the anti-dust panel.



DO NOT APPLY EXCESSIVE FORCE TO THE SCREW. THE LASER SHELL IS IN PLASTIC AND EXCESSIVE PRESSURE COULD CRACK THE PLASTIC AND POSSIBLY SHORT CIRCUIT THE LASER.

- Adjust the length of the laser line by screwing or unscrewing the screws **B**. See **Fig. Laser Adjustment**.

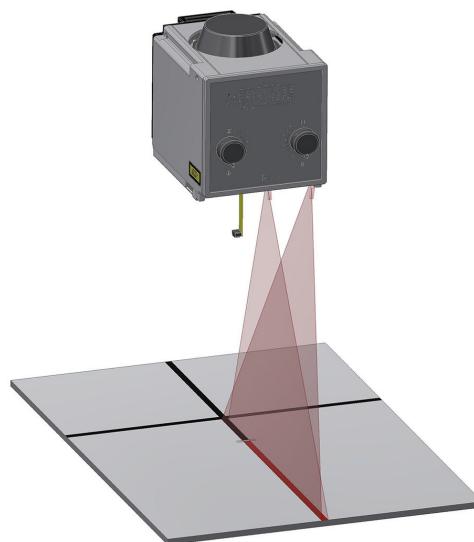


Fig. Laser Line

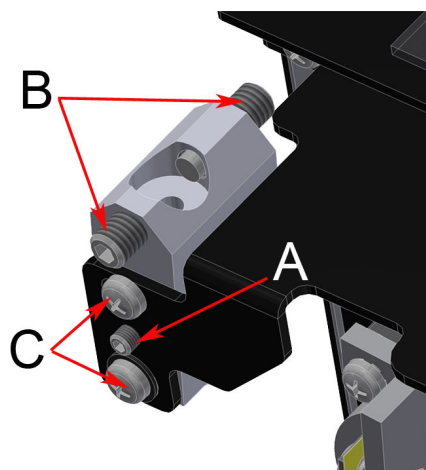


Fig. Laser Adjustment

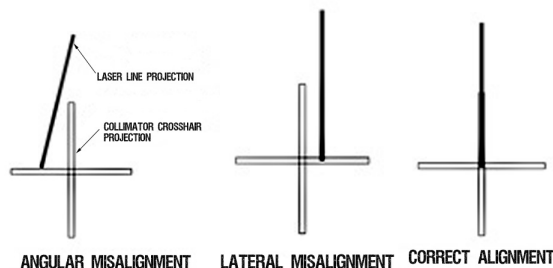


Fig. Laser Alignment


- Shift the laser system by loosening the two **C** screws holding the laser system base to the beam limiting device front plate.
 - Move the base until the laser beam falls over the perpendicular bisector line on the anti-dust panel, see **Fig. Laser Adjustment**.
 - Tighten the screws **C**.

Substitution

- Disconnect supply.
- Remove the cover, see Chapter- **COVER REMOVAL**.
- Unscrew the fixing Allen screws **A**, see **Fig. Laser Adjustment**.
- Disconnect the timer cables from the terminal board - white 0 V, red 5 V.
- Remove the laser and substitute with an identical item.
- Tighten the screws.
- Check the laser alignment, see Chapter- **ADJUSTMENTS**.
- Remount the cover, see Chapter- **COVER REMOVAL**.

Classification EN 60825-1 par. 1 -5: Class 1 laser product => Red laser line.

LABEL 3A

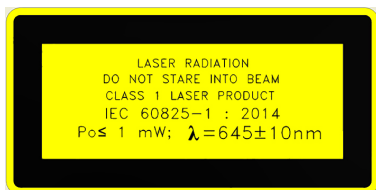

LABEL 4



LABEL 5



LABEL 6


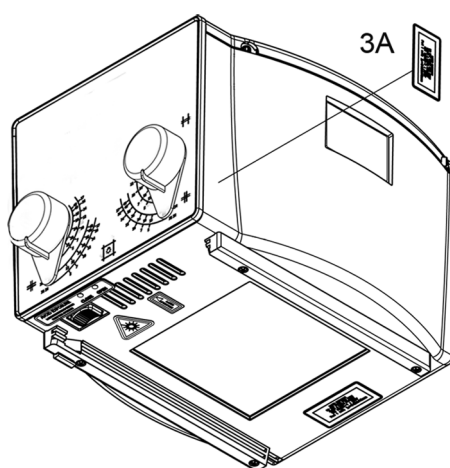


Fig. Standard and DHHS Model Labels, LED and Laser (Red Laser Line, class 1)


RO 587/2 Two lasers (one mounted externally) forming a crosshair to center the patient to the collimator: Class 1

Second laser which serves to center the patient using the cross projection.

The collimator lasers are classified as Class 1 (1 m W - wavelength = 645 nm, +/- 10 nm) and are used for collimator/image receptor center alignment.

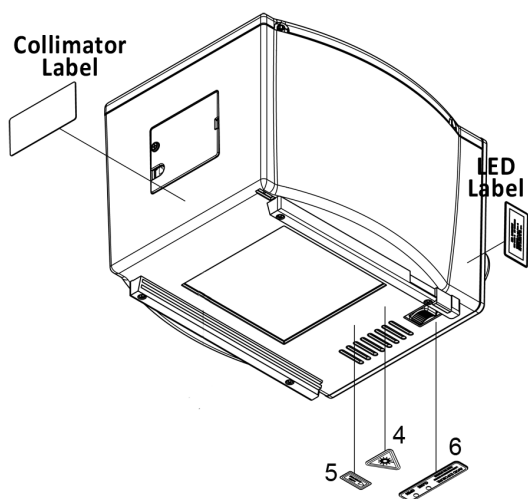

CAUTION: CLASS 1 LASER SYSTEM DO NOT STARE INTO THE BEAM.

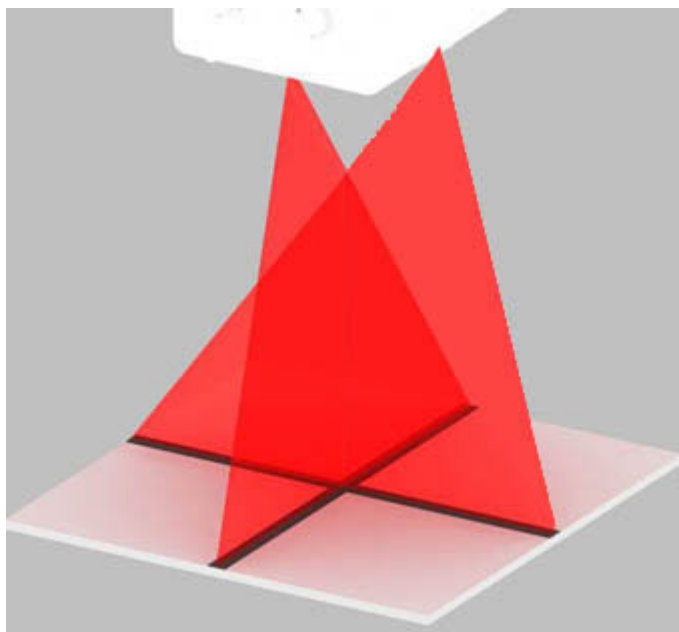

CALIBRATION CONTROL

- Draw a cross (two lines at 90°) on a sheet of paper and use this as reference.
- Switch the light ON and set the sheet of paper at 1 m from the focus.
- Make sure the cross on the paper coincides exactly with the two lines silk-screened on the plastic window.
- Examine the laser projections on the paper: the laser is correctly calibrated when the projections coincide with the lines drawn on the paper, otherwise an adjustment will be necessary.

ADJUSTMENT

Horizontal

- Disconnect the collimator supply.
- Remove the laser cover located on the collimator cover by unscrewing the two screws **A**, see **Fig. Horizontal Laser**.
- To correct horizontal misalignment, loosen the screws **B**, see **Fig. Horizontal Laser** and move the laser support until the projected laser line coincides with the cross on the sheet of paper. Tighten screws **B**.

Vertical

- Disconnect power supply.
- Remove the knobs and front panel, see Chapter- **COVER REMOVAL**.
- To correct vertical misalignment, loosen the Allen screw **A**, see **Fig. Vertical Laser** on the laser support; rotate the laser and align the laser line over the cross on the sheet of paper.
- To move laterally the laser loosen the screws **B** and shift the laser support until the projected laser line coincides with the cross on the sheet of paper.
- Tighten **B** screw s.

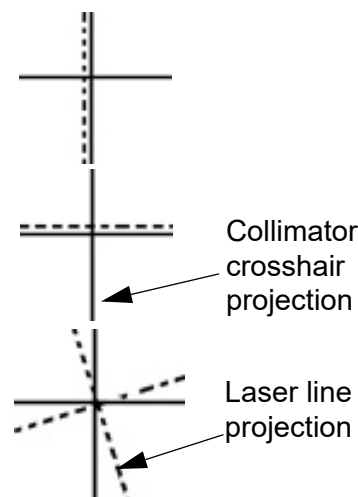


Fig. Laser Projections

WARNINGS



DO NOT APPLY EXCESSIVE FORCE TO THE SCREW. THE LASER SHELL IS IN PLASTIC AND EXCESSIVE PRESSURE COULD CRACK THE PLASTIC AND POSSIBLY SHORT-CIRCUIT THE LASER.

SUBSTITUTION

The Laser Substitution Kit is composed of the following components: the laser (A), the extension cable (B) and the laser label (C) pictured below in **Fig. Laser Substitution Kit**. This Kit is designed to substitute any laser Ralco provides.

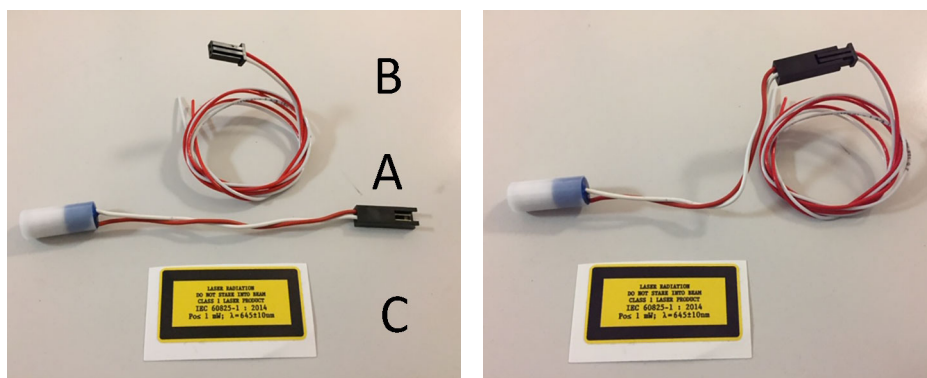


Fig. Laser Substitution Kit

Prior to replacing the laser, confirm which version of laser is installed on your collimator which will ensure the substitution is performed correctly.

- The previous laser version is connected to the board via 2 cables (red and white), see **Fig. Previous Laser Version** below.

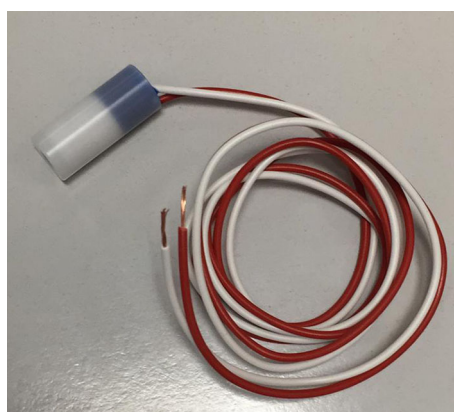


Fig. Previous Laser Version

- The new laser is connected to the board via the connector, see **Fig. New Laser Version** below.



Fig. New Laser Version

Replacing the Previous Laser Version

To replace the previous version of laser, all components of the Laser Substitution Kit (RS 2589) (A, B and C) must be used.

To substitute the laser, proceed as follows:

- Disconnect the collimator supply.
- Remove the cover, see Chapter **COVER REMOVAL** in the Instruction Manual for your specific model.
- Prior to disconnecting the laser cables from the board, identify the cables and their position on the terminal board, see Chapter **INSTALLATION**, paragraph **Wiring Diagram** in the Instruction Manual for your specific model.
- Carefully remove the laser, the extension cable and the label from their packaging.
- Ensure the extension cable (**B**) is firmly connected to the laser (**A**), see **Fig. Laser Substitution Kit** above.
- Substitute the laser with the identical item using component (**A**) of the Laser Substitution Kit, see **Fig. Laser Substitution Kit** above.
- Adjust the length of the extension (**B**) cable by cutting the 2 cables (red/white).
- Connect both cables to the board.
- Apply the new laser label (**C**) to the collimator cover, see Chapter **SPARE PARTS** in the Instruction Manual for your specific mode.
- Verify the Laser alignment, see Chapter **ADJUSTMENT** or **OPTIONAL ITEMS** in the Instruction Manual for your specific model.

Replacing the New Laser Version


To replace the new laser version, only part **A** and **C** of the Kit must be used.

To substitute the laser, proceed as follows:

- Disconnect the collimator supply.
- Remove the cover, see Chapter **COVER REMOVAL** in the Instruction Manual for your specific model.
- Detach the connector of the faulty laser from the collimator wiring.
- Carefully remove the laser, the extension cable and the label from their packaging.
- Substitute the laser with the identical item using component (**A**) of the Laser Substitution Kit, see **Fig. Laser Substitution Kit** above.
- Connect the new laser to the collimator wiring.
- Apply the new laser label (**C**) to the collimator cover, see Chapter **SPARE PARTS** in the Instruction Manual for your specific mode.
- Verify the Laser alignment, see Chapter **ADJUSTMENT** or **OPTIONAL ITEM** in the Instruction Manual for your specific model.

Classification EN 60825-1 par. 1 -5: Class 1 laser product => Red laser line.

LABEL 3A

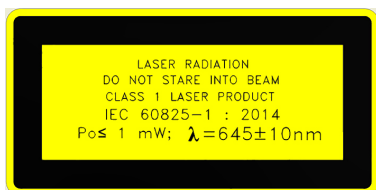

LABEL 4



LABEL 5



LABEL 6

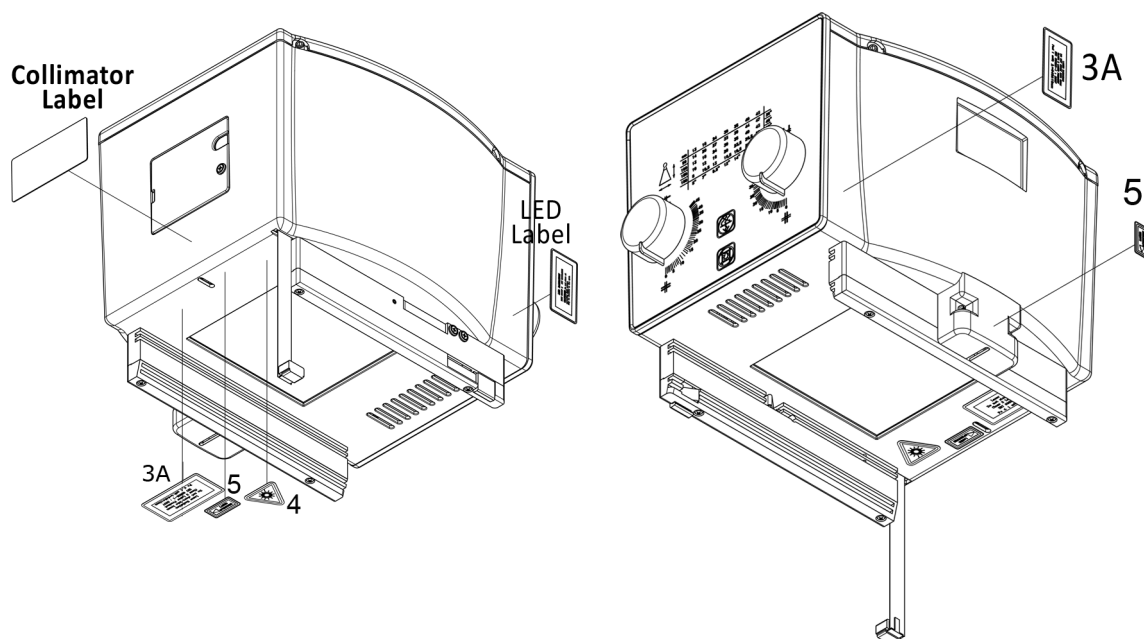


Fig. Standard Model Labels, LED and Laser (Red Laser Line, class 1)

RO 597 Asymmetric longitudinal and transversal shutters

This option has been designed to control the shutters independently (Stitching mode).

RO 609 Self-Centering Top-Cover Bracket with Zero Position Output Signal

Zero position (+/-3°) output signal is provided by a microswitch with signal being managed by the customer.

Installation

WARNINGS



THE COLLIMATOR MUST BE INSTALLED TO THE X-RAY TUBE THROUGH A MOUNTING FLANGE. RALCO PROVIDES VARIOUS FLANGE OPTIONS WHICH MAY NOT BE INTERCHANGEABLE. ONLY FLANGES PROVIDED WITH THE COLLIMATOR MAY BE UTILIZED. THE END-USER MAY INSTALL THEIR OWN FLANGE, HOWEVER RALCO CANNOT GUARANTEE COMPATIBILITY. ANY PREEXISTING FLANGE ON THE END-USER SYSTEM MUST NOT BE USED.



WHEN A FLANGE IS PROVIDED WITH THE COLLIMATOR BEARING A MATCHING SERIAL NUMBER, ENSURE THEY ALWAYS REMAIN COUPLED. IT IS MANDATORY THEY REMAIN TOGETHER AND THE CORRECT FLANGE PART NUMBER IS USED.



THE CONTENTS OF THE INSTRUCTIONS BELOW SHOULD BE STRICTLY ADHERED TO. RALCO IS NOT LIABLE FOR ANY PROPERTY DAMAGE OR RESULTING HARM IF NON-RALCO COMPONENTS OR NON-COMPATIBLE RALCO COMPONENTS ARE USED DURING THE INSTALLATION PROCESS.

TUBE COMPATIBILITY

1. Using the dimensions in **Fig. Tube Compatibility** below, ensure the near port shutters of the collimator are placed in the X-Ray tube port without interference.
2. The distance between the X-Ray tube focus and the flange mounting plane (collimator upper plate) must be: 80 mm (3.14"), tolerance +/- 1 mm (0.04 ").

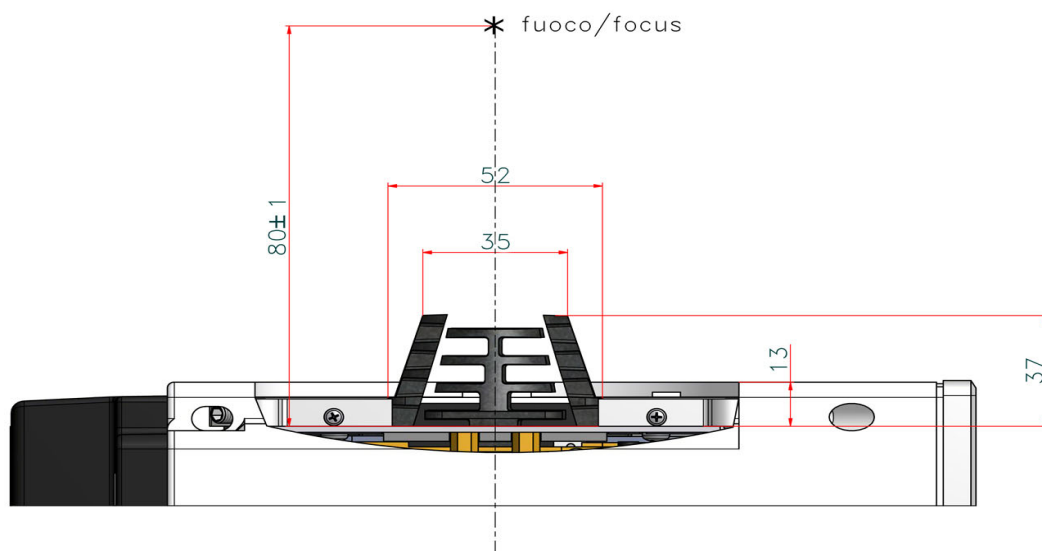


Fig. Tube Compatibility

3. Carefully remove the collimator and the mounting flange from their packaging.
4. Use the X-Ray tube housing datasheet to determine the distance (A) from the focal spot to the X-Ray tube port, see **Fig. Collimator Installation**.
5. Subtract the resulting distance from the source flange distance (B) and determine the number of spacers (1.5 mm) which, combined with the thickness of the mounting flange, will make up the difference (C). Allowable tolerance is 1 mm. (0.04"), see **Fig. Collimator Installation**.
6. Once the mounting plane distance has been confirmed, continue with the mounting flange installation to the X-Ray tube.

**The flange fixing screws and the spacers of the previous flange may be reused if the flange thickness is the same.*

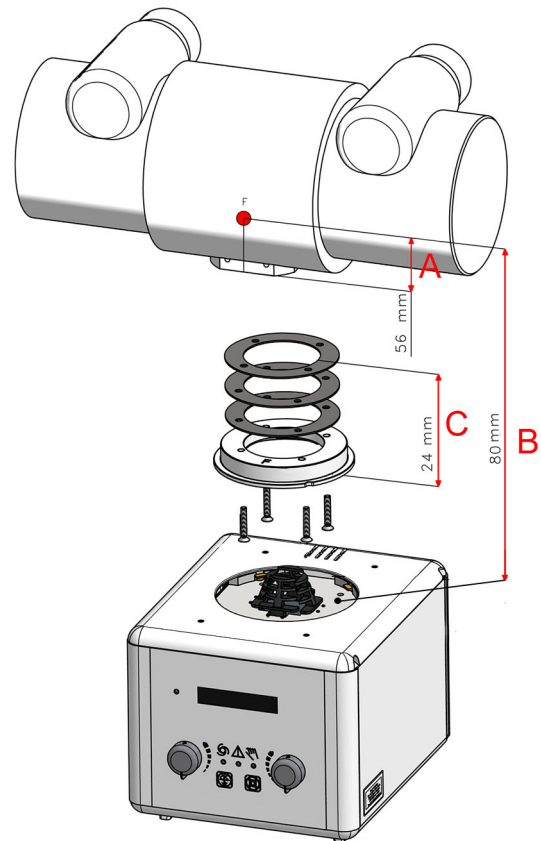


Fig. Collimator Installation

**illustrative purpose only*

Mounting the Flange to the X-ray Tube

NOTE



THE FOLLOWING MOUNTING INSTRUCTIONS ARE ONLY APPLICABLE FOR COMPATIBLE FLANGES WITH THIS OPTIONAL ITEM. IF YOU ARE UNSURE THE MOUNTING FLANGE YOUR COLLIMATOR IS EQUIPPED WITH IS COMPATIBLE, PLEASE CONSULT YOUR PERSONALIZATION PAGE PROVIDED WITH THIS MANUAL TO LOCATE THE FLANGE PART NUMBER (RO REFERENCE). FOLLOW THE MOUNTING INSTRUCTIONS SET FORTH UNDER THE SPECIFIC RO REFERENCE IN THIS MANUAL.

WARNING



FLANGES MAY BE PROVIDED BY RALCO OR BY THE SYSTEM MANUFACTURER. FLANGES MAY NOT BE INTERCHANGEABLE. ONLY THE FLANGE PROVIDED WITH THE COLLIMATOR BEING INSTALLED WITH A SPECIFIC PART CODE MAY BE UTILIZED. ANY PRE-EXISTING FLANGES MAY NOT TO BE USED. IF THERE ARE ANY QUESTIONS REGARDING COMPATIBILITY, PLEASE CONTACT RALCO.

CAUTION



RALCO CANNOT GUARANTEE COMPLIANCE WITH RADIATION STANDARDS CONCERNING SAFETY IF THIS CONTROL HAS BEEN OMITTED.

1. Place the flange on the X-Ray tube port, see **Fig. Flange Installation**.
2. Mount the mounting flange and spacers (optional) to the X-Ray tube port using 4 screws.**

****Please ensure no conflicting information nor dangerous conditions exist due to adhering to these instructions or those provided by the X-Ray tube manufacturer. When in doubt please contact X-Ray tube manufacturer and/or Ralco.**

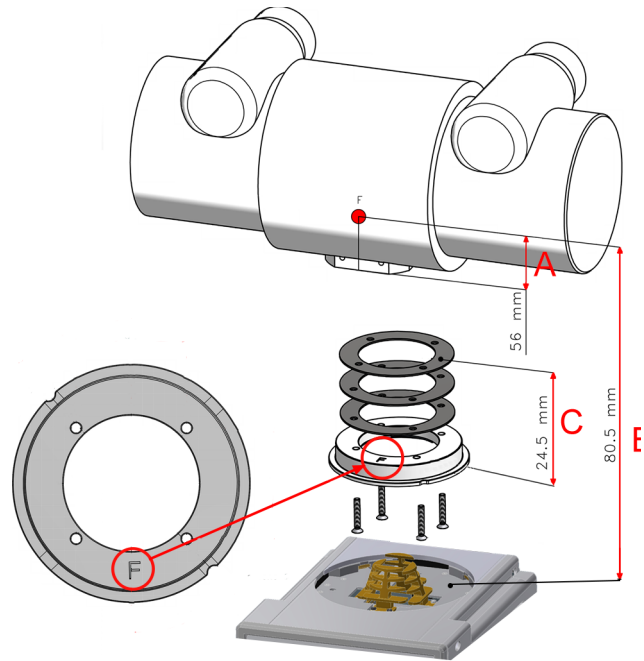


Fig. Flange Installation
**illustrative purpose only*

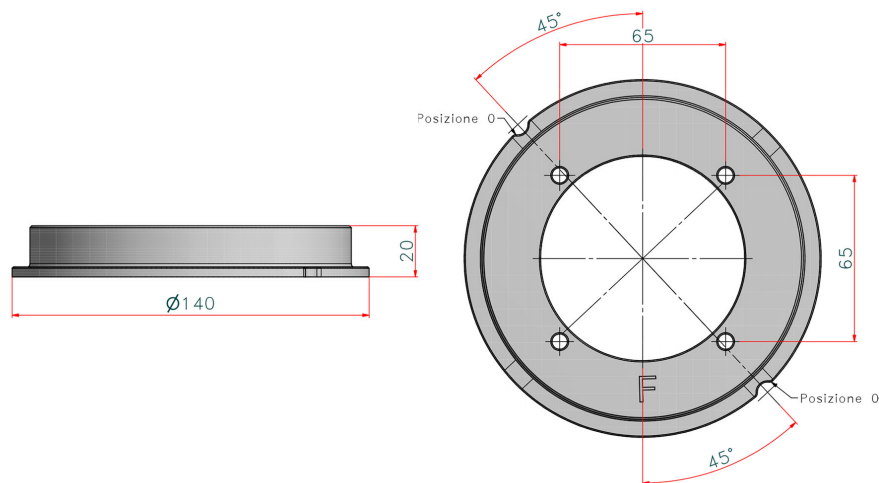


Fig. Mounting Flange
**illustrative purpose only*

CAUTION



THE FLANGE MUST BE PLACED WITH THE LETTER "F" FACING TOWARD THE X-RAY TUBE PORT. INCORRECT POSITION MAY CAUSE THE COLLIMATOR AND FLANGE TO MALFUNCTION.



ENSURE THE SCREW HEAD IS INDEED CORRECT FOR THE FLANGE SELECTED. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE ALL SAFETY MEASURES ARE IMPLEMENTED TO ENSURE THE SCREWS ARE OPTIMALLY TIGHTENED INCLUDING THE USE OF APPROPRIATE LOCKTITE.



TIGHTEN THE 4 SCREWS TO THE X-RAY TUBE HEAD SECURELY, STRICTLY ACCORDING TO THE INSTRUCTIONS OF THE X-RAY TUBE MANUFACTURER. DO NOT EXCEED 0.45 NM OF FORCE.

Alignment of X-Ray Tube Focus and Collimator

Ralco guarantees the correct collimator functionality, format compliance and light/X-Ray field alignment only if the mounting flange and the collimator have been installed exactly in the centre of the X-Ray beam.

All Ralco collimators are aligned on our test bench utilizing specific references/values for our X-Ray tube focus, detector and Source to Image Detector Distance (SID). The customer must know and verify all known variables which may influence the X-Ray tube focus and collimator alignment. These may include, the X-Ray tube focus position tolerance, distance from X-Ray tube focus to collimator mounting plane, or the SID.


ALIGNMENT DEVICE

This device is used to ensure the collimator mounting flange is correctly aligned to the X-ray beam. The X-ray tube manufacturer provides a tolerance for the placement of X-ray tube focus.

Ralco recommends to use the Focal Alignment Device (jig) to ensure the correct flange alignment with the centre of the X-Ray beam, see **Fig. Focal Alignment Device**. By making an exposure, it is possible to verify the perpendicularity and concentricity using fixed references on the X-Ray image.

Once the mounting flange is aligned the collimator light/X-ray field should also be aligned (within specific tolerances).

Please consult the technical specifications of your X-Ray tube to find the maximum tolerance for the position of the focus. Should the use of an alignment device not be possible, Ralco collimators allow for the regulation of the light field.

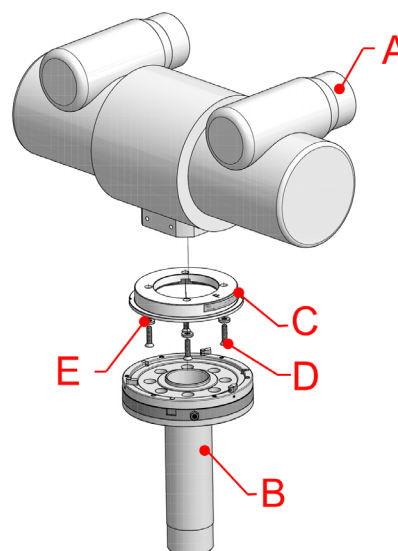

A - X-Ray Tube, B - Focal Adjustment Device
C - Mounting Flange, D - Screw, E - Washer

Fig. Focal Alignment Device

Mounting the Collimator to the Resin Flange (RO 610 or RO 619)

NOTE



THE MOUNTING FLANGE PROVIDED (IF PURCHASED) WITH THE COLLIMATOR IS SUBJECTED TO TESTING PURSUANT TO ALL APPLICABLE STANDARDS.



MOUNTING BRACKET TABS CONFORM TO EN60601.

WARNINGS



PURSUANT TO APPLICABLE STANDARDS, RALCO HAS TESTED THE COLLIMATOR AND FLANGE APPLYING STATIC LOADS. RALCO IS NOT IN A POSITION TO KNOW THE DYNAMIC FORCES OF ALL END-USER SYSTEMS. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE DYNAMIC FORCES OF THE SYSTEM DO NOT CREATE A DANGEROUS CONDITION.



IT IS THE RESPONSIBILITY OF THE SYSTEM MANUFACTURER TO ENSURE AND MITIGATE ANY DANGEROUS CONDITIONS WHICH MAY OCCUR DUE TO THE DYNAMIC FORCES CREATED BY THE SYSTEM. THE END-USER MUST PERFORM A SYSTEMATIC AND STRUCTURAL ANALYSIS DURING THE INSTALLATION AND USUAL MAINTENANCE.



SHOULD ANY DAMAGE TO THE COLLIMATOR OR FLANGE OCCUR A RISK ANALYSIS AND DAMAGE ASSESSMENT NEEDS TO BE CONDUCTED IMMEDIATELY. CONTACT RALCO IMMEDIATELY SHOULD THIS OCCUR. RALCO IS NOT LIABLE FOR RESULTING PROPERTY DAMAGE AND/OR HARM DUE TO AN UNREPORTED INCIDENT.



RALCO HAS DESIGNED AND TESTED THE COLLIMATOR FOR A LIFETIME OF 10 YEARS. AFTER THIS TIME PERIOD, IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE THE PROPER FUNCTIONING OF THE COLLIMATOR AND FLANGE. LIABILITY FOR ANY DANGEROUS CONDITIONS WHICH MAY BE PRESENT AFTER THE 10YEAR LIFETIME OF THE COLLIMATOR AND FLANGE RESTS WITH THE END-USER.



TO ENSURE THE SAFETY OF THE COLLIMATOR AND FLANGE AFTER 10 YEARS OF USE, RALCO HAS INSTITUTED A PROGRAM TO ASSESS THE SAFETY OF THE COLLIMATOR AND FLANGE. AFTER APPLYING A CHECK LIST OF QUALITY CONTROLS AND REFURBISHMENT ACTIVITIES (AT END-USER EXPENSE), RALCO MAY CERTIFY THE COLLIMATOR AND FLANGE FOR ADDITIONAL YEARS OF USE.

The mounting bracket has 2 tabs with springs in positions 2 and 4, while the 2 tabs in position 1 and 3 cover the surface of the outer ring of the flange, see **Fig. Mounting Bracket**.

1. Prepare the collimator to be installed by unscrewing the 4 hexagonal socket screws until the four tabs are completely withdrawn from the collimator top mounting plane, see **Fig. 4 Mounting Screws**.
2. Ensure the microswitch is completely retracted, see **Fig. Microswitch**.



Fig. Microswitch

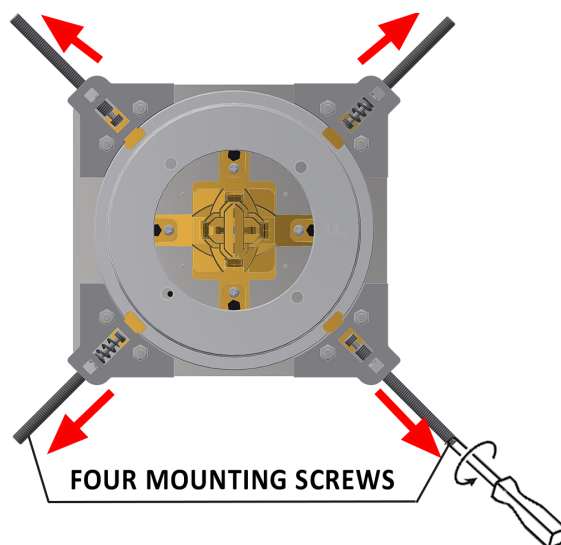


Fig. 4 Mounting Screws

CAUTION



WHEN UNSCREWING THE HEXAGONAL SOCKET SCREW WHICH CONTROL THE TABS,
DO NOT USE FORCE EXCEEDING 0,45 NM.
UNSCREW WITH CARE SO AS NOT TO DAMAGE THE HEXAGONAL SCREW HEAD AND TABS.

3. With the tabs fully retracted, tabs in position 1 and 3 which overlap the outer ring must be tightened completely up to the end stroke with 0.45Nm of torque (at least 7 turns), see **Fig. Mounting Bracket**.
4. With the tabs fully retracted, tabs in position 2 and 4 which overlap the outer ring must be tightened exactly 5 ½ turns, see **Fig. Mounting Bracket**.
5. If you are not able to respect the turns something is incorrect, repeat above instructions, if issues persist, please contact Ralco.
6. Ensure the microswitch is completely engaged by tightening the set screw, see **Fig. Microswitch**.
7. The collimator tabs glide on the flange outer ring and the collimator rotates towards the X-ray tube axis. If, however:
 - The release force from the position 0° and the collimator rotation is too low, the two tabs of the tab 2 and 4 need to be tightened ½ turn.
 - The release force from the position 0° and the collimator rotation is too high, the two tabs of the tab 2 and 4 need to be loosened ½ turn.

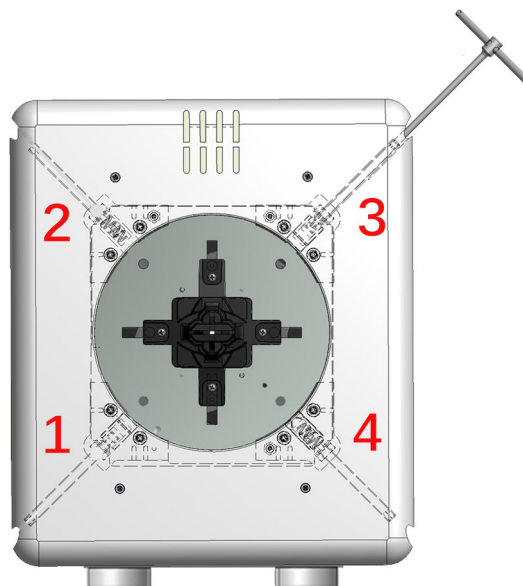


Fig. Mounting Bracket
*illustrative purpose only

Mounting the Collimator to the Metal Flange (RO 611)

NOTE



THE MOUNTING FLANGE PROVIDED (IF PURCHASED) WITH THE COLLIMATOR IS SUBJECTED TO TESTING PURSUANT TO ALL APPLICABLE STANDARDS.



MOUNTING BRACKET TABS CONFORM TO EN60601.

WARNINGS



PURSUANT TO APPLICABLE STANDARDS, RALCO HAS TESTED THE COLLIMATOR AND FLANGE APPLYING STATIC LOADS. RALCO IS NOT IN A POSITION TO KNOW THE DYNAMIC FORCES OF ALL END-USER SYSTEMS. IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE DYNAMIC FORCES OF THE SYSTEM DO NOT CREATE A DANGEROUS CONDITION.

WARNINGS



IT IS THE RESPONSIBILITY OF THE SYSTEM MANUFACTURER TO ENSURE AND MITIGATE ANY DANGEROUS CONDITIONS WHICH MAY OCCUR DUE TO THE DYNAMIC FORCES CREATED BY THE SYSTEM. THE END-USER MUST PERFORM A SYSTEMATIC AND STRUCTURAL ANALYSIS DURING THE INSTALLATION AND USUAL MAINTENANCE.



SHOULD ANY DAMAGE TO THE COLLIMATOR OR FLANGE OCCUR A RISK ANALYSIS AND DAMAGE ASSESSMENT NEEDS TO BE CONDUCTED IMMEDIATELY. CONTACT RALCO IMMEDIATELY SHOULD THIS OCCUR. RALCO IS NOT LIABLE FOR RESULTING PROPERTY DAMAGE AND/OR HARM DUE TO AN UNREPORTED INCIDENT.



RALCO HAS DESIGNED AND TESTED THE COLLIMATOR FOR A LIFETIME OF 10 YEARS. AFTER THIS TIME PERIOD, IT IS THE RESPONSIBILITY OF THE END-USER TO ENSURE THE PROPER FUNCTIONING OF THE COLLIMATOR AND FLANGE. LIABILITY FOR ANY DANGEROUS CONDITIONS WHICH MAY BE PRESENT AFTER THE 10YEAR LIFETIME OF THE COLLIMATOR AND FLANGE RESTS WITH THE END-USER.



TO ENSURE THE SAFETY OF THE COLLIMATOR AND FLANGE AFTER 10 YEARS OF USE, RALCO HAS INSTITUTED A PROGRAM TO ASSESS THE SAFETY OF THE COLLIMATOR AND FLANGE. AFTER APPLYING A CHECK LIST OF QUALITY CONTROLS AND REFURBISHMENT ACTIVITIES (AT END-USER EXPENSE), RALCO MAY CERTIFY THE COLLIMATOR AND FLANGE FOR ADDITIONAL YEARS OF USE.

1. Prepare the collimator to be installed by unscrewing the 4 hexagonal socket screws until the four tabs are completely withdrawn from the collimator top mounting plane, see **Fig. 4 Mounting Screws**.
2. If installing a manual collimator, adjust the collimator shutters to the fully open position using both knobs.

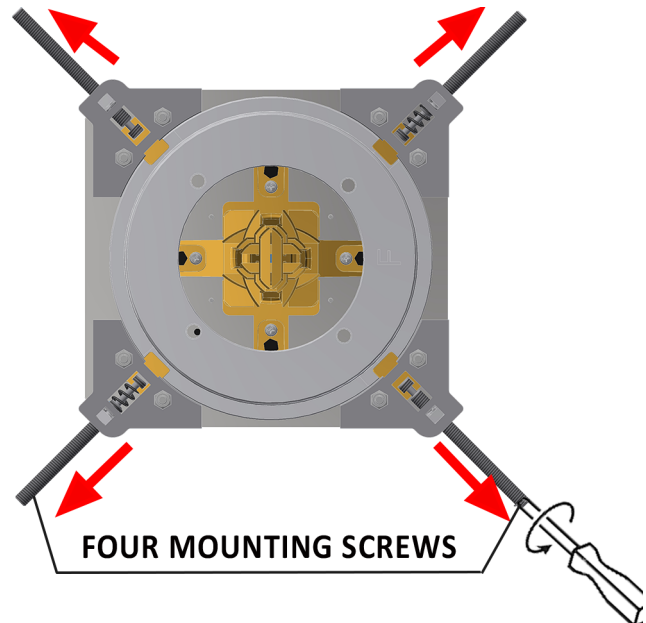


Fig. 4 Mounting Screws

CAUTION



WHEN UNSCREWING THE HEXAGONAL SOCKET SCREW WHICH CONTROL THE TABS, DO NOT USE FORCE EXCEEDING 0,45 NM. UNSCREW WITH CARE SO AS NOT TO DAMAGE THE HEXAGONAL SCREW HEAD AND TABS.

3. With the tabs fully retracted, the hexagonal socket screws of the mounting bracket must be tightening equally up to the end stroke with 0.45 Nm torque (at least 7 turns), see **Fig. Mounting Bracket**.
4. The 4 tabs of the hexagonal socket screws overlap on the flange outer ring in the same manner.
5. The collimator tabs adhere to the flange outer ring. Depending on the optional flange purchased, the collimator may rotate or be fixed (no rotation).
6. Once the collimator is coupled to the flange in the method described above, verify the distance between the collimator housing and the mounting flange is equal in all directions and the collimator face is parallel to the axis of the table. Loosen the screws and adjust as necessary.
7. The collimator should be coupled to the flange firmly. If the collimator is loose, please repeat the above mounting instructions, and if issues persist, please contact Ralco.

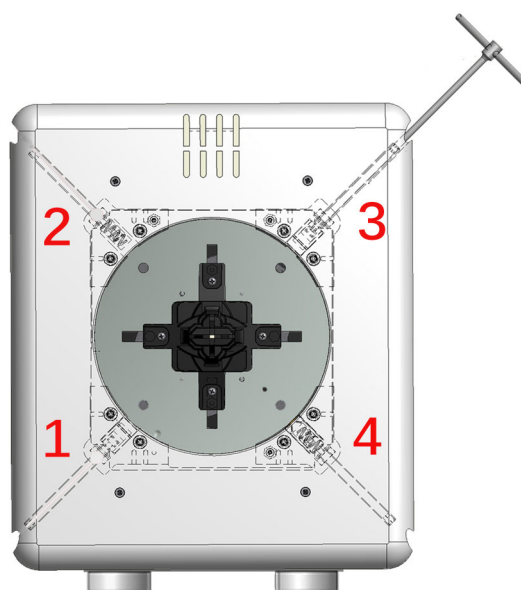


Fig. Mounting Bracket
*illustrative purpose only

Verification of Correct Installation

WARNING



IT IS THE DUTY OF THE INSTALLER TO ENSURE NO RISK OF THE COLLIMATOR FALLING EXISTS. ENS URE THE FOLLOWING, AS IN EACH SCENARIO BELOW SERIOUS RISK OF INJURY AND/OR PROPERTY DAMAGE MAY EXIST DUE TO NON-ADHERENCE.

1. The 4 tabs should overlap the flange outer ring, see **Fig. Correct Overlap**.
2. Ensure the mounting flange is flat against the collimator mounting plane, see **Fig. Correct Overlap**.
3. Ensure the 4 tabs are not in contact with only the mounting flange edge, see **Fig. Incorrect Overlap**.
4. Once the collimator is mounted, if not already, return the collimator/tube head to the intended use position. Rotate and/or gently pull the collimator to ensure correct coupling.
5. If the collimator is loose, something is incorrect. Repeat above mounting instructions, and if issues persist, please contact Ralco.

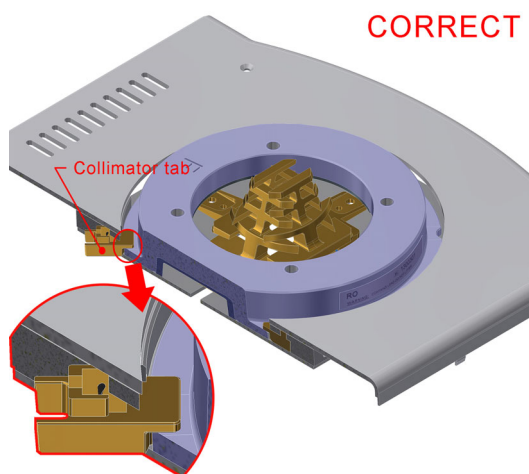


Fig. Correct Overlap



Fig. Incorrect Overlap


RO 619 Resin Rotating Mounting Flange with Position Indicator

This accessory is a part of the position control system designed to indicate the collimator 0° position (+/- 3°) through a microswitch.

Resin rotating mounting flange:

- 15 mm thickness
- +/- 90° detent
- 140 mm diameter

Please refer to section **RO 609 Self-Centering Top-Cover Bracket** in this chapter of the Instruction Manual for the correct flange/collimator mounting instructions.

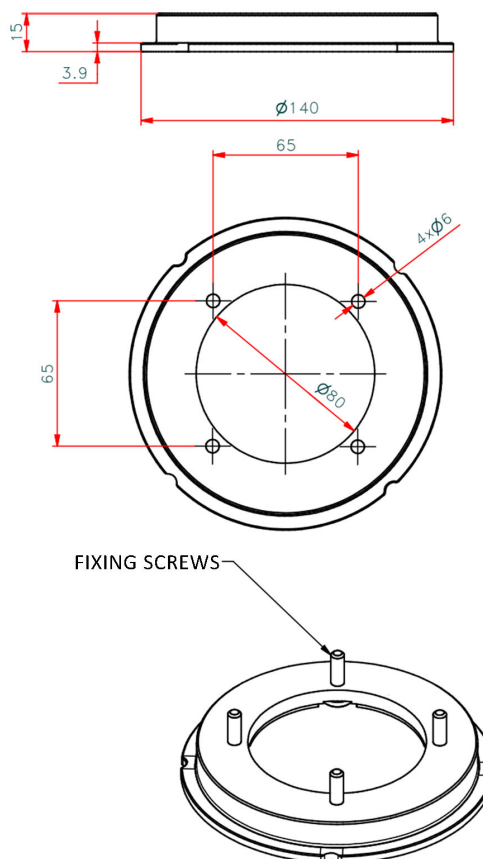

MAINTENANCE

To ensure constantly safe performance of the collimator and its compliance with applicable regulations, a maintenance program is indispensable.

It is the Owner's responsibility to supply or arrange for this service.

CLEANING RECOMMENDATIONS

- The collimator housing must be cleaned as prescribed by the sanitary regulations followed by the operator.
- Disconnect supply.
- Use non abrasive cleaning products.
Care must be taken to prevent liquid from entering the collimator. **Please Note: The collimator cover is not watertight.**
- **Do not** reapply power if inflammable liquids have leaked into the collimator.
See the following Maintenance Instructions.
- Clean the varnished and aluminium surfaces with a damp cloth only, using a neutral cleansing agent then dry the surfaces with a soft cloth.
- Clean chrome surfaces with a dry soft cloth.

CAUTION



DO NOT SPRAY WATER OR DETERGENT DIRECTLY OVER THE COLLIMATOR. THE UNIT'S LIQUID PROTECTION LEVEL IS IPx0.

DISINFECTION

The disinfection method use must conform with the currently applicable norms and directives covering disinfection and protection against explosion hazards.

CAUTION



NEVER USE CAUSTIC SUBSTANCES, SOLVENTS OR ABRASIVE DETERGENTS. IF PRODUCTS THAT COULD FORM EXPLOSIVE GAS MIXTURES ARE USED, ALLOW THE GAS TO EVAPORATE BEFORE STARTING THE SYSTEM.

- Disconnect supply.
- Disinfect the unit including accessories and cables with a dampened cloth.
- Do not spray the unit with the disinfectant because it could leak into the collimator.

RECOMMENDED MAINTENANCE PROGRAM

Ralco suggests a yearly servicing program, however shorter intervals are advisable when the collimator is subject to heavy workloads.

- Re-calibration of the collimator will be necessary whenever the X-ray tube is changed or at each substitution of the source light (lamp or LED, if assembled) used to simulate the light field.

- Calibration procedures (if provided) must be performed as described in this manual.
- Check once a week that the screws and tabs which serve to secure the collimator to the flange/tube adapter are correctly tightened.
- Remove the covers and panels from collimator. Inspect the moving parts for signs of wear or damage.
- Check the electric system and substitute parts that show wear.
- Check the plastic anti-dust window and substitute it if necessary.
- Clean the collimator with a soft cloth paying particular attention to the plastic anti-dust window. Do not use abrasive or inflammable cleaning products.
- Wipe away all excess oil and remount the cover.

CAUTION



MAKE SURE TO TIGHTEN THE ALLEN SCREWS SECURING THE COLLIMATOR OR THE CONTROL TABS.
APPROPRIATE TIGHTENING OF THE 4 ALLEN SCREWS ENSURES SECURE MOUNTING OF THE COLLIMATOR. TIGHTENING FORCE USED MUST NOT EXCEED 0.50 NM.

NOTE



IF THE COLLIMATOR IS TO BE MOUNTED ON A ROTATING FLANGE, USE A TIGHTENING FORCE BETWEEN MIN. 0,50 NM AND MAX. 0.75 NM.

GENERAL

WARRANTY

This product has been manufactured and tested to the highest quality standards by Ralco, srl. Ralco undertakes to replace and repair any collimator during a period of 24 months for mechanical and 12 months for electrical parts (motors, potentiometers, electrical boards, lasers) from the date of invoice (shipment date from Ralco).

The warranty applies provided the product has been handled properly in accordance with its operating instructions and its intended use.

Warranty covers cost of all components and labor involved, **unless**:

- Product documents have been altered in any way or made illegible;
- The model or production number on the product has been altered, deleted, removed or made illegible;
- Repairs or product modifications and alterations have been performed by unauthorized and unqualified persons;
- Unauthorized repairs and/or modifications have been performed;
- Damage caused by misuse or neglect, incorrect installation or accidental damage;
- Damage occurred during transit due to shipping company, or incorrect packing by customer;
- Unoriginal spare parts and accessories have been used.

In-warranty spares will be available only upon return to Ralco, at the customer's expense, of the parts considered to be faulty to allow Ralco to assess the cause of the fault.

Components Not Covered by this Warranty:

- Consumable items such as lamp bulbs, lexan panels and mirrors (if applicable);
- Cosmetic damage such as scratches;
- Any missing components when product arrives for repair.

REPAIRS

In the event the customer finds any Non Conformity in the product, please contact Ralco via e-mail at the address: repairs@ralco.it (Ralco Repair Assistance).

To successfully resolve any Non Conformity, the following information must be provided:

- The model and serial number of the collimator found on the label;
- A detailed description of the problem (in Italian or English);
- Whether you want a repair, refurbishment, or model upgrade of the product.

Ralco and the customer will work together to resolve the problem by either providing instructions, and/or field service or by sending replacement parts.

In the event the collimator needs to be repaired at Ralco, you will be provided with a RMA (Return Merchandise Authorization) number as your consent to the return. If a product is returned to Ralco without a RMA or without a description of the problem, the customer will incur a €100 processing fee.

If the product is returned under warranty and no defect is found, the customer incurs the cost of evaluation, testing and shipping.

The shipping of the unit is at the customer's expense if the unit is out of warranty.

The warranty period resets only on replaced parts once repairs are completed (shipment date from Ralco back to customer), see paragraph, **WARRANTY**.

Ralco reserves the right to decide if the product is to be repaired or substituted.

Please send defective material to the following address:

RALCO SRL
VIA DEI TIGLI 13/G
20853 BIASSONO (MB) - ITALIA
FAX: ++39-039-2497.799
EMAIL: RALCO@RALCO.IT

END OF LIFE DISPOSAL

Your collimator contains materials which can be recycled and reused. Specialised companies can recycle your product to increase the amount of reusable materials and to minimize the amount of materials to be disposed of.

The product contains lead which can be highly contaminating if dispersed incorrectly. The following symbol signifies that the product conforms to the environmental requirements of directives 202/95/EC, 2002/96/EC, 2003/108/EC; it must be disposed of correctly at the end of its life-cycle.



The collimator does not contain polluting materials or products with the exception of the lead that composes the shutters - avoid direct contact with lead especially for prolonged periods.

It is required that you observe Local Laws regulating the disposal of the collimator using certified environmental management entities. Should this prove impossible, return the collimator to Ralco at the purchaser's expense and Ralco will undertake its correct disposal.

If you are replacing the unit with new equipment, you may return the old collimator to Ralco. Please contact us if you require further information.

DISASSEMBLY

WARNINGS



CARE MUST BE TAKEN NOT TO LET THE COLLIMATOR FALL.

- Disconnect supply to the collimator.
- Remove the cover and disconnect the supply cables.
- Loosen the fixing Allen screws on the upper part of the collimator connected to the flange mounted to the X-ray tube.

TRANSPORT AND STORAGE

ATTENZIONE



ANY DAMAGE TO THE COLLIMATOR DUE TO INCORRECT OR UNSUITABLE PACKAGING IS THE RESPONSIBILITY OF THE CUSTOMER. IF POSSIBLE, THE USE OF ORIGINAL RALCO PACKAGING IS RECOMMENDED. IF THIS IS NOT POSSIBLE, PLEASE FOLLOW THE INSTRUCTIONS PROVIDED WITHIN THE INSTRUCTION MANUAL. IF THE COLLIMATOR IS NOT PACKAGED CORRECTLY, ALL WARRANTIES WILL BE VOIDED.

In order to properly package the collimator for shipping the following materials are needed:

- Plastic bag;
- Sturdy cardboard box properly sized for the collimator;
- Protective packaging (bubble wrap, bubble bags, air pillows, polyfoam etc.) -
- Packaging tape;
- Strapping.


Procedure:

1. Ensure all covers are remounted properly on the collimator if previously removed.
2. Place the collimator in the plastic bag to avoid packing material from entering the collimator.
3. Place the collimator inside the cardboard box.
4. Use protective packaging to fill any empty spaces inside the cardboard box so that the collimator is stable during shipment.
5. Seal the cardboard box firmly so it will not open during shipping using high-quality packaging tape.
6. Use strapping to ensure proper closure.
7. It is strongly recommended to pallet the packaging (especially when single collimators are shipped) to ensure proper handling.
8. Ensure of the correct storage conditions:
 - Ambient Temperature = from -40°C to +70°C
 - Relative Humidity = from 10% to 95%
 - Atm. Pressure = from 500 a 1060 hPa.

FRAGILE

X-RAY EQUIPMENT

DISPOSITIVO RADIOLOGICO - X射线设备

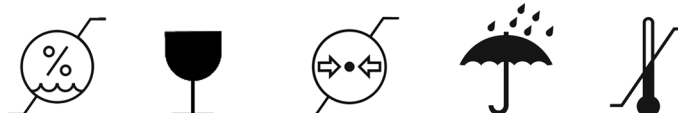

SAFETY/RESPONSIBILITY

Ralco adheres to the directives governing manufacturers of electro-medical equipment:

Directive 2007/47/CE para.10 -Legislative Decree n° 46 para.10

Ralco shall not be held responsible when instructions provided in the present manual are not complied with. Ralco shall not be held responsible if the collimator relates to one or several of the following instances:

- The unit is of Ralco construction, built to client specifications with no CE marking.
- The unit has been modified by the OEM or end user.
- The unit has been installed without respecting the instructions provided in this manual.
- The unit is used without respecting the instructions provided in this manual.
- The unit has not been subject to routine functional inspection.
- The unit has not been subject to routine maintenance.
- The unit has been repaired with unoriginal spare parts.
- Ralco shall decline all responsibility for any damage, direct or indirect, caused to persons or things by inappropriate accessories.

WARNINGS



INFORMATION REGARDING ACCIDENTS THAT HAVE OCCURRED WHILE USING THE RADIOLOGICAL COLLIMATOR MUST BE REPORTED IMMEDIATELY TO RALCO SRL.

RESIDUAL RISKS

The collimator has been constructed to current standards to meet the safety requisites of directive 2007/47/CE . However, due to the presence of x-rays, the type of application implies a residual risk derived from possible faults that could occur during operation of the unit.

The Instructions contained in the this Manual will ensure the correct use of the device and reduce the causes of possible hazards.

The residual risks of the device are reasonable; they have been assessed and approved in the related Risk Management Plan contained the Technical Report.

Information For The Manufacturer

The following form is provided for your comments and suggestions with regards to the collimator so that we may ensure and improve the quality of our production.

Please e-mail comments and/or suggestions to: repairs@ralco.it

Date:

Customer:

Information regarding possible accidents that may have occurred while using the collimator.

Directive 2007/47/CE states that accidents (such as death or grievous injury to a patient) that involve the collimator described herein, must be reported to the Ministry of Health and to the Manufacturer.

The present form is provided to report to Ralco srl post-free.

Date:

Customer:

