

RAPIXX DR System 3325V1i X WiFi Set

Digital X-ray detector system

Model/ID: 4567-9-1000

Instructions for Use & Installation Manual

ID no. 5567-0-0002





NOTE

All sheets of this document contain proprietary and confidential information of PROTEC X-ray Systems GmbH and is intended for exclusive use by current PROTEC X-ray Systems GmbH customers. Copying, disclosure to others or other use is prohibited without the express written consent of PROTEC's law department. Knowledge of violations of these regulations must be reported immediately to PROTEC X-ray Systems GmbH.

© 2025 PROTEC X-ray Systems GmbH, Oberstenfeld

Comments and questions about the documentation, please contact:

PROTEC X-ray Systems GmbH

In den Dorfwiesen 14 | 71720 Oberstenfeld
Germany

Phone: (+ 49) 7062 – 92 55 0

Fax: (+ 49) 7062 – 92 55 60

E-Mail: protec@protec-med.com

Internet: www.protec-med.com

Table of contents

	Page
Table of contents	3
Revision Status	5
Radiation Warning	6
To the User	6
1 Device Description	7
1.1 Introduction	7
1.2 Description.....	7
1.2.1 System Components	7
1.2.2 Hardware and Network System Requirements	8
2 Safety Instructions	9
2.1 General Safety Instructions.....	9
2.1.1 Device Operation	9
2.1.1.1 Operating Type	9
2.1.2 Operating Personnel	9
2.1.3 Explosion Protection	10
2.1.4 Interaction with Other Devices.....	10
2.1.5 Electromagnetic Environment and Influencing of Devices	10
3 Installation	11
3.1 Checklist	12
3.2 Installation of CONAXX 2	12
3.3 Set-up of the network connections	12
3.4 Connection of the components.....	13
3.5 WLAN connection establishment with the detector	13
3.6 First start of CONAXX 2.....	14
3.6.1 Necessary settings in CONAXX 2	14
3.6.2 Licensing of CONAXX 2	14
3.6.3 Activation of CONAXX 2 Modules.....	16
3.7 Dose indicator settings for detector	16
4 Calibration of the DR System	17
5 Control Elements and Displays	18
5.1 Control Elements and Displays of the Detector	18
5.1.1 Display Power Supply	18
5.1.2 Display Connection.....	18
5.1.3 Display Status.....	18
5.2 Displays Battery Charging Station	19
5.3 Control Elements and Displays of the CONAXX 2 Acquisition Software.....	19
6 Handling	21
6.1 Requirements before and during Operation.....	21
6.2 Operation.....	21
6.2.1 Start-up order for daily use.....	21
6.2.2 Operation of the CONAXX 2 acquisition software.....	21
6.2.3 Operation of the detector with accessories	22
6.2.3.1 Battery handling and battery changing.....	22
6.3 Function of the digital X-ray detector system	22
6.3.1 Switching on the RAPIXX DR System.....	22
6.3.2 Switching off the RAPIXX DR System.....	22
6.3.3 Switch-on order for battery changing	23
6.3.4 Troubleshooting while connection establishment.....	23
6.3.5 X-ray generator errors during operation	23
6.3.6 Modality errors during operation	24

6.3.7	Positioning the organ when using automatic exposure detection (AED).....	24
7	Safety and Maintenance.....	25
7.1	Introduction	25
7.2	Reusability	25
7.3	Cleaning and Disinfection	25
7.3.1	Cleaning.....	25
7.3.2	Disinfection	26
7.4	Inspection and maintenance	26
7.4.1	Daily Monitoring before and during the Examination Operation.....	26
7.4.2	Regular Monitoring	26
7.4.3	Maintenance	26
7.4.4	Warranty.....	27
7.4.5	Product Service Life	27
7.4.6	Further Information	27
7.4.7	Disposal Notes	27
8	Power Supply	28
9	Technical Data.....	29
9.1	Dimensions Detector	29
9.2	Technical Data Detector.....	30
9.3	Protection Type and Protection Class.....	30
9.4	Automatic Cut-off dose	30
9.5	Environmental Conditions	30
9.5.1	Environmental Conditions during Operation.....	30
9.5.2	Environmental Conditions for Shipping and Storage	30
10	Description of Symbols, Labels and Abbreviations.....	31
10.1	Symbols	31
10.2	Type Labels.....	32
10.2.1	RAPIXX DR System.....	32
10.2.2	Detector.....	33
10.2.3	Battery charging station (charger) and detector battery.....	33
10.2.4	INTERFACE BOX	34
10.2.5	CONAXX 2.....	34
10.3	Positions of the Signs and Labels.....	34
10.3.1	Detector.....	34
10.3.2	Battery charging station.....	35
10.3.3	Battery	35
10.3.4	INTERFACE BOX	35
10.4	Abbreviations.....	35

**NOTE**

The information contained in these instructions for use conforms to the configuration of the system equipment as of the date of manufacture. Revisions made after the delivery are incorporated in a new version of this document. Current versions of the document can be accessed at any time via the PROTEC website.

Revision Status

Revision	Date	Updated pages	Comments	Author
1.0	2023-07-26	all	Original issue	FR/DP
2.0	2025-07-18	All	First edition PROTEC X-ray Systems GmbH	ML

Radiation Warning



WARNING!

In these accompanying documents, a system or a component for such a system is documented, which is used for the intended generation of X-rays in medical diagnostics.

X-rays are ionizing radiation which can cause damage to living organisms (e.g., cancer or mutations).

X-rays pose a potential risk to patients and employees.

Therefore, the aim of any radiation application and given medical issue is to minimize the radiation exposure for both groups of persons.

The persons responsible for the application must have the necessary expertise in accordance with the regulations and guidelines and apply the procedures for the safe operation of such systems.

National regulations must also be observed during planning and installation.

X-rays are generated in the X-ray tube by strong deceleration of previously accelerated electrons, which emits energy in the form of electromagnetic waves. The intensity depends on the set parameters of voltage (kV), current (mA) and time (s) at the X-ray generator. The X-rays are emitted only at a beam exit window of the tube and are limited by the collimator placed directly below it.

To the User



NOTE

The user of these accompanying documents is required to carefully read through and carefully consider the instructions, warnings and cautions contained therein before starting operation.

Even if you have already operated similar systems, changes in the design, production and functional routine of the system described here may have been made, which have a significant influence on the operation.

Even if the product was the subject of a hazard analysis and the design corresponds to the current state of the art, residual risks remain during clinical use. These are represented in the following instructions for use by limits of use, contraindications, warnings and precautions. Assembly and service works on the system described here must be carried out by authorised and qualified personnel from PROTEC X-ray Systems GmbH. Assembly personnel and other persons who are not employees of the technical service department of PROTEC X-ray Systems GmbH are requested to contact the local branch of PROTEC X-ray Systems GmbH before assembly or service work is started.

For assembly and service works, it is necessary to use the "Technical Description" of the product and to observe the points it contains.

1 Device Description

1.1 Introduction

The instructions for use summarize the most important information for efficient and effective operation of the RAPIXX DR System.



NOTE

Before working with the RAPIXX DR System, it is mandatory to read the applicable original manuals for the system components with detailed safety and handling instructions. These documents are leading and valid in their current version.

1.2 Description

The digital X-ray detector systems of the RAPIXX-series are intended for conventional radiography. The detectors of the RAPIXX-set serve as an image receiver that displays X-ray images in digital form on a monitor or display. The CONAXX 2 acquisition software included in the set is responsible for image reception, image processing and image subsequent processing.

1.2.1 System Components

The RAPIXX DR System consists of the following system components:

- Detector RAPIXX 3325V1i X WiFi
 - Power supply unit for detector incl. power cable
 - Charging station incl. power supply unit
 - 2 batteries
 - Data cable 3,5m
- CONAXX 2 acquisition system software (on data carrier)
- Network cable 5m
- Access Point (optional)
- Documentation RAPIXX DR System (on data carrier)

Optional system components

- INTERFACE BOX (trigger mechanism detector)
- Power set iRay (power supply detector)
- CONAXX 2 Module
 - CONAXX 2 X-Ray Journal
 - CONAXX 2 Gridline Suppression
 - CONAXX 2 DICOM Print
 - CONAXX 2 Generator Connection
 - CONAXX 2 Patient CD
 - CONAXX 2 DICOM Query
 - CONAXX 2 DICOM Worklist
 - CONAXX 2 Diagnostic Viewer
 - CONAXX 2 Dual Panel

Optional accessories

- Detector protective housing
- PC or Notebook

Accessories that can influence the EMC conditions

- Network cable (take note of the max. cable length in the component documents)
- RAPIXX data connection cable (take note of the max. cable length in the component documents)
- WLAN router or Access Point (only use devices approved by PROTEC)

1.2.2 Hardware and Network System Requirements

When using the digital X-ray detector system, it should be ensured that the country-specific requirements for data protection and IT security are met.

The system requirements for the optional system components (RAPIXX-series) can be found in the current document supplied, "EN_5330-0-0026_CONAXX2_System_Requirements.

PROTEC X-ray Systems GmbH

In den Dorfwiesen 14 | 71720 Oberstenfeld
Germany

Phone: (+ 49) 7062 – 92 55 0

Fax: (+ 49) 7062 – 92 55 60

E-Mail: protec@protec-med.com

Internet: www.protec-med.com

2 Safety Instructions



NOTE

Contains information that must be observed during operation.

xxx



CAUTION!

Contains information which, if not observed, can cause property damage.

xxx



WARNING!

Contains information which, if not followed, can cause personal injury.

xxx



WARNING!

Warning of radioactive substances or ionizing radiation. Contains information which, if not observed, can cause personal injury.

xxx

Settings and calibrations that are not described in these instructions for use must be carried out using the technical description of the individual components by PROTEC service department or a service company authorized by them.



NOTE

All supplied instructions must be observed and the safety instructions contained therein must be carefully read and adhered to.



CAUTION!

The instructions for use contain all the information relevant to safety in order to generally put the digital X-ray detector system into operation. The device may only be operated by appropriately trained and trained personnel. In this context, operation is ensured by clear symbols on the control elements. All further information and instructions can be found on the supplied data carrier (USB, CD or DVD). This information applies in its entirety as an appendix to these instructions for use and must be observed.

2.1 General Safety Instructions

2.1.1 Device Operation

In case of a malfunction, do not use the digital X-ray detector system anymore and notify PROTEC service department or a service company authorized by them.

2.1.1.1 Operating Type

The digital X-ray detector system is intended for continuous operation.

2.1.2 Operating Personnel



NOTE

Only trained and authorized personnel are allowed to work on the digital X-ray detector system.

**NOTE**

The operating personnel must be familiar with all warning signs attached to the digital X-ray detector system. They are used for your own safety and that of others and ensure proper operation.

2.1.3 Explosion Protection

The digital X-ray detector system is not designated for use within areas with explosive hazards.

2.1.4 Interaction with Other Devices

Interactions with other devices are not known.

2.1.5 Electromagnetic Environment and Influencing of Devices

**CAUTION!**

The use of other accessories, other converters and other cables than those specified by PROTEC or provided in the documentation of the component manufacturer can result in increased electromagnetic interference or reduced electromagnetic immunity of the device and lead to defective operation.

**CAUTION!**

The use of the digital X-ray detector system immediately next to other devices or with other devices in a stacked form should be avoided, as this could result in defective operation. If use in the manner described above is nevertheless necessary, the digital X-ray detector system and the other devices should be observed to ensure that they are working properly.

**NOTE**

The properties of this device, determined by emissions, allow its use in industrial areas and in hospitals (CISPR 11, class A). When used in residential areas (for which class B is usually required by CISPR 11), this device may not provide adequate protection for radio services. The user may need to take remedial measures such as relocating or realigning the device.

The digital X-ray detector system is intended for use in an environment in professional veterinary facilities (e.g., veterinary clinics, veterinary practices ...).

3 Installation



NOTE

The installation of the digital X-ray detector system must be performed by PROTEC customer service or a service authorized by it.



CAUTION!

The digital X-ray detector system must not be installed in the following locations to avoid malfunction, damage, fire or injury:

- **Where there are large temperature fluctuations.**
 - **Near heat sources e.g., a heater.**
 - **In an environment containing salt or sulfur.**
 - **In the vicinity where water can escape.**
 - **Where corrosive gas may be generated.**
 - **In a dusty environment.**
 - **Where the device is exposed to frequent or excessive vibrations or shocks.**
 - **Where the device is exposed to direct sunlight.**
 - **Where the ambient conditions for temperature, humidity and air pressure cannot be maintained.**
-

For detailed information, please refer to the installation manuals of the individual components.



NOTE

Use only supplied network cables or well-shielded cables of category CAT6 or higher.



NOTE

The anti-scatter grid recommended for this type of detector depends on factors such as grid motorization. Therefore, the grid to be used should be coordinated with PROTEC X-ray Systems GmbH in advance.



WARNING!

PROTEC X-ray Systems GmbH assumes no liability for the selection of the anti-scatter grid.



WARNING!

If the detector of the RAPIXX DR System is operated via data cable and with direct patient contact, it must be ensured that the data cable is equipped with a network isolator in accordance with EN 60601-1.

3.1 Checklist

If the installation has already been prepared by PROTEC, only the following subchapters of this chapter need to be observed:

- **Fehler! Verweisquelle konnte nicht gefunden werden. Fehler! Verweisquelle konnte nicht gefunden werden.**
- **Fehler! Verweisquelle konnte nicht gefunden werden.** Connection of the components
Afterwards continue with the chapter "4. Calibration of the DR System".

3.2 Installation of CONAXX 2

Install the supplied CONAXX 2 software. Make sure that the item "RAPIXX 3325V1i X WiFi" is selected in the drivers.

Detailed information about the CONAXX 2 installation can be taken from the document "CONAXX 2 Installation Manual".

3.3 Set-up of the network connections


On delivery, each detector is configured with the IP 192 . 168 . 8 . 8. Please adjust the network configuration on the computer to access the detector. Go to „CONTROL PANEL“, open „NETWORK- AND SHARING CENTER“ and afterwards „CHANGE ADAPTER SETTINGS“.

In case of using an Access Point, select the network adapter where the Access Point is connected with the right mouse button and open „PROPERTIES“.

In case of using the WLAN adapter of the computer for direct contact to the panel, select the WLAN adapter with the right mouse button and open „PROPERTIES“.


Select the „INTERNET PROTOCOL VERSION 4 (TCP/IP)“ entry from the list and click on „PROPERTIES“. The IP-address can be manually entered in the following window. Select the address 192 . 168 . 8 . 188 and set 255 . 255 . 255 . 0 as subnet mask.

The configuration of the network card is now completed.

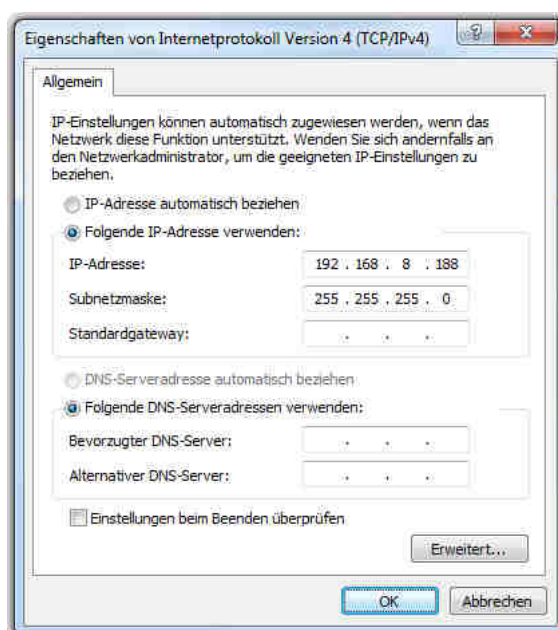

3.4 Connection of the components

Connect the components as described in the following schematic diagram.

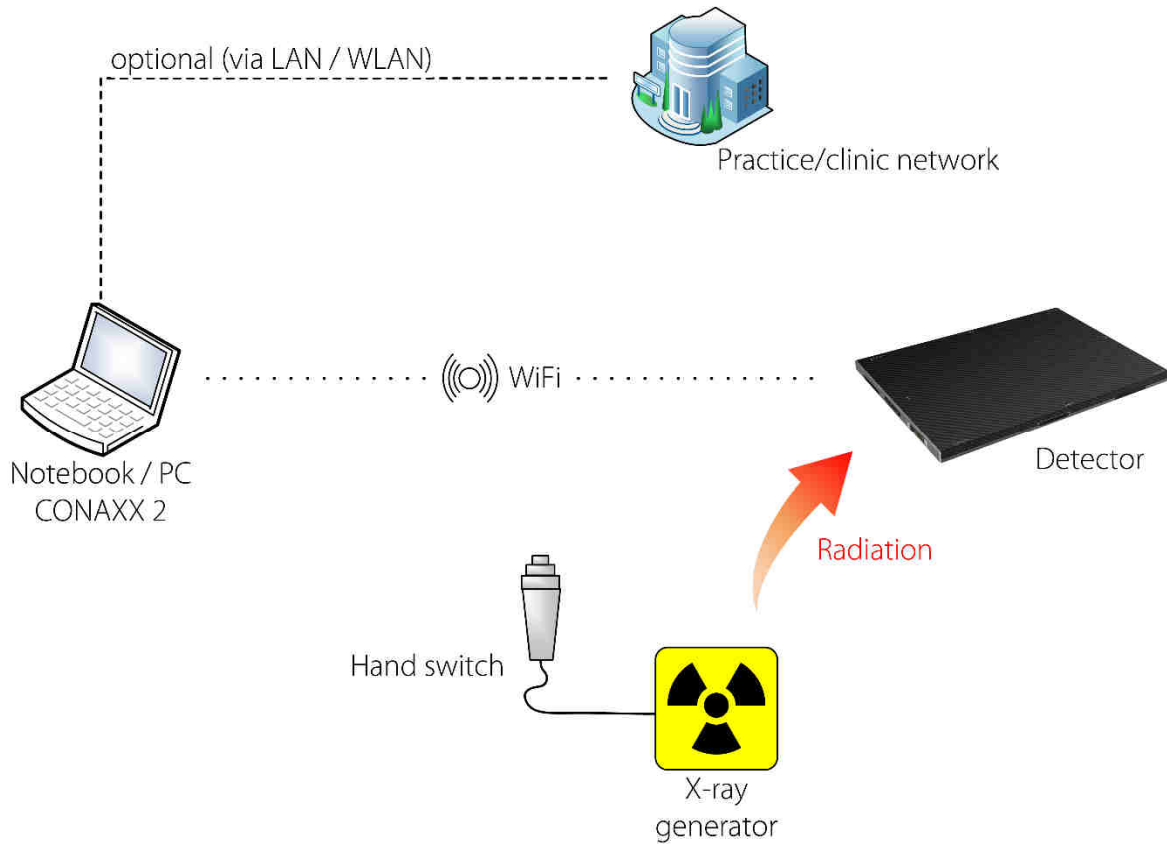

3.5 WLAN connection establishment with the detector

If the WLAN adapter built into the computer is used for a direct connection to the panel and not an Access Point, the WLAN connection must be established once to the detector.

Open the list with the available wireless networks. The detector WLAN is designated with the detector serial number, e.g., "KV0704A294138". The password to use the detector WLAN is "12345678".



3.6 First start of CONAXX 2

Start CONAXX 2 with a double click on the desktop icon. Check the document "CONAXX 2 User Manual" for more details.

3.6.1 Necessary settings in CONAXX 2

After the first start of CONAXX 2 open the configuration by clicking the button "Configuration" in the start menu of CONAXX 2 and change into the section "System > Modality".

Select in the area "Selected Modality" the detector that you would like to use, e.g., "RAPIXX 3325 V1i X WiFi".

Depending on how the detector is built-in it might be necessary to rotate the acquired images in CONAXX 2. This can be changed in "Rotations" settings.

In case of using "AUTOMATIC EXPOSURE DETECTION" (AED) activate the according option.




NOTE

The acquisition time window of the detector is 1000 milliseconds at delivery. If a longer acquisition time window is required for the X-ray images, please contact PROTEC's support.

Afterwards restart CONAXX 2.

3.6.2 Licensing of CONAXX 2

After the restart of CONAXX 2 you will be informed that the license does not fit to your system.

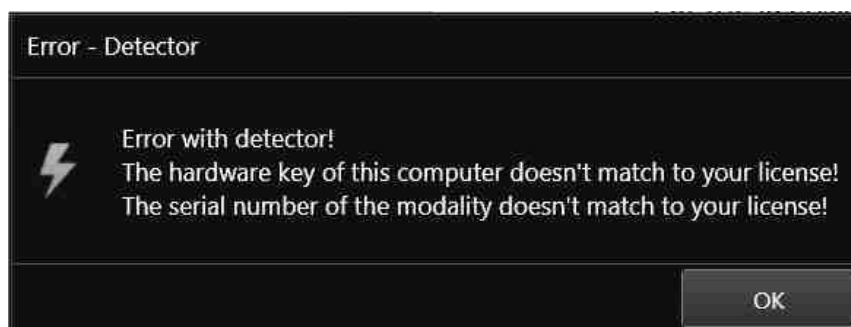


Confirm this message.



NOTE

For the licensing procedure it is necessary that CONAXX 2 can communicate with the connected detector. Please do not change the system configuration (network card, connected detectors, ...) otherwise the license may become invalid. In case of using more than one detector please connect all detectors to the computer.

Open the configuration of CONAXX and navigate to the "SYSTEM > REGISTRATION" section to create the license request. In the section "GENERATE HARDWARE KEY" select the function "SAVE KEY AS..." to create a license request. In the next step please choose the type of licensing.

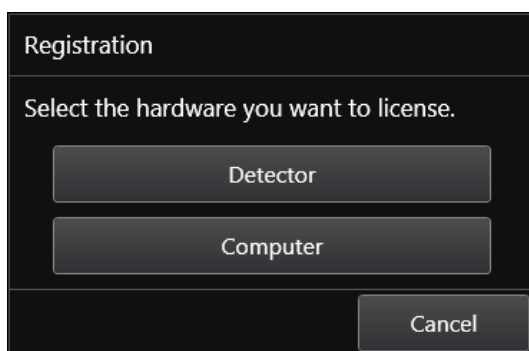

Two different types of licensing are supported:

Type of licensing	Properties
Detector-based	This type links the license with the detector.
Computer-based	This type links the license with the computer.

If no selection option appears, the detector only supports computer-based licensing.



NOTE

The module "Advanced image processing (AIP)" is generally linked to the computer. It is not possible to use it on other computers, even if the detector-based licensing is used.

There are two ways to send the hardware key:

1. Online with the licensing page of the dealer backroom or
2. by E-mail

For the online licensing, please login to the backroom and navigate to the page "LICENSING". Select the desired license and upload the hardware key with the function "SET". Download the license afterwards with "GET".

For the licensing by E-mail, send the hardware key to mis@protec-med.com. You will then get the valid license key by E-mail.

This license file must be imported in the section "SYSTEM > REGISTRATION" of the CONAXX 2 configuration.

Afterwards restart CONAXX 2. CONAXX 2 is now operational. The process with the DR System is described in the document "CONAXX 2 User Manual".

**NOTE**

For detailed information on CONAXX 2 licensing please refer to the document "CONAXX 2 User Manual".

3.6.3 Activation of CONAXX 2 Modules

After licensing, make sure that the CONAXX 2 modules you purchased are activated.

For example:

- Advanced image processing (AIP)
- Grid suppression (optional)
- Diagnostic Viewer (optional)
- DICOM Worklist (optional)
- ...

For detailed descriptions of the modules, please refer to the CONAXX 2 User Manual.

3.7 Dose indicator settings for detector

For each detector the document "RAPIXX calibration values" is included. Follow the instructions in the document to setup the dose indicator.

4 Calibration of the DR System



NOTE

For the calibration make sure that the detector runs at least 2 hours.

General notes for calibration:

- Set the required SID of the detector manufacturer to 120 cm.
- Make sure that the detector is completely irradiated (no collimation).
- Ensure that there are no objects or dirt in the beam path.
- Use 70 kV for all calibration exposures.
- Calibrate on the table.
- Perform a gain calibration every 6 months.

Start the program "RAPIXX Calibration" via the start menu in the folder "CONAXX2\DR-Panel". Make sure that the connection to the detector in CONAXX 2 has been successfully configured and established beforehand. Via "Start" the calibration can be started.

The gain calibration will be started with "Start" in the section "Gain" and each exposure must be prepared with the button "Prepare". Please follow the screen instructions and perform 5 exposures without any object. After the gain calibration, a status window will be shown.

The defect calibration will be started with "Start" in the section "Defect" and each exposure must be prepared with the button "Prepare". Please follow the screen instructions and perform 9 exposures without any object. After the defect calibration, a status window will be shown.

The "RAPIXX Calibration" software can be closed.



NOTE

If problems occur during the calibration of the detector, please calibrate according to the supplied instructions of the detector manufacturer.

5 Control Elements and Displays

5.1 Control Elements and Displays of the Detector



	Description	Function
A	Display status	Displays different status
B	Display connection	Displays connection status
C	Display power supply	Displays power supply
D	Switch on/off button	Switching the detector on and off
E	Connection charging/power cable	24V DC connection

5.1.1 Display Power Supply

LED	Description	Status
	Off	Detector switched off – no DC input
	Green	Detector switched on – DC input
	Orange (flashing)	Detector switched on – no DC input – Battery charge $\geq 7\%$ and $< 15\%$
	Green (flashing)	Detector switched off – DC input – Battery charge $\geq 15\%$ and $< 95\%$
	Green & Orange (flashing)	Detector switched off – DC input – Battery charge $< 95\%$

5.1.2 Display Connection


LED	Description	Status
	Off	Detector switched off – Wired connection broken and wireless connection not ready
	Blue	WLAN connection is enabled
	Green	Wired connection is enabled (Service Mode)

5.1.3 Display Status

LED	Description	Status
	Blue	Detector switched off
	Green	Exposure possible
	Off	Error state

For detailed information on the detector, please refer to the enclosed User Manual of the detector.

5.2 Displays Battery Charging Station

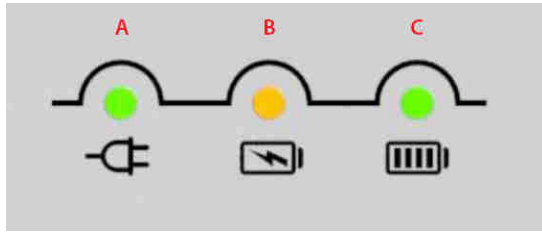

A	Display power supply
B	Display charging process
C	Display fully charged battery

LEDs	Description	Status
	All off	No power supply
	Display A on	- power supply - multiple batteries inserted
	Display A on B and C flashing alternately 2 times	Self-test on battery insertion
	Display A and B on	Battery is charging
	Display A and C on	Battery charged, charging process stopped
	Display A on B and C flashing alternately	Defective charging process

For detailed information of the charging station, please refer to the enclosed User Manual of the detector.

5.3 Control Elements and Displays of the CONAXX 2 Acquisition Software

This area provides status information and tools for various components in the CONAXX 2 main window:



- System:

This function shows status information of the system.
This includes, for example, information about hard disk capacities.



- Modality:

This function opens the toolbox for the connected modality.
It provides functions to recover or close the connection to the modality or calibration functions. Furthermore, status information such as battery charge level is displayed.

Special detector status information:



- The detector is powered by an external power supply unit.



- The detector is powered by an external power supply unit and the battery is charging.



- The battery still has at least 75% of its charge.



- The battery still has at least 50% of its charge.





- The battery still has at least 25% of its charge.



- The battery still has at least 20% of its charge.



- The battery has less than 20% of its charge. No X-ray exposure is possible.

-  - The connection to the detector is interrupted. No X-ray exposure is possible.
-  - **X-ray generator:**
This function opens the x-ray generator control.
X-ray exposures can be made without taking images via the modality.

For detailed information of the CONAXX 2 acquisition software, please refer to the CONAXX 2 User Manual.

6 Handling

6.1 Requirements before and during Operation

It must be ensured that the surfaces in contact with patients are disinfected before the X-ray examination of each patient (see chapter 7.3.2).




WARNING!

In the event of a technically incorrect image acquisition (e.g., missing image content, strip structures), the entire X-ray system must be restarted and a test exposure must be taken without patient. Only if the exposure is correct, the system can be used again properly. If the test exposure is still defective, please contact PROTEC customer service or a service authorized by them.

6.2 Operation

6.2.1 Start-up order for daily use

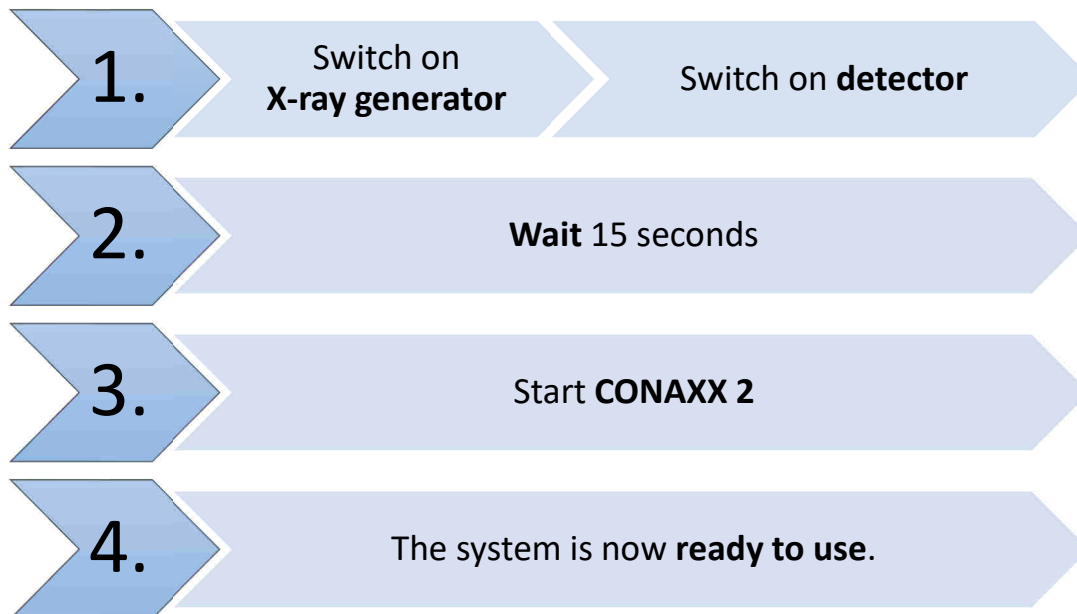

If CONAXX 2 cannot connect to the modality or generator after launching the program, there will appear an error message. In this case, CONAXX 2 must be restarted to initiate a new connection attempt.

More detailed information can be taken from the error message displayed by CONAXX 2.

If no error message appears after starting the program, the connection was established successfully and the system can be used.

6.2.2 Operation of the CONAXX 2 acquisition software

For detailed information of the CONAXX 2 acquisition software, please refer to the CONAXX 2 User Manual.



NOTE

In order to keep quality and battery power optimal for wireless detector systems, it is recommended to press the "Start exposure" button in the CONAXX 2 software only after all preparation for the exposure (e.g., patient positioning) have been made and the X-ray system is ready to be triggered.

6.2.3 Operation of the detector with accessories

For detailed information of the detector, please refer to the enclosed original instructions *UserManual_Luna1013XE* of the detector manufacturer.



WARNING!

If the detector of the DR System is operated via data cable and with direct patient contact, it must be ensured that the data cable is equipped with a network isolator in accordance with EN 60601-1.

6.2.3.1 Battery handling and battery changing

For detailed information on handling the battery and charger, as well as on changing the battery, please refer to the enclosed original instructions *UserManual_Luna1013XE* of the detector manufacturer.



NOTE

Before changing the battery, it is recommended to switch off the detector. After the battery change, the detector can be switched on again.



CAUTION!

Only 1 battery may be charged in the charging station at the same time. If more than 1 battery is inserted, the charging station is not working.

6.3 Function of the digital X-ray detector system

6.3.1 Switching on the RAPIXX DR System

The digital X-ray detector system is first switched on by switching on the detector. This is done either by pressing the on/off button on the detector with inserted battery or by inserting the battery into the compartment provided on the back the detector, which then switches on automatically. After waiting 15 seconds, the CONAXX 2 acquisition software can be started by double-clicking on the desktop icon.

If no error message appears while starting the program, the connection has been established correctly and the system can be used. If no connection can be established, proceed as described in the section "Troubleshooting while connection establishment".


6.3.2 Switching off the RAPIXX DR System

To shut down the system, the following components must be switched off one after another.

1. Detector

The detector can be switched off in two ways:

- a. via the on/off button on the detector
- b. via the "switch off" function in the "Modality" status area of the CONAXX 2 software.

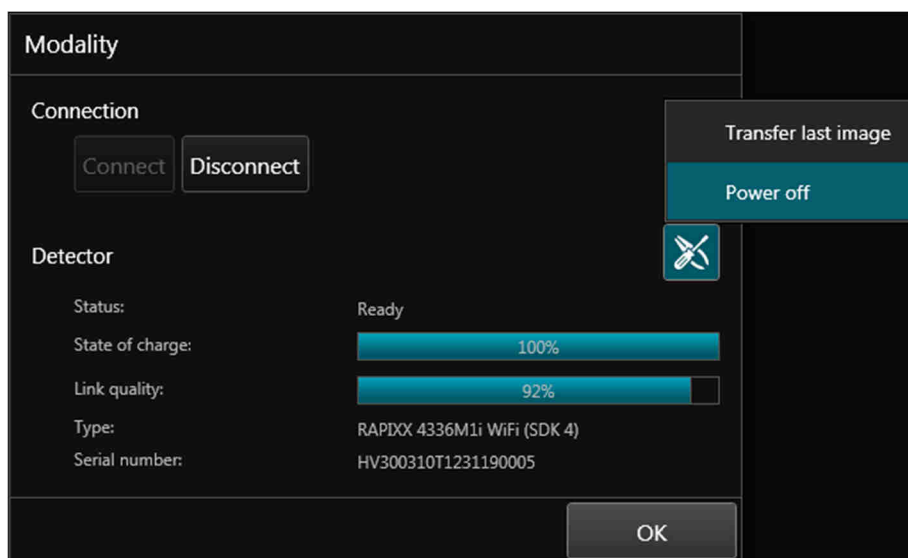

2. CONAXX 2 software

The software can be shut down via the "Exit" menu item in the main menu.

6.3.3 Switch-on order for battery changing


1. Switch off the detector
2. Change the battery
3. As soon as the new battery charge level is displayed in the status window, the window can be closed and work can be continued.

If no error message appears during the automatic connection, the connection has been established correctly and the system can be used.

If no connection can be established, proceed as described in the section "Troubleshooting while connection establishment".

6.3.4 Troubleshooting while connection establishment

If the connection cannot be established, it can be established manually in the toolbox in CONAXX 2.

1. Open „Toolbox“ ()
2. Press the „Disconnect“ button
3. Make sure that the detector is switched on
4. Press „Connect“ button

If no error message appears, the connection has been established correctly and the system can be used.



NOTE

If problems with the WLAN connection occur permanently, please note the following information:

Is there an obstacle in the WLAN transmission path that weakens the signal?

Is there an interfering transmitter near the WLAN connection (smartphone, DECT phone, baby monitor, wireless speaker, etc.)?

Is the WLAN switched on at the detector?

6.3.5 X-ray generator errors during operation

If an error occurs with the X-ray generator during daily operation, the error message provides information on how to solve the problem.

If the connection to the X-ray generator is interrupted during operation with CONAXX 2, it must be restarted. A new connection attempt is not initiated until the restart. If no error message appears during the restart, the connection was established correctly and the generator can be used.

6.3.6 Modality errors during operation

If an error occurs with the modality during daily operation (e.g., communication error, power blackout), the error message provides information on how to solve the problem.

If the connection to the modality is interrupted during the runtime of CONAXX 2, CONAXX 2 must be restarted. A new connection attempt is not initiated until the restart. If no error message appears after the restart, the connection has been established successfully and the modality can be used.

6.3.7 Positioning the organ when using automatic exposure detection (AED)



WARNING!

The detector is equipped with full area dose detection. Therefore, no special positioning of the organ on the detector is necessary.

To ensure the function of the automatic exposure detection (AED), the following factors must be observed for each X-ray exposure:

- **The X-ray radiation must cover at least an area of 3 x 5 cm.**
- **The collimated area must not be completely covered with highly absorbent metal/material, e.g., by a prosthesis, lead apron, etc.**

If the collimated area is completely covered with a body organ, it must be ensured that the correct dose is applied for the organ. Underexposure may result that the automatic exposure detection is not triggered.

7 Safety and Maintenance



WARNING!

Caution, risk of electric shock!
Switch off the RAPIXX DR System before cleaning or disinfecting. This disconnects the digital X-ray detector system from the power source and avoids the risk of electric shock.

7.1 Introduction

In this chapter you will find information about safety and maintenance that are necessary to ensure the correct and reliable function of the device after installation.

7.2 Reusability

The RAPIXX DR System can be reused without any special preparation procedures. However, it must be ensured that the surfaces in contact with patients are disinfected before the X-ray examination of each patient (see also chapter 6.1).

The RAPIXX DR System must no longer be used with patients if it shows signs of wear (e.g., metal abrasion, wear of insulations) or dangerous technical defects (e.g., missing, defective or bent parts) or if the resulting image quality (e.g., artifacts in the image) is insufficient. In this case, please contact PROTEC service department or a service company authorized by them immediately.

7.3 Cleaning and Disinfection



WARNING!

Make sure that no liquid enters the interior of the housing during cleaning and disinfection in order to prevent electrical short circuits and/or corrosion.



CAUTION!

Possible material changes!
Do not use any corrosive, solvent or abrasive cleaning agents that may damage the surface of the device or the coating.
Do not use an excessive amount of ethanol (or neutral cleaner), to prevent liquids from the surface from entering the inside of the detector or its accessories. This will prevent damage and peeling of labels on the detector or its accessories.
Do not use a solvent such as thinners or benzene, as it reacts with the surface of the detector.
Never immerse or flood any parts during cleaning.

7.3.1 Cleaning

The cleaning of the digital X-ray system is very easy due to the very good quality surface coating. This is usually only done with a soft dry cloth.

Do not use corrosive, dissolving or abrasive cleaning agents, which could damage the device surfaces or the coating.

To clean the detector and its accessories, we recommend using cleaning agents that evaporate quickly and therefore prevent liquids from entering inside of the housing.

Commercially available medical rapid disinfection wipes (e.g., Dr. Schumacher Descosept Sensitive Wipes) can be used.

If ready-to-use cloths are not used, cloths lightly soaked with neutral cleaner can be used. These must be carefully wrung out that no liquids from the surface can enter the interior of the detector or its accessories.

Detailed instructions can be found in the enclosed original instructions for the system components.

7.3.2 Disinfection

The disinfection must be performed in accordance with the current applicable legal requirements and guidelines corresponding for disinfection and explosion protection.

For disinfection of surfaces in contact with patients, we recommend commercially available medical rapid disinfection wipes (e.g., Dr. Schumacher Descosept Sensitive Wipes).



WARNING!

**No highly flammable disinfectants may be used! For safety reasons, spray disinfection must not be carried out, as the spray mist could penetrate the device and cause short circuits or corrosion.
If disinfectants are used that can form explosive gas mixtures, the device may only be switched on again when the gas mixtures have evaporated!**

Detailed instructions can be found in the enclosed original instructions for the system components.

7.4 Inspection and maintenance



WARNING!

**No maintenance or repair work may be performed while the digital X-ray detector system is being used with a patient!
All maintenance and repair work may only be performed by personnel trained or authorized by PROTEC.**

7.4.1 Daily Monitoring before and during the Examination Operation

The detector surface must be daily checked before operating start if there are any visible damages. In case of visible damages, a technical homogeneity X-ray (without any patient) should be taken to check if they are visible inside the image.

For detailed instructions, refer to the enclosed original manuals for the system components.

7.4.2 Regular Monitoring

In the interest of the safety of the patient, operator and external third parties, checks that maintain the operational safety and functionality of the device are required to be undertaken in regular 12 months intervals by the PROTEC service department or a PROTEC authorized service technician. This includes the control of the image quality as well.

It is recommended to calibrate the digital X-ray detector system every 6 months at least.

Detailed instructions can be found in the enclosed original manuals of the system components.

7.4.3 Maintenance

The required maintenance must be carried out by PROTEC service department or a service company authorized by them in order to ensure the safe and reliable functionality of the device. The maintenance intervals depend on the frequency of use. Prior to the examination operation, the user must satisfy himself that all appliances listed in the operating instructions and serving safety are in working order and that the RAPIXX DR System is ready for operation.

Detailed instructions can be found in the enclosed original manuals of the system components.

**WARNING**

Wear parts must be replaced with original parts.

**NOTE**

In the case that the required maintenance is not completed as intended, PROTEC X-ray Systems GmbH is no longer responsible for injuries to the operator and third parties, provided that the damage is the result of improper or missing maintenance.

7.4.4 Warranty

**NOTE**

The current warranty conditions can be found in your order documents.

Repairs and spare parts in the event of improper use are also excluded.
Warranty work may only be carried out by trained specialists.

7.4.5 Product Service Life

Detailed information on the product service life can be found in the enclosed original instructions for the system components. After the product has reached the end of its service life, further use is at your own risk.

7.4.6 Further Information

Detailed information and further descriptions for "Safety and Maintenance" of the detector can be found in the enclosed original manual of the detector manufacturer *User Manual_Luna1013XE*.

7.4.7 Disposal Notes

The digital X-ray detector system contains various plastics, chemical elements and heavy metal. When disposing exchange and spare parts, or if necessary, the entire device, the valid rules and regulations must be observed. Please contact your contractual partner or service company or commission a company specialized in disposing the respective components.

8 Power Supply



WARNING!

To avoid the risk of an electric shock, this device may only be connected to a supply network with a protective earth conductor.


9 Technical Data

9.1 Dimensions Detector


Dimension (L x W x H): 362.4 mm x 269.35 mm x 14.5 mm

Effective area: ca. 331.8 mm x 252.8 mm

Front side (active side)

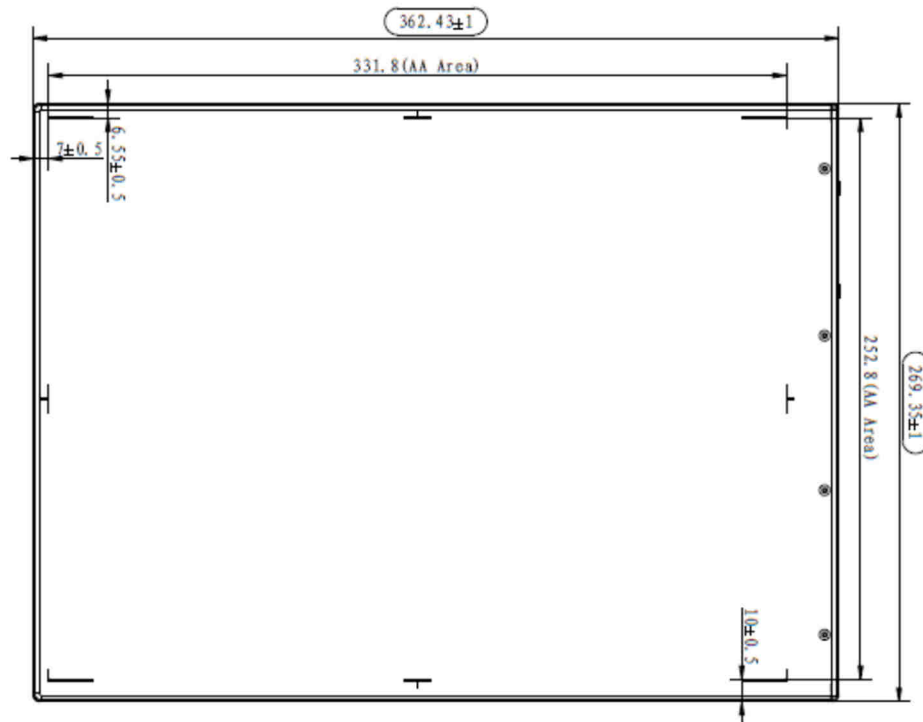

Back side (inactive side)

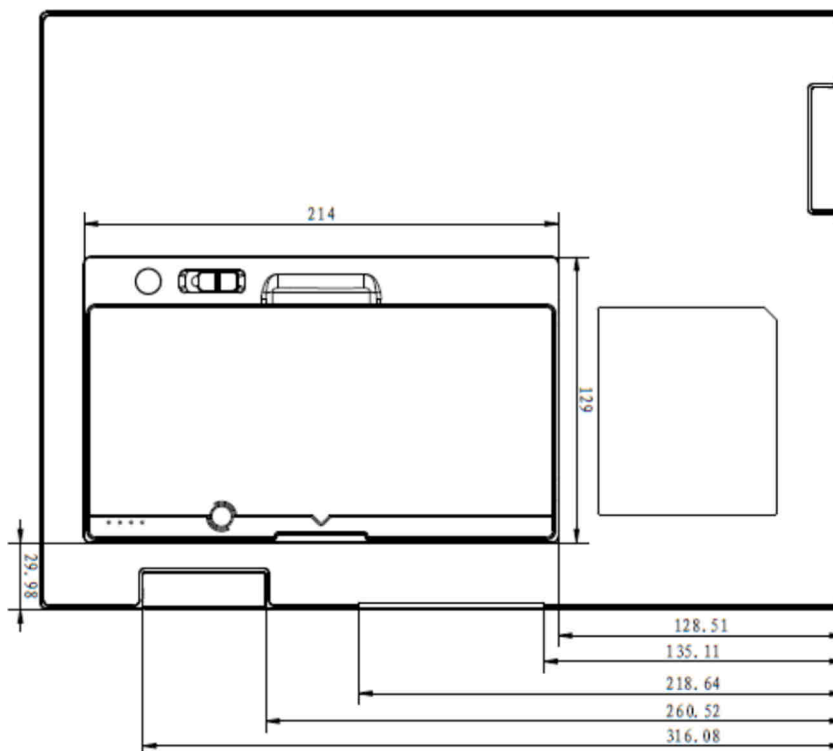

9.2 Technical Data Detector

Resolution	100 μm
Scintillator	CsI
Effective area	ca. 331.8 mm x 252.8 mm
Protection class	IP67
Weight	2.285 kg incl. battery
WLAN Band	2.4 GHz \leq 35 MHz 5 GHz \leq 50 MHz

For information of the detector and its accessories, refer to the enclosed original instructions of the detector manufacturer *UserManual_Mars1417X*.

9.3 Protection Type and Protection Class

For detailed instructions on the protection type and protection class, refer to the enclosed original instructions for the system components.

9.4 Automatic Cut-off dose

The recommended automatic cut-off dose on the X-ray system with the digital X-ray detector system is 2.7 μGy .

9.5 Environmental Conditions

9.5.1 Environmental Conditions during Operation












Ambient temperature	+10°C to +35°C
Relative humidity	5% to 90%
Atmospheric pressure	700 hPa to 1060 hPa











9.5.2 Environmental Conditions for Shipping and Storage

Ambient Temperature	-20°C to +55°C
Relative humidity	5% to 95%
Atmospheric pressure	600 hPa to 1060 hPa

10 Description of Symbols, Labels and Abbreviations


10.1 Symbols

	Atmospheric pressure limit
	Temperature limit
	Humidity limit
	Keep dry
	Protect from sunlight
	Fragile, handle with care
	Handle with care
IPxx	Protection type
	This way up
	Attention, observe accompanying documents
	Refer to Instructions for use
CE	CE-certification, with number of notified body if applicable
	Manufacturer
MD	Medical Device
REF	Order reference


	Serial number
	Unique Device Identification
	Production date
	Classification according to EN 60601-1 (type B applied part)
 www.protec-med.com/download	This symbol indicates the need to observe the instructions for use. This is provided in an electronic format (eIFU) on our website.
	Disposal instructions; WEEE (Waste of Electrical and Electronic Equipment)
	Protective earthing
	System (CONAXX 2)
	Modality (CONAXX 2)
	X-ray generator (CONAXX 2)

10.2 Type Labels

10.2.1 RAPIXX DR System

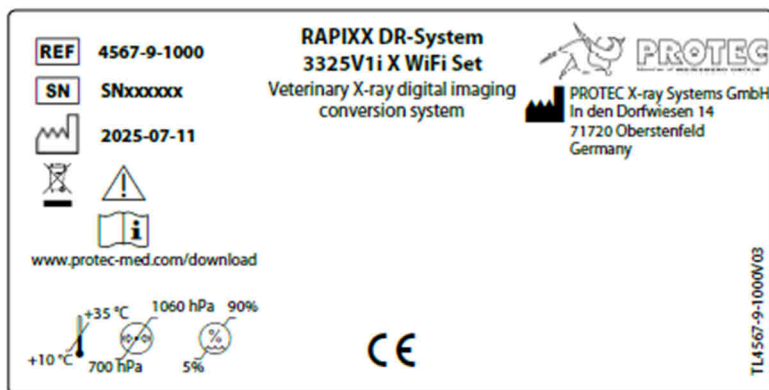

Example Type Label RAPIXX DR System

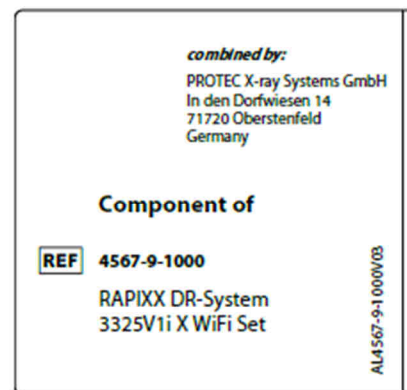

Example component RAPIXX DR System

10.2.2 Detector



无线数字平板探测器

产品型号: Luna1013XE
 接入电源: 适配器供电输入 24V \equiv 0.75A
 电池供电输入 11.55V \equiv 1.6A

奕瑞影像科技(太仓)有限公司
 中国江苏省
 太仓港经济技术开发区兴港路33号 2023-05




Wireless Digital Flat Panel Detector

Model: Luna1013XE
 Power: Adapter Port Input 24V \equiv 0.75A
 Battery Port Input 11.55V \equiv 1.6A

iRay Technology Taicang Ltd.
 No.33 Xinggang Road Taicang Port Economic and Technological Development Zone Taicang 215434 Jiangsu China 2023-05
 www.iraygroup.com

FC Contains FCC ID: 2ACHK-01070189

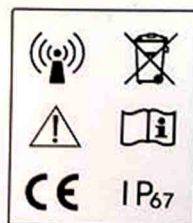

Example type label iRay Luna1013XE detector

10.2.3 Battery charging station (charger) and detector battery

充电器

产品型号: CHARGER-COMBO
 接入电源: 输入 100-240V~50/60Hz
 1.0A MAX
 输出1 12.6V \equiv 2.5A
 或输出2 13.2V \equiv 2.5A
 或输出3 8.7V \equiv 2.5A

奕瑞影像科技(太仓)有限公司
 中国江苏省
 太仓港经济技术开发区兴港路33号
 其他内容详见说明书 Made in China

CE 0197
 C US IP20
 IS-13252(PART 1)-2010V
 IEC 60950-1:2005
 R-41231680
 www.bis.gov.in

Model: CHARGER-COMBO
 Power: Input 100-240V~50/60Hz
 1.0A MAX
 Output1 12.6V \equiv 2.5A
 OR Output2 13.2V \equiv 2.5A
 OR Output3 8.7V \equiv 2.5A

iRay Technology Taicang Ltd.
 No.33 Xinggang Road, Taicang Port Economic and Technological Development Zone, Jiangsu, China
 www.iraygroup.com

EC REP iRay Europe GmbH
 In den Dorfriesen 14, 71720 Oberstenfeld Germany

SN 02061096T524230046

Example type label iRay battery charging station

可充电锂离子电池组

产品型号: BATTERY-03
 额定电压: 11.55V
 充电电压: 12.6V
 额定容量: 4400mAh(25°C)
 额定功率: 4400mAh(25°C)
 零件号: 04112V10174
 零件号: 1.0A-06-01C

奕瑞影像科技(太仓)有限公司
 中国江苏省
 太仓港经济技术开发区兴港路33号

CE 0197
 C US IP20
 IS-13252(PART 1)-2010V
 IEC 60950-1:2005
 R-41231680
 www.bis.gov.in

Model: BATTERY-03
 Rated Voltage: 11.55V
 Limited Charging Voltage: 12.6V
 Rated Capacity: 4400mAh(25°C)
 Standard: GB15313-2014
 Model: 04112V10174
 Model: 1.0A-06-01C

iRay Technology Taicang Ltd.
 No.33 Xinggang Road, Taicang Port Economic and Technological Development Zone, Jiangsu, China
 www.iraygroup.com


EC REP iRay Europe GmbH
 In den Dorfriesen 14, 71720 Oberstenfeld Germany

CAUTION:
 • Please do not short-circuit the battery terminals.
 • Please do not use the battery in a fire or other high-temperature environment.
 • Please do not use the battery in a humid or corrosive environment.
 • Please do not use the battery in a high-voltage or high-current environment.
 • Please do not use the battery in a high-vibration environment.
 • Please do not use the battery in a high-pressure environment.
 • Please do not use the battery in a high-magnetic field environment.
 • Please do not use the battery in a high-radiation environment.
 • Please do not use the battery in a high-temperature environment.
 • Please do not use the battery in a high-humidity environment.
 • Please do not use the battery in a high-corrosive environment.
 • Please do not use the battery in a high-voltage or high-current environment.
 • Please do not use the battery in a high-vibration environment.
 • Please do not use the battery in a high-pressure environment.
 • Please do not use the battery in a high-magnetic field environment.
 • Please do not use the battery in a high-radiation environment.

SN 01290013C027330482

Example type label iRay battery

10.2.4 INTERFACE BOX

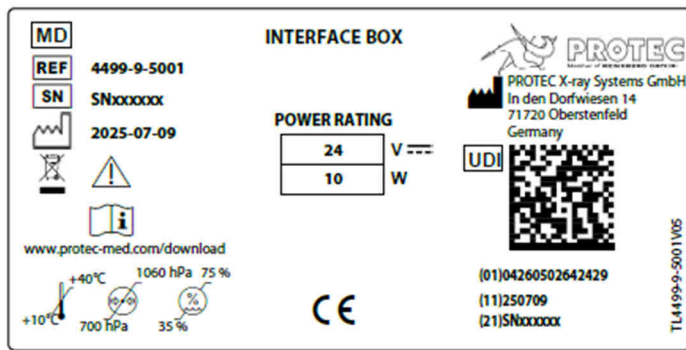


Example type label INTERFACE BOX

10.2.5 CONAXX 2



Example type label CONAXX 2

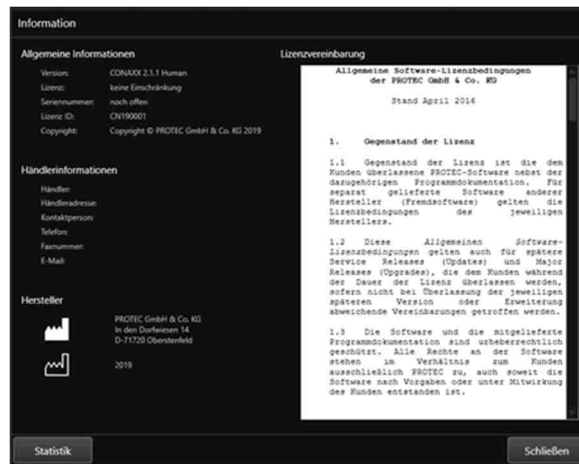

Example specification in the CONAXX 2 interface

10.3 Positions of the Signs and Labels

10.3.1 Detector



The type labels are located on the underside of the detector.

10.3.2 Battery charging station



The type label is located on the underside of the battery charging station.

10.3.3 Battery



The type label is printed on the bottom of the battery.

10.3.4 INTERFACE BOX



The type labels are located on the underside of the INTERFACE BOX

10.4 Abbreviations

mm	Millimeter
cm	Centimeter
m	Meter
µm	Micrometer
µGy	Microgray
kg	Kilogram
°C	Degree - Celsius
hPa	Hectopascal

DIN	German Industry Standard
EN	European Standard
CE	CE-Mark
Hz	Hertz
A	Ampere
SN	Serial number
EMC	Electromagnetic compatibility
HF	High frequency
LP/mm	Line pairs per millimeter
CsI	Cesium iodide
IP	International Protection